


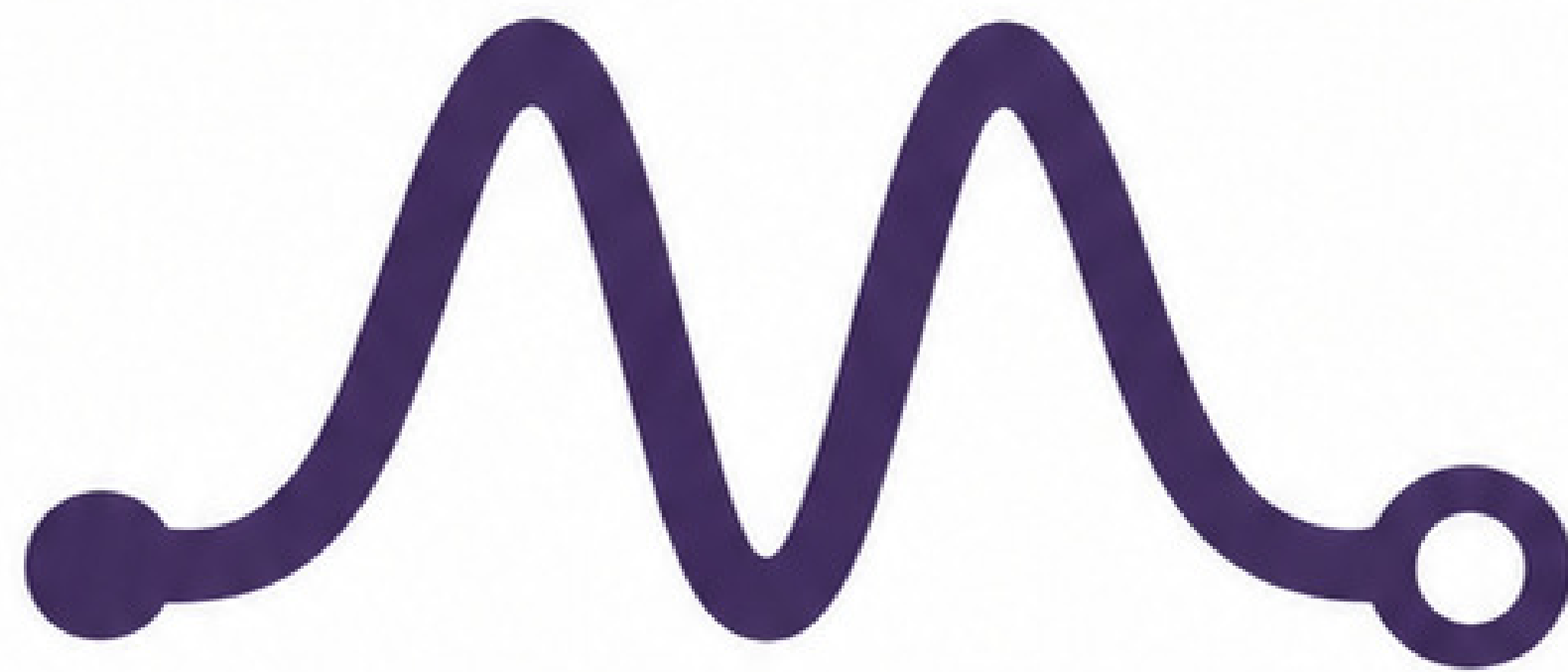

NeuroMyst

Aero



Read Me

NeuroMyst IS

- A low power device for general wellness.
- A TES device that applies tDCS, tACS, or other waveforms.

NeuroMyst IS NOT

- A medical device.
- Intended to treat, cure, or diagnose a medical condition.

Warning! Do NOT use if you


- Have a history of headache, migraine, or seizures
- Are or may be pregnant
- Are <18 years of age
- Are intoxicated or incapacitated
- Have a neurological impairment or an active electronic implant and have not first consulted your physician

For your safety, Do NOT

- Apply electrodes to the chest or unhealthy skin
- Apply stimulation that is uncomfortable or painful

You might experience

- Mild skin sensations
- Harmless light flashes
- Very mild skin redness

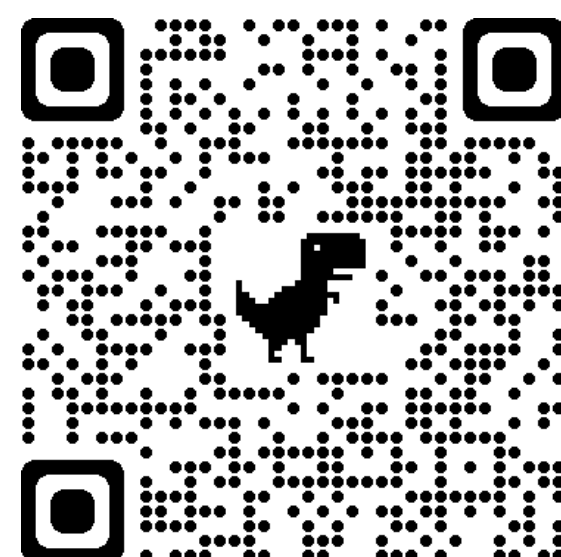


Watch Me


Point your phone camera at the tiny dots and click the yellow link.

Learn More


Website

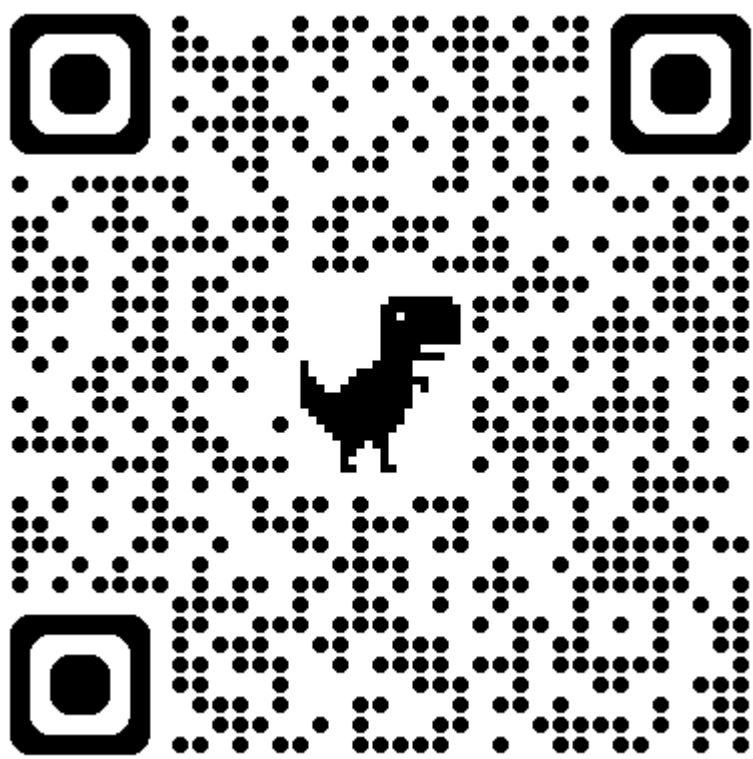


Aero Setup

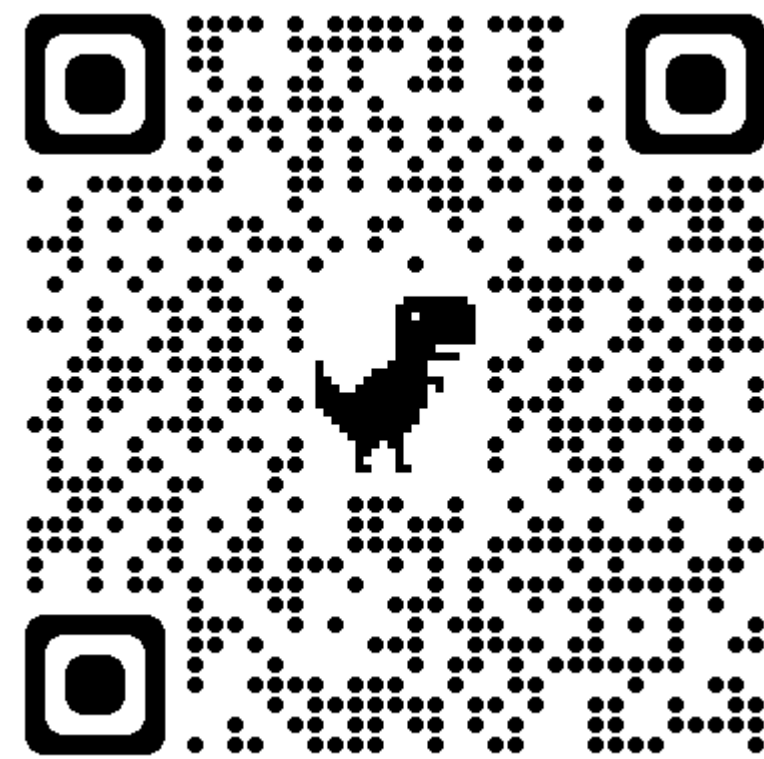

iOS App

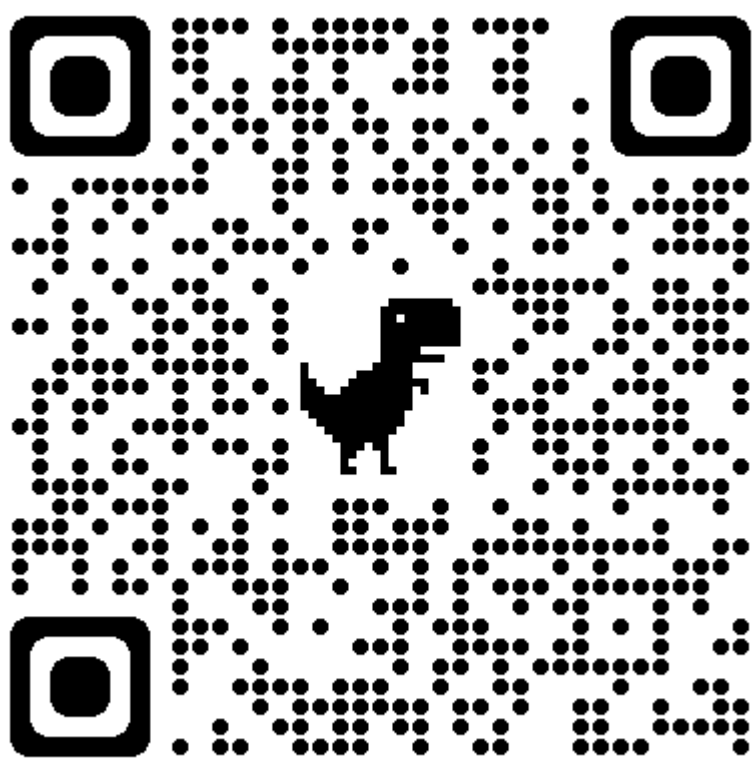


Android App

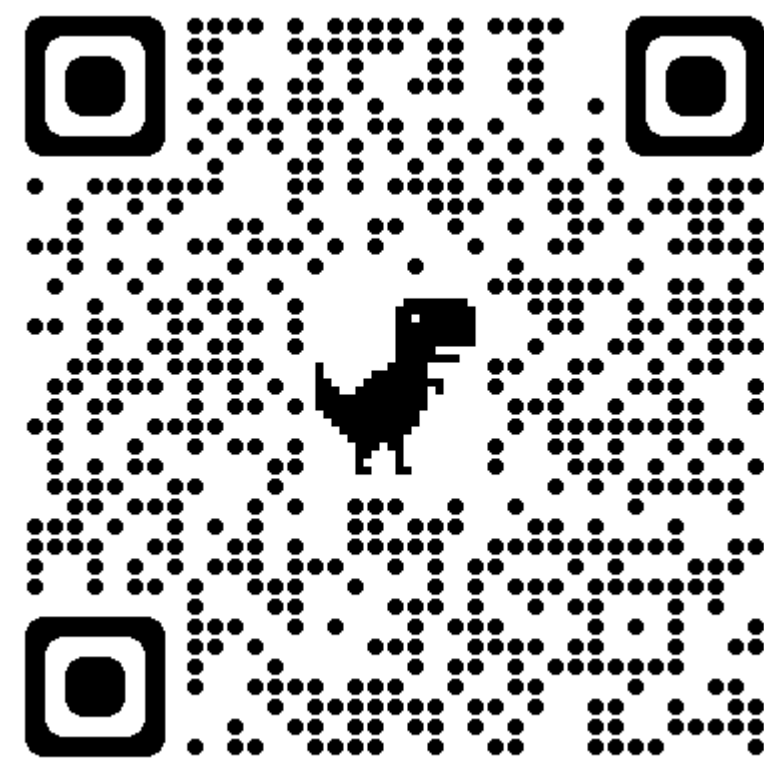

Point your phone camera at a QR code and click the yellow link.

STEP 1 OF 4

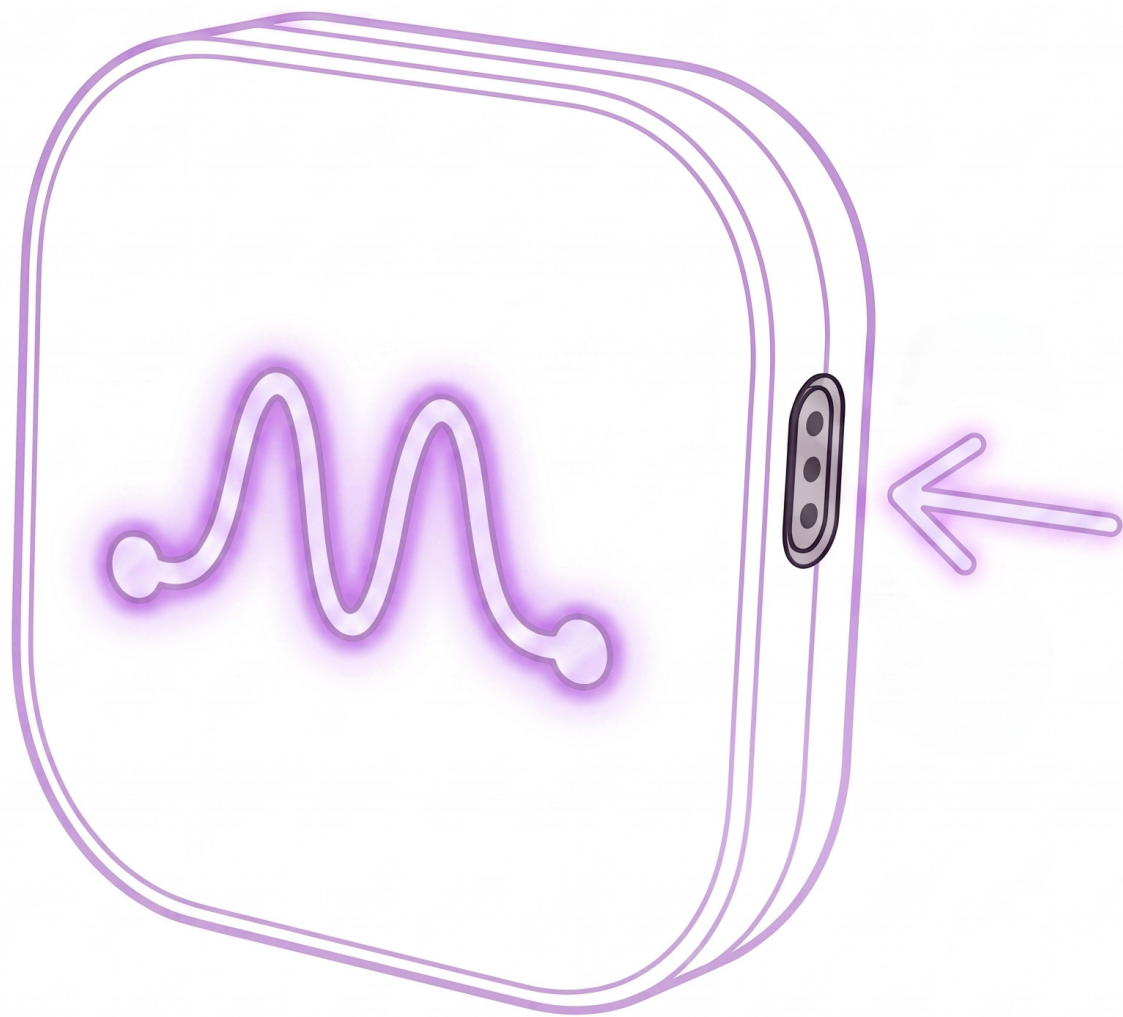


A. Wake up Aero

Aero will pulse purple near your phone or with a side button push.




B. Turn on Bluetooth

Enable Bluetooth on your phone and keep Aero nearby.

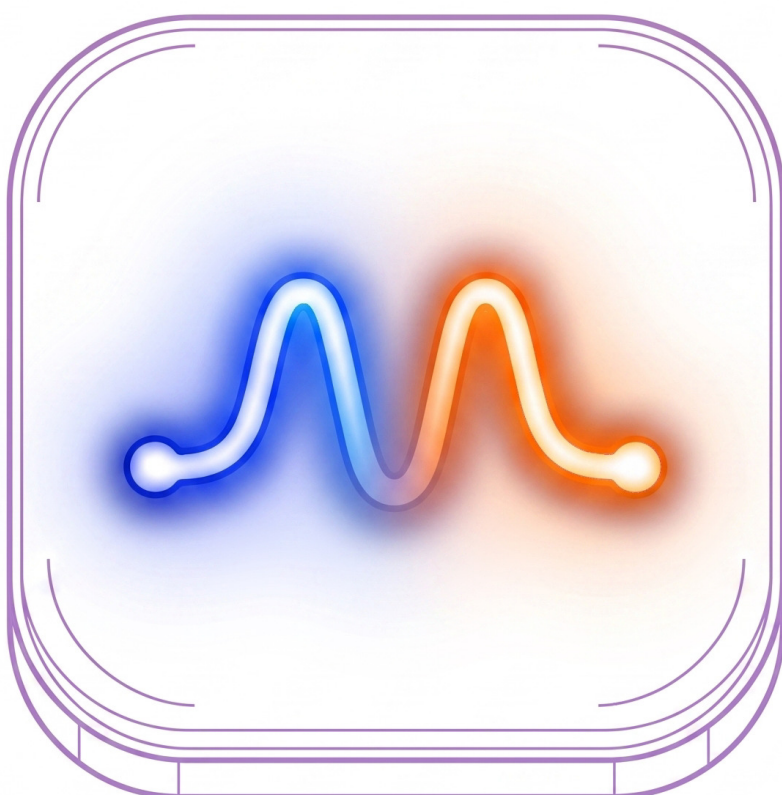

C. Connecting...


Aero will pulse blue and orange while connecting to your phone.

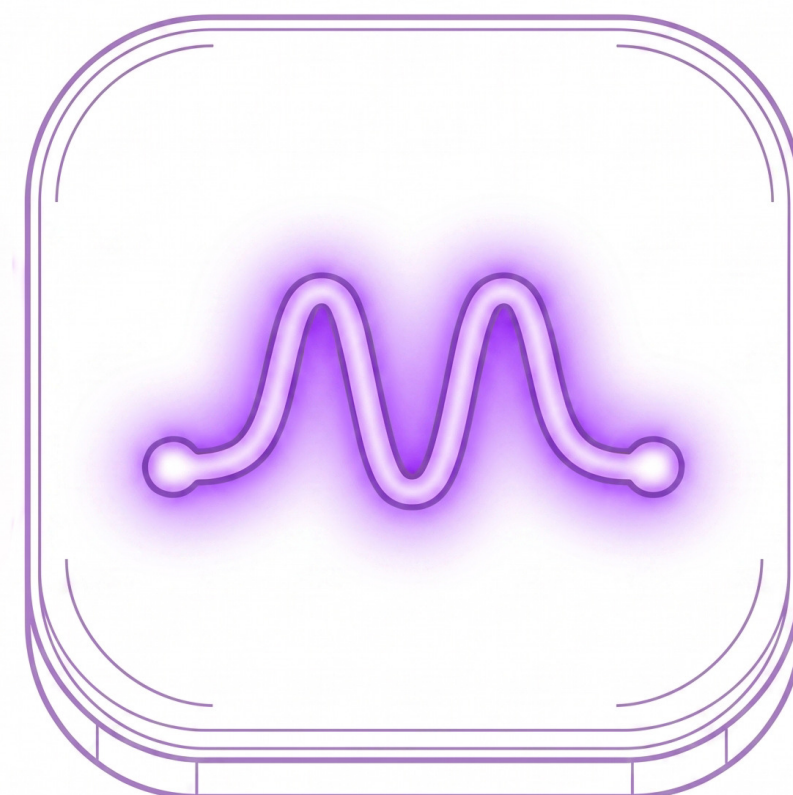

D. Aero Connected

Aero will again pulse purple once connected to your phone.


STEP 2 OF 4

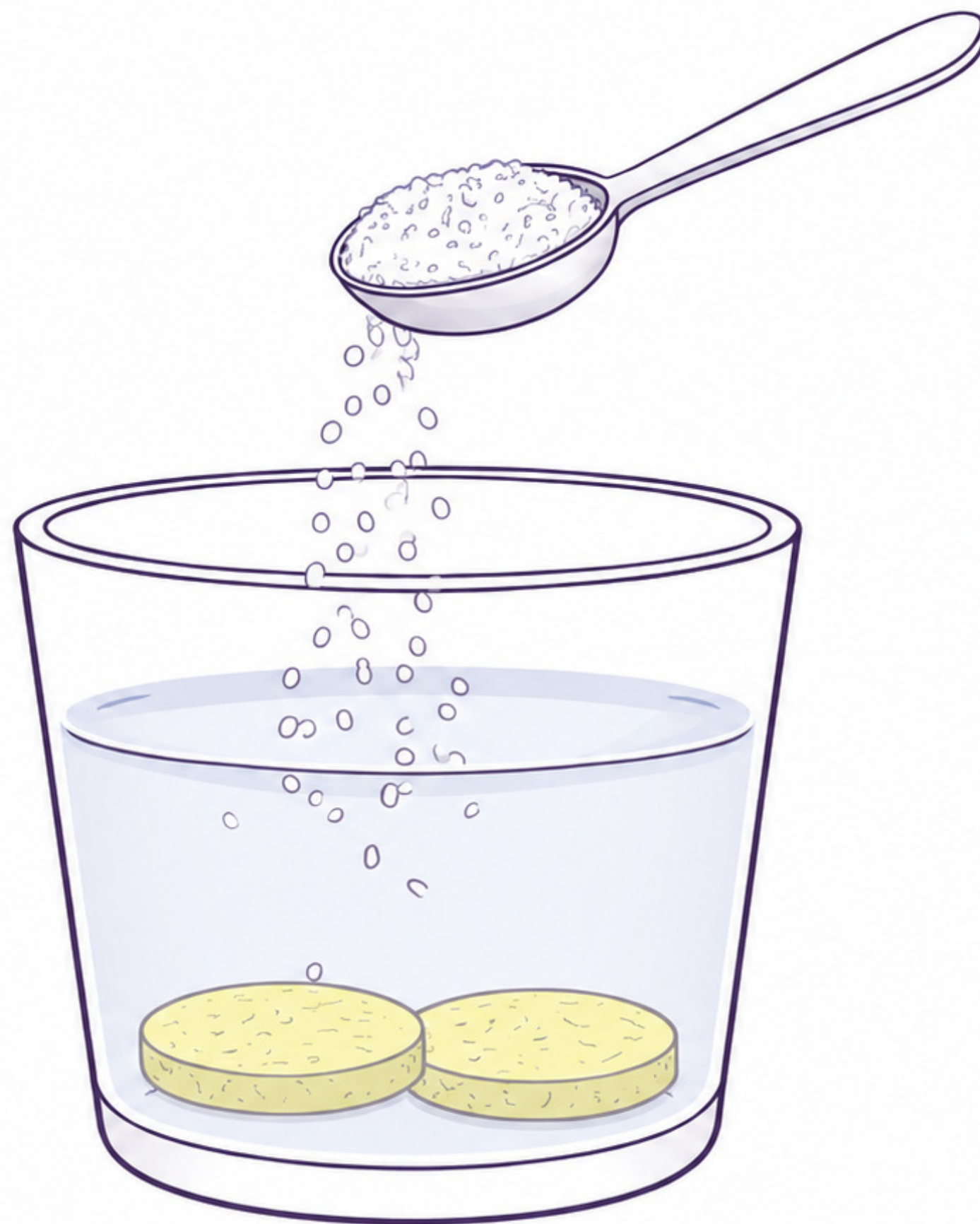

Wet the Electrodes

Lightly wet both sponges with salt water ($\frac{3}{4}$ tsp of salt per $\frac{1}{2}$ L).


STEP 3 OF 4

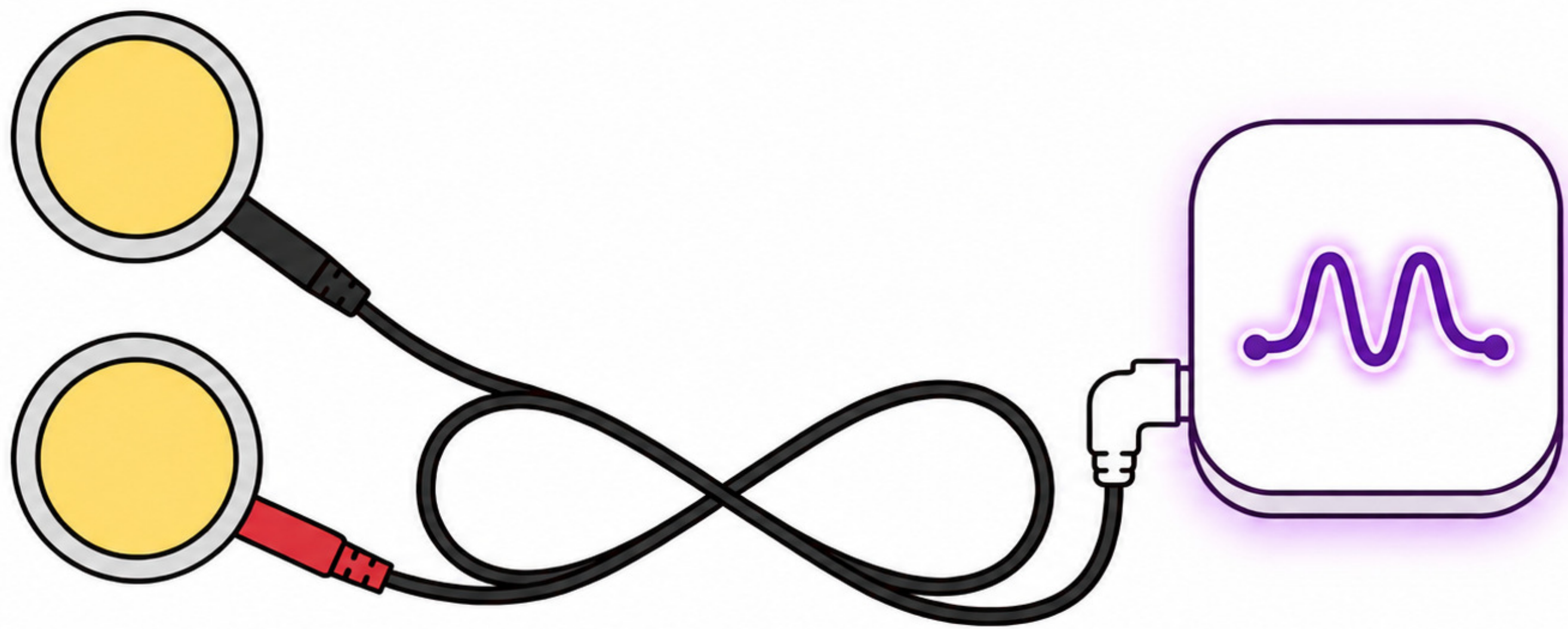

Attach the Electrodes

Assemble the sponge electrodes, insert one red/black pin into each electrode, and connect the lead wire to Aero.

STEP 4 OF 4

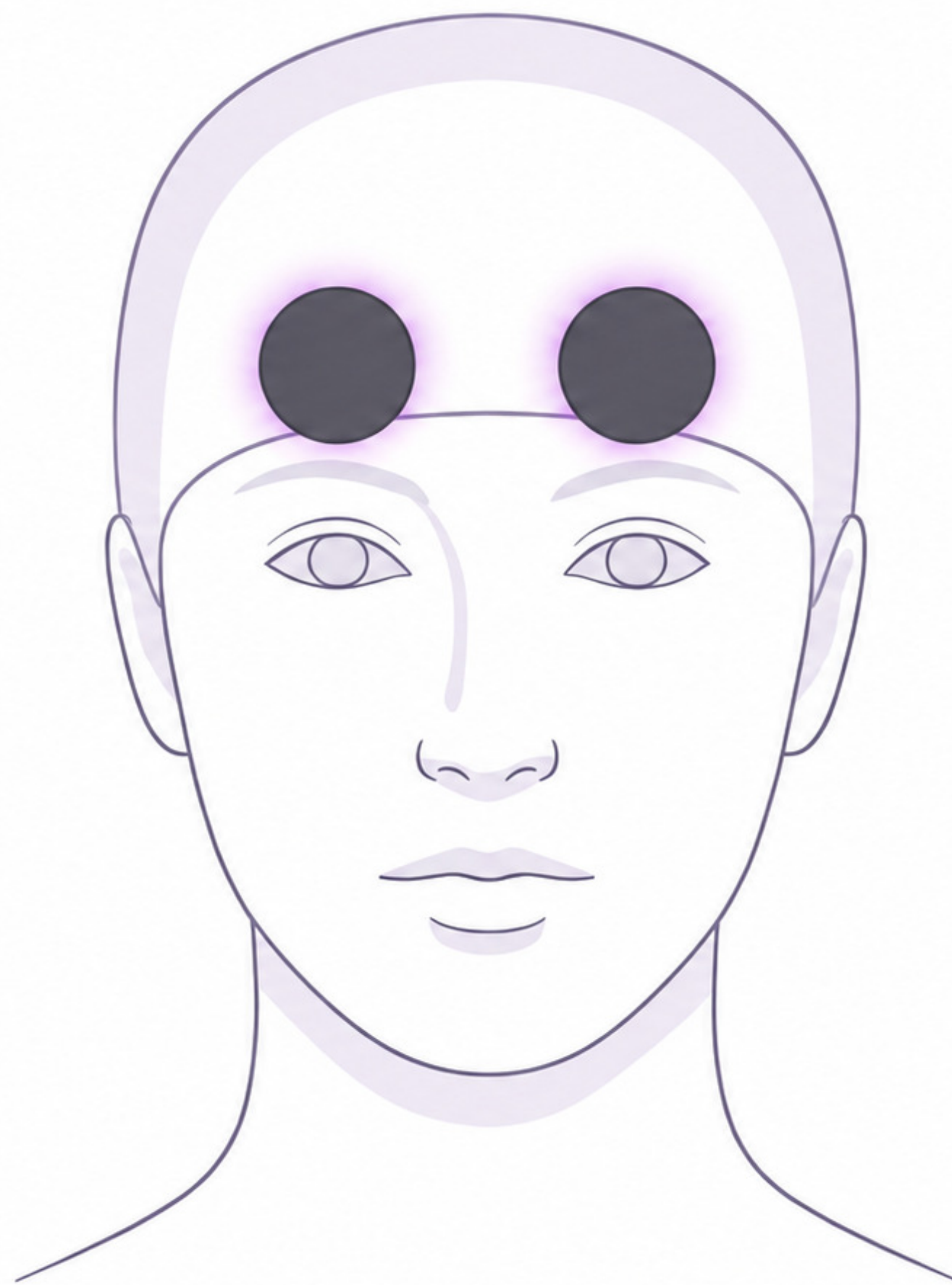

Place & Secure

Put on the skullcap, and then slide the electrodes underneath the cap to the target montage locations.

Terms & Warranty

Limited Output Device. NeuroMyst is not a medical device. It is a consumer wellness device for transcranial electrical stimulation (TES). Results may vary based on the user, setup, and other factors. Keep Aero nearby and charged so it turns on automatically. If needed, tap the right button to wake it.

Assumption of Risk. Using TES involves exposure to low electrical currents (up to 4 mA) and may cause temporary skin or visual sensations. By using this device, you accept all known and unknown risks associated with its use.

Voluntary Agreement. By using NeuroMyst, you confirm that you are 18 years or older, have read and understood this manual, and accept responsibility for your use of the device.

One Year Limited Warranty. NeuroMyst is covered by a one-year limited warranty against defects in materials and workmanship. The warranty does not cover normal wear, misuse, or unauthorized repairs. The product is otherwise provided "as is." Some warranty limitations may not apply depending on your state or province.

FCC Information

FCC ID: 2BXDG-A1000

FCC Warning

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation.

NOTE 1: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE 2: Any changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.