

Instruction manual

Aid2



Unleash your hearing, embrace the world



Product Specification

Bluetooth version: 5.3

Battery input: DC 5V

Earphone battery: 50mAH

Charge case battery: 250 mAh

Working time (Hearing mode): About 8~9 hours

Working current: < 7mA

Maximum OSPL90: < 115dB + 3dB

EQ Input Noise: <15dB



Product Bluetooth and hearing aid mode cannot be used at the same time.

(When using Bluetooth mode, the hearing aid function is automatically turned off, and the Bluetooth volume is set to the maximum.)



Operation instructions

Power on default mode: Hearing enhancement mode

Touch and hold L (or R) side 3 sec then release: Switch Hearing enhancement mode/Bluetooth mode

Hearing mode

Touch once on L/R: Hearing Volume+ (volume from small to large loop)

※ A total of 4 levels of volume, hear the "du, du du" Sound, the volume is maximum

Bluetooth mode

Touch twice on L: Previous song

Touch twice on R: Next song

Touch once L/R: Play/Pause Answer /End a call

Touch three L/R: wake up siri.





Product schematic



① Charge Case

② LED Display

③ Charging Port

④ Earphone Indicator

⑤ Touch Area

⑥ Earphone Charging Port

Charge case

When charging: LED display flashes and shows real electricity

Charge earphone: LED display shows real electricity

Earphone

Charging: The indicator light will turn green while charging, and blue after fully charged for a few seconds before turning off.

Earphone pairing: Green and blue flash in both 2 earbuds, and once paired successfully, lights turn off.

Connect successfully: lights turn off.

Charging time: about 40-60minut

Suggestion

1. Please fully charge the new machine before use, when charging with charge case, please use the

5V/1A charger. Please don't use the fast charger which over 5V, to avoid damage on the charge case.

2. To ensure the battery life of the earphone, the earphones and the charge case should be charged once in a month.

Pairing for operating

Step1. Touch and hold the left or right earphones for 3 seconds to switch Bluetooth mode, hear the pairing voice prompt, the pairing is successful, and the earphone's lights go off.

Step2. At this point, open the mobile phone's Bluetooth to search earphones' Bluetooth name then connect with it.

※ When the earphones connect to the same device again, open the lid of charge case it will turn on

automatically then reconnect with the device, voice prompt " Connected ".

Turn on the earphone

Method A. Touch and hold the area for 5 seconds.

Method B. Open the lid of the charge case, they will automatically power on, and enter automatic pairing.

Turn on the earphone

Method A. Touch and hold the area for 5 seconds.

Method B. Put earphones into the charge case, they will automatically shut down and enter charging.

Calling

When connected with mobile phone, touch once anyearphone to answer current call. Touch 2 seconds to reject the call. If you want to end the current call, touch once any earphone to stop.



FAQ

A. When the earphones can't be turned on

Check if the earphone has power (after the charge case is connected to the USB, and the earphone is charged in the charge case for more than 1 hour).

B. When the earphones have only one side sound or earphones can't pair successfully

1. Make sure that both earphones are powered and can be turned on normally.
2. Touch 5 seconds to turn on the earphones or take the earphone out of the charge case (when the charge case is powered) the earphone will automatically power on, earphones automatically enter the pairing mode. When the earphone's LEDI will turn off. the pairing is successful.

C. When the mobile phone cannot find the device

1. Confirm that the two earphones are turned on.
2. Check if the earphone is connected to other devices.
3. Change an area and reconnect it again.

D. When the earphone is stuck or disconnect while talking or playing music

1. The earphone is low on battery.
2. The distance between the mobile phone and the earphone is more than 10 meters.
3. There is large area of signal obstruction between the mobile phone and the earphone, such as a wall.
4. The left and right ears are not paired and synchronized, and need to be shut down and re-paired.

E. After using up the battery of the charging case, the earbuds may automatically turn on and produce a high-pitched noise while in the case.

In this case, you can manually turn off the earbuds by long-pressing the touch button or charge the case to resolve the issue.



Attention

1. For the first use, please take out the earphones to remove the insulating film at the charging position of the earphones, put the earphones into the charging case and use them after charging is completed.
2. Please ensure that the earbuds are properly worn before turning on the assistive listening function, as improper usage may cause feedback.

3. After use, please turn off the assistive listening function by touching the control button and then store the earbuds into the charging case. Failure to turn off the assistive listening function before storing may cause feedback.
4. Please don't change the built-in battery by yourself.
5. Please store device in dry area away from water or rain.
6. Please don't expose the earphone to excessive heat to avoid battery damage.
7. Please don't disassemble the earphones.



Warning

1. This product cannot fully replace a medical hearing aid and is not recommended for use by minors.
2. Do not play music while driving or cycling, as it may lead to traffic accidents

