Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.

1. IMPORTANT SAFETY NOTES

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

	Safety note indicating possible damage to the device/accessory
	Warning instruction indicating a risk of injury or damage to health
	Indicates that caution should be exercised if devices and controllers in the vicinity of this symbol are being operated, or that the current situation requires prudence or intervention on the part of the user to avoid undesired consequences
	Addresses practices not related to personal injury, such as product and/ or property damage. Product information Note on important information
	Observe the instructions Read the instructions before starting work and/or operating devices or machines
	Application parts type BF
	Direct current The device is suitable for use with direct current only
	Permissible storage and transport temperature and humidity
	Permissible operating temperature and humidity
	Device protected against foreign objects ≥ 12.5 mm and against water dripping at an angle
	Serial number

! WARNING

- Do not use the EKG monitor on newborns, children or pets.
- People with restricted physical, sensory or mental skills should be supervised by a person responsible for their safety and receive instructions from this person on how to use the device.
- If you have any of the following conditions, it is essential you consult your doctor before using the device: cardiac arrhythmia, circulatory problems, diabetes, pregnancy, pre-eclampsia, hypotension, chills, shaking.
- Do not use the device if you have implanted electrical implants (e.g. pacemaker).
- Do not use the device if you have metal implants.
- Do not use the device with a defibrillator.
- Do not use the device during an MRI scan.
- Do not use the device with persons with sensitive skin or allergies.
- Do not self-diagnose or self-medicate on the basis of the device measurement without consulting your doctor.
- The device displays changes in the heart rhythm, which may have various different causes. These may be harmless, but may also be triggered by illnesses or diseases of differing degrees of severity. Please consult a medical specialist if you believe you may have an illness or disease.
- The device is not a substitute for a medical examination of your heart function or for medical electrocardiogram recordings, which require more complex measurements.
- Do not use the device at the same time as other medical electrical devices (ME equipment). This could cause the measuring device to malfunction and/or an inaccurate measurement.
- Do not use or store the device in the following ambient conditions:
 - Near fires or open flames
 - Locations exposed to strong vibration
 - Locations exposed to strong electromagnetic fields

! CAUTION

- Do not expose the device to any static electricity. Always ensure that you are free of static electricity before operating the device. Static electricity can damage electronic components acc. to Shella: Redundant statement, delete Contact with grounded objects or devices will discharge the energy that has built up and cause electrostatic discharge.
- Do not place the device in pressure vessels or gas sterilisation devices.
- Do not use the device outside of the specified storage and operating conditions. This could lead to incorrect measurements or damage to device.
- Do not drop, step on or shake the device.
- Do not dismantle the device as this may result in damage, defects or malfunctions.
- Do not allow the electrodes of the device to come into contact with other conductive parts (including earth).

Battery Handling Safety Precautions

- Do not remove or attempt to remove the non-user-replaceable battery.
- Risk of explosion! Never throw batteries into a fire.
- Batteries must not be short-circuited.
- Only use chargers specified in the instructions for use.
- Fully charge the battery prior to initial use and at least every 3 months.
- Protect batteries from excessive heat.
- Keep batteries away from children and pwenets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Do not disassemble, open or crush the batteries.

2. INTENDED USE

The device is a handheld, personal electrocardiograph unit, which can measure electrical activities of the heart easily and conveniently. It is immediately available at any time to manually record transient cardiac events, suitable for home health care use, which can detect, display and store ECG signal, and if possible, provide average heart rate message after ECG measurement. The users can use it themselves to check their heart condition. It is suitable for the adult users, who suffers from cardiovascular diseases, or the adult people who are caring about their heart working conditions during their daily life. This device is not intended for use as a conventional diagnostic tool, but use as a healthcare tool which can provide doctor the recorded data as references. The product is not a conventional diagnostic tool.

3. PACKAGE CONTENTS

- 1 x EKG Monitor
- 1 x USB Cable
- 1 x Instruction Manual
- 1 x Quick Start Guide

4. DEVICE DESCRIPTION

- LCD Display
- Bottom electrode
- Top electrodes
- Power Button
- USB Charging Port

- Time
- Bluetooth®
- Battery
- Average heart rate of EKG measurement
- Heartbeat during the measurement
- Measuring time
- EKG Rhythm
- Measurement countdown

5. USAGE

In the entire text of this manual, the term "electrocardiogram" will be abbreviated as "ECG" or "EKG" for easier reference.

5.1 Charging the battery


Connect the USB charging cable included in delivery to the USB charging port and to a USB power supply or computer. Whilst charging, the current battery status is displayed on the display.

- The device cannot be used while it is being charged.

As soon as the battery is fully charged, appears on the display and you can unplug the charging cable from the power supply and the charging port.

5.2 Measuring

- To turn on the device press and hold the Power Button for about two seconds until the device emits a beep.
- Hold the device using one of the ways illustrated below. Choose the most appropriate measuring location on your body for your particular use. The most suitable method/ procedure depends on the heart configuration (shape of the heart) of each individual user. If it is not possible to take stable measurements using a given measurement method, this could have a harmless cause such as the shape of the heart. However, the cause may also be an illness or disease.

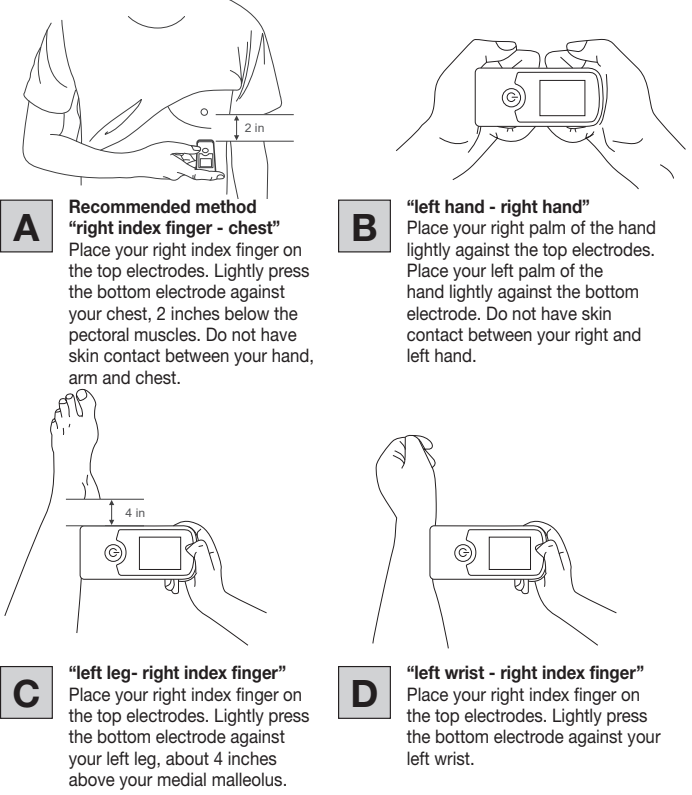- When the heart rate becomes stable, the EKG rhythm on the display turns green and the 30-second-measurement starts automatically. You can view the measurement countdown in the lower right of the LCD Display.
- After each measurement concludes, the results will appear in the LCD Display.
- To start another measurement, place the device against your skin again and hold firmly.
- To shut the device off, press and hold the Power Button for about two seconds until the LCD Display shuts off. The device will also shut off automatically if it is not used for about one minute.

! How to measure correctly:

- If you have dry skin, moisten it with a damp cloth or with disinfectant alcohol before measuring.
- Sit comfortably and upright.
- Never use the device on top of clothing.
- Press only lightly on the electrodes of the device, do not press too hard.
- During measurement do not have skin contact between your right and left hand (method B) or your hand and chest (method A).
- Make sure both electrodes fully contact the skin until measurement completed.
- Do not speak or move during measuring.
- Repeat measurement if unsure of results.
- To ensure accurate comparison of values, always use the same measurement method.

5.3 Interpreting the results

When each measurement concludes, a message will appear in the LCD Display informing you of the results. For any message other than „No Abnormalities“, repeat the measurement and consult your physician for advice. The measurement taken by you are for your information only – they are no substitute for a medical examination.

5.4 Memory

To enter memory mode, press the power button, when the device is turned on. Press the Power Button again to cycle through all saved results (up to 100).

6. USE WITH APP

- System requirements:
iOS ≥ 14.0
Android ≥ 7.0
Bluetooth® ≥ 4.0

- Download the free "Mini ECG" app from the Apple App Store or Google Play.

- Activate Bluetooth® in your smartphone's settings.
- Start the app.
- The device will connect to the app automatically.

- Click on the device in the app and follow the instructions.
- In „History,“ a list of measurements is displayed. Click on a measurement to enter the EKG detail page, where you can download and share the data.

7. CARE, MAINTENANCE AND DISPOSAL

- Never submerge the device in water or other liquids. Do not clean the device with acetone or other volatile solutions. Clean the device with a cloth dampened with water or a mild cleaning liquid. Then dry the device using a dry cloth.
- When storing the device, make sure no heavy objects are placed on top of it.

8. DISPOSAL

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

9. TROUBLESHOOTING GUIDE

Problem	Possible cause	Solution
The device will not switch on or appears on the display	The battery is empty	Charge the battery.
Wireless communication failure	There is a problem with the connection between the device and the app.	Turn off the device, close the connection between the device and the app. Reactivate Bluetooth® in your smartphone's settings and start the app.
Heart Rate becomes not stable, EKG rhythm on the display does not turn green	The EKG signal is unstable or too weak	- Moisten your skin - Sit comfortably and upright - Do not move or speak - Do not have skin contact between your right and left hand - Enter an environment with few electromagnetic noise - Repeat the measurement as per the instructions

10. TECHNICAL SPECIFICATIONS

Type	ME 75
Measuring system	Single-channel ECG in freely selectable front positions / ECG signal ground (earth) referenced.
Bandwidth	1 – 20 Hz
Sample rate	250 Hz
Measurement range for heart rate	30 to 300 beats per min
Memory	100 memory spaces max
Power supply	Internal Lithium-ion Battery Nominal capacity: 430 mAh Nominal voltage: 3.7V
External power supply (adapter not included)	Input: 100-240 V AC 50/60Hz Output: 5 V DC 1A
Battery lifetime	approx. 300 measurements
Storage / Transport conditions	-40 °F - 131 °F (-40 °C - 55 °C), ≤ 95% relative humidity
Operating conditions	41°F - 104°F (5 °C - 40 °C), < 80% relative humidity
Weight	approx. 0.13 lb (60 g)
Dimensions	approx. 3.94 x 1.77 x 0.59 in (100 x 45 x 15 mm)

The serial number is located on the device.

11. FCC COMPLIANCE INFORMATION

EKG Monitor ME 75
Responsible Party – U.S. Contact Information
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020
United States
1-800-536-0366
info@beurer.com

FCC Compliance Statement
This device complies with Part 15 of the FCC Rules.Operation is subject to the following two conditions:
(1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

12. WARRANTY

Limited Lifetime Warranty For Original Purchaser Your Beurer EKG Monitor, model ME 75, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners. We will, at our option, repair or replace the Beurer EKG Monitor, model ME 75, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/ power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED

TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/ or sealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www. shop-beurer.com

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020
USA
www.shop-beurer.com

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

Assembled in China

EMC Guidance

The ME EQUIPMENT or ME SYSTEM is suitable for home environments.

Warning: Don't use near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment , including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Technical description:

- All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the excepted service life.
- Guidance and manufacturer's declaration -electromagnetic emissions and Immunity

Emissions test	Compliance
RF emissions, CISPR 11	Group 1,
RF emissions, CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not applicable
Voltage fluctuations/ flicker emissions, IEC 61000-3-3	Not applicable

Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC 61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz

NOTE UT is the a.c. mains voltage prior to application of the test level.

	Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Test Level (V/m)	Compliance level (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380-390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27	27
	450	430-470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28	28
	710	704-745	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9	9
	780	787						
	810							
	870							
	930	900-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28	28
	1720		GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25 UMTS	Pulse modulation 217 Hz	2	0.3	28	28
	1845							
	1970	1700-1990						
	2450	2400-2570	Bluetooth®, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28	28
	5240							
	5500	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9	9
	5785							

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