

Smart Sock



BH08

User Manual

Please read the user manual carefully before using this appliance.

Safety Instructions


- Strangulation Hazard—Children have STRANGLED in cords. Keep this cord out of the reach of children (more than 3 ft (0.9 m) away).
 - NEVER place the cord in or near the crib or playpen.
 - Only use the AC adapters provided.
 - Never use the unit near water, including near a bathtub, sink, laundry machine tube, wet basement, etc.
 - Do not install near the product heat sources such as radiators, heat registers, stoves, or other devices that produce heat.
 - Only use attachments and accessories specified by the manufacturer.
 - Unplug this device when unused for long periods of time.
 - Unplug the product during lightning storms.
- 01 ● Do not disassemble or modify the equipment without authorization.

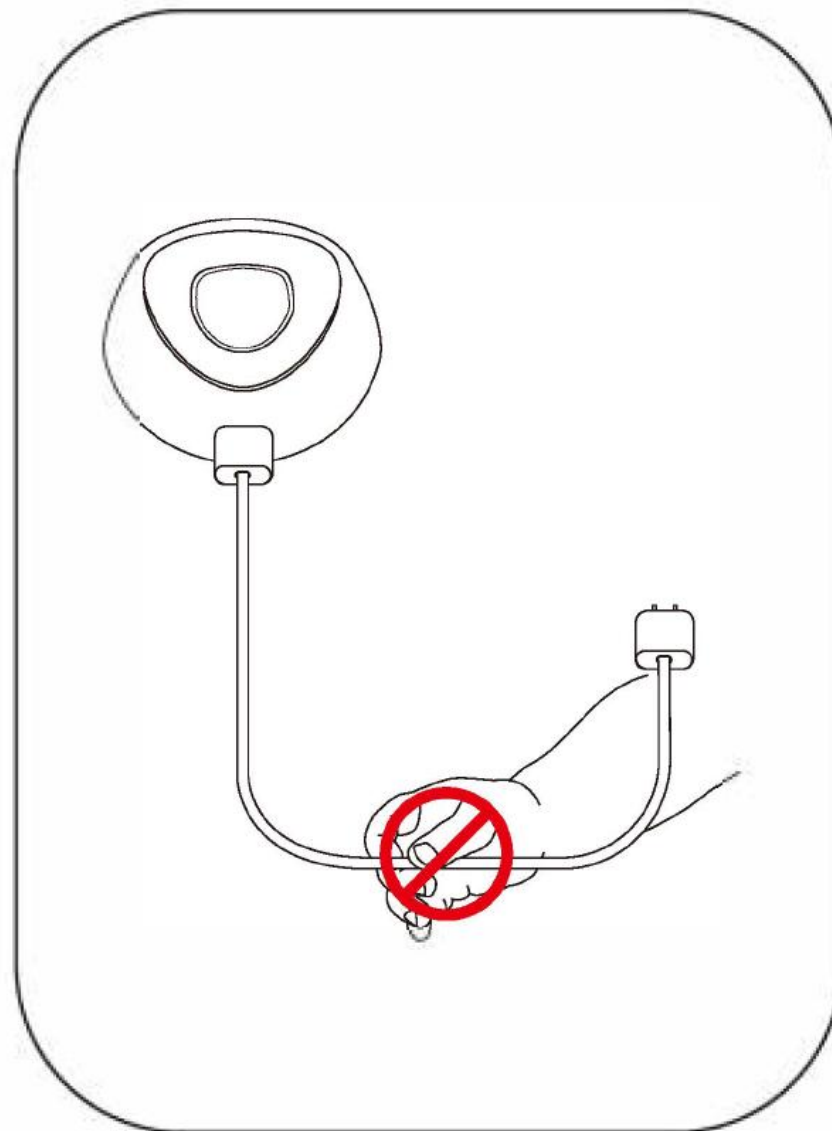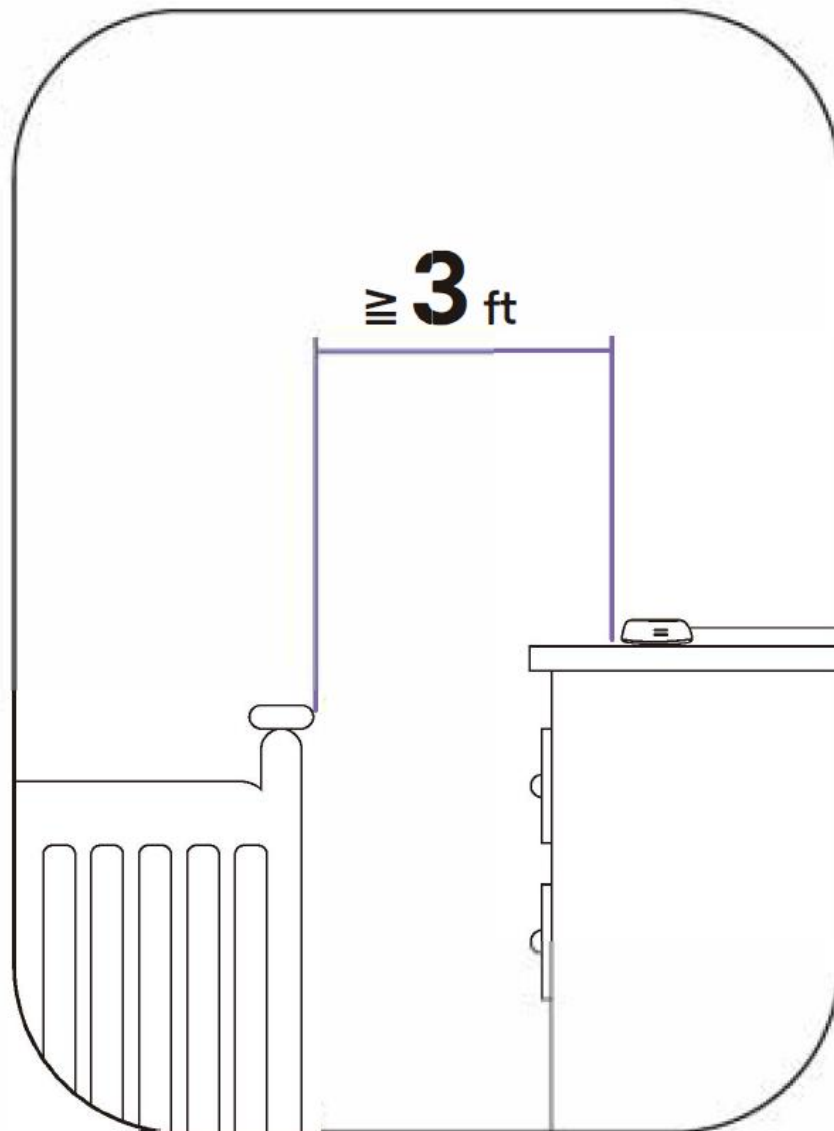


Cautions

- NOT toys. Do not allow children to play with them.
 - This product is NOT intended to replace adult supervision. You MUST check your child's activity regularly.
 - This product is not intended for use in medical devices, nor for the diagnosis, treatment, or prevention of disease.
 - Please ensure socks are dry and clean before wearing.
 - Do not use on infants with skin allergies or open wounds.
 - Please check the operating range and condition of the device carefully at the initial installation, and on a regular basis after that.
 - Check the reception regularly. Test the Charger before use and after changing the Charger's location.
 - For indoor use only.
- 03 ● Clean the Charger with a dry cloth only.


What's in The Box

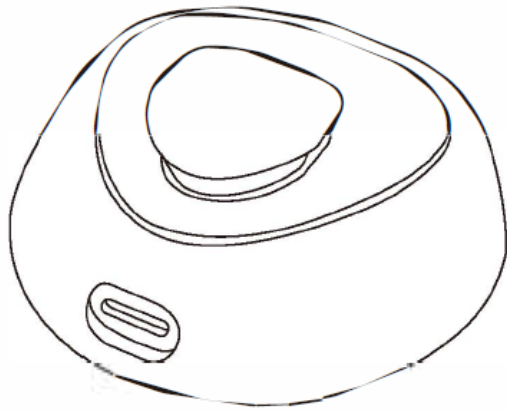


Sock Sensor * 1
Charging Station * 1

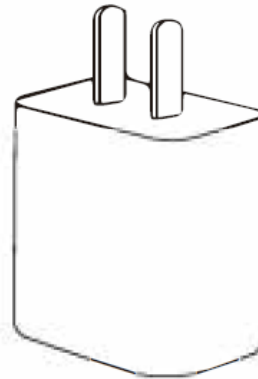

Power Adapter * 1

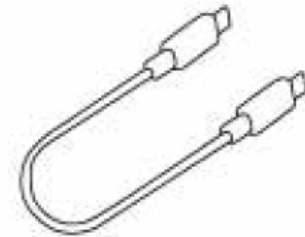

Type-C Charging Cable * 1

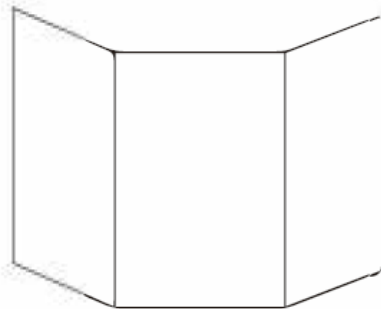

Quick Start Guide (FAQ) * 1



User Manual * 1




Size S Sock * 2



Size M Sock * 2

Product Overview

Charging Station

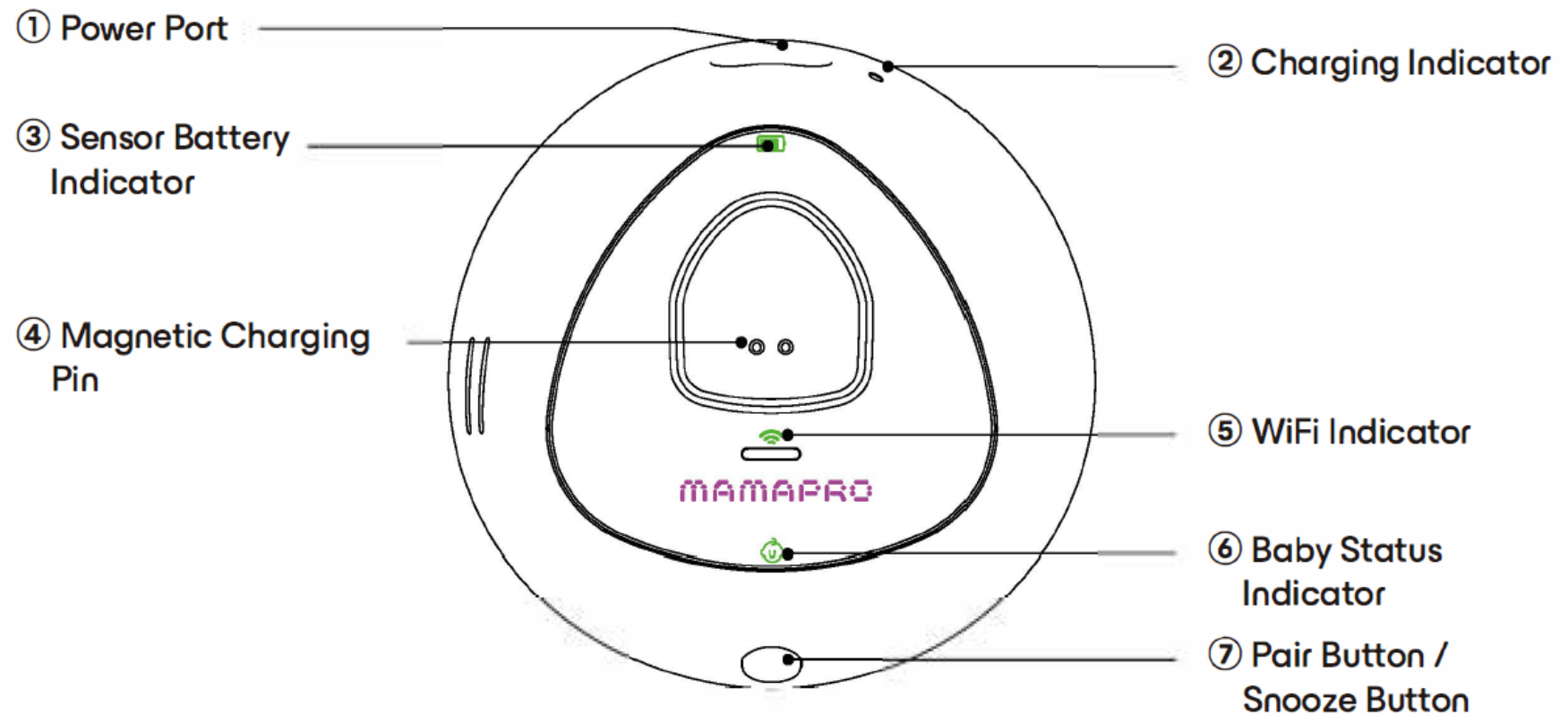


Icon	Indicator	Description
	Charging Station Battery	• Solid Green : ① Power source connected ② Charging Station charged fully • Solid Red : Charging station charging • Light Off : No power connected
	Sock Sensor Battery	• Solid Red: Sensor is charging • Flashing Red : Sensor battery low • Light Off : Sensor charged fully
	WiFi	• Solid Red: ① Not connected to WiFi ② WiFi network issue, WiFi disconnected • Flashing Red : Connecting to WiFi • Light Off : Connected to WiFi
	Baby Status	• Flashing Red : Abnormal status • Light Off : Usage and detection data normal

Smart Sock

Solid Red : Charging and connected

Solid Green : Charged and connected

Flashing Green for 3 seconds : Entering measurement mode

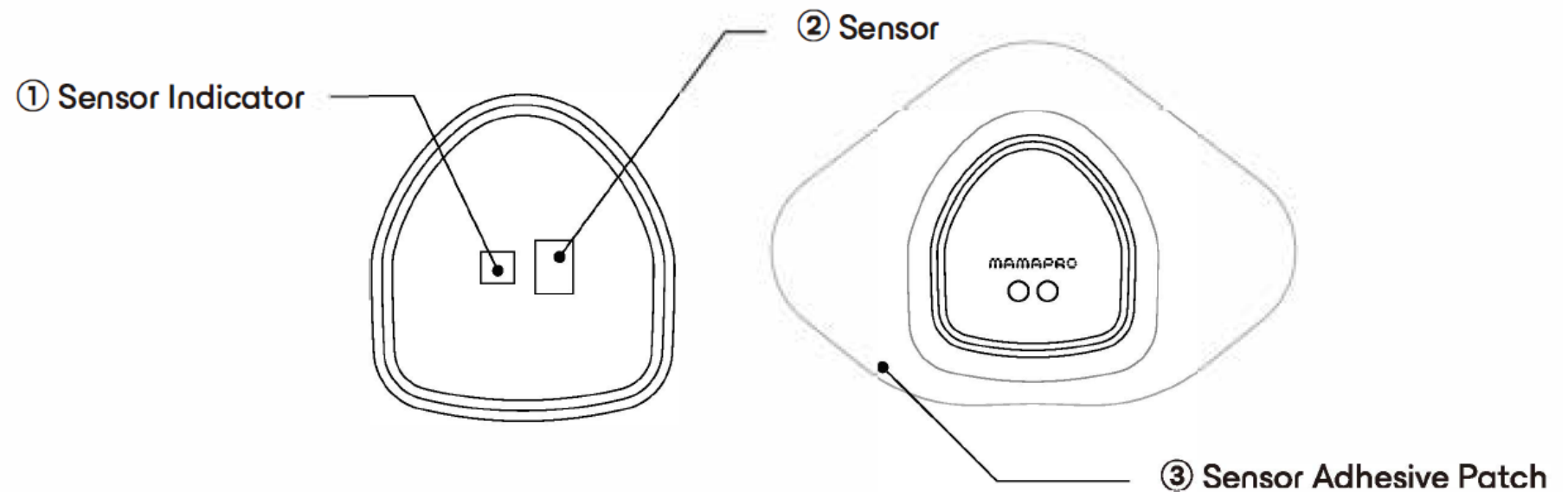

Product Specifications

Product Name : Smart Sock

Model Number : BH08

Input Voltage : DC 5V/1A

Power Adapter Input Voltage : 110-240VAC, 50/60Hz

Storage Temperature : -20°C to 60°C/ -4°F to 140°F

Operation Temperature : 0°C to 40°C/ 32°F to 104°F

How to Use

Preparation Before Use

1. The Smart Sock must connect to an iOS or Android device with the MamaPro App installed; minimum system requirements are detailed in the app store description.
2. 2.4GHz wireless network connection is required for live readings via the MamaPro app, remote access to the App and some App features.

Note: Hotspot Wi-Fi and 5GHz bands are not supported.

To Set Up the Smart Sock

1. Get the APP and account

Download and install the MamaPro app from the App Store (for iOS devices) or Google Play (for Android devices), then sign up for an account.




Download on the APP Store
GET IT ON Google Play

To Set Up the Smart Sock

2. Powering On the Charging Station

Place the Sensor on the Charging Station. Plug in the Charging Station to a power source, wait until the Charging Indicator turns white and vibrates to power on, then place the Sock Sensor into the charging port on the Station. The WiFi Indicator will remain solid red, indicating it has entered network configuration mode.

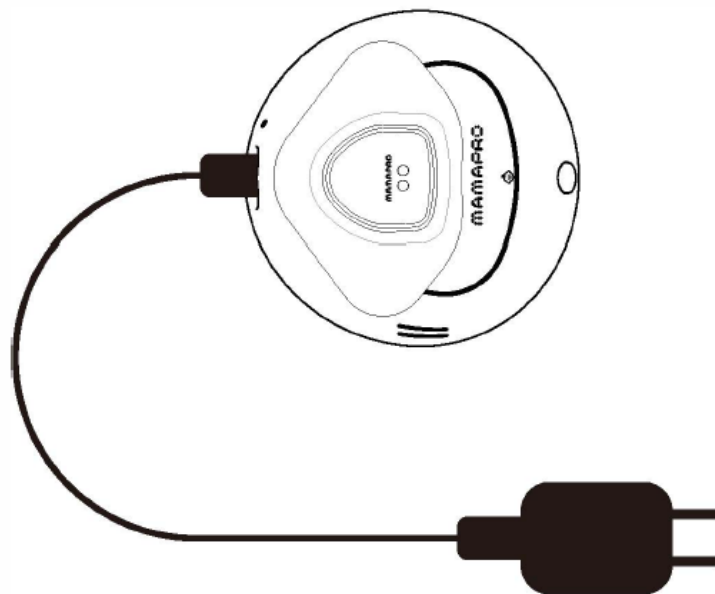


NOTE :

- Only use the power adapter included in the package.
- Do not place the Charging Station and power cord within the reach of your baby.
- Before connecting the device, the Sensor must be placed on the Charging Station to activate it; otherwise, pairing with the device will not be successful.

3. Set Up the Smart Sock


Follow the app instructions and tap "Get Started" or the "+" in the top right corner to add a device.

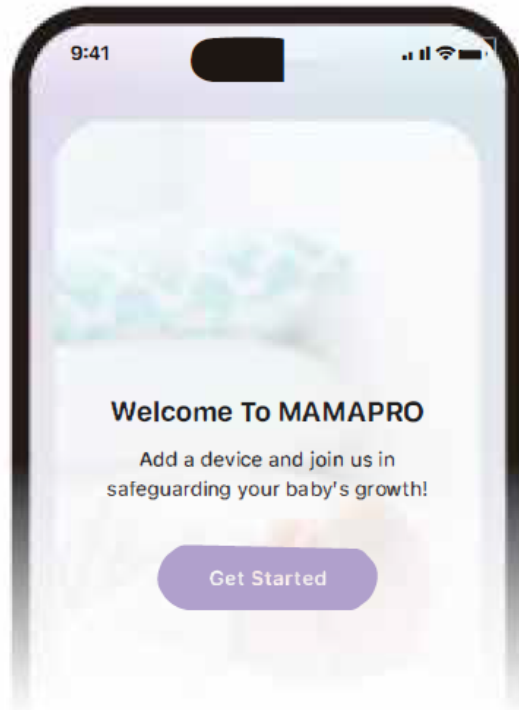

NOTE: The smart socks transmit data via a secure, encrypted 2.4GHz Wi-Fi connection. The baby camera must only be connected to a dedicated network requiring a security password.

Charging

- After each use, place the Sock Sensor back in the Charging Station to recharge.
- Once powered on, the Charging Station will automatically connect to the configured Wi-Fi network.

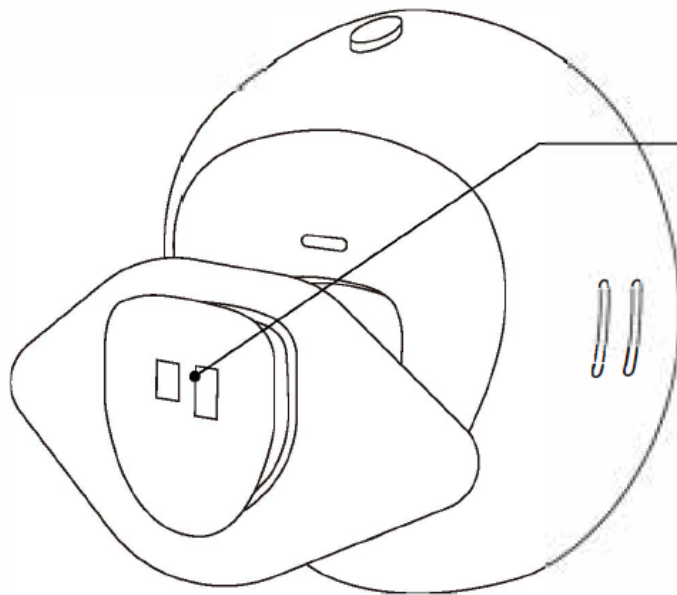

Sensor Indicator Description:

Solid Red : Charging and connected

Solid Green : Charging and connected

Flashing Green for 3 seconds : Entering measurement mode.




Solid Red




Solid Green

Wear And Use

1. Select the appropriate sock size (S / M).
2. Attach the sensor to the sock, aligning the Sensor orientation with the Sock's direction, and press firmly to secure the adhesive patch.

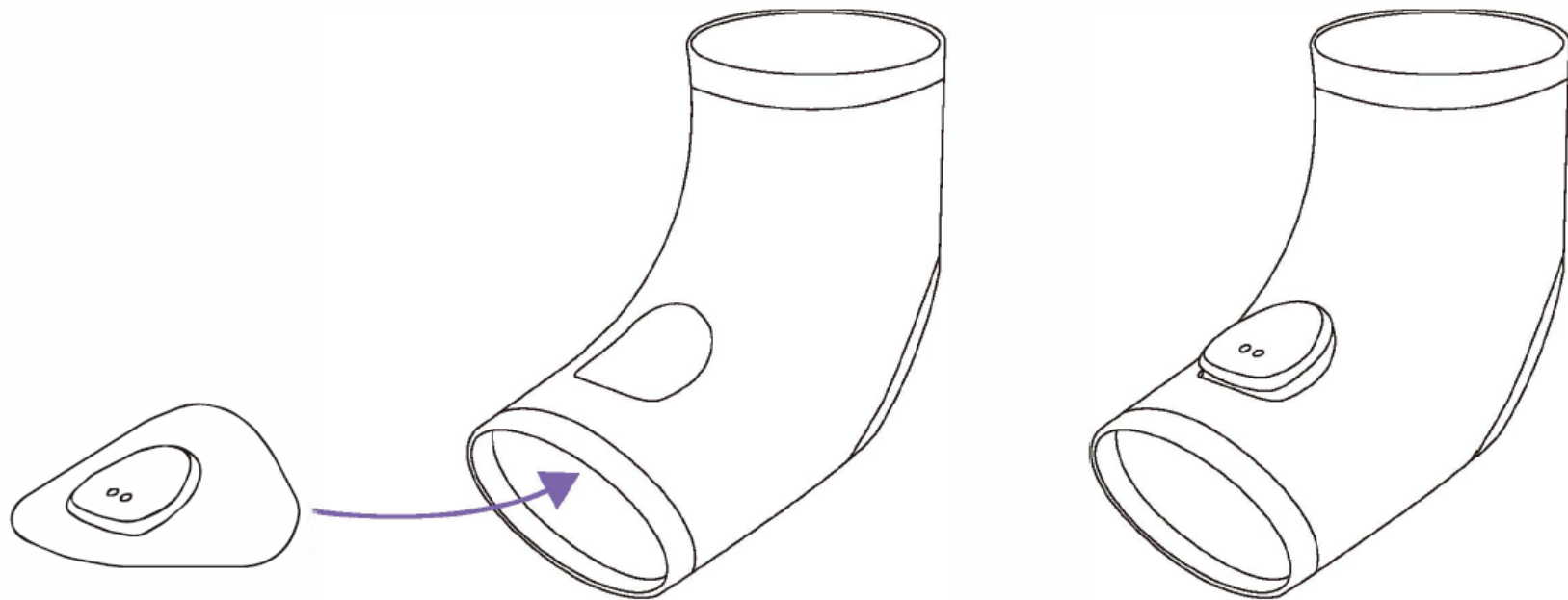

3. Put the Sock on your baby, ensuring the Sensor module rests against the center of the top of their foot.
4. After putting on the Sock, open the App to confirm the device is connected.
5. Once monitoring begins, you can view heart rate, blood oxygen levels, respiration, and sleep status. (Sleep status is viewable only after the baby falls asleep).

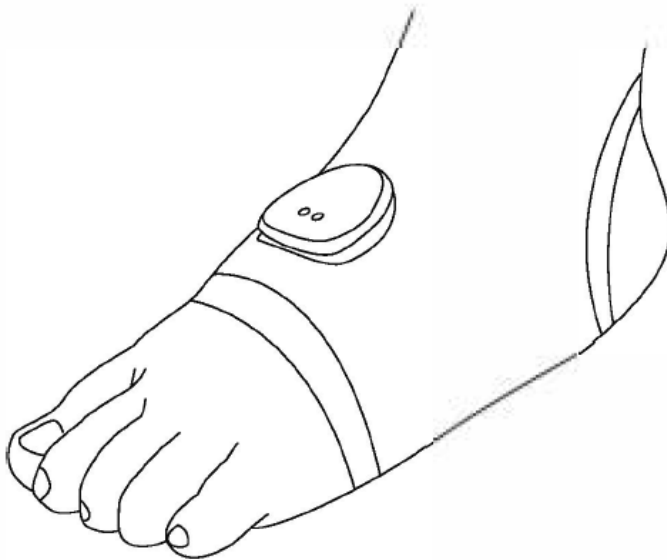


NOTE: If redness, swelling, or skin irritation occurs, remove the socks immediately.

- For infants with sensitive skin, frequently alternate which foot wears the socks.
- When charging, place the socks on the charging base and ensure the battery indicator light on the base turns red.

Charging Station Alarms

Charging Station Alarms


The Charging Station alerts you with sound, light, and vibration, keeping you informed of your baby's status at all times so you can promptly detect if they feel unwell or need assistance. You can also receive notifications on your phone via the MamaPro App.

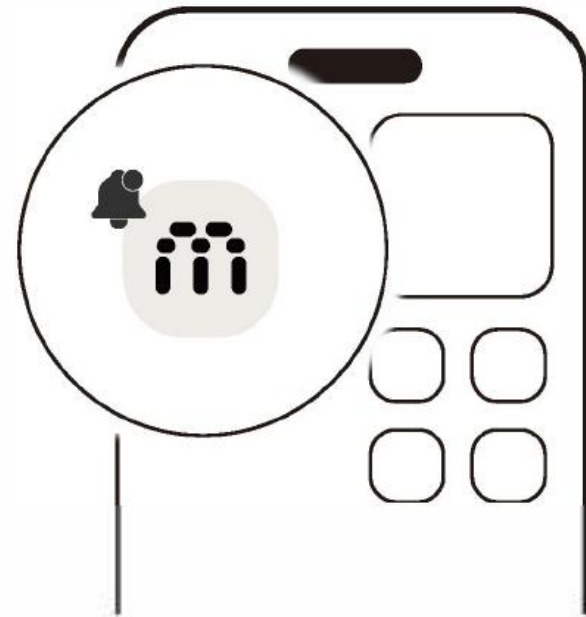

Turn off the Alarms

To snooze the alarms, simply press the Snooze Button.

You can also set Do Not Disturb mode through the MamaPro App.

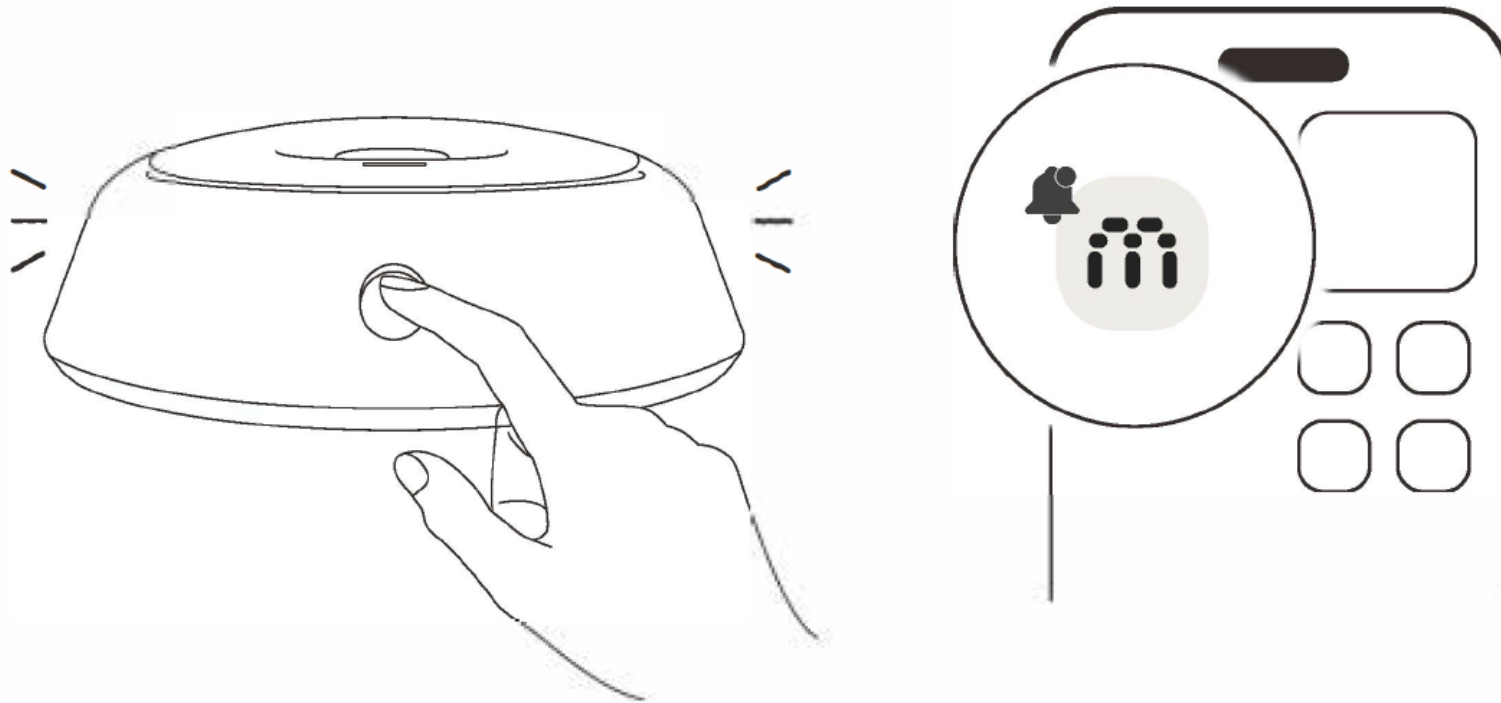

Clean and Storage

- Remove the Sock Sensor before washing the Sock body.
- Hand wash the Sock body and Sensor Patch using mild detergent and cold water.
- Clean the Sock and Sensor Patch at least every two weeks to prevent skin irritation caused by dirt buildup.
- Do not wash the Sensor or charging case.
- Avoid prolonged exposure to humid environments.

FCC Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

US Importer :
Company : Interfocus Inc
Address : 650 Castro Street Ste. 120-458,
Mountain View, CA 94041, United States
Service Call : +1(888)379-3991
Email : service@mamapro.com