

eBeat

Instructions for Use

DOCUMENT TYPE:	Instructions for Use
DOCUMENT NUMBER:	T0008383
DOCUMENT VERSION:	1.0
PRODUCT NAME:	eBeat
MODULE T-CODE:	T0008253
RELEASE DATE:	March 30, 2026

© 2026 TEKTELIC Communications Inc., all rights reserved.

All products, names, and services are trademarks and registered trademarks of their respective companies.

TEKTELIC Communications Inc.

7657 10th Street NE

Calgary, AB, Canada T2E 8X2

Phone: (403) 338-6900

1. Revision History

Revision	Issue Date	Editor	Comments
1.0	March 30, 2026	Megan Tremaine, Emma Tholl	Initial release.

Table of Contents

1.	Revision History	2
2.	Glossary	7
2.1.	Warnings and Cautions	7
2.2.	List of Symbols.....	11
2.3.	List of Terms and Acronyms	13
3.	Product Description	14
3.1.	Device Introduction.....	14
3.2.	Indications for Use	14
3.3.	Contraindications	15
3.4.	Device Description	15
3.4.1.	Hardware Components.....	16
3.4.2.	Software Components	19
4.	Operating Information.....	20
4.1.	Required Equipment	20
4.2.	Unpacking and Inspection.....	20
4.3.	Setup Instructions	21
4.3.1.	Gateway Setup.....	21
4.3.2.	Powering On the Device and Replacing the Battery.....	21
4.4.	Checkout Procedure	23
4.5.	Operating Instructions	24
4.5.1.	Wearing the Device.....	24
4.5.2.	Push-Button Functionality	25
4.5.3.	Visual Indicator Lights.....	26
4.5.4.	Reporting Modes	26
4.6.	TEKTELIC eDoctor Mobile Application Functionality	26
4.6.1.	TEKTELIC eDoctor Mobile Application Download and Registration	26
4.6.2.	Authorization	28
4.6.3.	Setup wizard	28

4.6.3.1.	Adding a Gateway.....	28
4.6.3.1.1.	Adding Gateway using QR Code	29
4.6.3.1.2.	Adding Gateway manually using gateway ID	30
4.6.3.1.3.	Adding gateway manually using T-code, Revision and Serial number	31
4.6.3.1.4.	Editing gateway	33
4.6.3.1.5.	Gateway Connection	33
4.6.3.1.6.	Skip Gateway Connection.....	34
4.6.3.2.	Adding Device	35
4.6.3.2.1.	Adding device using QR	36
4.6.3.2.2.	Adding device manually using T-code, Revision and Serial number	36
4.6.3.2.3.	Add device manually using device EUI	37
4.6.3.2.4.	Editing device	38
4.6.3.3.	Activate device and complete setup wizard.....	38
4.6.3.3.1.	Skip Device Connection	39
4.6.4.	Manage Patient.....	40
4.6.4.1.	Adding new patient	40
4.6.4.2.	Editing patient information	41
4.6.4.3.	Delete Patient.....	41
4.6.4.4.	Patient Device Data	42
4.6.4.4.1.	View device data	43
4.6.4.4.2.	Reporting mode setting.....	45
4.6.5.	Manage Gateways	45
4.6.5.1.	View Gateway details	46
4.6.5.2.	Gateway editing.....	47
4.6.5.3.	Add new gateway	47
4.6.5.4.	Gateway deleting.....	48
4.6.6.	Manage devices	48
4.6.6.1.	View device details	48
4.6.6.2.	Device editing	48
4.6.6.3.	Add new device	48

4.6.6.4.	Device deleting	49
4.6.7.	Account Settings	49
4.6.7.1.	Forgot Password	49
4.6.7.2.	Change Name	50
4.6.7.3.	Change Password.....	50
4.6.7.4.	Application settings	50
4.6.7.5.	Logout	50
4.7.	TEKTELIC eDoctor Web Application Functionality	51
4.7.1.	User authorization	51
4.7.2.	Manage Patient.....	51
4.7.2.1.	Adding new patient	52
4.7.2.2.	Editing patient information	52
4.7.2.3.	Delete patient.....	53
4.7.2.4.	Patient device data	54
4.7.2.4.1.	View device data	55
4.7.2.4.2.	Reporting mode setting.....	56
4.7.2.4.3.	Change theme for data visualization.....	57
4.7.2.5.	Application settings	57
4.7.2.6.	Full screen mode	58
4.7.2.7.	Export patient's data	59
4.7.3.	Manage Devices	61
4.7.4.	Forgot Password	61
4.7.5.	Logout	62
4.8.	Notifications	63
4.8.1.	Push-Button Assistance Request	63
4.8.2.	Device Wear Status.....	63
4.8.3.	Measurement Thresholds.....	64
4.8.4.	Poor SQI	64
4.9.	Software and Firmware Updates	64
4.10.	Cleaning Instructions	65

4.11.	Maintenance.....	66
4.12.	Expected Service Life	66
4.13.	Medical Device Disposal	67
5.	Troubleshooting Information	68
6.	Technical Description	69
6.1.	Software Bill of Material (SBOM)	69
6.2.	Essential Performance.....	69
6.3.	Use Environments	70
6.4.	Monitoring Specifications	70
6.4.1.	Device/System Security	70
6.4.1.1.	Network Ports.....	72
6.4.1.2.	Protection of Critical Functionality.....	72
6.4.1.3.	Data Flow and Security Controls	73
6.4.2.	Pulse Rate.....	73
6.4.3.	Respiration Rate.....	73
6.4.4.	Blood Oxygen Saturation (SpO ₂)	74
6.4.5.	Skin Temperature.....	74
6.4.6.	Heart Rate Variability (HRV)	75
6.4.7.	Signal Quality Indicator (SQI)	75
6.4.8.	Battery Capacity.....	76
6.5.	Electrical Safety	76
6.6.	MRI Safety Information.....	77
6.7.	Electromagnetic Compatibility (EMC)	77
6.8.	FCC Compliance Statements	80
7.	Precautions	82
8.	Warranty.....	83
9.	Contact Us.....	85

2. Glossary

2.1. Warnings and Cautions

⚠ WARNING – A **WARNING** statement indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

⚠ CAUTION – A **CAUTION** statement indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury, or damage to the equipment.

⚠ WARNING – For proper use of this device, read and follow these instructions for use.

⚠ WARNING – Adhere to all WARNINGS and CAUTIONS in these instructions for use to maintain patient safety.

⚠ WARNING – The arm strap may pose a choking hazard. Do not chew or place in mouth.

⚠ CAUTION – The eBeat contains a lithium coin cell battery. Do not burn or puncture battery. Please ensure disposal instructions are followed.

⚠ WARNING – This product contains a coin cell battery. Swallowing a coin cell battery can cause severe internal chemical burns in as little as 2 hours and may result in serious injury or death. Even used batteries can cause severe injury or death. If a battery is swallowed or placed inside any part of the body, seek immediate medical attention or contact a local poison control center.

⚠ CAUTION – Keep new and used batteries away from children. If the battery compartment does not close securely, stop using the product and keep it out of reach of children.

⚠ CAUTION – Risk of fire or explosion if battery is replaced by an incorrect type.

⚠ CAUTION – Use only recommended accessories supplied by TEKTELIC.

⚠ CAUTION – Do not modify this equipment without authorization of the manufacturer.

⚠ CAUTION – If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.

⚠ WARNING – Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

⚠ WARNING – Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

⚠ WARNING – Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the TEKTELIC eBeat Device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

⚠ WARNING – Do not wear if sensitive to nylon, stainless-steel, or polycarbonate.

⚠ CAUTION – Avoid prolonged or excessive contact with fluids. If the device is exposed to significant amounts of fluid, remove it from service and contact the manufacturer.

⚠ CAUTION – Review all cleaning instructions carefully before cleaning the device.

⚠ WARNING – Ensure the device has been cleaned before return to the manufacturer for repair or maintenance, or before disposal, to reduce the risk of infection.

⚠ WARNING – Do not remove the enclosure, current-carrying components are present. Service, repair, and maintenance must only be performed by the manufacturer.

⚠ WARNING – Device failure may occur due to use-life exceeded. Perform regular inspection of the device and arm strap.

⚠ WARNING – Remove this device before patient defibrillation.

⚠ WARNING – Only wear the device on intact skin. Inspect the area of contact with the device and strap at least every 24 hours. If irritation is observed, remove the device immediately.

⚠ CAUTION – Remove the device if skin irritation or discomfort occurs.

⚠ WARNING – Care provider users should ensure that their hospital network has firewalls, network security protections, data security protections, and anti-virus protections subject to HIPAA compliance within their network.

⚠ WARNING – TEKTELIC strongly advises the user NOT to modify, alter, or reset the software without consent from TEKTELIC. Exceptions to this are when the App instructs the user to perform a software update or if a TEKTELIC technical assistance representative instructs/guides the user through maintenance and/or troubleshooting.

⚠ WARNING – Firmware updates are performed through FUOTA using manufacturer-authorized update mechanisms. Users must not attempt unauthorized modification of device firmware.

⚠ WARNING – Do not force discharge, recharge, disassemble, heat above 70°C or incinerate. Doing so may result in injury due to venting, leakage, or explosion resulting in chemical burns.

⚠ CAUTION – Remove and immediately recycle or dispose of used batteries according to local regulations and keep away from children. Do NOT dispose of batteries in household trash or incinerate.

⚠ CAUTION – Do not attempt to recharge non-rechargeable batteries.

⚠ CAUTION – It will take 25 minutes for the device to warm to 20°C from the minimum storage temperature.

⚠ CAUTION – It will take 25 minutes for the device to cool to 20°C from the maximum storage temperature.

⚠ CAUTION – Please wait for the skin temperature readings to stabilize after putting the device on or if there have been changes in environmental conditions.

⚠ CAUTION – Damage to the device may degrade sensor and electrode performance.

⚠ CAUTION – The device should not be near or in contact with high temperatures (i.e., Fireplace).

⚠ CAUTION – Keep the device out of reach of pets and children. Do not place the device in the mouth, as it contains electrical components and a battery.

⚠ CAUTION – Remove and immediately recycle or dispose of batteries from equipment not used for an extended period according to local regulations.

⚠ CAUTION – Do not mix old and new batteries, different brands or types of batteries, such as alkaline, carbon-zinc, or rechargeable batteries.

⚠ CAUTION – Ensure the batteries are installed correctly according to polarity (+ and -).

⚠ CAUTION – Mobile devices and PCs used to access the eDoctor web app or the eDoctor mobile app should be secured when not in use.

⚠ CAUTION – Users should regularly apply/allow security updates to your device's operating system (OS).

⚠ CAUTION – TEKTELIC does not make any commercial use of or disclose any private information of eBeat users.

⚠ CAUTION – Follow applicable hospital data backup and disaster recovery processes and ensure that data from the web and mobile applications is periodically backed up to maintain availability of information. TEKTELIC technical support can be contacted to assist with disaster recovery.

⚠ CAUTION – Safeguard your login information with the utmost care. Never share your username and password with any third party.

⚠ CAUTION – Exercise prudence when installing mobile applications. Only install apps from trusted sources, such as official app stores. Avoid downloading and installing applications from unverified or unofficial sources.

⚠ CAUTION – Use strong, unique passwords for your web/mobile accounts. Combine a mix of upper and lower-case letters, numbers, and symbols. Consider using a reputable password manager.

⚠ CAUTION – TEKTELIC personnel will never request your password or login information. Notify TEKTELIC immediately if you receive unexpected or confusing emails, phone calls, or other forms of communication

⚠ CAUTION – Federal law restricts this device to sale by or on the order of a physician.

⚠ CAUTION – Do not touch the battery and the patient at the same time during battery replacement.

⚠ CAUTION – Do not touch the gateway/smart phone/computer and patient at the same time.

⚠ CAUTION – The device has not been clinically validated in pregnant or breastfeeding patients, and therefore performance in those groups is unknown.

⚠ CAUTION – The device has not been clinically validated in patients with pacemakers, and therefore performance in those groups is unknown.

⚠ WARNING – The eBeat device and arm band are MR Unsafe. Do not expose the device or the arm band to a magnetic resonance (MR) environment. The device and arm band present projectile hazards. Patients should not receive an MR exam while the device and arm band are fastened to the patient.




California Proposition 65:

⚠ WARNING – This product can expose you to chemicals including lead, nickel, and carbon black, which are known to the State of California to cause cancer, birth defects, or other reproductive harm. For more information, go to www.P65Warnings.ca.gov.

2.2. List of Symbols

Table 2-1: Symbols and Meaning

Symbol	Meaning	Source of Symbol
	Warning/Caution.	ISO 15223-1: 2021, Symbol 5.4.4
	Type BF Applied Part.	ANSI/AAMI ES60601-1:2005 & A1:2012 & A2:2021, Table D.1, Symbol 20
		ASTM F2503-23, Fig. 9
	Consult Instructions for Use.	ISO 15223-1:2021, Symbol 5.4.3
	FCC Declaration of Conformity.	47 CFR § 2.1074 (b)

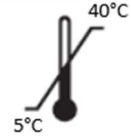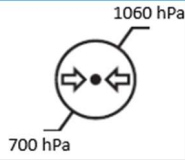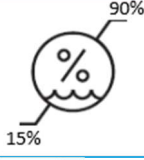	Ingestion hazard. This product contains a button cell or coin battery.	ANSI/UL 4200A-2023, Fig. 7B.1
	Temperature Limitation. (5°C to 40°C)	ISO 15223-1:2021, Symbol 5.3.7
	Atmospheric Pressure Limitation. (700 hPa to 1060 hPa)	ISO 15223-1:2021, Symbol 5.3.9
	Humidity Limitation. (15% to 90%)	ISO 15223-1:2021, Symbol 5.3.8
	Date of Manufacture YYYY-MM-DD	ISO 15223-1:2021, Symbol 5.1.3
	Manufacturer.	ISO 15223-1:2021, Symbol 5.1.1
	Serial Number.	ISO 15223-1:2021, Symbol 5.1.7
	Unique Device Identifier.	ISO 15223-1:2021, Symbol 5.7.10
Rx ONLY	Prescription Use Only.	21 CFR § 801.109 (b)(1)
IP22	IP Rating.	ANSI/AAMI ES60601-1:2005 & A1:2012 & A2:2021, Table D.3, Symbol 2
	Do Not Iron. (Arm Strap)	ISO 7000, Symbol 3113
	Do Not Dry Clean. (Arm Strap)	ISO 7000, Symbol 3114
	Do Not Bleach. (Arm Strap)	ISO 7000, Symbol 3124
	Dry Flat. (Arm Strap)	ISO 7000, Symbol 3080
	Hand Wash Normal. (Arm Strap)	ISO 7000, Symbol 3125

2.3. List of Terms and Acronyms

- **BF Applied Part** – Body Floating, a component making electrical contact with a patient's body, but not directly with the heart.
- **BPM** - Beats per minute.
- **CAUTION** – A **CAUTION** statement indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury, or damage to the equipment.
- **Device** - TEKTELIC eBeat sensor supported by the eDoctor application.
- **Device EUI** - A 16-letter/digit globally unique Extended Unique Identifier (EUI-64) is assigned by the manufacturer, or the owner, of the end-device (ex. 647FDA00000XXXXX).
- **FUOTA** – Firmware Update Over the Air.
- **Gateway** - IoT based device that connects other appliances to a wider network. They act as a bridge between these devices and the wider network, allowing them to communicate with each other and exchange data.
- **Gateway ID** - A 16-letter/digit unique gateway identifier, which is assigned by the manufacturer (ex. 647FDAFFFE0XXXXX).
- **Healthcare Professional** – Adult responsible for viewing the applications and setting up patients.
- **IoT** – Internet of Things.
- **LED** – Light-emitting diode.
- **LoRaWAN** – Long Range Wide Area Network.
- **Network Operator** – Adult responsible for setting up a LoRaWAN network. This role is offered as a service by TEKTELIC Communications Inc.
- **Patient** – Adult wearing the TEKTELIC eBeat Device.
- **RPM** - Respirations per minute.
- **Serial Number** - A 9-letter/digit unique identifier which is assigned by TEKTELIC for each gateway and device models that contains manufactory information (ex. 0000X0000).
- **Setup Wizard** - simple and intuitive onboarding procedure for new users of eDoctor application.
- **T-code** - An 8-letter/digit unique identifier which is assigned by TEKTELIC for each gateway and device models according to its region or specification (ex. T000XXXX).
- **WARNING** – A **WARNING** statement indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

3. Product Description

3.1. Device Introduction

TEKTELIC eBeat is the ideal solution for the monitoring of patient health, physical readiness, and overall well-being. This upper-arm wearable is designed to track key vital and wellness parameters including skin temperature, pulse rate, heart rate variability, respiration rate, and blood oxygen level (SpO₂), while also monitoring battery status and signal quality to support reliable measurements. Powered by a replaceable CR2477 coin-cell battery, the eBeat offers up to 3+ months of operation between battery changes. It is worn on an adjustable, washable arm-band strap for discreet use in clinical or home environments, wirelessly transmitting data over LoRaWAN to TEKTELIC's web and mobile applications for display and review.

Normal use of the device includes three roles: the network operator, the healthcare professional, and the patient. The network operator is responsible for the configuration of the gateway(s) and device(s) on the mobile and web applications. This role is offered as a service by TEKTELIC Communications Inc. The healthcare professional is the main viewer of the data displayed on the mobile and web applications and will configure patients on the applications (i.e. Add new patients, pair a patient with a device, add patient information, remove patients, etc.). The patient is responsible for simply wearing the device.

3.2. Indications for Use

The TEKTELIC eBeat Device is intended for reusable bedside, mobile, and central multi-parameter, physiologic patient monitoring of adult patients in professional healthcare facilities, such as hospitals or skilled nursing facilities, or their own home. It is intended for monitoring of patients by trained healthcare professionals. The TEKTELIC eBeat Device is intended to be reused on single or multiple patients. It has been designed for prolonged use and is transit-operable, meaning it is intended to be worn on the body and is operable during movement.

The TEKTELIC eBeat Device is intended for intermittent monitoring of the following parameters in adults:

- Pulse rate (PR)
- Heart rate variability (HRV)
- Respiration rate (RR)
- Blood oxygen saturation (SpO₂)
- Skin temperature (ST)

3.3. Contraindications

The eBeat Device is not intended for pediatric use.

The eBeat Device is not intended for use in high-acuity environments, such as ICU or operating rooms.

The eBeat Device is not for emergency use or emergency situations. In a medical emergency or if your condition suddenly worsens, do not rely on the eBeat Device. Follow your local emergency procedures and seek immediate medical help.

The eBeat Device is not a substitute for an ECG monitor. The eBeat Device is not intended for use on acutely ill cardiac patients where continuous ECG monitoring is more appropriate.

The eBeat Device is not suited for adults who are sensitive to contact with stainless steel, nylon, or polycarbonate.

The eBeat Device is not intended for health condition diagnosis or treatment.

3.4. Device Description

This section will describe the hardware components and software components of the TEKTELIC eBeat Device. A diagram representing the eBeat system can be seen in Figure 3-1 below.

The system consists of:

- An eBeat device
- An adjustable upper arm strap
- A coin cell battery (3.0 V non-rechargeable CR2477 battery)
- A 1.5mm hex screwdriver (not included, needed for battery removal/replacement)
- A TEKTELIC LoRaWAN gateway (purchased separately)
- TEKTELIC network server
- eDoctor mobile application
- eDoctor web application
- Mobile device (not included)

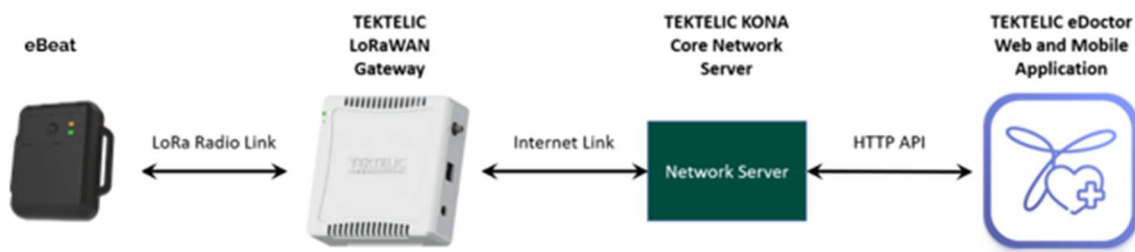


Figure 3-1: eBeat System Diagram

3.4.1. Hardware Components

⚠ CAUTION – Use only recommended accessories supplied by TEKTELIC.

⚠ CAUTION – Do not modify this equipment without authorization of the manufacturer.

⚠ CAUTION – If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.

Figures 2-2 and 2-3 illustrate the eBeat enclosure and its external interfaces. The front of the enclosure includes two indicator lights, one green and one orange, as well as a single push button.

The eBeat has an IP classification of IP22. This classification indicates that the enclosure is protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling water droplets when the enclosure is tilted up to 15°.



Figure 3-2: Front View of eBeat External Interfaces



Figure 3-3: Back View of eBeat External Interfaces



Figure 3-4: eBeat Upper Arm Strap

The eBeat sensor uses small lights and sensors to measure changes in blood flow just beneath the skin of the upper arm. These measurements are used to track key health metrics.

When the device is worn, it first checks that it is properly positioned on the arm and then adjusts the light output to match the user's skin. Once this setup is complete, the sensor uses the collected signals to calculate health metrics such as pulse rate (PR), respiratory rate (RR), heart rate variability (HRV), blood oxygen saturation (SpO₂), and signal quality (SQI).

Additional sensing components include a temperature sensor for measuring skin temperature and an accelerometer for determining when the device is in use.

The eBeat specifications are listed in Table 3-1.

Table 3-1: eBeat Specifications

Parameter	Specification
Use Environment	Indoor professional healthcare facilities
Medical Device Classification	USA FDA Class II Device
Size	eBeat: 65 x 55 x 20 mm Arm Strap: 450 x 28 x 5 mm
Weight	eBeat: 45 g Arm Strap: 10 g
Power Source	Battery operated (one 3.0 V non-rechargeable CR2477 battery)
Network Technology/Frequency Band	LoRaWAN using North American Region: US902-928 Transmit (Tx): 902 MHz – 915 MHz Receive (Rx): 923 MHz – 928 MHz
Maximum Transmit Power (ERP)	14 dBm
Air Interface	LoRa (Chirp Spread Spectrum)
Battery Lifetime	> 3 months*
Number of Visual Indicators	2 Visual Indicators (one orange and one green) Green: Indicates that the device is ready for use Orange: Flashes when the push button has been successfully pressed and the assistance request has been sent
Number of Push Buttons	1 push button that allows the wearer to request assistance
Sensing Functions	Skin temperature, pulse rate, heart rate variability, respiration rate, SPO ₂
IP Classification	IP22
Use Life	eBeat Device: 2 years Arm Strap: 2 years
Temperature Range	Operating: 5°C (41°F) to 40°C (104°F) Storage: -25°C (-13°F) to 70°C (158°F)
Applied Parts of the System	BF Applied parts. Temperature contact, strap fabric

*Actual battery life depends on configuration, data rate, and operating temperature.

3.4.2. Software Components

The eBeat is designed to operate with the TEKTELIC eDoctor Web and Mobile Applications.

The TEKTELIC eDoctor Application is an easy-to-use tool used to store up to three months worth of data and display the latest measurements from the eBeat Device.

The Mobile Application allows the network operator to create a LoRaWAN network by adding gateways and devices. Once the network has been established, the healthcare professional can assign devices to patients. When the device is worn by a patient, their reported parameters will be visualized on both the Web and Mobile Applications.

The TEKTELIC eDoctor Mobile Application is available for download on the [App Store](#) and [Google Play](#).

To access the TEKTELIC eDoctor Web Application, click the link and log in to the [TEKTELIC eDoctor Web Application](#).

Please ensure the following minimum system requirements are met:

Table 3-2: Minimum Browser Versions

Browser Name	Minimum Version
Google Chrome	Version 123+
Mozilla Firefox	Version 124+
Safari	Version 16+

Table 3-3: Minimum Mobile Operating Systems

Mobile Operating System	Minimum Version
Android	Version 10+
iOS	Version 13+

Table 3-4: Minimum Desktop Operating Systems

Desktop Operating System	Minimum Version
Windows	Version 10

4. Operating Information

⚠ WARNING – The arm strap may pose a choking hazard. Do not chew or place in mouth.

⚠ CAUTION – Do not modify this equipment without authorization of the manufacturer.

⚠ WARNING – Firmware updates are performed through FUOTA using manufacturer-authorized update mechanisms. Users must not attempt unauthorized modification of device firmware. Unauthorized attempts to modify the device firmware outside the manufacturer-authorized update process may result in device malfunction, compromised security, or loss of performance.

⚠ WARNING – Only wear the device on intact skin. Inspect the area of contact with the device and strap at least every 24 hours. If irritation is observed, remove the device immediately.

⚠ CAUTION – Remove the device if skin irritation or discomfort occurs.

⚠ CAUTION – Remove and immediately recycle or dispose of used batteries according to local regulations and keep away from children. Do NOT dispose of batteries in household trash or incinerate.

This section describes the included product, required equipment for setup, unpacking and inspection instructions, setup instructions, and operating instructions.

For concise operating information, please refer to the eBeat Quick Start Guide included in the device packaging or visit the TEKTELIC website for an introductory video.

4.1. Required Equipment

To onboard and configure the TEKTELIC gateway(s), eBeat device(s), and patient(s), please download the TEKTELIC eDoctor mobile application from the [App Store](#) or [Google Play](#).

A 1.5mm hex screwdriver is required to lock/unlock the battery lid located on the back of the enclosure. No other special equipment or tools are required to power or operate the eBeat.

4.2. Unpacking and Inspection

The following should be considered during the unpacking of a new eBeat:

1. Inspect the shipping carton and report any significant damage to TEKTELIC.
2. Unpacking should be conducted in a clean and dry location.
3. Do not discard the shipping box or inserts as they will be required if a unit is returned for repair or maintenance. Store in a safe and dry location after unpacking.

4.3. Setup Instructions

The setup procedure is to be completed by the network operator in an indoor environment. This procedure includes setting up the LoRaWAN network, creating the application account, as well as onboarding the gateways and devices. This setup is offered as a service by TEKTELIC Communications Inc.

The device can be powered on by either the network operator or the healthcare professional, depending on what is most convenient.

4.3.1. Gateway Setup

To turn on the gateway, follow the steps below:

1. Connect power cable from the gateway to an outlet.
2. Connect the gateway to either ethernet or cellular.
 - a. For ethernet, connect the ethernet cable from the gateway to the ethernet socket.
 - b. For cellular, insert a SIM card into the specified SIM port on the gateway. Please refer to [Kona Gateway Cellular Backhaul FAQ](#) for additional support.
3. Once steps 1 and 2 have been completed, the gateway should begin flashing green.

Further setup for onboarding and configuring the gateway on the app will be described in Section 4.6.3.1 below.

4.3.2. Powering On the Device and Replacing the Battery

⚠ CAUTION – Do not touch the battery and the patient at the same time during battery replacement.

⚠ CAUTION – Do not touch the gateway/smart phone/computer and patient at the same time.

The eBeat is shipped with a CR2477 coin cell battery installed and engaged, but with a battery sticker on the negative terminal preventing the battery from turning the device on. Removing

the sticker and re-inserting the battery will activate the device. After the sticker has been removed, the battery must be removed to turn off the device.

Prior to initial use, inspect the battery for signs of corrosion, such as white, green, or powdery residue on the battery or contacts. Corrosion can affect device performance and safety. If corrosion is present, or if the device has not been used for an extended period, replace the battery with a **NEW** CR2477 (3V Lithium Manganese Dioxide) coin cell. Remove used batteries promptly and recycle or dispose of them according to local regulations.

Recommended battery brand: Panasonic.

Do not mix old and new batteries, different brands or types of batteries, such as alkaline, carbon-zinc, or rechargeable batteries.

To activate the unit or to replace the battery when required, follow these steps:

Note: Ensure the device has been onboarded to the mobile application by the network operator before proceeding.

1. Locate the battery lid on the back of the eBeat. Remove the 4 screws holding the lid in place and then remove the battery lid from the device.



Figure 4-1: Removing Battery Lid from eBeat

2. Remove the battery from the battery holder. For activation of a newly received device, peel the sticker off the negative terminal of the battery. For battery replacement, use only a **new** CR2477, 3V Lithium Manganese Dioxide coin cell. **Recommended battery brand:** Panasonic.

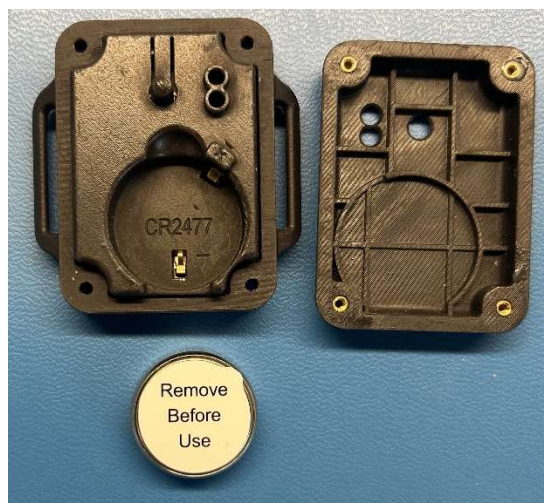


Figure 4-2: Remove Battery Sticker

3. Place the battery inside the eBeat battery compartment ensuring the battery is installed correctly according to polarity (+ and –). The positive terminal of the battery must be visible after insertion, as shown in Figure 4-1. Replace the battery lid and the 4 screws. Tighten the screws to hold the lid securely in place.
4. After a few seconds, the visual indicators on the eBeat should start blinking, showing that the eBeat is trying to join the LoRaWAN network. The visual indicators will stop blinking once the eBeat has joined the LoRaWAN network.

Note: Time from device powering on until ready for normal use may take up to 5 minutes. The device will only report measurements while it is being worn.

Note: The eBeat may take a few minutes to fully power off because it uses supercapacitors for internal energy storage. This is normal behavior.

4.4. Checkout Procedure

Once the patient has completed using the device, the healthcare professional can assist in removing the strap and device from the patient. Afterward, the healthcare professional is expected to follow the cleaning procedures outlined in the Cleaning Instructions section.

4.5. Operating Instructions

The following operating instructions assume the setup procedure has already been followed. This means the LoRaWAN network has been fully setup, the gateways and devices have been onboarded, and the device has been powered on.

The following sections will describe how to wear the device, how to use the push buttons, as well as how to navigate and use different features on the mobile and web applications.

4.5.1. Wearing the Device

⚠ CAUTION – It will take 25 minutes for the device to warm to 20°C from the minimum storage temperature.

⚠ CAUTION – It will take 25 minutes for the device to cool to 20°C from the maximum storage temperature.

⚠ CAUTION – Please wait for the skin temperature readings to stabilize after putting the device on or if there have been changes in environmental conditions.

⚠ WARNING – Do not wear if sensitive to nylon, stainless-steel, or polycarbonate.

⚠ WARNING – Only wear the device on intact skin. Inspect the area of contact with the device and strap at least every 24 hours. If irritation is observed, remove the device immediately.

⚠ CAUTION – Remove the device if skin irritation or discomfort occurs.

1. Attach the arm band to the eBeat by feeding the ends of the strap through the slots on either side of the device and fastening the strap to itself.
2. Slide your arm into the strap until the device is positioned at your upper arm. Make sure the eBeat is at the back of your arm. Ensure that the eBeat is in direct contact with your skin. Tighten the strap so that the device stays in place. The user is advised to wear the sensor snugly. An easy rule of thumb is to adjust the strap on your arm to a tightness where you can still slip two fingers under the band.

Note: The eBeat device and strap should sit comfortably without sagging.



Figure 3-7: Device and strap being worn correctly

Note: Inspect the arm strap to ensure it is not displaying any signs of deteriorations from prolonged use such as loose fabric, rips, holes, etc. If the strap displays any of these signs, please dispose the strap and use a new strap instead.

3. After a few minutes, the device will begin reporting data to the application. By default, the device will report data every 5 minutes when worn.

Note: For accurate monitoring, ensure the device is worn snugly and correctly positioned. The device must make direct contact with the skin and the strap should not sag. Improper fit or loose contact, particularly during sleep, may result in inconsistent readings or temporary data gaps.

4.5.2. Push-Button Functionality

The eBeat is equipped with a push button that allows the wearer to request assistance. Pressing the button sends a request to the application, where the viewer is notified and prompted to acknowledge the request. If the button is pressed during an active measurement, the request will be sent once the measurement is complete, which may result in a brief delay before the notification appears in the application.

4.5.3. Visual Indicator Lights

The eBeat device uses two indicator lights, a green light and an orange light, to show its status and confirm certain actions.

When the device is powered on, the lights may briefly turn on or blink while the device starts up. This is normal and indicates the device is preparing for use. Once the startup process is complete, the green light briefly blinks and then turns off. No lights will be visible during normal operation.

When the push button is pressed to request assistance, the orange light flashes to confirm that the request has been sent.

4.5.4. Reporting Modes

The eBeat supports two reporting modes: Normal Reporting Mode and Enhanced Reporting Mode. By default, the device operates in Normal Reporting Mode, reporting data every five minutes. Enhanced Reporting Mode increases the reporting frequency to once per minute.

4.6. TEKTELIC eDoctor Mobile Application Functionality

TEKTELIC eDoctor mobile application is an easy-to-use tool for monitoring patients' vital signs in real time. This app is a perfect choice for remote health monitoring and tracking any changes in medical conditions. eBeat will significantly improve operational processes in the medical sphere, reducing time wasted on regular checks and filling out the forms with updated measurements. With this app, users can access measurements from eBeat device.

4.6.1. TEKTELIC eDoctor Mobile Application Download and Registration

The creation of an eDoctor Mobile Application account is offered by TEKTELIC Communications Inc. To create an account, please reach out to the [TEKTELIC Support Portal](#).

In order for TEKTELIC Communications Inc. to create an eDoctor Mobile Application account for you, they will require the following information:

- An email account that the eDoctor Mobile Application account can be created under
- The gateway ID (GWID) of the gateway(s) that should be added to the account. The GWID can be found on the bottom label of the gateway.
- The device EUI (DevEUI) of the device(s) that should be added to the account. The DevEUI can be found on the bottom label of the device.

From here, you will be sent an email at the email address provided to TEKTELIC that asks you to change the password to your eDoctor Mobile Application account. Once the password has been changed, you will gain access to the eDoctor Mobile Application account. This email address and newly created password will also provide access to the eDoctor Web Application.

For instructions on how to add a patient, please refer to the Adding new patient section.

If you are setting up the TEKTELIC eDoctor mobile application on your own, please follow the steps described in the following sections:

1. Download the TEKTELIC eDoctor mobile application from the [App Store](#) or [Google Play](#) and then open it.
2. Tap on the "Sign Up" button to begin the registration process.

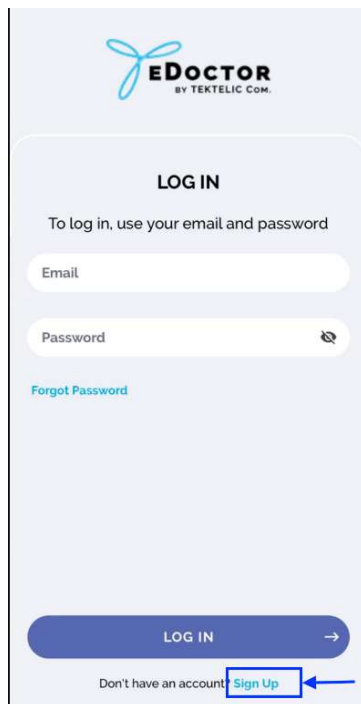


Figure 4-3: Starting the registration process

3. Enter your email address and password in the designated fields. Make sure to provide a valid and accessible email address. Password should be at least 8 characters long with a mix of letters, numbers and symbols.
4. Please read the privacy policy carefully by clicking on "Privacy Policy". After reading and agreeing to the rules, click on the checkbox.
5. Once you have entered your email address and password, tap on the "Register" button.

6. Check your email and find a message with confirmation code. If you do not see the email in your inbox, please check your spam or junk folder. If you don't see the code in your spam or junk folder, click "Resend code" to receive a new code.
7. Return to the App, enter confirmation code and tap on the "Continue" button.

Note: If the code matches, a confirmation message will appear, indicating that your registration is successful.

Note: If the code doesn't match, an error message will appear, asking you to enter the correct confirmation code. Please double-check the code or click "Resend code" to receive a new code.

4.6.2. Authorization

To log in to the application:

1. Open the app to access the login screen.
2. Enter the email and password you used during registration.
3. Tap the "Log in" button.

Once logged in and your profile is set up, you can begin using the app's features.

If login fails, double-check your email and password and try again.

4.6.3. Setup wizard

After registering and logging in for the first time, you'll be guided through the Setup Wizard.

The Setup Wizard is a step-by-step process that helps you connect and configure your gateways and devices. It's designed to simplify the initial setup and ensure a smooth start with the application.

To begin, tap "**GET STARTED**" to add your first gateway.

4.6.3.1. Adding a Gateway

To add a gateway, tap the "**ADD GATEWAY**" button. You can either scan the QR code on the device or enter the details manually, including the Gateway ID, Serial Number, Revision, and T-Code.



Figure 4-4: Setup wizard start view

4.6.3.1.1. Adding Gateway using QR Code

To add a gateway using a QR code, use the in-app camera to scan the QR code on your gateway.

Make sure camera permission is enabled for the app. If you previously denied it, you can allow it by going to your phone's settings:

Settings → Apps → eDoctor → Permissions → Camera.



Figure 4-5: Scan gateway QR

4.6.3.1.2. Adding Gateway manually using gateway ID

To add a gateway manually by entering the gateway ID you should click on “Don’t have the QR code or can't scan?”.

← ADD GATEWAY Log out

GW ID: 647FDAFFFE008***

Find the Gateway ID on the gateway or its packaging, e.g. 647FDAFFFE000000

Gateway ID

SUBMIT

Figure 4-6: Adding gateway using ID view

After scanning the QR code (or choosing manual entry), a screen will appear prompting you to enter the Gateway ID.

Note: The Gateway ID must be 16 characters long and start with “647FDAFFFE”.

Once entered, click the **"Submit"** button. You’ll then be taken to the internet connection setup screen.

4.6.3.1.3. Adding gateway manually using T-code, Revision and Serial number

If you have forgotten your gateway ID or it is damaged, you can add a gateway using Serial number and T-code. If this happens, click on “Add gateway with T-code, Revision and Serial number”. After this, a screen will appear prompting the user to enter the T-code, Revision and Serial number.

← ADD GATEWAY Log out

T0006***
D0
2249J1***

1. Find the T-code on the gateway or its packaging, e.g. T0000000
2. Find the Revision (Rev) on the gateway or its packaging, e.g. D0
3. Find the Serial number on the gateway or its packaging, e.g. 0000J0000

T-Code

Revision

Serial Number

SUBMIT

[Add gateway with Gateway ID](#)

Figure 4-7: Adding gateway using serial number, revision and T-code

Note: The T-code should be 8 characters long and start with “T”.

Note: The Revision should have up to 4 characters (Latin letters and digits only).

Note: The Serial Number should be 9 characters long and contain a letter in the middle.

Click on the "Submit" button after entering T-code, Revision and Serial number. After submit your entering, you will be taken to the internet connection setup.

4.6.3.1.4. Editing gateway

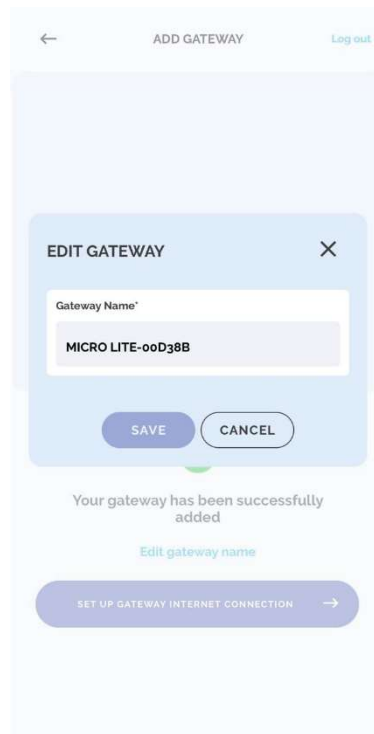


Figure 4-8: Edit gateway view

After you successfully add a gateway, you can change its name by clicking on "**Edit gateway name**".

To save the entered name, click "Save" button, and to cancel and return to the previous name, click "Cancel".

Note: The Gateway Name should be minimum 3 and maximum 255 characters long.

Note: The Gateway Name should only contain Latin letters, numbers and special characters: e.g., ! @ # ?].

4.6.3.1.5. Gateway Connection

Go further to the internet connection setup by clicking on the "**SET UP GATEWAY INTERNET CONNECTION**" button.

Connect the Ethernet cable to the gateway and wait while the gateway is connecting. It may take up to 5 minutes. After completing the setup of the Internet connection, you will receive an

“The Internet connection has been established” message and you can add a device by clicking on the **"ADD DEVICE"** button.



Figure 4-9: Successful internet connection view

If after 5 minutes the gateway does not pass the Internet connection stage, you will receive an error screen and can restart the process by clicking on the "TRY AGAIN" button.

If you want to use a cellular internet connection on the gateway with LTE, please follow the guide for setting it up - [Cellular Configuration](#).

4.6.3.1.6. Skip Gateway Connection

If you do not want to complete the Internet connection setup at this step, you can skip the gateway connection process and proceed directly to adding a device. To do this, click on the **"ADD DEVICE"** button.

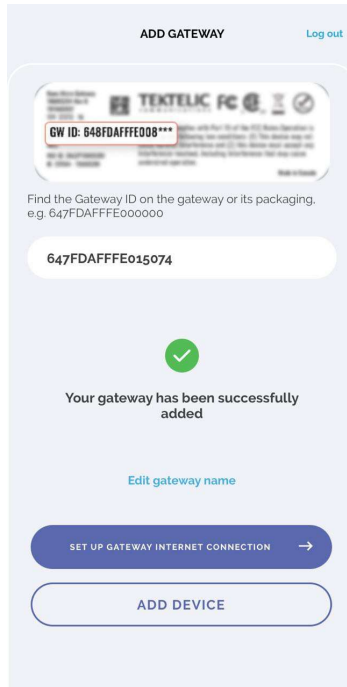


Figure 4-10: Skip gateway internet connection view

You will be redirected to the device setup screen without establishing the Internet connection for the gateway

Please note that if you skip the gateway connection setup, you will still be able to configure the internet connection later in the application when needed.

4.6.3.2. Adding Device

To add a device, click **“ADD DEVICE”**:

To add a device, scan its QR code or enter the serial number, revision and t-code, or device EUI, manually.

4.6.3.2.1. Adding device using QR

To add a device using QR you should scan with the camera window in the app, the QR code of your device.



Figure 4-11: Scan device QR code

4.6.3.2.2. Adding device manually using T-code, Revision and Serial number

To add a device manually by entering the T-code, Revision and Serial number you should click on "Don't have the QR code or can't scan?". After this, a screen will appear prompting you to enter the T-code, Revision and Serial number.

Figure 4-12: Adding device using serial number, revision and T-code view

Note: The T-code should be 8 characters long and start with “T”.

Note: The Revision should have up to 4 characters (Latin letters and digits only).

Note: The Serial Number should be 9 characters long and contain a letter in the middle.

Click on the "Submit" button after entering your device T-code, revision and Serial number. After submit your entering, you will be taken to the device establishing connection.

4.6.3.2.3. Add device manually using device EUI

Click on “Add device with Device EUI” if you only know the Device EUI of the device. After this, a screen will appear prompting you to enter the Device EUI.

← ADD DEVICE Log out

DEVICE COMMUNICATIONS
DEVEUI: 647FDA0000013***

Find the Device EUI on the device or its packaging. e.g.
647FDA0000000000

Device EUI

SUBMIT

[Add device with T-code and Serial number](#)

Figure 4-13: Adding device using device EUI

Note: The Device EUI should be 16 characters long and start with “647FDA”.

Click on the "Submit" button after entering your Device EUI. After submit your entering, you will be taken to the device establishing connection.

4.6.3.2.4. Editing device

After successfully add a device, can change its name by clicking on "**Edit device name**" or go further to the activate device by clicking on the "**ACTIVATE DEVICE**" button.

Note: The Device Name should be minimum 3 and maximum 255 characters long.

Note: The Device Name should only contain Latin letters, numbers and special characters: e.g., ! @ # ?].

4.6.3.3. Activate device and complete setup wizard

For device activation:

1. Open the device case and take out the battery.
2. Remove the battery sticker.

3. Reinsert the battery and close the device case.
4. Wait for the device to start collecting the data. It may take up to 5 minutes. After completing the device activation, will receive an "The connection has been established" message and you can complete setup wizard by clicking on the "**COMPLETE SETUP WIZARD**" button.



Figure 4-14: Successfully activate device view

⚠ If after 5 minutes the device does not pass the Internet connection stage, you will receive an error screen and can restart the process by clicking on the "TRY AGAIN" button.

After successfully completing all the steps and by clicking on the "**COMPLETE SETUP WIZARD**", will proceed to the main functionality of the application.

4.6.3.3.1. Skip Device Connection

If you do not want to activate the device during this step, you can skip the device activation and complete the Setup Wizard. To do this, click on the "**COMPLETE SETUP WIZARD**" button.

You will finish the Setup Wizard without activating the device. This option allows you to complete the setup process and return later to activate the device when needed.

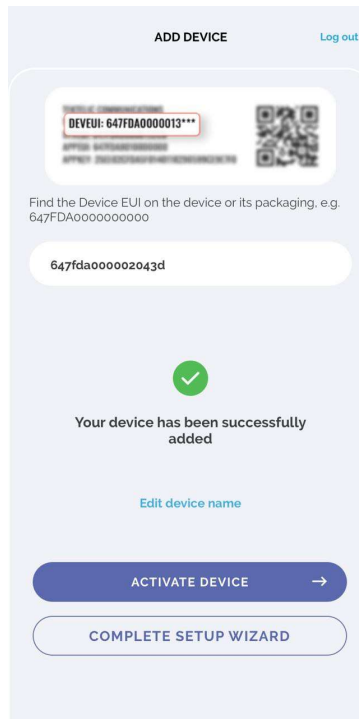


Figure 4-15: Skip device internet connection view

Note: If you skip the device activation, the device will not start transmitting data until you perform the device activation.

4.6.4. Manage Patient

The "**Patient**" page allows the user to manage patients (view device data, change patient information and delete patients).

4.6.4.1. Adding new patient

After at least one device and one gateway have been added to the application, a new patient can be created and assigned a device.

To add a new patient in the app, follow these 4 simple steps:

1. Tap the "+" icon

Tap the "+" icon in the lower right corner of the screen to begin.

2. Scan or enter the patient's ID

Scan the patient's QR code, or tap "QR is not available" to manually enter the patient's Medical ID (an internal identifier used in your organization), then tap "Next".

Note: The Patient QR code may contain any combination of letters and/or numbers that uniquely identifies the patient. The specific format may vary depending on the hospital or healthcare system.

3. Scan or select a device

Scan the QR code of the device, or tap "QR is not available" and select a device from the dropdown list of previously added ones. Tap "Next" to continue.

4. Enter patient details

Enter the first name, last name, age, and gender. Optionally, set threshold values for measurements.

Note: Pulse and Respiration Rate thresholds must be entered as whole numbers. Skin Temperature thresholds can include decimal values; use a dot (.) as the decimal separator.

If everything is correct, tap "Submit" to complete patient creation. If changes are needed, tap "Back" to edit the information.

Note: The device data screen updates every minute. Please wait at least one minute for the data to appear.

4.6.4.2. Editing patient information

To edit patient information such as the first name, last name, or measurement thresholds:

1. Select the patient's card.
2. Go to the "Information" section.
3. Tap the edit icon in the lower right corner of the screen.

Make the necessary changes and click the "SAVE" button to apply the updates.

4.6.4.3. Delete Patient

To delete a patient:

1. Select the patient's card.
2. Click on the corresponding trash can icon in the right-top corner of the screen.
3. Confirm deleting by clicking on "OK".

Note: The history of data for patient after its removal will also be deleted.

4.6.4.4. Patient Device Data

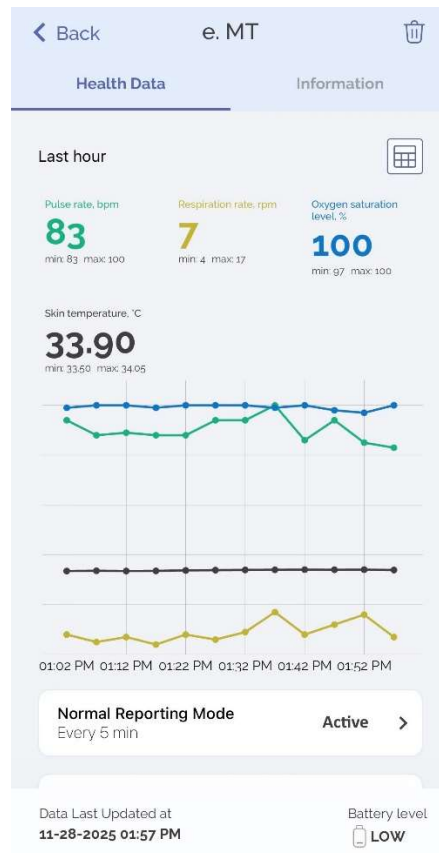


Figure 4-16: View of patient data reported from the eBeat device

After creating a patient, you can view their device-reported data by selecting the patient's card. The screen will automatically update to reflect new values reported by the device.

Above the graph, the most recently received value for each parameter within the past hour is shown. Next to each parameter, the minimum and maximum values received during that hour are also displayed.

If a reported value for pulse rate, skin temperature, respiration rate, or SpO₂ falls outside the set thresholds, it will be highlighted in red in the Last Hour view.

Also, the corresponding values outside the established thresholds are highlighted in red in table view screen.

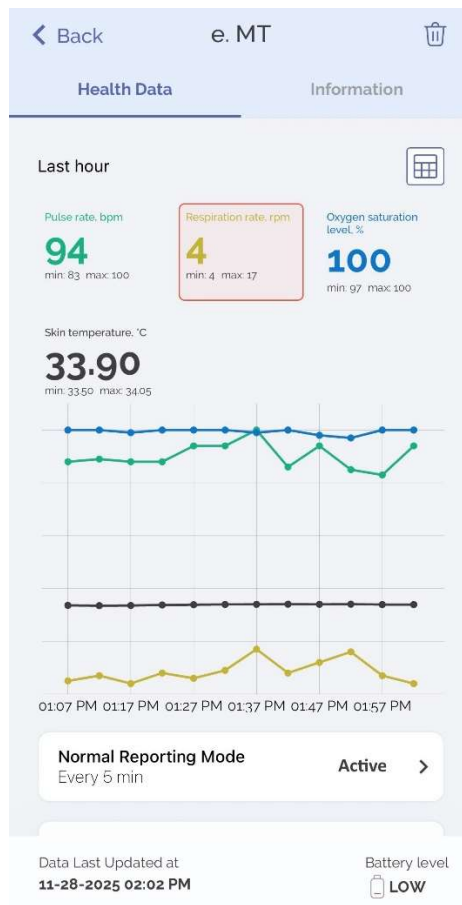


Figure 4-17: Highlighting patient's thresholds

4.6.4.4.1. View device data

After the patient card is opened, you can view the latest data from the device. The eBeat device reports skin temperature, pulse rate, respiration rate, SpO₂, last report time, as well as the battery level of the device itself. To view data for a certain period, you must click on the table icon in the upper right corner of the last measurement data. In Figure 4-18 below, skin temperature is used as an example.



Figure 4-18: Data history screen

Clicking on the desired parameter will take you to the data history screen for that parameter. You can customize the period of data display using the dropdown selector in the upper right corner of the screen. You can select two main periods – last hour (which is set by default after opening the history screen) and last 24 hours.

You can also customize measurement display using the dropdown selector in the upper left corner of the screen. You can select five measurements – pulse rate (which is set by default after opening the history screen), skin temperature, respiration rate, SpO₂, and heart rate variability.

For return to viewing measurement data for last hour, click on chart icon in the upper right corner above the selection period of data display.

4.6.4.4.2. Reporting mode setting

New data from the device will be received after the period set in the device settings. By default, Normal Reporting Mode is set with a reporting period of 5 minutes when worn.

The Normal Reporting Mode interval can be customized anywhere from every 2 minutes to every 60 minutes, depending on user requirements. This can be changed by clicking on the configuration icon and entering the desired reporting period.

1. Click on the corresponding icon.
2. Set the report value in minutes (minimum value 2, maximum value 60).
3. Confirm the selected value clicking "Save" button.

The user can also increase the device data reporting speed to every 1 minute by switching on the Enhanced Reporting Mode (by clicking on the "Enhanced Reporting Mode button", turning it purple).

4.6.5. Manage Gateways

The "**Gateways**" page allows you to manage gateways (view details, add new gateway, change gateway name and delete gateways).

You can open the "**Gateways**" page by clicking on the corresponding icon in the top menu.

GATEWAYS			
←			Log out
Status	Name	Gateway ID	Type
✓	Gateway-000001	647FDA...	KONA Micro
Status	Name	Gateway ID	Type
⚠	Gateway-000001	647FDA...	KONA Micro
Status	Name	Gateway ID	Type
?	Gateway-000001	647FDA...	KONA Micro

+

Figure 4-19: Gateway page view

4.6.5.1. View Gateway details

To check the detailed information about your gateway, simply click on gateway in the table and the system will open the detailed view where you can see the gateway details, its ID, model and name.

Figure 4-20: Gateway details view

4.6.5.2. Gateway editing

For editing gateway name:

1. Click on name field.
2. Enter new gateway name that does not repeat with an already existing name.
3. Click on the “**SUBMIT**” button. After changing the name of the gateway, a pop-up message will appear.

Note: The Gateway Name should be minimum 3 and maximum 255 characters long.

Note: The Gateway Name should only contain Latin letters, numbers and special characters: e.g., ! @ # ?].

4.6.5.3. Add new gateway

To add a new gateway, click on the plus icon (in the lower right corner of the screen) and follow the instructions in the Adding a Gateway section, described above. Unlike adding a gateway using the setup wizard, you cannot immediately set up the Internet connection for your

gateway. Instead, you can add another existing gateway by clicking on 'Add one more gateway +' or finish and return to the gateways page by clicking the 'Finish' button.

4.6.5.4. Gateway deleting

To delete a gateway that already exists, simply select the corresponding icon in the right-top angle on gateway details screen you would like to delete.

⚠ After gateway deleting, all data transmissions from devices that were in coverage of the gateway, will be disrupted.

4.6.6. Manage devices

The "**Manage Devices**" page allows the user to manage devices (view details, change device name, add new device and delete devices).

You can open the "**Manage Devices**" page by clicking on the corresponding icon in the top menu.

4.6.6.1. View device details

To check the detailed information about your device, simply click a record in the table and the system will open the detailed view where you can see the device details, its id, type and name.

4.6.6.2. Device editing

For editing device name:

1. Click on name field.
2. Enter new device name that does not repeat with an already existing name.
3. Click on the "**SUBMIT**" button. After changing the name of the device, a pop-up message will appear.

Note: The Device Name should be minimum 3 and maximum 255 characters long.

Note: The Device Name should only contain Latin letters, numbers and special characters: e.g., ! @ # ?].

4.6.6.3. Add new device

To add a new device, click on the plus icon (in the lower right corner of the screen) and follow the instructions in the Adding Device section described above. Unlike adding a device to the

setup wizard, you cannot immediately set up the connection of your device but add another existing device clicking on "Add one more device +" or finish and return to the devices list by clicking on "Finish" button.

4.6.6.4. Device deleting

To delete a device that already exists, simply select the corresponding icon in the right-top angle on device details screen you would like to delete.

To delete a device, you must delete the patient to which this device is attached.

⚠ The history of data from the device after its removal will also be deleted.

4.6.7. Account Settings

You can also change and customize your account settings, such as setting/changing your account name and password and also account disabling.

You can open the Account settings by clicking on the corresponding icon in the top menu.

4.6.7.1. Forgot Password

To recover your password in the application, you must:

1. Proceed to login screen by opening the app.
2. Click on the "Forgot Password" button.
3. Enter the email address that you previously used to create an account.
4. Click on the "Reset password" button.
5. Check your email and find a message with confirmation code. If you do not see the email in your inbox, please check your spam or junk folder.
6. Return to the App, enter confirmation code and tap on the "Continue" button.
7. Enter password and confirm password in the designated fields and click on "Submit" button to set new password for your account.
Password should be at least 8 characters long with a mix of letters, numbers and symbols.
8. If the code matches, a confirmation message will appear, indicating that your password reset is successful.
9. Click "OK" button and log in to the application using your email and new password.

4.6.7.2. Change Name

To change the account name setting, click on "Not Specified" if the account name is not yet set, or click on the previously entered name to change it.

4.6.7.3. Change Password

To change the account password:

1. Click on the "Change password" option.
2. Enter the old password, new password, and confirm the new password.
3. After clicking the "Save" button, your account password will be updated.

Note: Password should be at least 8 characters long with a mix of letters, numbers and symbols.

4.6.7.4. Application settings

To select the appropriate units of measurement according to your preferences:

1. Select the desired unit (Metric or Imperial).
2. Click on the "Save" button.

Note: Metric unit is the default value when creating an account.

The Metric system uses degrees Celsius (°C) to measure skin temperature. The Imperial system uses degrees Fahrenheit (°F).

4.6.7.5. Logout

To logout from an app, click on the corresponding icon in the upper right corner of the screen. You can re-login to your account by entering your email and password.

4.7. TEKTELIC eDoctor Web Application Functionality

TEKTELIC eDoctor Web is a user-friendly solution for real-time monitoring of patients' vital signs and an ideal choice for remote health monitoring and tracking any changes in medical conditions. The eDoctor Web is designed to streamline access to eBeat measurements, offering users continuous control over their health parameters.

TEKTELIC eDoctor Web is your way to facilitate routine medical processes, make healthcare accessible, and increase patients' safety in remote areas. To use the web application, go to the [TEKTELIC eDoctor web](#) and log in.

4.7.1. User authorization

For authorization in the eDoctor web, you should:

1. Enter the email and password in the appropriate fields from a previously created account in the mobile application.
2. Click on the "Log in" button.

Note: Once you have completed the profile setup, you can start exploring and using the eDoctor web features and functionalities.

Note: In case of unsuccessful login to the web application, try to recheck and re-enter your email and password.

4.7.2. Manage Patient

The "**Patient**" page allows you to manage patients (view patient data, change patient information and delete patients).

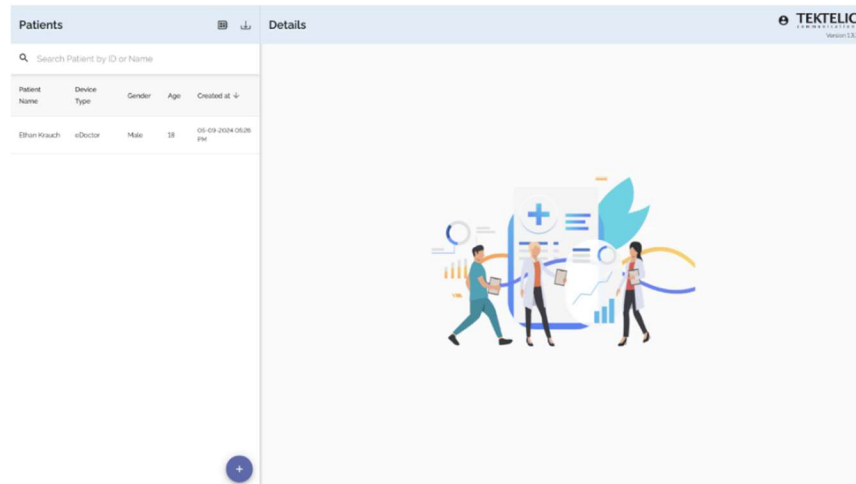


Figure 4-21: Patient page view

4.7.2.1. Adding new patient

After at least one device and one gateway have been added to the application, a new patient can be created and assigned a device.

For adding a new patient, you need to follow 4 simple steps:

1. Click on the "+" icon in the bottom of the screen.
2. Fill in the patient's information (first name, last name, age, gender), fill in the threshold fields for measurements (optional) and click "Next" button.
Note: Thresholds for pulse rate and respiration rate measurements can only contain integer values, and thresholds for skin temperature measurement can contain both integer and decimal values. To fill a decimal value, you should use only the dot symbol
3. Assign device for patient, selecting available device from dropdown list.
4. Carefully review the patient information you entered and if everything is correct and click the "Submit" button to finish creating the patient. If you see that you have entered incorrect data, then edit the patient's info.

4.7.2.2. Editing patient information

For editing patient information (first name, last name and measurements thresholds) select the patient's card, then go to the "Details" block and click on the edit icon.

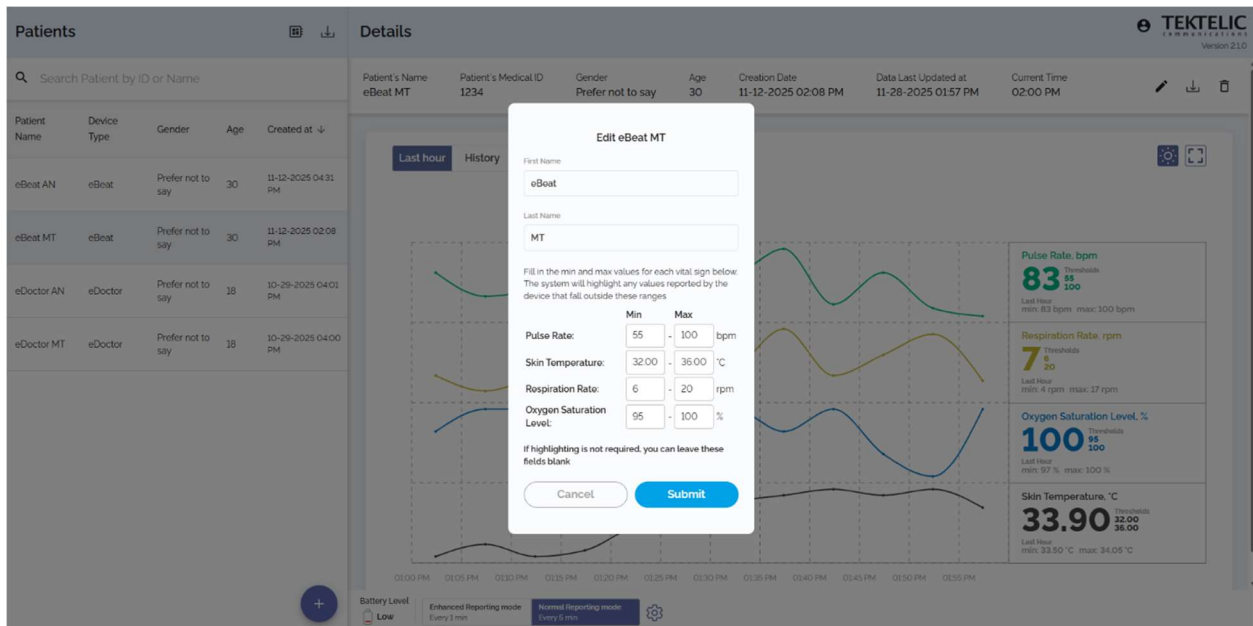


Figure 4-22: Editing patient info view

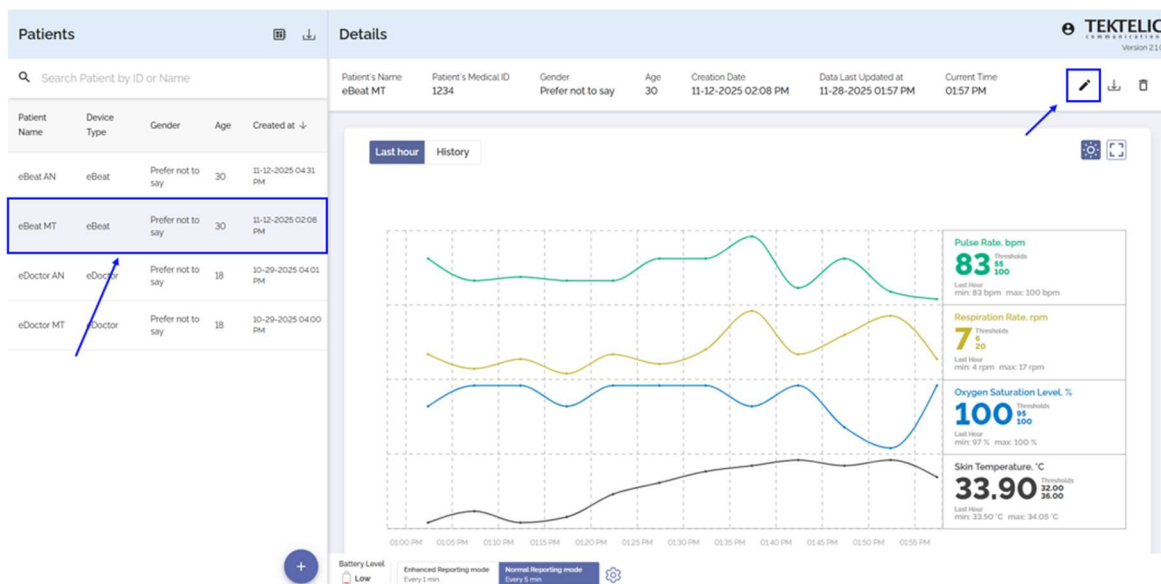


Figure 4-23: Selecting patient editing info view

Change first name, last name or measurements thresholds and click on **“Submit”** button.

4.7.2.3. Delete patient

To delete patient, simply select the patient's card, click on the corresponding icon in the right-top corner of the screen and confirm deleting by clicking on "Submit".

4.7.2.4. Patient device data

Once a patient is created, the Last Hour screen will open and display data from the device, including the main report data for the eBeat device. The eBeat device report displays skin temperature, pulse rate, respiration rate, SpO₂, the last report time, and the device’s battery level.

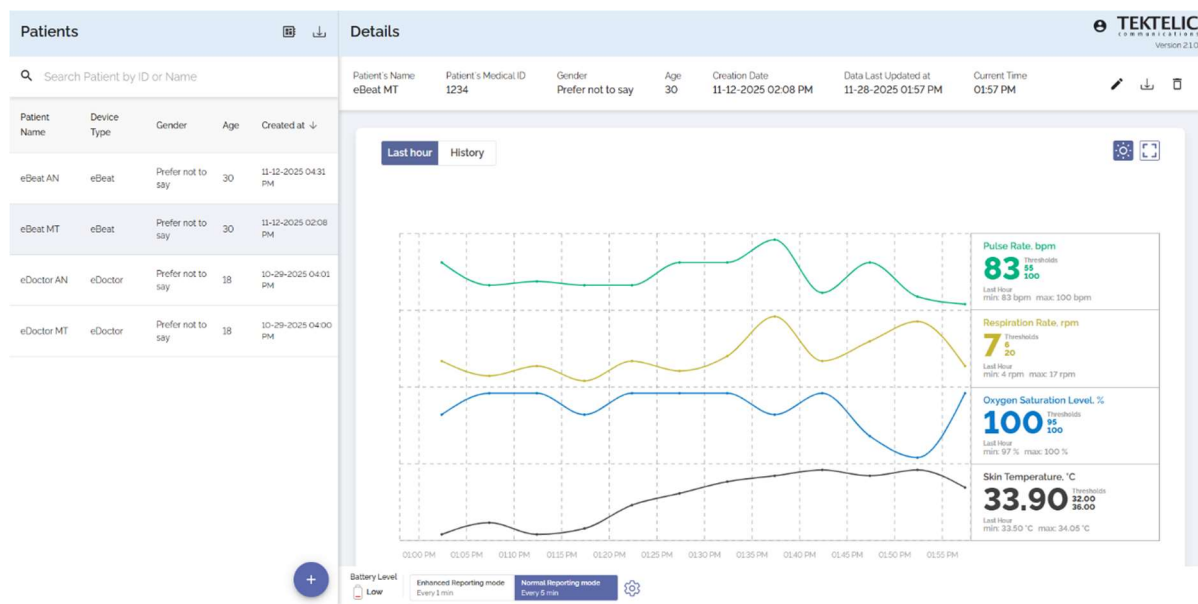


Figure 4-24: Patient data from eBeat device view

The Last Hour screen displays data received over the last hour. The data will be updated on the screen every minutes.

To the right of the graph, you can see the last received value of a specific parameter for the last hour. Also, next to each parameter its minimum and maximum value obtained over the last hour and the set threshold values are displayed.

If the obtained value for pulse rate, skin temperature, respiration rate, or SpO₂ measurements is outside the established thresholds, the corresponding value is highlighted in red on the Last Hour screen.



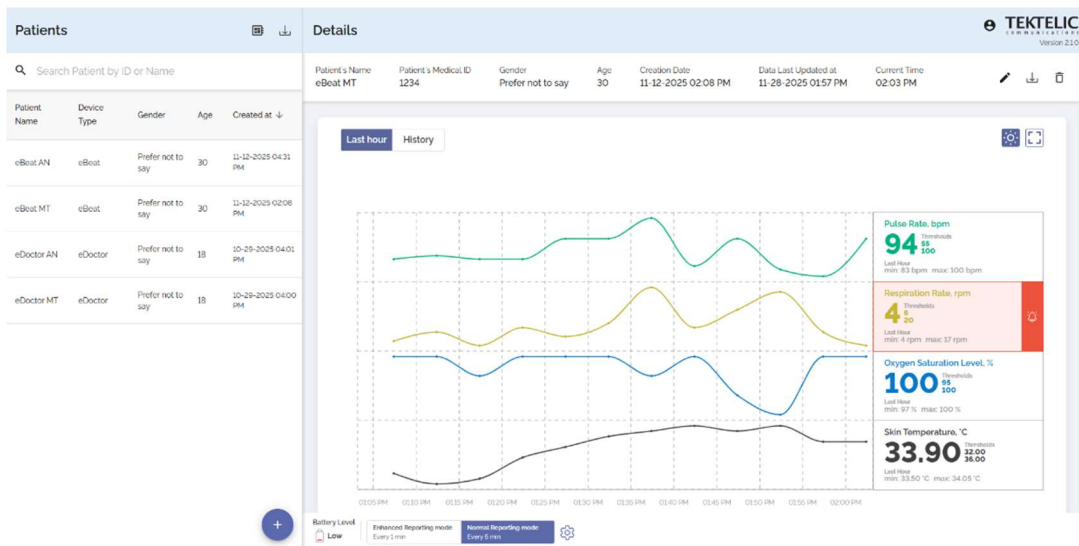


Figure 4-25: Highlighting patient's thresholds

Also, the corresponding values outside the established thresholds are highlighted in red in table view screen.

4.7.2.4.1. View device data

To view detailed information for the last three months, you need to go to the History screen.

To customize the information on the chart, select up to two measurements, the desired date range for the last three months and the aggregation option.

Three aggregation options are available: by minutes, hourly, daily.

Per-minute aggregation displays all values of the selected measurement for the selected period. Hourly aggregation displays the average received value per hour for the selected measurement over the selected period.

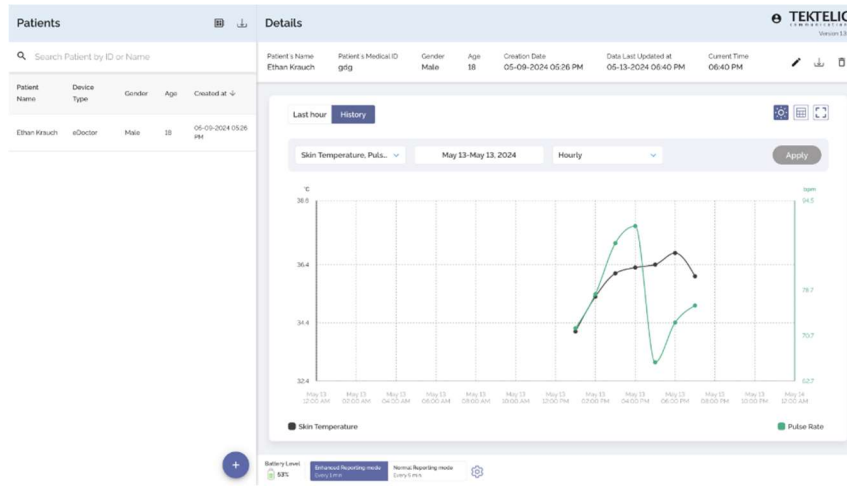


Figure 4-26: Data history chart view

Daily aggregation displays the average value received per day for the selected measurement over the chosen time period.

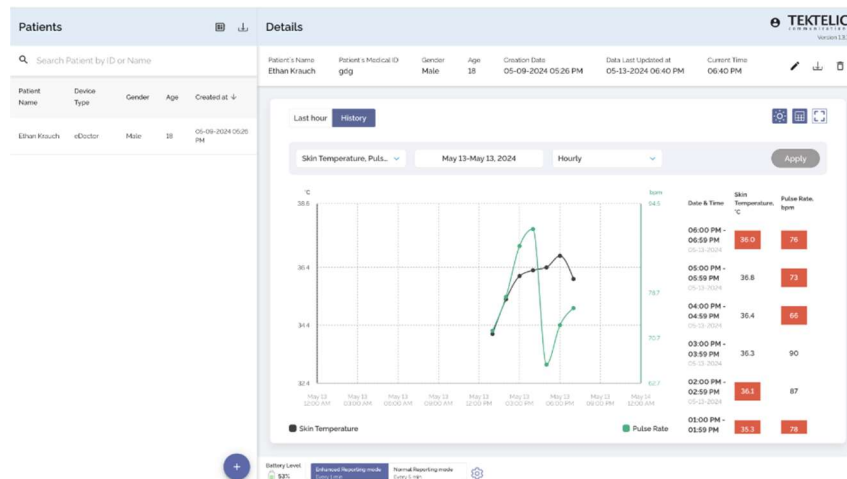


Figure 4-27: Data history table view

The data history for the selected parameter is displayed in both table and chart views. You can view the aggregated average (Avg) values for the selected period in the table or by hovering over data points on the chart. Use the “Table View” button to toggle the table display on or off.

4.7.2.4.2. Reporting mode setting

New data from the device will be received after the period set in the device settings. By default, Normal Reporting Mode is set with a reporting period of 5 minutes when worn.

The Normal Reporting Mode interval can be customized anywhere from every 2 minutes to every 60 minutes, depending on user requirements. This can be changed by clicking on the configuration icon and entering the desired reporting period.

4. Click on the corresponding icon.
5. Set the report value in minutes (minimum value 2, maximum value 60).
6. Confirm the selected value clicking "Save" button.

The user can increase the device data reporting speed to every 1 minute by switching on the Enhanced Reporting Mode (clicking on the "Enhanced Reporting Mode button", turning it purple).

4.7.2.4.3. Change theme for data visualization

To switch between light and dark themes for the visualized data, click on the corresponding icon above the graph.

The theme change will be applied to the account in the web application.

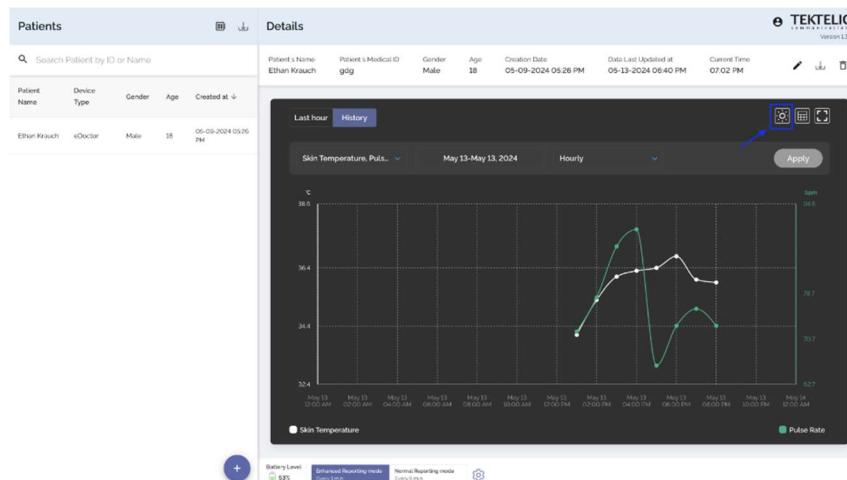


Figure 4-28: Change theme for data visualization

4.7.2.5. Application settings

To select the appropriate units of measurement according to your preferences:

1. Click on the corresponding icon in the upper right corner of the screen and select "Application settings".

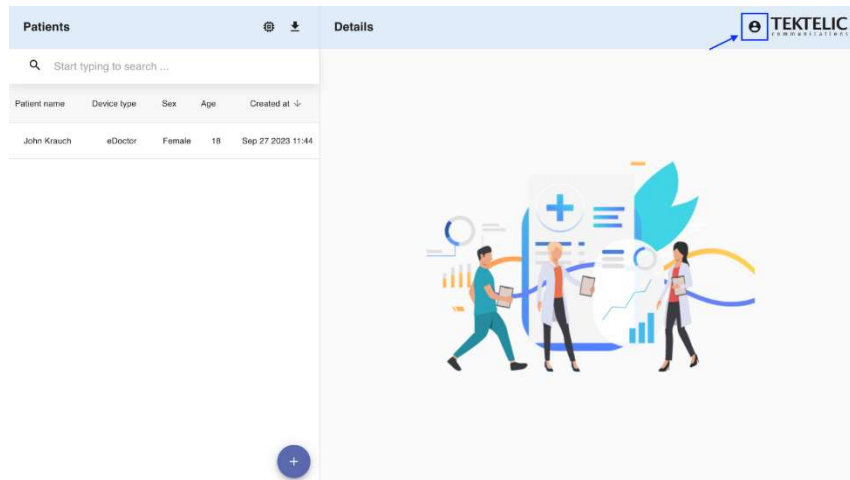


Figure 4-29: Account settings icon view

2. Select the desired unit (Metric or Imperial).
3. Click Save button.

Note: Metric unit is the default value when creating an account.

The Metric system uses degrees Celsius (°C) to measure skin temperature. The Imperial system uses degrees Fahrenheit (°F).

4.7.2.6. Full screen mode

To open full-screen mode, click the corresponding icon above the graph. Full-screen mode is available on both the “Last Hour” and “History” screens.

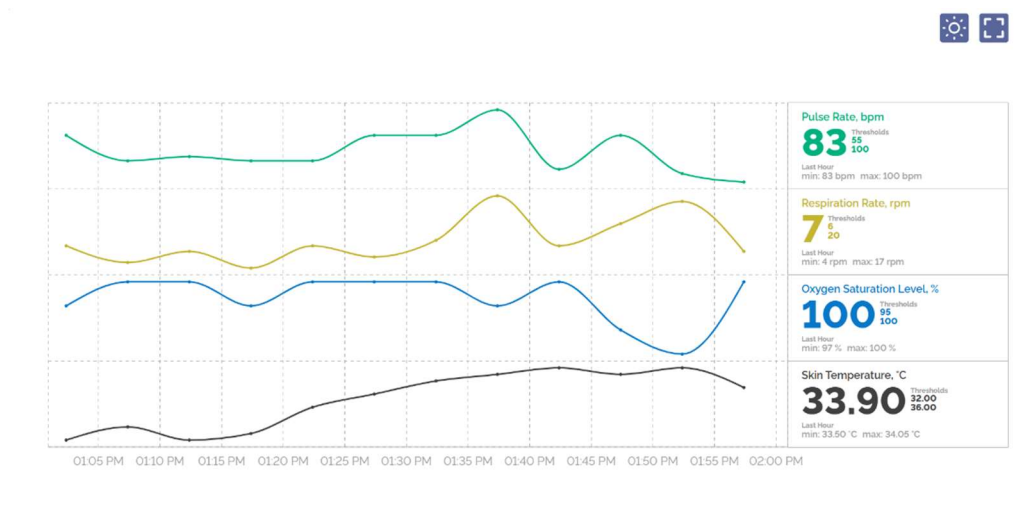


Figure 4-30: Full screen mode

4.7.2.7. Export patient's data

To obtain a table with patient/patients' data, you can use data export to a csv file. This file contains the history of the data from the patient's device and the patient's personal data.

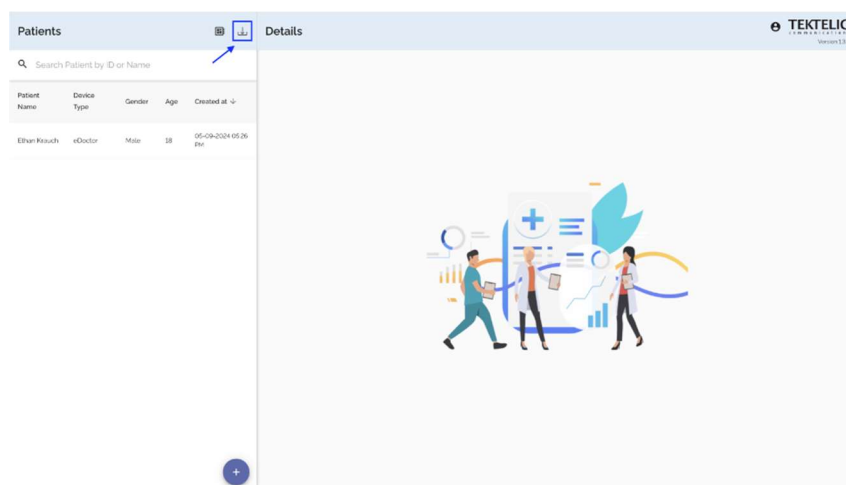


Figure 4-31: Export csv file with all patient's data

To download the data of all patients into a csv file, use the corresponding icon above the list of all patients.

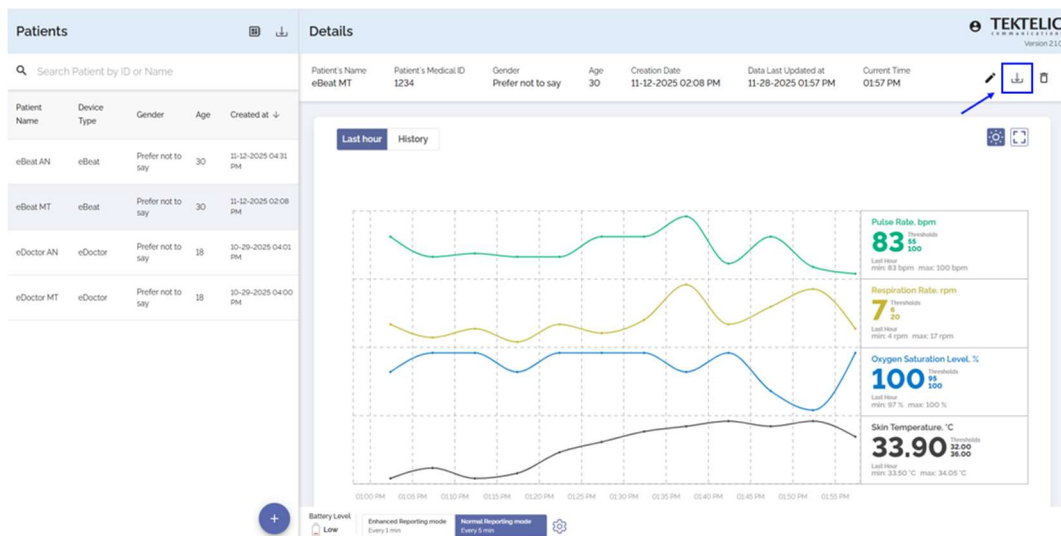


Figure 4-32: Export csv file of a specific patient's data

To download the data of a specific patients into a csv file, select the needed patient's card and use the corresponding icon in the patient details block.

4.7.3. Manage Devices

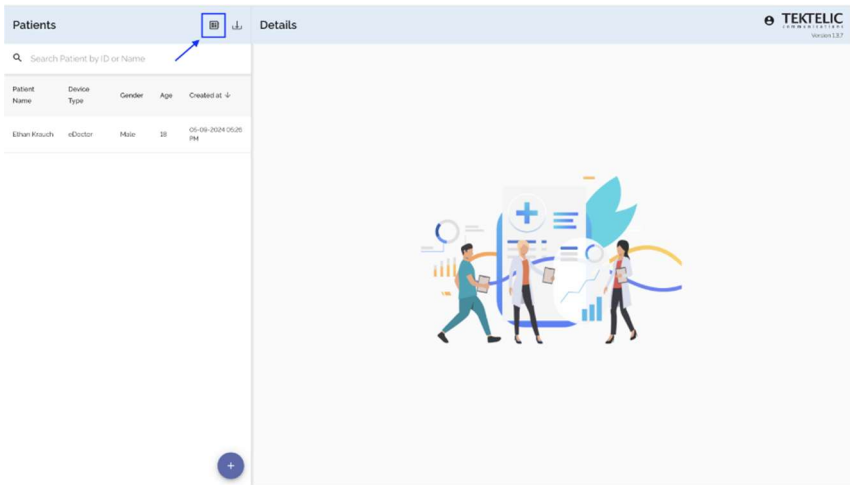


Figure 4-33: Devices list

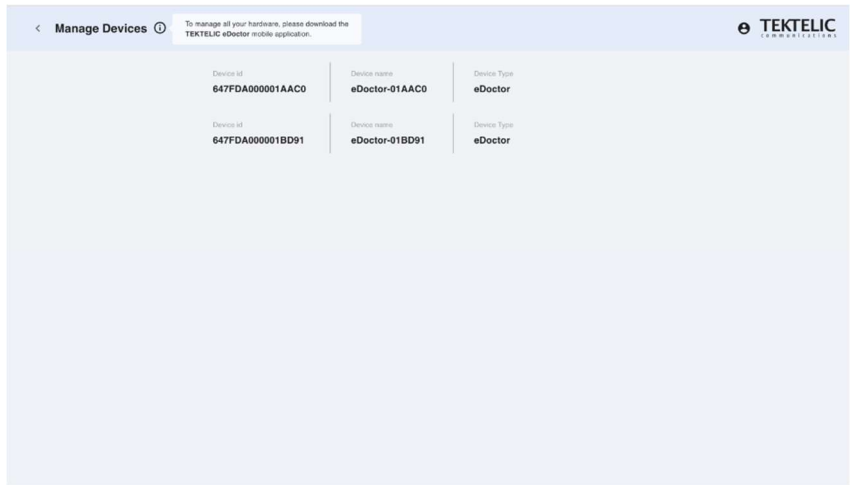


Figure 4-34: Manage devices list

The "**Manage Devices**" page allows you to view devices on account. To manage all your hardware, please download the TEKTELIC eDoctor mobile application. You can open the "**Manage Devices**" page by clicking on the corresponding icon in the top menu.

4.7.4. Forgot Password

To recover your password in eDoctor web, you must:

- 1. Proceed to login screen by opening the web application.



2. Click on the "Forgot Password" button.
3. Enter the email address from the account you are trying to log in to.
4. Click on the "Reset password" button.
5. Check your email and find a message with confirmation code. If you do not see the email in your inbox, please check your spam or junk folder.
6. Return to the web application and enter verification code.
7. Enter new password and confirm password in the designated fields and click on "Submit new password" button to set new password for your account.

Note: Password should be at least 8 characters long with a mix of letters, numbers and symbols.

8. After entering all fields correctly, the password will be successfully changed, and you will be redirected to eDoctor web login screen.
9. If the verification code entered was incorrect, a corresponding pop-up notification will appear.

4.7.5. Logout

To logout from eDoctor web, click on the corresponding icon in the upper right corner of the screen (see Figure 4-35 below).

You can re-login to your account by entering your email and password.

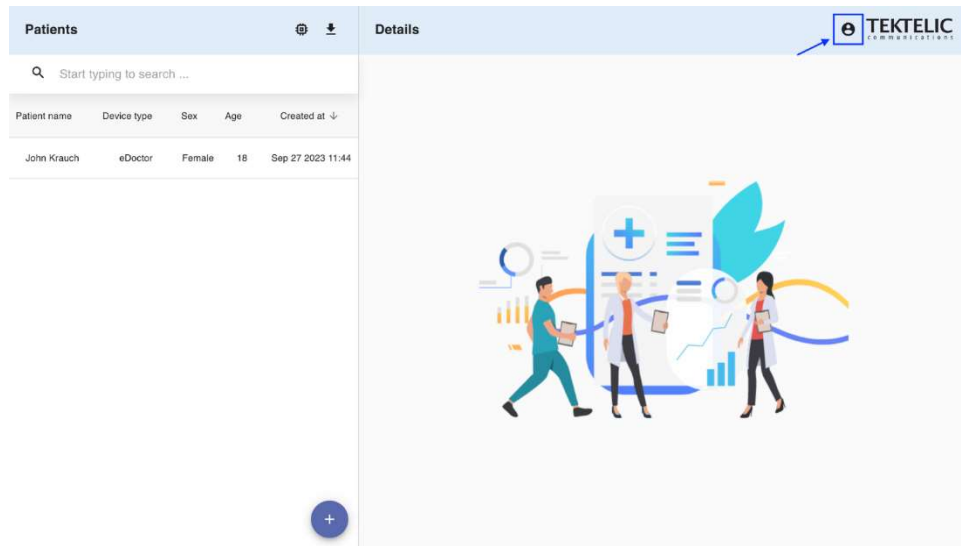


Figure 4-35: Logout from eDoctor web application

4.8. Notifications

Several types of notifications can be configured and displayed in the application. These notifications relate to the push-button assistance request, device wear status (device not worn or worn incorrectly), measurement thresholds, and poor SQI conditions. Each notification type and its meaning are described below.

4.8.1. Push-Button Assistance Request

When the push-button on the eBeat device is pressed and the corresponding uplink is received by the application, the patient and their data page are highlighted to indicate that the patient has requested assistance. If notifications are enabled, this alert will also appear in the notification section. A viewer of the application can acknowledge the notification, which removes the alert from the patient's page and indicates to other viewers that the patient has already been assisted.

4.8.2. Device Wear Status

Two types of notifications can be generated based on the device's wear status: Device Not Being Worn and Device Being Worn Incorrectly.

- **Device Not Being Worn:** Triggered when the device sends an uplink indicating it is entering sleep mode, informing the application that the sensor is no longer in contact with the user.

- **Device Being Worn Incorrectly:** Triggered when the device fails calibration five or more times consecutively. When this notification is received, it is recommended to readjust the device on the patient, as the sensor is having difficulty preparing to take accurate measurements.

4.8.3. Measurement Thresholds

When creating a new patient or editing an existing patient's profile, measurement thresholds can be configured to highlight a specific metric in red whenever it falls outside the defined range. If notifications are enabled, the threshold alert will also appear in the notification section. A viewer of the application can acknowledge the notification, which removes the alert from the patient's page and indicates to other viewers that the issue has already been addressed.

4.8.4. Poor SQI

Poor Signal Quality Indicator (SQI) notifications alert users when the device is unable to obtain reliable measurements. A notification is triggered only when five consecutive measurements for a given metric receive low-quality SQI values.

Poor SQI notifications apply exclusively to the eBeat device and the following metrics:

- Pulse Rate
- SpO₂
- Respiration Rate

If notifications are enabled, the alert appears in the application's notification section. A viewer can acknowledge the notification to remove it from the patient's page and indicate to other viewers that the issue has been addressed.

If a Poor SQI notification is received, it is recommended to adjust the device on the patient, as it may not be worn optimally, or to minimize patient movement, since excessive motion can interfere with signal quality.

4.9. Software and Firmware Updates

TEKTELIC Communications Inc. notifies users when software or firmware updates are available. Notifications are sent via email to the address associated with the user's registered account.

In addition, for mobile applications, update availability may be indicated through the respective mobile operating system's application store mechanisms. On iOS devices, the App Store may display a badge indicating available updates after a new version has been approved and published. On Android devices, the Google Play Store may generate system notifications indicating that application updates are available.

Users are responsible for maintaining accurate and up-to-date contact information and for ensuring that application updates are applied in accordance with their organization's policies.

4.10.Cleaning Instructions

⚠ CAUTION – Avoid prolonged or excessive contact with fluids. If the device is exposed to significant amounts of fluid, remove it from service and contact the manufacturer.

⚠ CAUTION – Review all cleaning instructions carefully before cleaning the device.

Table 4-1: Cleaning Supplies Required

Item Number	Equipment / Applications
1	TEKTELIC eBeat Device
2	TEKTELIC eBeat Arm Strap
3	Hospital-Grade Decontamination Wipes
4	Bowl of Cold Water
5	Mild Detergent (Dishwashing Liquid)

The TEKTELIC eBeat Device and arm strap are both non-sterile, reusable, and can be used on multiple patients.

The TEKTELIC eBeat Device and arm strap must be decontaminated between uses on different patients and before being returned to the manufacturer for maintenance or repair. This practice will limit the risk of infection and help ensure safety.

Cleaning Procedure for the eBeat Device:

1. Remove the eBeat device from the arm strap.
2. Ensure the battery lid is securely locked on the device before beginning to clean it with a disinfecting wipe.
3. Wipe down all surfaces of the eBeat device with a new, unused, hospital-grade disinfecting wipe. If the wipe dries out while cleaning, continue the cleaning with another new, unused disinfecting wipe.

4. Once the eBeat device has been fully wiped down with a disinfecting wipe, set it in a clean area to dry.

Cleaning Procedure for the Arm Strap:

1. Remove the eBeat device from the arm strap.
2. Place the strap in a bowl full of cold water with a small amount of a mild detergent, such as dishwashing liquid.
3. Handwash the arm strap and allow it to air dry lying flat.

It is the responsibility of the healthcare professional to ensure this cleaning procedure is completed correctly. Additional infection control precautions should be followed as directed by your local operating institution's guidelines.

Healthcare professionals and/or patients should regularly inspect the TEKTELIC eBeat Device and arm strap for cleanliness and decontaminate as needed.

4.11. Maintenance

Maintenance must **ONLY** be completed by TEKTELIC staff.

⚠ WARNING – Ensure the device has been cleaned before return to the manufacturer for repair or maintenance, or before disposal, to reduce the risk of infection.

⚠ WARNING – Do not remove the enclosure, current-carrying components are present. Service, repair, and maintenance must only be performed by the manufacturer.

⚠ WARNING – Device failure may occur due to use-life exceeded. Perform regular inspection of the device and arm strap.

4.12. Expected Service Life

The expected service life of the TEKTELIC eBeat Device and arm strap is 2 years. During this expected service life, the device may require battery changes every 2-3 months depending on use.

The expected service life is defined as the length of time at which the device and arm strap are suggested to be replaced to prevent performance degradation.

When the expected service life of the device is reached, the device may no longer meet the pulse rate, respiration rate, and/or skin temperature accuracy specifications. Security updates, patches, and vulnerability mitigations may no longer be available. Devices used beyond their supported service life may be subject to increased cybersecurity risk. Exceeding the expected service life will not pose an electrical safety hazard.

4.13. Medical Device Disposal

⚠ WARNING – Ensure the device has been cleaned before return to the manufacturer for repair or maintenance, or before disposal, to reduce the risk of infection.

⚠ CAUTION – The eBeat contains a lithium coin cell battery. Do not burn or puncture battery. Please ensure disposal instructions are followed.

⚠ CAUTION – Remove and immediately recycle or dispose of batteries from equipment not used for an extended period according to local regulations.

When the TEKTELIC eBeat device has reached its expected service life, remove and immediately recycle or dispose of used batteries according to local regulations and keep away from children. Do NOT dispose of batteries in household trash or incinerate.

When the TEKTELIC eBeat device and/or arm strap has reached its expected service life, please dispose of it in the garbage.

5. Troubleshooting Information

Table 5-1: Troubleshooting Problems and Options

Basic Feature	Problem	Options
General Troubleshooting	Difficulty with set-up, operation, or application(s).	• Refer to the “Setup Instructions” section of the IFU. • Refer to the “Operating Instructions” section of the IFU. • Refer to the “TEKTELIC eDoctor Mobile Application Functionality” section of the IFU. • Refer to the “TEKTELIC eDoctor Web Application Functionality” section of the IFU.
Accessory Troubleshooting	Fitting/attaching the arm strap.	• Refer to the “Wearing the device” section of the IFU.
General Troubleshooting	Powering on the device or replacing the battery.	• Refer to the “Powering on the device/replacing the battery” section of the IFU.
General Troubleshooting	Device remains blinking for more than five minutes.	• Ensure the device is provisioned on the Network Server. • Ensure the local LoRaWAN Gateway has also been provisioned on the eDoctor Mobile Application. • Ensure the local LoRaWAN Gateway is online and within range of the device.
Health-Data	Health metrics are not displaying correctly.	• Refer to the “Wearing the device” section of the

		IFU. Ensure the strap has been adjusted correctly.
General Troubleshooting	Setting up the gateway with cellular backhaul (SIM card).	Refer to the Kona Gateway Cellular Backhaul FAQ.
General Troubleshooting	Suspected cybersecurity incident (e.g. unauthorized access, abnormal behavior)	- Disconnect the affected system(s) from the network - Notify your organization's IT/security team - Follow the procedure outlined in the TEKTELIC Vulnerability Disclosure Policy

For further technical support, please visit the [TEKTELIC Support Portal](#).

If any malfunctions or mistakes occur, please contact your TEKTELIC local representative on the [TEKTELIC Website](#).

6. Technical Description

6.1. Software Bill of Material (SBOM)

TEKTELIC Communications Inc. maintains a machine-readable SBOM for the eBeat device and associated applications.

The SBOM includes a list of software components and dependencies to support cybersecurity risk management.

The SBOM is available to authorized users upon request. Please refer to the TEKTELIC Support Portal for access.

6.2. Essential Performance

Essential performance for the eBeat device is defined as: "The device shall report accurate pulse rate, respiration rate, skin temperature, heart rate variability, and SpO₂ values." To ensure essential performance the following conditions should be considered:

⚠ CAUTION – Avoid prolonged or excessive contact with fluids. If the device is exposed to significant amounts of fluid, remove it from service and contact the manufacturer.

⚠ CAUTION – Damage to the device may degrade sensor and electrode performance.

⚠ CAUTION – The device should not be near or in contact with high temperatures (i.e., Fireplace).

⚠ WARNING – Only wear the device on intact skin. Inspect the area of contact with the device and strap at least every 24 hours. If irritation is observed, remove the device immediately.

For more information on the pulse rate, respiration rate, skin temperature, heart rate variability, and SpO₂ accuracy specifications please refer to the Pulse Rate, Respiration Rate, Skin Temperature, Heart Rate Variability, and SpO₂ sections.

6.3. Use Environments

The TEKTELIC eBeat Device is designed for use in indoor environments. The storage temperature conditions range from -25°C to 70°C. The operational conditions range from 5°C to 40°C. The most common use environments would include in professional healthcare facilities, such as hospitals or skilled nursing facilities, or their own home.

It will stop reporting after 1 minute of not being worn.

6.4. Monitoring Specifications

This section will describe the security specifications for the eBeat device, eDoctor web application, eDoctor mobile application, and server application. It also describes the transducer, measurement range, accuracy (if applicable), and resolution of all reported parameters. Reported parameters without accuracy are considered wellness parameters.

6.4.1. Device/System Security

Table 6-1: eBeat Device Security Specifications

eBeat Device Security Specifications	
Encryption Standard	AES 128 encryption as per the LoRaWAN Specification
Data Storage	Data stored in the device flash memory includes:

	- Parameter settings. - LoRaWAN settings, including secure keys. - Flash memory is protected against unauthorized readout using device-level read protection.
Data Exchange	HTTPS protocol, transferred through TLS connection, encrypted by algorithm RSA 2048
Firmware Update Control	Firmware updates for the TEKTELIC eBeat device are controlled through manufacturer-authorized FUOTA mechanisms. Only authenticated and authorized firmware updates are permitted.

Table 6-2: eDoctor Web Application Security Specifications

eDoctor Web Application Security Specifications	
Encryption Standard	User-related data is stored in an AWS Cognito user pool
Data Storage	PostgreSQL database hosted on AWS cloud server
Data Exchange (Between User and Software)	HTTPS protocol, transferred through TLS connection, encrypted by algorithm RSA 2048

Table 6-3: eDoctor Mobile Application Security Specifications

eDoctor Mobile Application Security Specifications	
Encryption Standard	User-related data is stored in an AWS Cognito user pool
Data Storage	Temporarily stores information the user works with in the application
Data Exchange (Between User and Software)	HTTPS protocol, transferred through TLS connection, encrypted by algorithm RSA 2048

Table 6-4: eDoctor Server Application Security Specifications

eDoctor Server Application Security Specifications	
Encryption Standard	Amazon RDS
Data Storage	User data is stored in AWS Cognito. Information about devices, gateways, patients, and patients' parameters, user account settings, and device configurations are stored in the PostgreSQL database.
Data Exchange (Between User and Software)	HTTPS protocol, transferred through TLS connection, encrypted by algorithm RSA 2048

6.4.1.1. Network Ports

The TEKTELIC eDoctor application uses secure HTTPS (port 443) connections to communicate with web browsers and mobile applications. LoRaWAN gateways deployed with the eBeat solution may use MQTTS over port 8883 for secure communication. No inbound network ports are required to be opened within the user's or healthcare facility's network for normal system operation.

6.4.1.2. Protection of Critical Functionality

The TEKTELIC eBeat system incorporates security features intended to safeguard the device, its core functionality, and associated user data. These protections are implemented by TEKTELIC and are enabled as part of normal system operation.

Communications between the eBeat device, LoRaWAN gateway, and backend infrastructure are secured using industry-standard encryption methods. Each device uses unique identifiers and cryptographic keys to ensure that data is exchanged only with authorized system components.

Data transmitted across system elements, including cloud services and user-facing applications, is protected through secure communication protocols (such as TLS). These protections help prevent unauthorized access to system data and help ensure that device operation is not altered or interfered with.

6.4.1.3. Data Flow and Security Controls

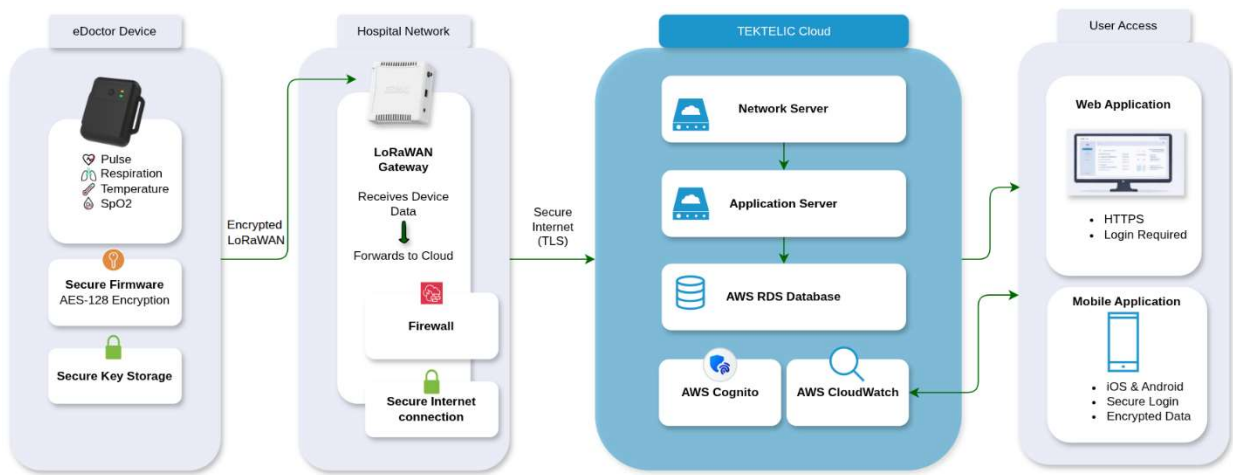


Figure 6-1: Data Flow and Security Controls

6.4.2. Pulse Rate

Pulse Rate shows how fast the heart is beating and helps indicate changes in activity level, stress, or overall heart health. The unit of measurement for pulse rate is in beats per minute (bpm).

Table 6-5: Pulse Rate Specifications

Pulse Rate Specifications	
Measurement Method	PPG-derived pulse rate algorithm
Measurement Range	30 bpm to 240 bpm
Resolution	1 bpm
Accuracy	+/- 3 bpm

Measurements falling outside the measurement range are reported as invalid.

In the last 10 seconds of every minute, the pulse rate is sampled multiple times. The pulse rate reported is the rounded statistical average of the per-minute pulse rate over the last three minutes.

6.4.3. Respiration Rate

Respiration Rate shows how fast a person is breathing and can help identify changes in comfort, activity, or breathing difficulty. The unit of measurement for respiration rate is respirations per minute (rpm).



Table 6-6: Respiration Rate Specifications

Respiration Rate Specifications	
Measurement Method	PPG-derived respiration rate algorithm
Measurement Range	4 rpm to 36 rpm
Resolution	1 rpm
Accuracy	+/- 2 rpm

Measurements falling outside the measurement range are reported as invalid.

In the last 20 seconds of every minute, the respiration rate is sampled multiple times. The respiration rate reported is the rounded statistical average of the per-minute respiration rate over the last three minutes.

6.4.4. Blood Oxygen Saturation (SpO₂)

SpO₂ shows how much oxygen is in the blood and helps identify if a person is getting enough oxygen during rest or activity. The unit of measurement for SpO₂ is percentage (%).

Table 6-7: SpO₂ Specifications

SpO ₂ Specifications	
Measurement Method	PPG-derived SpO ₂ algorithm
Measurement Range	70% to 100%
Resolution	1%
Accuracy	+/- 4%

Measurements falling outside the measurement range are reported as invalid.

In the last 10 seconds of every minute, SpO₂ is sampled multiple times. The SpO₂ reported is the rounded statistical average of the per-minute SpO₂ over the last three minutes.

6.4.5. Skin Temperature

Skin Temperature helps monitor changes in body temperature, which can be useful for detecting fever, heat stress, or environmental effects. The default unit of measurement for skin temperature is degrees Celsius (°C), it can be changed to Fahrenheit on the application.

Note: To ensure the accuracy stated in the table below is achieved, please allow for the skin temperature readings to stabilize.

Table 6-8: Skin Temperature Specifications

Skin Temperature Specifications	
Measurement Method	Skin-to-contact digital temperature sensor
Measurement Range	30°C to 43°C

Resolution	0.05°C
Accuracy	+/- 0.3°C

Measurements falling outside the measurement range are reported as invalid.

Skin Temperature Time Constant: The device will take 60 seconds to warm to the wearer's actual skin temperature.

Skin temperature is measured by the skin-to-contact digital temperature sensor every minute while being worn. The reported skin temperature is the skin temperature measurement performed in the last minute.

6.4.6. Heart Rate Variability (HRV)

HRV shows how much time varies between heartbeats and can give insight into stress levels, recovery, and overall wellness. The unit of measurement for HRV is milliseconds (ms).

Table 6-9: Heart Rate Variability Specifications

Heart Rate Variability Specifications	
Measurement Method	Derived from beat-to-beat (RR interval) variability
Available Windows	Past hour, past day, past week
Resolution	1 ms

Heart Rate Variability (HRV) has additional configuration options in the application, allowing users to choose whether they want their HRV value calculated over the past hour, past day, or past week of data. Shorter windows provide more immediate insight into recent physiological changes, while longer windows offer a smoother measure of long-term trends.

An HRV value is only displayed when the device has collected at least one full window of data (1 hour, 1 day, or 1 week).

HRV is calculated from the variability between successive heartbeats using fixed internal algorithms. The application computes HRV over the selected historical window and displays the resulting short-term or long-term HRV statistic.

6.4.7. Signal Quality Indicator (SQI)

Pulse Rate, SpO₂, and Respiration Rate each have an associated Signal Quality Indicator (SQI) value for every reported measurement. SQI shows how reliable a measurement is, helping users understand when movement or poor contact may affect accuracy. SQI is a categorical

quality measure assigned to Pulse Rate, SpO2, and Respiration Rate. It does not have a numeric unit.

Table 6-10: Signal Quality Indicator Specifications

SQL Categories	Meaning
High	Reliable signal quality
Medium	Acceptable but reduced signal quality
Low	Poor signal quality, measurement may be inaccurate

When the user hovers over a metric, the application displays a three-bar SQL indicator next to the value:

- Three colored bars (**green**): High SQL
- Two colored bars (**yellow**): Medium SQL
- One colored bar (**red**): Low SQL

Five consecutive low-SQL readings trigger a Poor SQL notification, indicating the device may be worn incorrectly or the patient is moving excessively.

6.4.8. Battery Capacity

Battery Capacity shows how much charge is left in the device so users can avoid interruptions in monitoring. The unit of measurement for battery capacity is percentage (%).

Table 6-11: Battery Capacity Specifications

Battery Capacity Specifications	
Measurement Method	Internal battery estimation algorithm
Measurement Range	0% to 100%
Resolution	1%

Measurements falling outside the measurement range are reported as invalid.

Battery capacity is estimated by the device's internal power-management algorithm. In the application, battery level is shown as part of the device status, allowing users to easily see when the eBeat device needs a battery replacement.

6.5. Electrical Safety

⚠ CAUTION – Do not touch the battery and the patient at the same time during battery replacement.

⚠ CAUTION – Do not touch the gateway/smart phone/computer and patient at the same time.

The TEKTELIC eBeat Device has been tested for and complies with electrical safety and IEC 60601-1:2005, AMD1:2012, AMD2:2021, IEC 60601-1-11:2015 and AMD1:2021 for devices intended for home use.

6.6. MRI Safety Information

⚠ WARNING – The eBeat device and arm band are MR Unsafe. Do not expose the device or the arm band to a magnetic resonance (MR) environment. The device and arm band present projectile hazards. Patients should not receive an MR exam while the device and arm band are fastened to the patient.

6.7. Electromagnetic Compatibility (EMC)

⚠ WARNING – Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

⚠ WARNING – Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

⚠ WARNING – Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the TEKTELIC eBeat Device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The TEKTELIC eBeat Device has been tested for and complies with IEC 60601-1-2:2014 + AMD1:2021 and is FCC qualified as a portal device. The TEKTELIC eBeat device has also been tested according to the recommendations of IEC TS 60601-4-2:2024 Medical electrical equipment – Part 4-2: Guidance and interpretation – Electromagnetic immunity: performance of medical electrical equipment and medical electrical systems.

Electromagnetic Emissions Declaration:

The eBeat device is designed for use in the specified electromagnetic environment outlined below. Users must ensure that the eBeat device is operated within this environment.

Table 6-12: Emissions Test Compliance and Guidance

Emissions Test	Compliance	Electromagnetic Environment Guidance
Electromagnetic Radiation Disturbance (Radiated Emissions) CISPR 11	Group 1	The eBeat uses RF energy only for its internal function.
Electromagnetic Radiation Disturbance (Radiated Emissions) CISPR 11	Class B	The eBeat is intended for use in domestic establishments.
Conducted Disturbances (Conducted Emissions) CISPR 11	N/A	Device is battery powered with no AC or DC power port.
Harmonic Current Emissions IEC 61000-3-2	N/A	Device is battery powered with no AC or DC power port.
Voltage Changes, Voltage Fluctuations and Flicker Emissions IEC 61000-3-3	N/A	Device is battery powered with no AC or DC power port.

Electromagnetic Immunity Declaration:

The eBeat device is designed for use in the specified electromagnetic environment outlined below. The compliance levels represent the levels at which the eBeat has been tested to ensure that it maintains basic safety, essential performance, and non-clinical performance. Users must ensure that the eBeat device is operated within this environment.

The eBeat may be unable to provide accurate results when subject to noise or interference affecting the measurement quality. Ensure that the environment is free from potential sources of interference to minimize the risk of inaccurate measurements. Examples of potential sources of interference include, but are not limited to, other electronic devices or strong electromagnetic fields.

Electrostatic discharge could cause the eBeat to restart. If this occurs, the eBeat is expected to recover and continue operation within 30 seconds. This has no impact on the essential performance of the eBeat.

The operator (i.e., the healthcare professional) should take appropriate measures to minimize the impacts of potential interference by following the guidance below.

Table 6-13: Immunity Test Compliance and Guidance

Immunity Test	Compliance	Electromagnetic Environment Guidance
Electrostatic Discharge IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Radiated RF EM fields IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the TEKTELIC eBeat Device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
Proximity fields from RF wireless communications equipment IEC 61000-4-3	See Table 6-14.	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the TEKTELIC eBeat Device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
Rated power frequency magnetic fields IEC 61000-4-8	N/A	Device and arm band do not contain any magnetically sensitive components.
Proximity Magnetic Fields IEC 61000-4-39	N/A	Device and arm band do not contain any magnetically sensitive components.
Conducted disturbances induced by RF fields IEC 61000-4-6	N/A	Patient coupling points do not have conducting cables.
Electrical fast transients/bursts IEC 61000-4-4	N/A	Device is battery powered with no AC or DC power port.

Surges IEC 61000-4-5	N/A	Device is battery powered with no AC or DC power port.
Voltage dips IEC 61000-4-11	N/A	Device is battery powered with no AC or DC power port.
Voltage interruptions IEC 61000-4-11	N/A	Device is battery powered with no AC or DC power port.
Electrical transient conduction along supply lines ISO 7637-2	N/A	Device is battery powered with no AC or DC power port.

Table 6-14: Proximity Fields from RF Wireless Fields Immunity Test Compliance

Test Frequency (MHz)	Modulation	Immunity Test Level (V/m)
385	Pulse Modulation: 18 Hz	27
450	FM: ± 5 kHz deviation, 1 kHz sine	28
710, 745, 780	Pulse Modulation: 217 Hz	9
810, 870, 930	Pulse Modulation: 18 Hz	28
1720, 1845, 1970	Pulse Modulation: 217 Hz	28
2450	Pulse Modulation: 217 Hz	28
5240, 5500, 5785	Pulse Modulation: 217 Hz	9

6.8. FCC Compliance Statements

Federal Communications Commission

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

To comply with FCC exposure limits for general population / uncontrolled exposure, this device has lowered its maximum power to comply with SAR limits for wearable devices and must not be co-located or operating in conjunction with any other transmitter.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy

and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

7. Precautions

In the interest of ensuring a secure and enjoyable experience with our mobile and web application, we strongly recommend the following precautions:

⚠ CAUTION – Mobile devices and PCs used to access the eDoctor web app or the eDoctor mobile app should be secured when not in use.

⚠ CAUTION – Users should regularly apply/allow security updates to your device's operating system (OS).

⚠ CAUTION – TEKTELIC does not make any commercial use of or disclose any private information of eBeat users.

⚠ CAUTION – Follow applicable hospital data backup and disaster recovery processes and ensure that data from the web and mobile applications is periodically backed up to maintain availability of information. TEKTELIC technical support can be contacted to assist with disaster recovery.

⚠ CAUTION – Safeguard your login information with the utmost care. Never share your username and password with any third party.

⚠ CAUTION – Exercise prudence when installing mobile applications. Only install apps from trusted sources, such as official app stores. Avoid downloading and installing applications from unverified or unofficial sources.

⚠ CAUTION – Use strong, unique passwords for your web/mobile accounts. Combine a mix of upper and lower-case letters, numbers, and symbols. Consider using a reputable password manager.

⚠ CAUTION – TEKTELIC personnel will never request your password or login information. Notify TEKTELIC immediately if you receive unexpected or confusing emails, phone calls, or other forms of communication.

By adhering to these precautions, you contribute to a safer and more secure experience while using our mobile and web applications.

8. Warranty

Products sold hereunder are covered by a warranty against defects in material and workmanship provided the Products and services are subjected to normal use and service. The applicable warranty period is twelve (12) months plus two weeks from the date of shipment. The foregoing warranty shall not apply to:

- 1) Products that have been modified by anyone other than TEKTELIC without TEKTELIC's prior written consent and/or subjected to improper handling, storage, installation, operation or maintenance;
- 2) Any item of the Products which is furnished by Purchaser as a component part of a purchased Product, or any equipment that is not manufactured by TEKTELIC and is purchased by TEKTELIC on behalf of Purchaser. For the removal of doubt, components selected or sourced by TEKTELIC shall be covered by the warranty;
- 3) Models or samples which are furnished to Purchaser as illustrations only of the general properties of TEKTELIC's products and workmanship;
- 4) Damage to Products caused by abrasive materials, corrosion due to aggressive fluids, lightning, improper voltage supply, mishandling or misapplication.

Except as expressly set forth herein, TEKTELIC expressly disclaims any and all representations, warranties, conditions, covenants, guarantees and/or assurances, whether express or implied, relating to or arising out of Products or services and/or this contract, including without limitation, any implied representations, warranties, conditions, covenants and/or guarantees as to merchantability, merchantable quality, non-infringement, fitness for a particular purpose, those arising by statute or otherwise, or from a course of dealing or usage or trade. To the extent permitted by applicable law, all warranties, representations, terms and conditions other than as expressly contained herein are excluded. Each party acknowledges that it has not relied on any other term, condition, representation, warranty, matter, statement or conduct in entering into this contract.

Purchaser's sole remedy for a breach of this limited warranty, for any other liability under this contract, or for any liability that cannot be excluded as provided by applicable law, shall be limited (at TEKTELIC's sole option) the repair, replacement or refund to Purchaser's account for any such Products which are returned by Purchaser during the applicable warranty period, provided that (i) TEKTELIC is promptly notified in writing during such warranty period upon discovery by Purchaser that the Products failed to conform to the express warranties set forth

above, (ii) such Products are returned to TEKTELIC Incoterms 2010 DDP TEKTELIC's plant shipping costs covered by the Purchaser, and (iii) TEKTELIC's examination of such Products shall disclose to TEKTELIC's satisfaction that such alleged deficiencies actually exist and were not caused by accident, misuse, neglect, alteration, improper installation, unauthorized repair or improper testing as set forth above. If TEKTELIC elects to repair or replace such Products, TEKTELIC shall have a reasonable time to make such repairs or replace such Products. The Products shall be returned to Purchaser location Incoterms 2010 DDP, cost covered by TEKTELIC. The repaired or refurbished units will receive a ninety (90) day warranty, or the remainder of the original product warranty, whichever is longer.

9. Contact Us

For questions or comments about the device and its application, or other issues related to the eBeat solution requiring assistance, please contact your TEKTELIC local representative which can be found on the [TEKTELIC Website](#).

For technical support please refer to the Troubleshooting section or visit the [TEKTELIC Support Portal](#).

To contact our local United States of America representative, please refer to the following name and address:

Rook Quality Systems

1155 Mount Vernon Hwy
Suite 800
Dunwoody GA 30338

Table 9-1: Manufacturer Symbol and Description

Symbol	Manufacturer
	TEKTELIC Communications Inc. 7657 10 St NE, Calgary, AB T2E 8X2