VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

Padded u-sling

MODEL:13220S,13220M,13220L

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

Padded u-sling

MODEL:13220S,13220M,13220L

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Padded u-sling.
Please read all of the instructions before using it. The information will help you achieve the best possible results.

WARNING:

Do not use your Vevor sling without first reading and understanding the operator's manual supplied with your Vevor sling. If you are unable to understand the warnings or instructions contact a healthcare professional before attempting to use your sling. Using your sling without properly understanding the instructions and the warnings can cause injury to the patient and/or caregiver, and cause damage to the sling.

 **WARNING:** Check the sling for damage or missing parts before using. In case of damage or incompleteness, DO NOT use the sling. Contact Vevor Customer Service for further instructions

 **WARNING:** Carefully inspect the sling before each use and after each laundering for wear and damage to seams, fabric, and handles. Torn, cut, frayed or damaged

slings can fail,resulting in serious personal injury to the user or caregiver.Use only slide sheets that are in good condition.Discard and destroy old,unusable slings and replace with a new one.

⚠ WARNING:DO NOT exceed the patient lift or the patient slings maximum weight capacity..

⚠WARNING:Notice for California Customers –California Proposition 65 Warning –This product contains a chemical known to the State of California to cause cancer and reproductive or develop– mental harm.

⚠WARNING: Vevor assumes no responsibility for any damage or injury caused by improper application or use of this product.

⚠ WARNING:Never leave a patient unattended while in a patient lift sling.

⚠ WARNING:Never attempt to re–position a patient by pulling on the sling loops.

General Guidelines for Using the Sling:

Use an appropriate Patient Sling that is recommended by the individual's doctor, nurse, or medical assistant for the comfort and safety of the individual being lifted and their specific medical need.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and any patient sling as this may cause the patient to slide out of the sling during transferring.

The loops of the sling are color coded and can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally. Make sure that there is sufficient head support when lifting a patient.

Sling Material

Sling Material: Polyester / Straps Material: Polypropylene

Using a Divided Leg Sling, U-Sling, or Toileting Sling

Note: Please make sure the patient's weight does not exceed the weight capacity for the patient sling or lift before you attempt to lift the patient.

Note: Use the following method to easily move the patient and avoid strain to yourself.

Note: The following method assumes a caregiver and an assistant.

⚠ WARNING: If the patient is in a wheelchair, secure the wheel locks in place to prevent the wheelchair from moving forward or back-wards

1. Lock both rear wheel locks on the wheelchair. Refer to Figure 1.

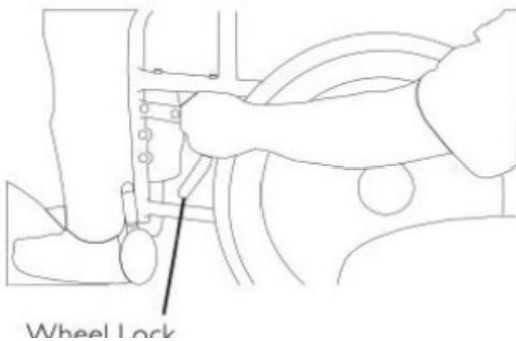
FIGURE1:Wheel Lock

2. With the patient sitting in a wheelchair one assistant is in front of the wheelchair and the other assistant in the back of the wheelchair, lean the patient forward. Refer to Detail "A" in Figure 2.

Note: The front assistant will be supporting the weight of the patient.

Note: Toileting slings have a waist buckle which helps hold the sling in

place during positioning.

3. Place the sling behind the patient (with “grab handle” on the outside) and bring the flaps out alongside the patient's legs.

Refer to Detail "B" in Figure 2.

Note: For the Divided Leg sling and U-Sling: the back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's head and the base of the spine.

FIGURE 2: Positioning the Sling

4. With the back of the sling positioned properly, push the bottom of the back section to the seat.
5. Lean the patient back into the wheel-chair with the assistant in the rear supporting his/her weight.

Note: For the Toileting Sling: the back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's mid back and the base of the spine

6. Lift the patient's legs (one at a time) and reach under the patient's leg to pull the front of the sling until it is behind the patient's knees (about 3 inches). The back of the sling should remain in position. Refer to Details "C" to "F" in Figure 3.

Note: The straps may be around, crossed between or underneath the patient's legs.

Refer to Detail "G" in Figure 3.

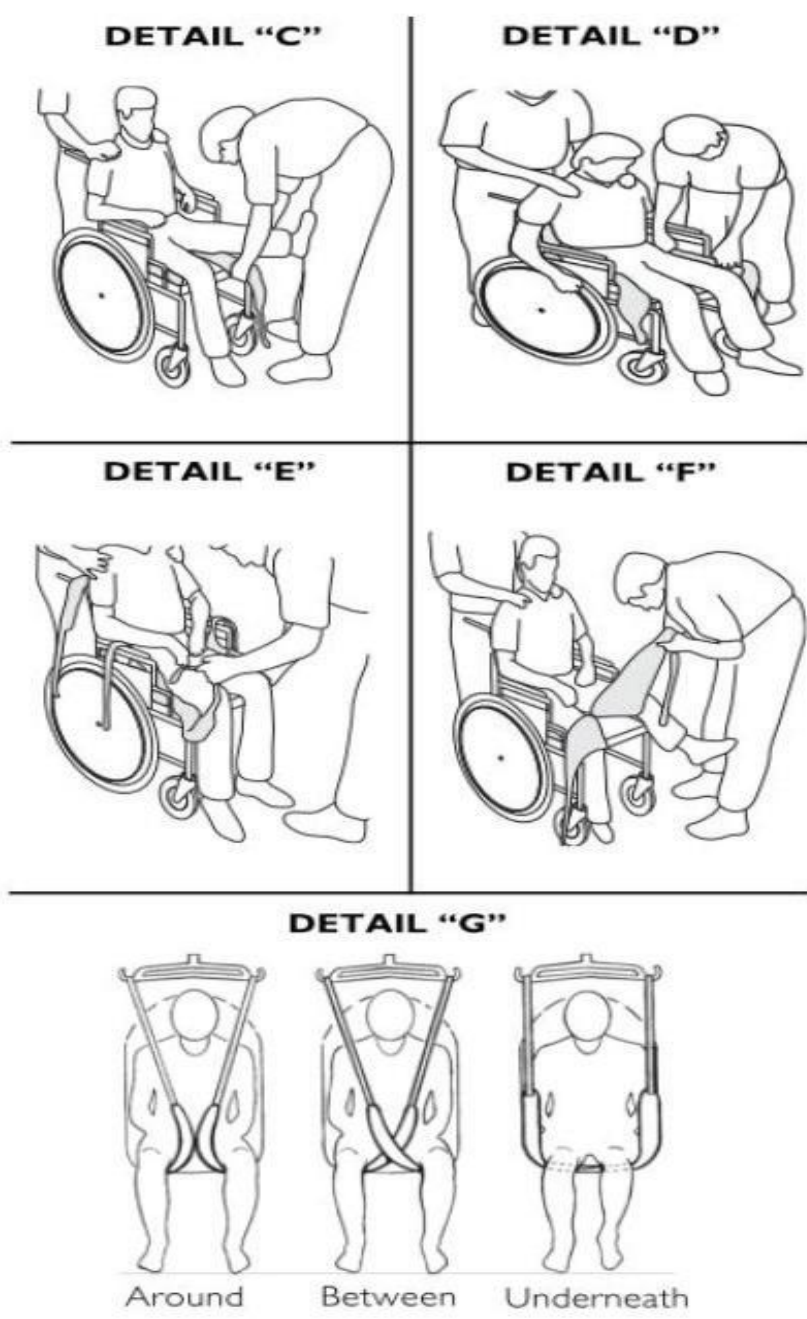
FIGURE 3: Positioning the Sling

Washing Instructions:

Machine wash in warm water or cold water at normal cycle. Maximum water temperature of 120°F (48°C). Air dry or tumble dry at cool or very low temperature. DO NOT use bleach or wash with other colors. Inspect after each wash. DO NOT dry clean.

▲ WARNING: Inspect patient slings for wear prior to each use. If signs of tearing, fraying or wear are found, discard the sling immediately; worn out slings are not safe for use and may result in injury or death. Do not alter slings in any way.

Lifetime Limited Warranty

Your Vevor sling is warranted to be free from defects in materials and workmanship for a period of one year from the date of purchase. This warranty is extended only to the original purchaser/user of our Vevor slings. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Vevor's option. This warranty does not include any labor or shipping charges incurred in replacement or repair of any such product. Vevor's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement. Useful life of this product is 6 months from date of purchase under normal use, however, heavy use or excessive washing may reduce the useful life of the product. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear, or incorrect washing and drying at temperatures higher than 120°F (48°C), or failure to adhere to the instructions in this manual. Vevor shall not be liable for any consequential or incidental damages whatsoever. Some states do not allow exclusion or limitation of incidental or consequential damage, or limitation on how long an implied warranty lasts, so the above exclusions and limitations may not apply to you. This warranty shall be extended to comply with state or provincial laws and requirements.

Manufacturer: Dongguan Chuangguo Daily Products Co. Ltd

Address: No.169.Kangyi Road, Qingxi Town,Dongguan

City,Guangdong.CN 523657

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,

Centurion House,

London Road, Staines-upon-Thames, Surrey,

TW18 4AX

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Mainzer Landstr.69, 60329 Frankfurt am

Main.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Produttore: Dongguan Chuangguo Daily Products Co. Ltd
Importato in AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importato negli USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH