


HORSEPAL Thermometer Quick Start Guide

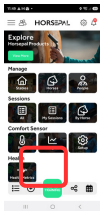

1

Turn ON & Open App


Turn on thermometer and
Tap Health Metrics
on the homepage of
the HORSEPAL App



2

Select Horse


Choose your horse.

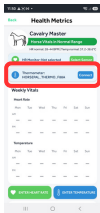

3

Select Thermometer


Tap the thermometer tab.

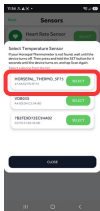

4

Press Connect

Return to previous page
and press Connect.

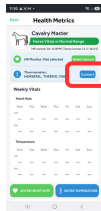

5

Confirm Connection

Select your thermometer.



6

Ready to Use

Thermometer is now
connected.

Warning Statment

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

c. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment

RF warning for portable device:
The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction

C.