VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

Positioning Bed Pad With Handles

MODEL:CGSL280

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

Positioning Bed Pad With Handles

MODEL:CGSL280

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Positioning Bed Pad With Handles. Please read all of the instructions before using it. The information will help you achieve the best possible results.

WARNING:

Do not use your Vevor Positioning Bed Pad without first reading and understanding the operator's manual supplied with your Positioning Bed Pad. If you are unable to understand the warnings or instructions contact a healthcare professional before attempting to use your Positioning Bed Pad. Using your Positioning Bed Pad without properly understanding the instructions and the warnings can cause injury to the patient and/or caregiver, and cause damage to the Positioning Bed Pad.

 **WARNING:** If any part of the Positioning Bed Pad is damaged or missing, contact VEVOR customer service immediately.

 **WARNING:** Carefully inspect the Positioning Bed Pad before each use and after each laundering for wear and damage to seams, fabric, straps, and strap loops. Torn, cut, frayed or broken Positioning Bed Pads can fail, resulting in serious personal injury to

the user or caregiver. Use only Positioning Bed Pads that are in good condition. Discard and destroy old, unusable Positioning Bed Pads.

 WARNING: DO NOT exceed the patient lift or the patient Positioning Bed Pad's maximum weight capacity.

 WARNING: Notice for California Customers—California Proposition 65
—This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

 WARNING: VEVOR assumes no responsibility for any damage or injury caused by improper application or use of this product.

 WARNING: Never leave a patient unattended while in a Patient Positioning Bed Pad

 WARNING: Never attempt to re-position a patient by pulling on the Positioning Bed Pad loops.

 WARNING: Check the Positioning Bed Pad for shipping damage before using. In case of damage, DO NOT use the Positioning Bed Pad. Contact VEVOR Customer Service for further instructions.

General Guidelines for Using the Positioning Bed Pad:

Use an appropriate Patient Positioning Bed Pad that is recommended by the individual's doctor, nurse, or medical assistant for the comfort and safety of the individual being lifted and their specific medical need.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and any patient Positioning Bed Pad as this may cause the patient to slide out of the Positioning Bed Pad during transferring.

The loops of the Positioning Bed Pad are color coded and can be used to place the patient in various positions. The colors make it easy to connect both sides of the Positioning Bed Pad equally. Make sure that there is sufficient head support when lifting a patient.

The VEVOR CGSL280 handled patient turning and positioning aid and the CGSL280 handled patient turning and positioning aid are designed to reduce the effort required to move, turn or transfer a patient. When used properly, they ease a multitude of patient mobilization procedures including: repositioning a

patient up in bed, turning of patients, moving or laterally transferring patient.

Using ergonomically inclined handles for gripping, caregivers can maneuver patients to the desired position without having to lift, which can cause injury to arms, back, and related muscles. The CGSL280 have a low friction bottom surface that allows the patient to be positioned or moved in any direction easily while reducing the shearing to the patient's skin. When using the CGSL280 for the first time it is recommended that the caregivers practice until they are comfortable using it properly and safely. The log roll technique is the preferred method of getting the CGSL280 under the patient.

Repositioning a Patient Up in Bed using the CGSL280

- 1) Make sure you have the proper number of caregivers required for the transfer or moving of patient;
- 2) Explain to the patient what they should expect and what you are going to be doing;

- 3) Make sure the CGSL280 has been inspected and all the handles and Positioning Bed Pad are not torn, damaged or show signs of severe wear;
- 4) If the patient is able it is recommended they bend their legs at the knees with their feet off the Positioning Bed Pad, so the legs will straighten out during the repositioning process;
- 5) Log roll the patient and place the transfer Positioning Bed Pad underneath them where the patients head and/or shoulders should be on the Positioning Bed Pad making sure the Positioning Bed Pad extends below the patient's bottom;
- 6) Make sure it is under the patient with the safety belt buckling around their torso; The adjustable safety belt is not required to be used and will not keep a patient from rolling off the transfer Positioning Bed Pad;
- 7) Make sure that the patient's head is supported by positioning the CGSL280 beneath the head or by having a caregiver support the head with his/her hands when moving the patient;
- 8) Make sure all the caregivers are positioned correctly;
- 9) Make sure the wheels of the bed or stretcher are locked;

- 10) To prevent injury, make sure the bed or stretcher is in the proper height, so the caregivers are not leaning over or bending down too far;
- 11) When sliding the patient up in the bed make sure the patients head is flat and not raised up on a pillow.;
- 12) Once the CGSL280 is under the patient correctly the caregivers are to grab the side handles;
- 13) Let the patient know what you are about to do and then count to three and have all caregivers simultaneously perform the move of the patient to the proper position;
- 14) Take extra caution to make sure not to hit the patients head on the headboard;
- 15) Once the patient is in the desired position remove the transfer Positioning Bed Pad from under the patient;

Lateral Transferring a Patient using the

- 1) Make sure you have the proper number of caregivers required for the transfer;
- 2) Explain to the patient what they should expect and what you are going to be doing;

- 3) Make sure the transfer Positioning Bed Pad has been inspected and all the handles and Positioning Bed Pad are not torn, damaged or show signs of severe wear;
- 4) Use the log rolling technique to place the transfer Positioning Bed Pad underneath them where the patients head and shoulders should be centered on the Positioning Bed Pad making sure the Positioning Bed Pad extends below the patient's bottom;
- 5) Position the adjustable safety belt around the patient's torso comfortably.
The safety belt is for patient's piece of mind, extra security and help with positioning but will not keep a patient from rolling off the Positioning Bed Pad;
- 6) Lower the side rails on the side of the bed where the transfer will take place. Position the other surface next to the bed making sure the surface heights are the same and any gap is minimal;
- 7) Raise the Side rails on the outside of each surface you are transferring to and from;
- 8) Make sure all the caregivers are positioned correctly;
- 9) Make sure the wheels of the bed or stretcher are locked;

10) To prevent injury, make sure the bed or stretcher is in the proper height, so the caregivers are not leaning over or bending down too far;

11) One or two caregivers on each side depending on the transfer Positioning Bed Pad you have will grasp the handles on both sides of the Positioning Bed Pad and pull the patient towards the surface the patient is transferring to;

12) Take extra caution when completing the transfer making sure the bed rails are in the up position except those between the two transfer surfaces;

Once the patient is in the desired position remove the transfer Positioning Bed Pad from under the patient. DO NOT:

EVER LEAVE TRANSFER POSITIONING BED PADS UNDER A PATIENT WHEN NOT IN USE AND SUPERVISED BY A CAREGIVER.

WASHING INSTRUCTIONS: Machine wash in warm or cold water. Maximum water temperature of 120°F(48°C). Air dry or tumble dry at cool or very low temperature. DO NOT use bleach or wash with other colors. Inspect after each wash. DO NOT dry clean.

WARNING: Inspect patient Positioning Bed Pads for wear prior to each

use.If signs of tearing,fraying,or wear are found discard the Positioning Bed Pad immediately;worn out Positioning Bed Pads are not safe for use and may result in injury or death. Do not alter Positioning Bed Pads in any way.

Manufacturer: Dongguan Chuangguo Daily Products Co. Ltd

Address: No.169.Kangyi Road, Qingxi Town,Dongguan City,Guangdong.CN 523657

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,
Centurion House,
London Road, Staines-upon-Thames, Surrey,
TW18 4AX

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Mainzer Landstr.69, 60329 Frankfurt am
Main.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Produttore: Dongguan Chuangguo Daily Products Co. Ltd
Importato in AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD Nuovo Galles del Sud 2122 Australia
Importato negli USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Luogo, Rancho Cucamonga, CA 91730
E-CrossStu GmbH