VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

STAIR CLIMBING CHAIR

MODEL: YHR-LD01

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

STAIR CLIMBING CHAIR

MODEL: YHR-LD01

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

Before Initial Operation

- Read the user manual and note all warnings and safety instructions.
- Note technical data as lifting capacity cannot be exceeded.
- Check all electronic functions, safety brakes, and the position of the climbing track.

While Operating the Unit

- Never change settings while climbing or descending stairs.
- Never put any part of a body, especially feet, under the climbing tracks.
- If the drive elements lock up, turn off the unit immediately.

WARNING

1. We strongly advise that only certificated trained personnel operate the Stair chair. Untrained personnel may cause injury or damage !
2. Pay attention to smooth handling.
3. When carrying patients with this machine, be sure to lock tightly, fasten the safety belt for safety and tell him to hold on to the handrail.
4. Don't move too fast when passing pothole areas.
5. When there is a failure, non-professional personnel shall not be disassembled at will.
6. You can add some lubricating oil to the transmission mechanism of the drive wheel to increase the service life.
7. When working, do not carry out other work on both sides of the car to avoid accidents.
8. After the completion of the operation, check whether the electric climbing car is normal, and turn off the power.
9. Do not knock, needle, tread, modify, sun the battery, do not place the battery in the microwave, high pressure and other environments.
10. Keep away from naked flames.
11. Wear non-slip footwear and always operate with both hands.
12. Do not use it in rain, wet conditions, or on snow or ice.
13. Do not use it on escalators or travelators.
14. The wheelchair stair lift should not be used for the transportation of

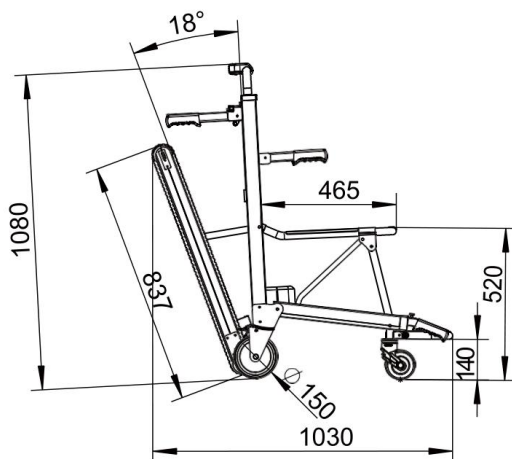
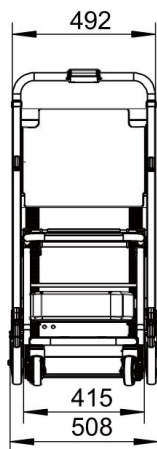
cargo.

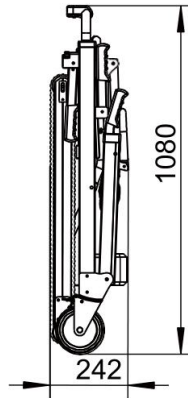
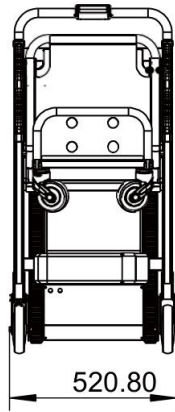
MODEL AND PARAMETERS

Model	YHR-LD01
Max. Load	150kg
Rubber-track Belts' Angle	20°
Input	AC 100-240V 50/60Hz
Motor	DC 24V / 120W
Battery	Lithium Battery; DC 24V / 13Ah
Charger	DC29.4V / 2A
Telescopic Gripping Frame Height	H108cm, H126cm, H143cm and H160cm
Chair Size / Height	47x42cm / 52cm
Pedal Size	37x21cm
Materials	Aluminium alloy
Weight	32.5kg

DIMENSION AND DIAGRAM

Unit: mm

COMPONENTS

1. Adjustable&Removable headrest	14.Front universal wheel
2.Front handle/Armrest	15.Seat with spongy cushion
3.Vest-type safety belts	16.Control panel
4.Folding control rod	17.Telescopic top handle

5.Foldable footrest	18.Red bar for Locking top handle
6.Front telescopic carry handle	19.Belt tensioner
7. Button for front carry handle	20.Top bar for leaning chair backward
8.Battery	21.Red bar for restoring track belts
9.The key to battery	22.Red pedal for opening track belts
10.Track belt	23.Lower bar for leaning chair backward
11.Rear carry handle	24.Motor protection cover
12.Lock of Rear carry handle	25.Rear wheels
13.Strap for knee	26.Brake of Rear wheels

BATTERY CHARGING

1. The battery charger should be tested on a portable appliance tested prior to use and as required thereafter in accordance with local guidance.
2. Battery technical details can be found on the battery.

3. Connect the DC power adapter **a1** to the battery charging port **a3** (Connect the other end **a2** to wall socket) to charge, and make sure the battery is fully charged before using the stair chair. When the indicator lamp **a4** on the battery turns green from red, it means that the battery is fully charged.
4. The charger recharges the battery to 100%. As soon as the charging process is completed, it switches to a retaining mode to avoid overcharging.
5. * If the battery is less than 25%, charging is required.

6. * If the battery is not fully 100% charged, it needs several cycles of charging and discharging to achieve its maximum storage capacity.
7. Please do not disassemble or modify the charger.
8. Please make sure that the charger is not exposed to humidity while charging the battery.
9. Never touch the plug with moist hands.
10. * When finished charging, remove the input AC plug, then the output plug. The battery should be charged after each use and can be charged on or off the chair.
11. * The battery should be checked and charged prior to use and at least every 2 weeks.

INSTALLATION AND REMOVAL

1. The slot on the inner side of the battery must align with the guide rail, Push the battery along the guide rail arrow direction b1 until it clicks into place. Turn the key clockwise 180° to lock the battery onto the chair.
2. Turn the key 9 anticlockwise 180° and push the battery forward along the direction b2 to unload the battery.

Remaining Capacity Inspection:

Press the button in the middle, there are four red lights, each one means 25% capacity. Please do not use STAIRCHAIR when there is only one red light before charging to full.

- * Lithium-ion battery is a consumable product.
- * When it displays low power or under voltage, charging is required.

Battery Switch:

Power-On:

Turn on the battery switch as shown f1, Press the POWER button, the display LED is on, and remaining of the battery is displayed.

Power-Off:

Press the POWER button, the battery monitor background lights off.
Turn off the battery switch as shown f2.

The battery with two keys and can be used to lock battery on unit.

USE THE STAIR CHAIR

Deployment:

Open the chair as shown

Open the armrests as shown

Open the rear handles as shown if required until they lock into position.

Expand the telescopic front lifting handles: Press the red button and pull the handles in the direction of the arrow until they lock into position.

To release the tracks: Depress the red locking pedal.

When the chair stands deployed, keep the telescopic frame at the lowest position.

When going upstairs or downstairs, the operator must adjust the height of the telescopic grip frame close to the operator's shoulder height.

Adjust Telescopic Grip Frame

To pull up:

Press the red locking bar inwards and pull the handle up to suitable height, then release the locking bar until the handle locks at a suitable height.

Lower the handle:

Press the red locking bar inwards, and lower the handle down to suitable height, then release the locking bar until the handle locks at a suitable height.

	
<p>Adjust and fix the headrest to suitable height with strap according to the passenger's height.</p>	<p>After the passenger is seated, adjust safety belt and fix it as shown if required.</p>

Folding:

	
<p>Fold the armrests up as shown</p>	<p>Press the red button and push the handles into the frame.</p>
	
<p>Storing the track belts: Depress the red bar downwards</p>	<p>Grip the seat with one hand and the frame with the other, depress the foldaway support bracket then after the bracket is out of the locking position fold up the seat.</p>
	<p>Fold lifting handles: Press locks and fold the handles down.</p>

Brakes:

	
Lock: Press the brake pedal.	Release: Press the top one to unlock.
Note: Before folding the chair, you should press the brake pedal.	

Control Panel:

	DOWN stairs (Press&Hold)
	POWER on/off
	LIGHT on/off
	UP stairs (Press&Hold)
	“+” increase speed
	“-” reduce the speed
<p>As shown, from left to right: go downstairs, power, lighting, go upstairs.</p> <p>Light Press the LIGHT button: The display LED is on. Re-press the LIGHT button: The display LED is off.</p> <p>Climbing Upstairs Upstairs: Keep pressing the UPSTAIRS button. Pause or Stop: Release the UPSTAIRS button.</p> <p>Climbing Downstairs Downstairs: Keep pressing the DOWNSTAIRS button. Pause or Stop: Release the DOWNSTAIRS button.</p>	

Adjusting the Track Tension:

	<p>If the climbing track becomes too tight or too loose, it can be adjusted as shown using the supplied hexagonal wrench.</p>
-------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------

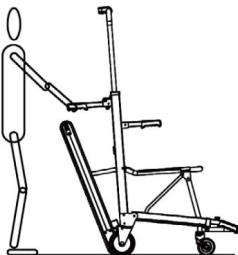
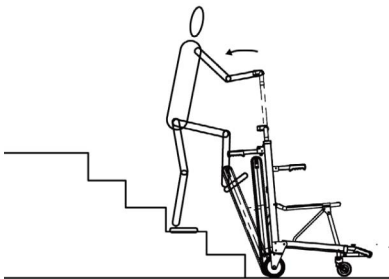
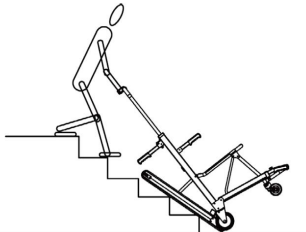
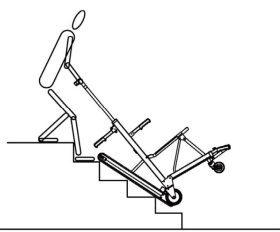
Stairchair use, Going Upstairs and Downstairs:

- * Before using STAIRCHAIR, Please be sure that you have been properly trained and familiar with the operation of the chair.
- * You may only be permitted to transport ONE person each time.
- * Please practice climbing stairs without person on the chair at low speed.
- * Before using the chair please check remaining of the battery and the track belts.

WHEELCHAIR USE



Release the brakes, grip the rear lifting handles, and push STAIRCHAIR forward or backward.

Up Stairs Operation:


	
<p>Deploy chair as shown, be sure the battery is switched on, adjust the top handle to suitable height.</p>	<p>Adjust the distance (30-40cm) between the rear castors and the staircase slowly tilt the handle backward until the track belts lean on the stair steps as shown.</p>
	
<p>Press POWER button to start lifting the chair, meanwhile press the gripping frame a little to make the track belts cling to stair steps.</p>	<p>If you need to pause or stop, release UP button.</p>

Continuously press the UP button and move up the stairs at the same time.

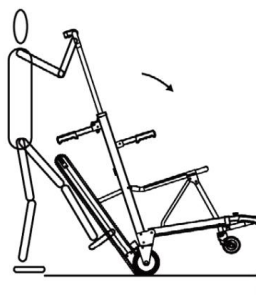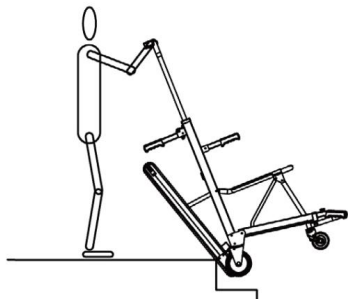Note: When the track belts lie on one or two stair steps, make sure to press the frame downwards to avoid the chair leaning forward.

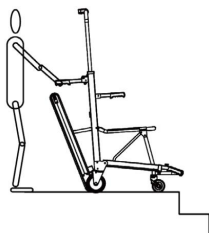
When all the wheels reach the staircase landing, release the UP button, grip the handle and slowly lean forward to make four wheels stand on the ground.


Grip the back handle push the chair to next stairs.

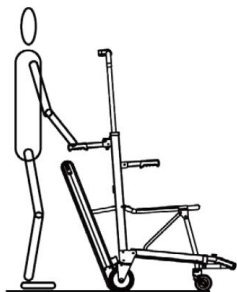
Down Stairs Operation:

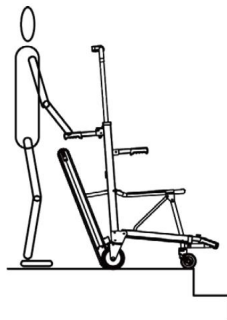
Deploy the chair as shown, make sure that the battery switch is on, adjust the top handle to suitable height.

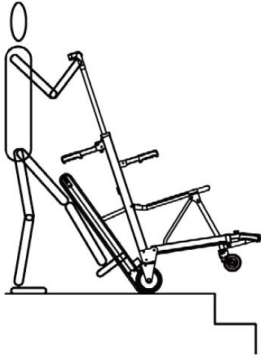
Push the chair to staircase, adjust the chair and make the track belts in the middle of the staircase.

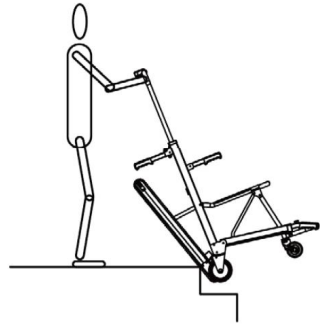
When the front wheels reach the staircase edge, tilt the chair backward until the track belts on the floor as shown, then press the DOWN button.

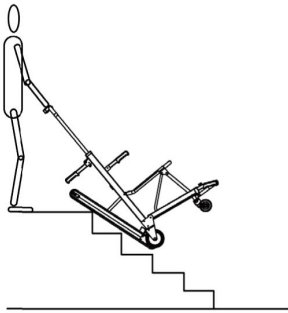
After the belts pass the first stair step, adjust the chair slowly to let the belts cover two/three steps along the staircase angle.

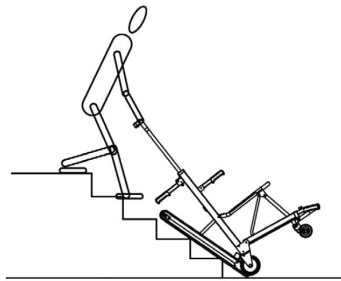If you need to pause or stop, release DOWN button.
Note: When the track belts cover one or two steps, press the handle downwards a little to avoid the chair leaning forward.

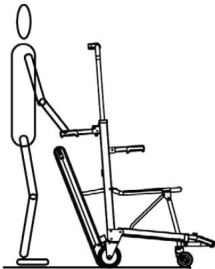When the track belts reach the floor at the bottom of the stair release the DOWN button, then slowly tilt the chair forward until the four wheels stand on the floor.

Grip the back handle, push the chair to next stairs.

TROUBLESHOOTING

Faults	Detection and Solutions
The unit won't start or can't go up or down stairs.	Ensure the power switch on the control panel is in the ON position. Charge the unit. If the battery has a charge remaining, and the unit is unable to start, please contact the manufacturer.
The battery can't be recharged.	Please contact the manufacturer.

MAINTENANCE

1. Keep clean as routine (incl. sterilization).
2. Check frequently for loose parts.
3. If not used for a long time, please fully charge the battery and take out the battery and store it in a cool and dry place. Beware of water entering the lithium battery.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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