# VEVOR<sup>®</sup>

## TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate [www.vevor.com/support](http://www.vevor.com/support)

### SICK BED

MODEL:D8D8Y

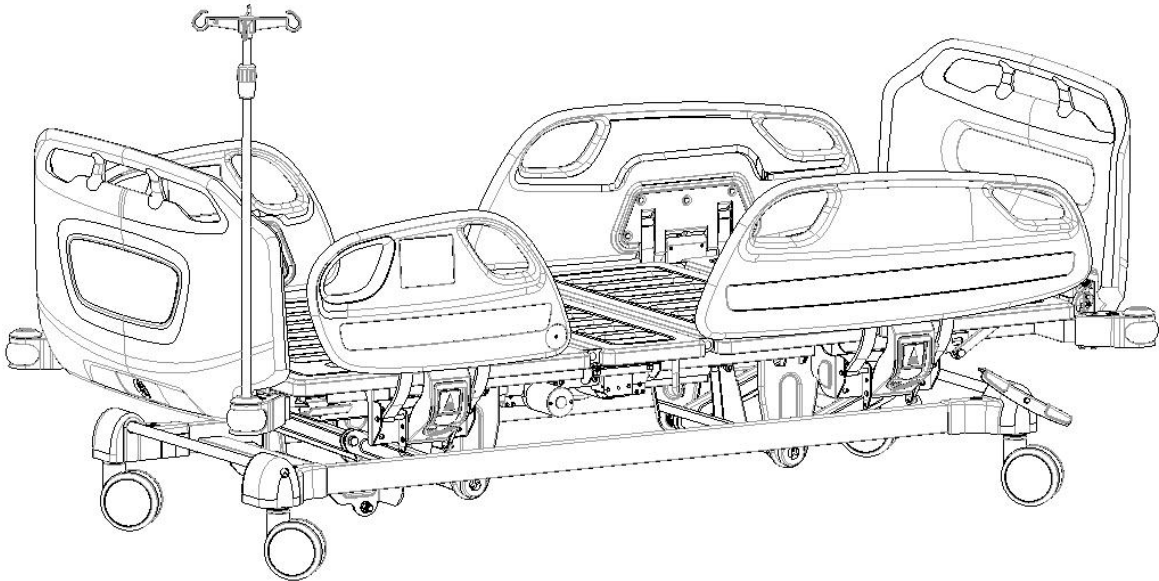
We continue to be committed to provide you tools with competitive price.  
"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us.  
You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR<sup>®</sup>

TOUGH TOOLS, HALF PRICE

## SICK BED

MODEL:D8D8Y

### NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:  
**Technical Support and E-Warranty Certificate**  
[www.vevor.com/support](http://www.vevor.com/support)

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

# Manual Introduction

Hospital beds are essential equipment for the well-being, treatment and recovery of bedridden patients. Hospital beds allow patients to remain in more comfortable positions as well as in positions sometimes recommended by doctors, which would be very difficult in a normal bed, where one would always have to be placing or removing supports. Hospital beds can be manual or automatic. The automatic hospital bed has its functions controlled by buttons on the side panels of the bed or via remote control. The manual hospital bed has 2 movements triggered by cranks, having the elevation of the headboard, which facilitates the moment of feeding, also avoiding refluxes and facilitating breathing and also has elevation of the part of the feet that will cause it to improve the blood circulation of the lower limbs, in view of the long period that the patient will spend lying down. On the other hand, patient safety is very important at this time of recovery, so all of our beds have a side protection rails, preventing the patient from going at night and running risks of falling.

## Note:

**Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.**

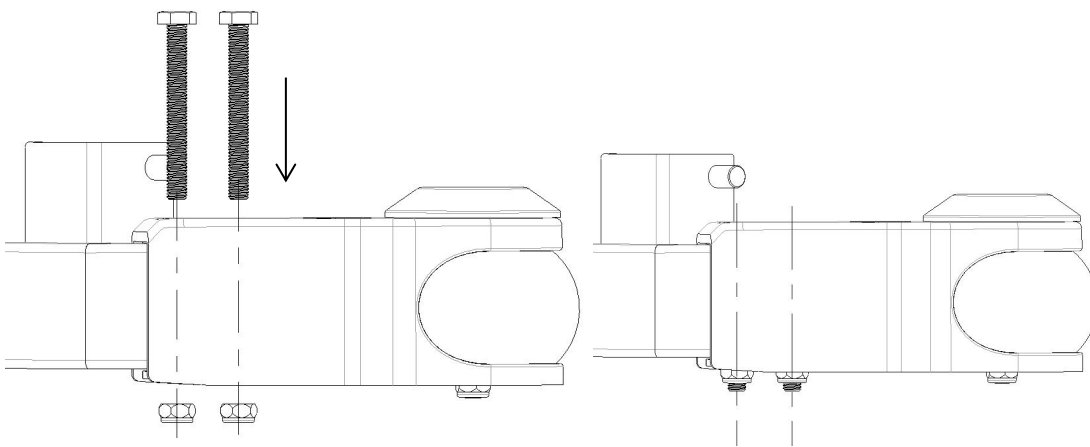
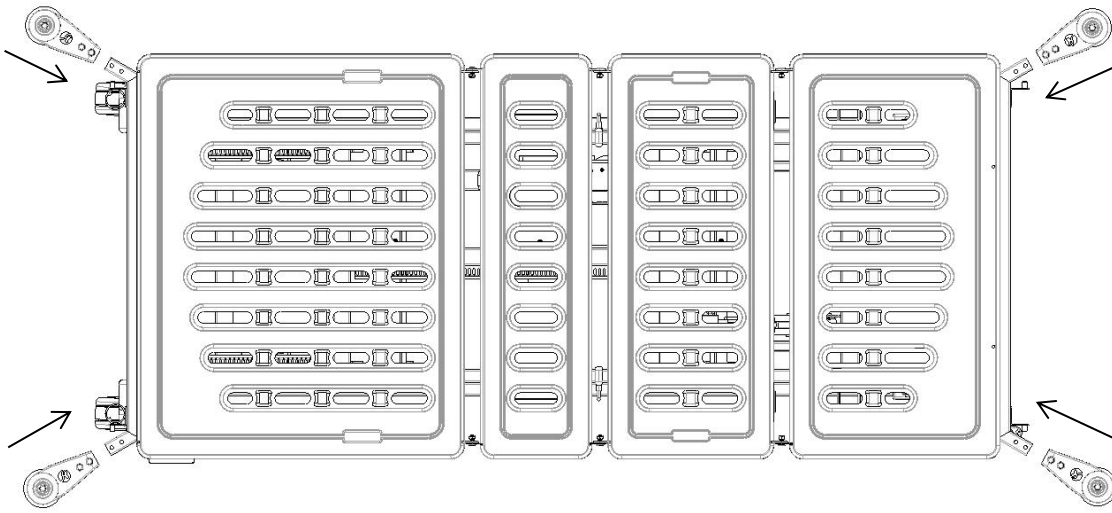
- ▶ This manual is applicable to D8d8y electric hospital bed. It contains the installation method, operating instructions and maintenance check points.
- ▶ The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- ▶ This user manual includes the description, instruction, maintenance and technical parameters of the product.
- ▶ Any questions, please contact the after-sale service center or distributor.
- ▶ Anyone using or operating this product should read and comply with it.
- ▶ Except for some unexpected situations, users are required to be familiar with the usage and precautions.

## 1. Installing guide

### 1.1 Bumper wheels installation


Take out the bumper wheels from the bag and use a 17-inch hex wrench to turn the bolt clockwise to install the bumper wheels;


The bumper wheel prevents direct collision of the bed and protects the bed.

## 1.2 Head board & Foot board assembly method

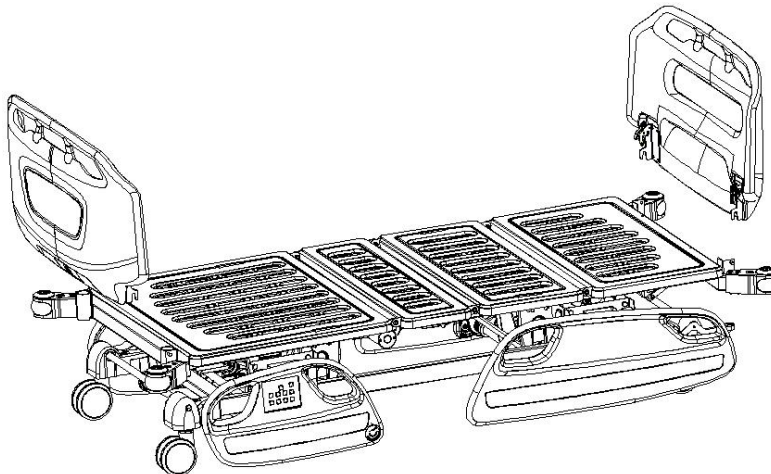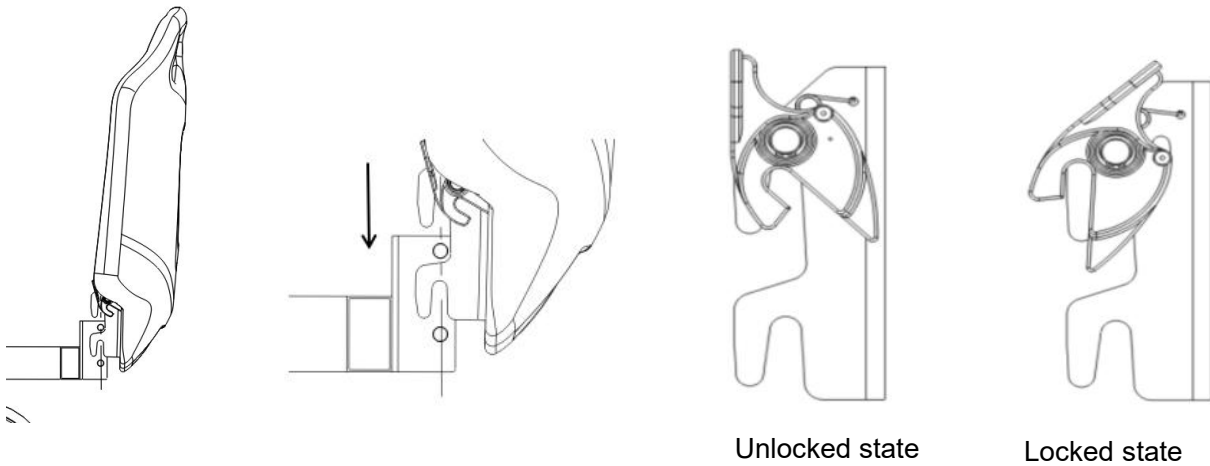
First align the Head board & Foot board with the positioning column on the bed frame, and then insert the Head board & Foot board vertically downwards. After hearing a "click", it indicates that the buckle is in the correct position, and finally check whether the buckle is in a locked state. The Head board & foot board installation diagram is as follows:

### 1.3 Drainage hook installation

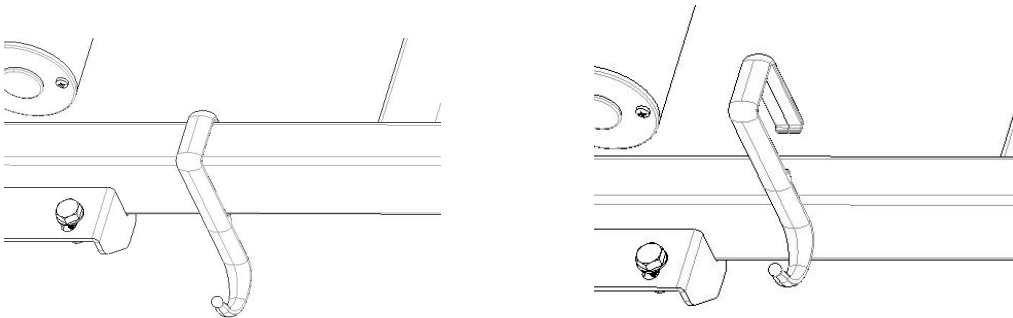
Place the concave buckle of the drainage hook downward, and then buckle it down on the bed frame beam, as shown in the figure below:


### 1.4 IV pole installation


Female part installation: Press the plastic hook(female part) down on the  $\Phi 16\text{mm}$  stainless steel pipe.

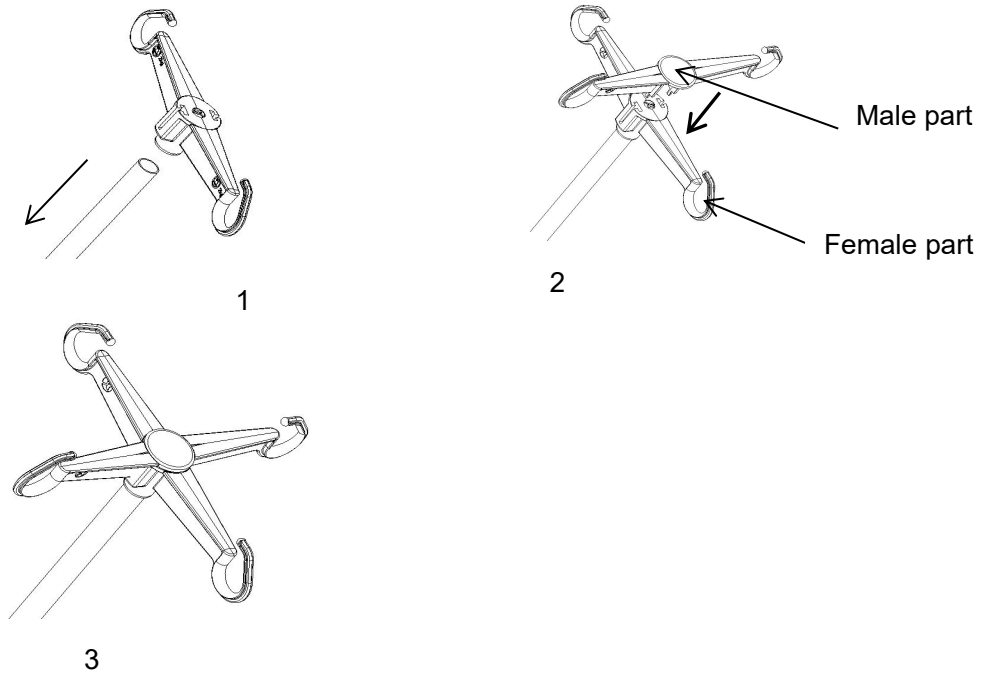Male part installation: Place the plastic hook (male part) directly above the female part in a cross shapes, and then aligns the buckles on the male part with the slots on the female part and press down. After hearing a "click", check whether the end of the buckle is tight, if it is tight, then complete the installation, if not, continue to press down until the buckle is tight.

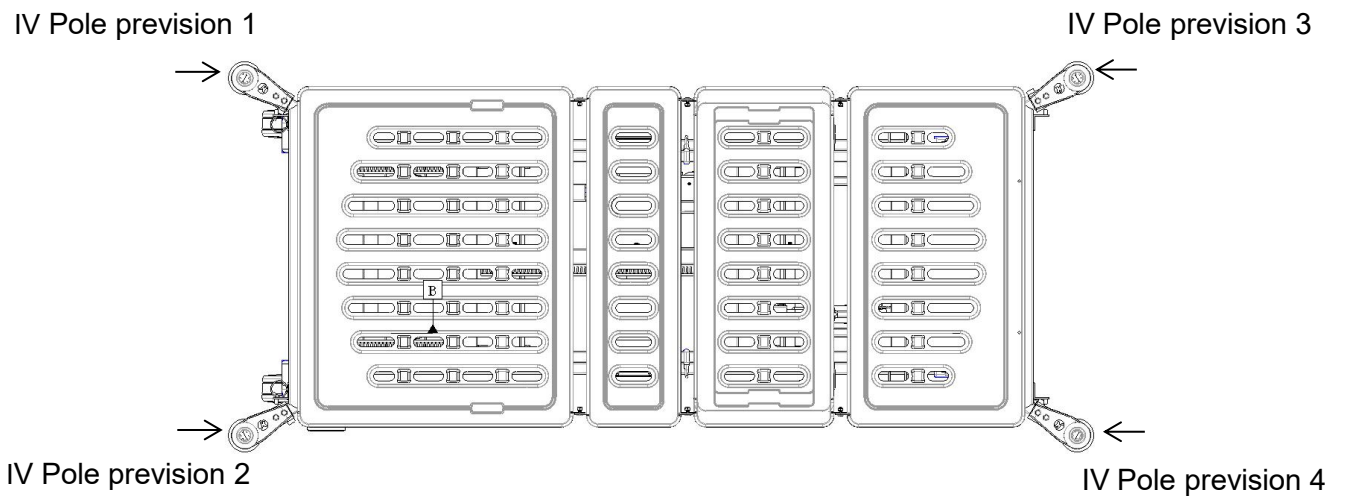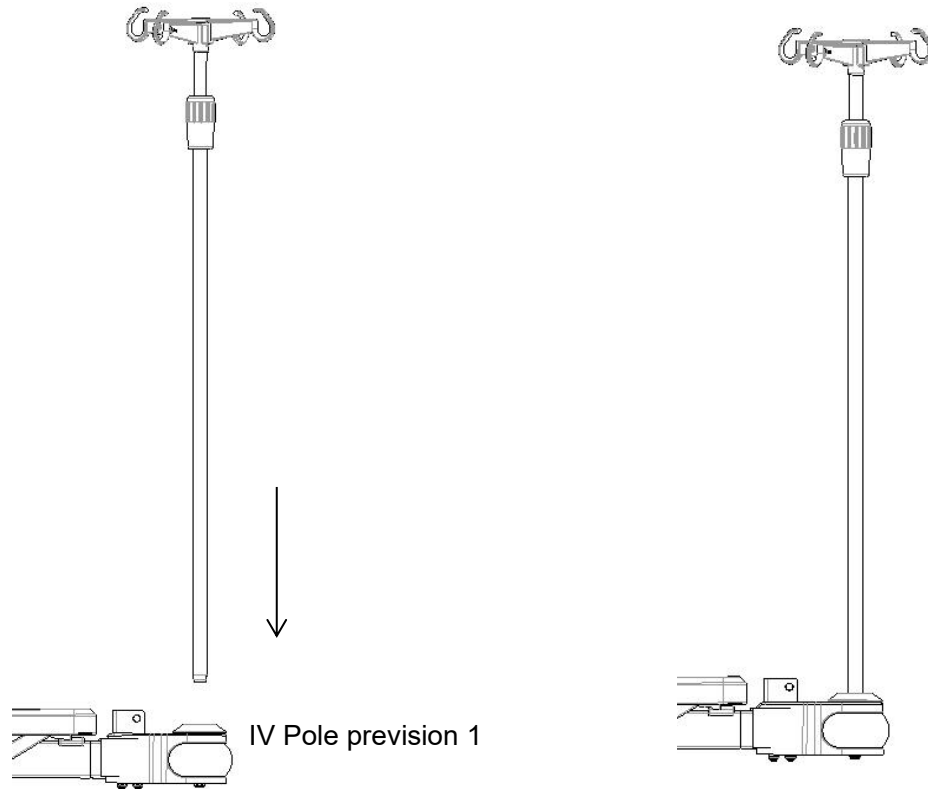
The IV pole can be installed in the four positions shown in the figure below:

## 2. Product description

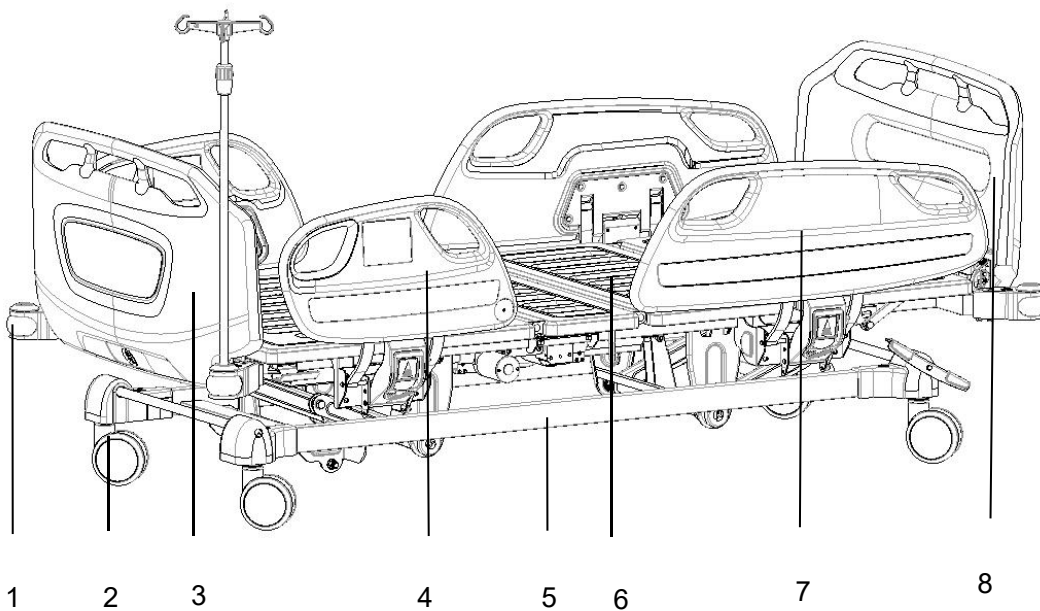- |                 |                     |                 |                   |
|-----------------|---------------------|-----------------|-------------------|
| 1-Bumper wheels | 2-Casters           | 3-Head board    | 4-Small Guardrail |
| 5-Chassis       | 6-Bed surface board | 7-Big Guardrail | 8-Foot board      |

Main technical parameters:

Bed surface size	Length	mm	1925±10
	width		900±10
	The lowest height of the bed from the ground		440±10
Dimensions	Length	/	2280±10
	Width		1090±10
	Height		(440-735)±10
Tilt range	Back board	/	0-75±5°
	Leg board		0-35±5°
	Reverse Trendelenburg		0-12±5°
	Trendelenburg		0-12±5°
Weight	Safe working load	KG	250

## 2.1 Use method of caster brakes

 When there is a patient on the bed, the guardrail should be raised before moving the bed.


 Except when the bed must be moved, the caster brake must be stepped on during the rest of the time. Need to depress all two caster brakes.

**This bed uses a centrally controlled side brake, and all four casters can be locked by controlling the brake lever on one side.**

**Press the red foot lever down to lock all casters.**


**Press the green foot lever down to unlock all casters.**

The following picture shows the two movement states of the caster:

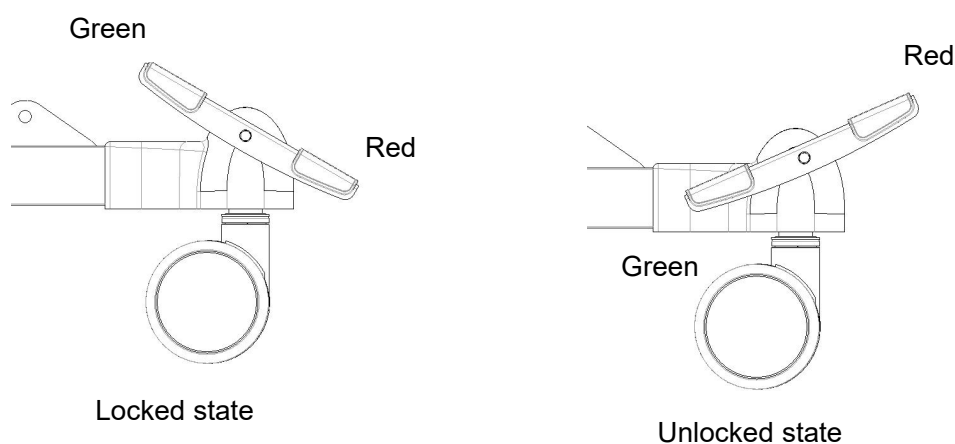
## 2.2 Use method of guardrails

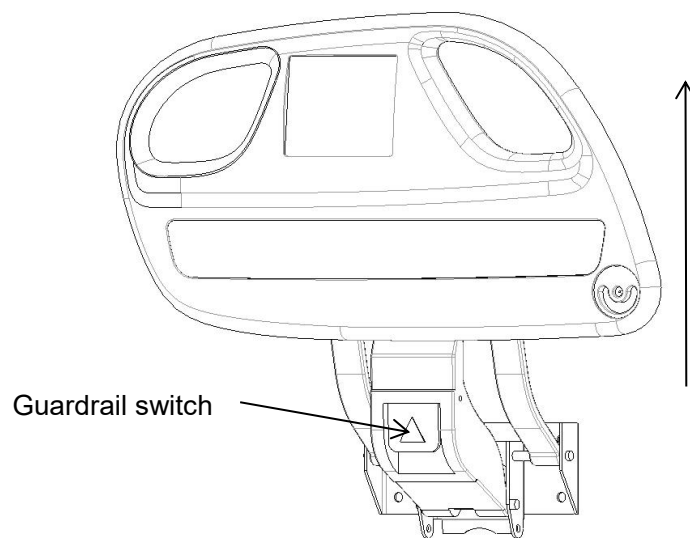**Raising guardrails:** Hold the upper guardrail pulled up, a "click" sound indicates the guardrail has been locked. The picture is as follows:


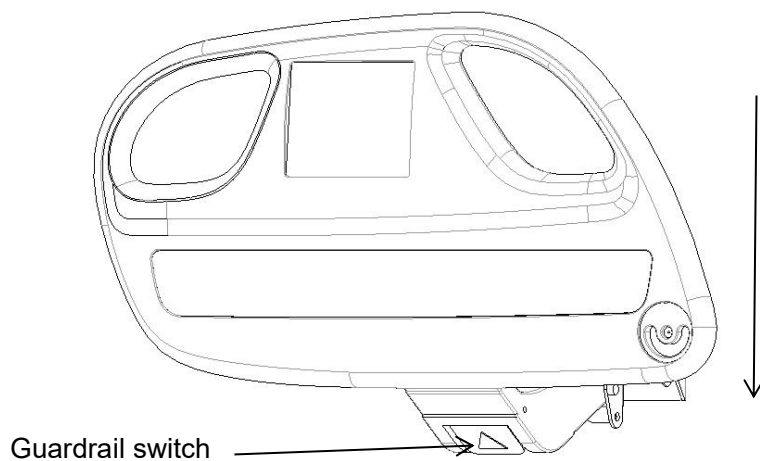
**Lowering the guardrail:** Pull Guardrail switch outwardly, and hold the upper guardrail while slowly putting down the guardrail. The picture is as follows:

## 2.3 CPR instant release function

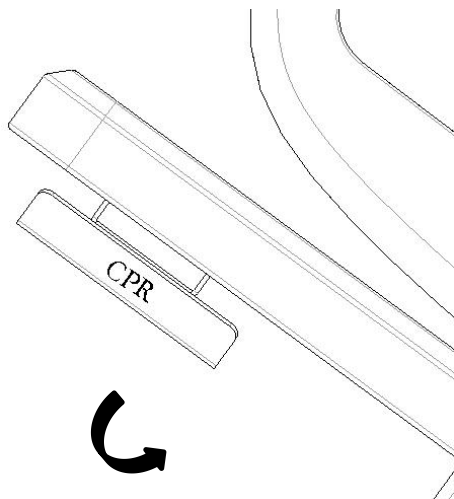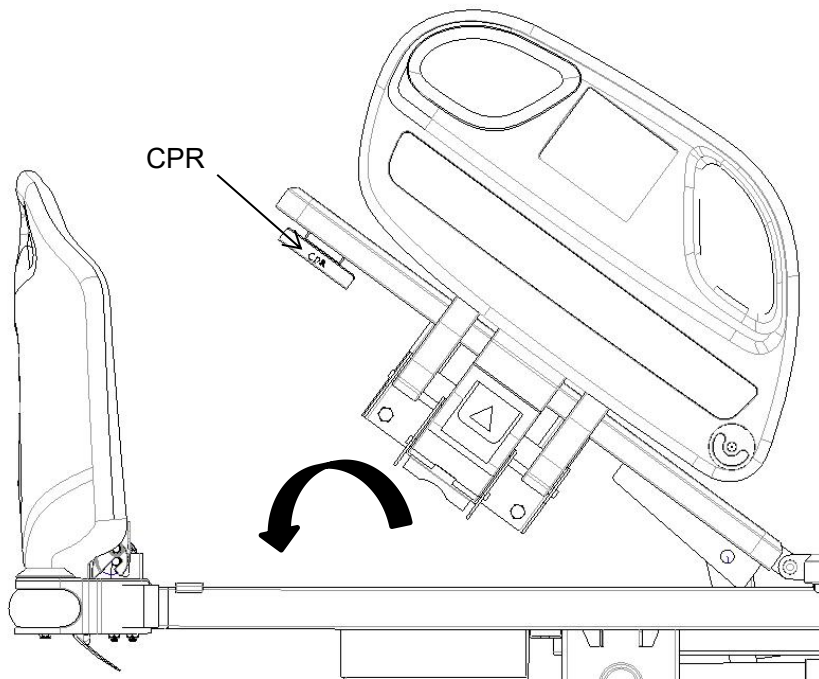This bed can mechanically lower the backboard quickly in the emergency process. Operation method: Lift up the red CPR switch to realize the rapid lowering of the backboard.

Quickly lower the backboard


## 2.4 Use method of IV pole

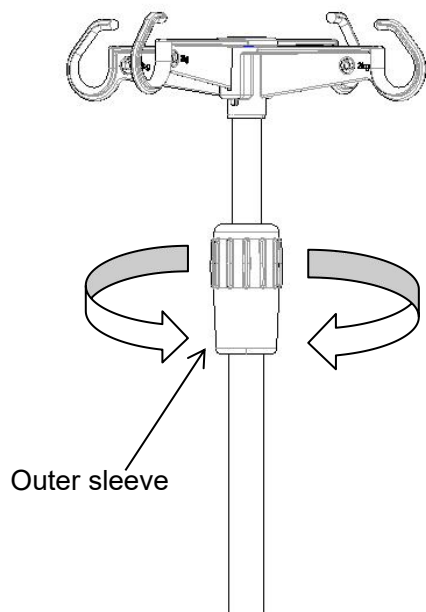Adjusting the height of the IV pole:

1. First loosen the outer sleeve counter clock carefully, insert the inner tube of the IV pole into the outer sleeve, and then adjust the length of the extension rod up and down;
2. Second adjust the required length, and then tighten the outer sleeve clock carefully.

The maximum bearing of each hook is 2kg, a total of 8kg.

(Height Adjustment :890--1540mm)

## 2.5 Hand controller style and description

### 2.5.1 Use method of hand controller

(Note: The hand controller is an optional accessory, according to the needs to determine whether it needs to be equipped with this configuration)

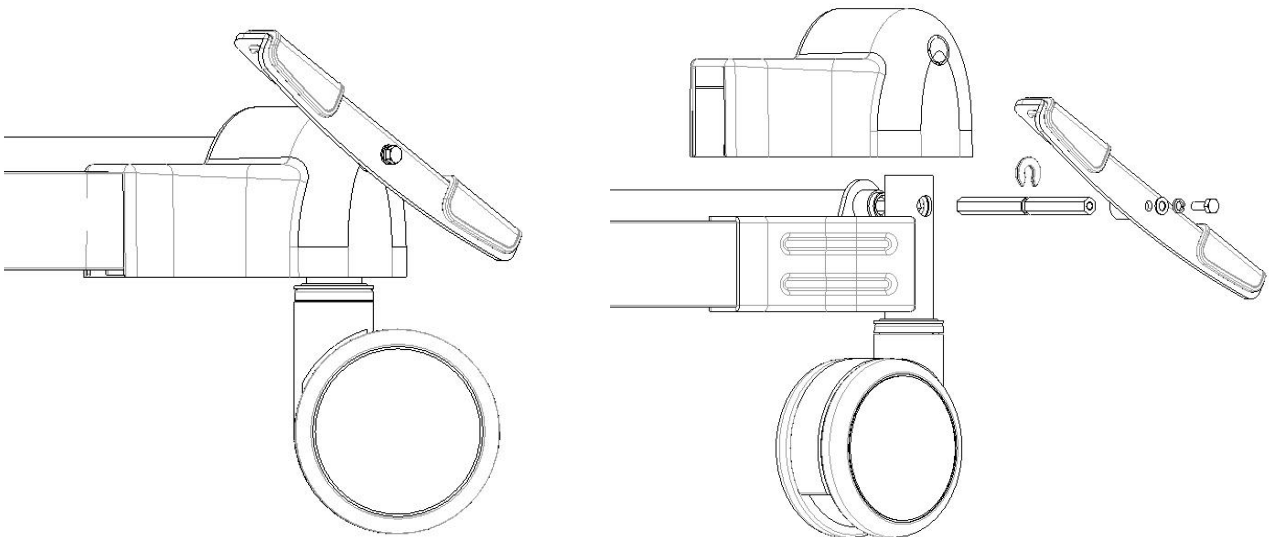
- : Back up button
- : Leg up button
- : Back and leg linkage up (auto-contour) button
- : Whole bed up button
- : Reversed trendelenburg
- : CPR
- : Back down button
- : Leg down button
- : Back and leg linkage down (auto-contour) button
- : Whole bed down button
- : Trendelenburg

### 3. Bed parts replacement and maintenance

#### 3.1 Casters disassembly and replacement

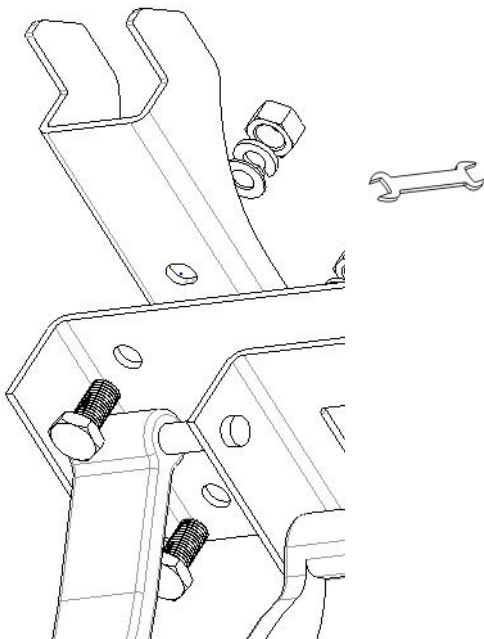
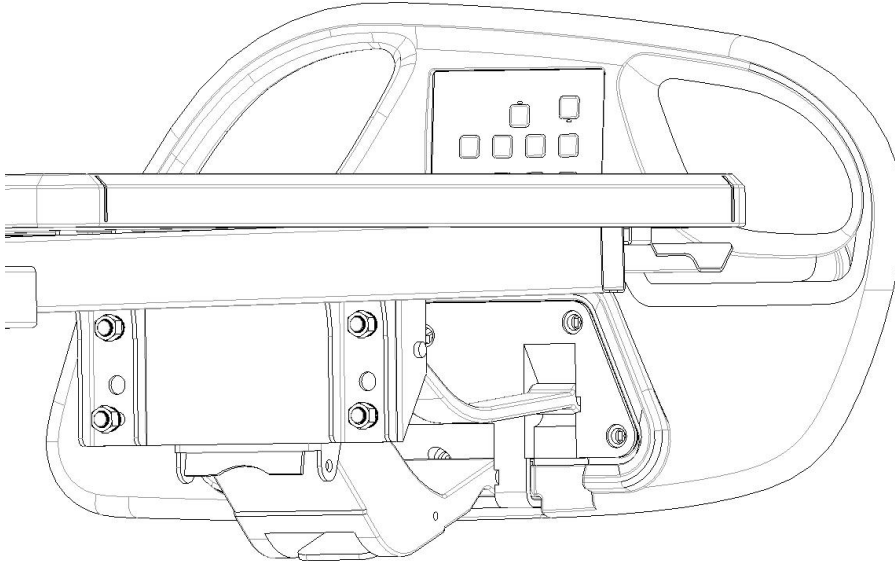
Use a 10-inch Allen wrench to unscrew the screw, take out the screw, flat spring washer, brake pedal, plastic retaining ring, hexagonal steel, etc., replace the spare caster, insert the bolt into the base and the caster mounting hole, and tighten the nut.

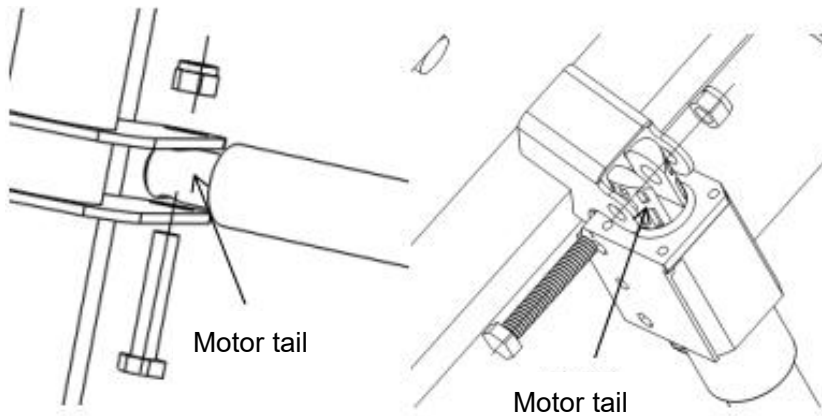
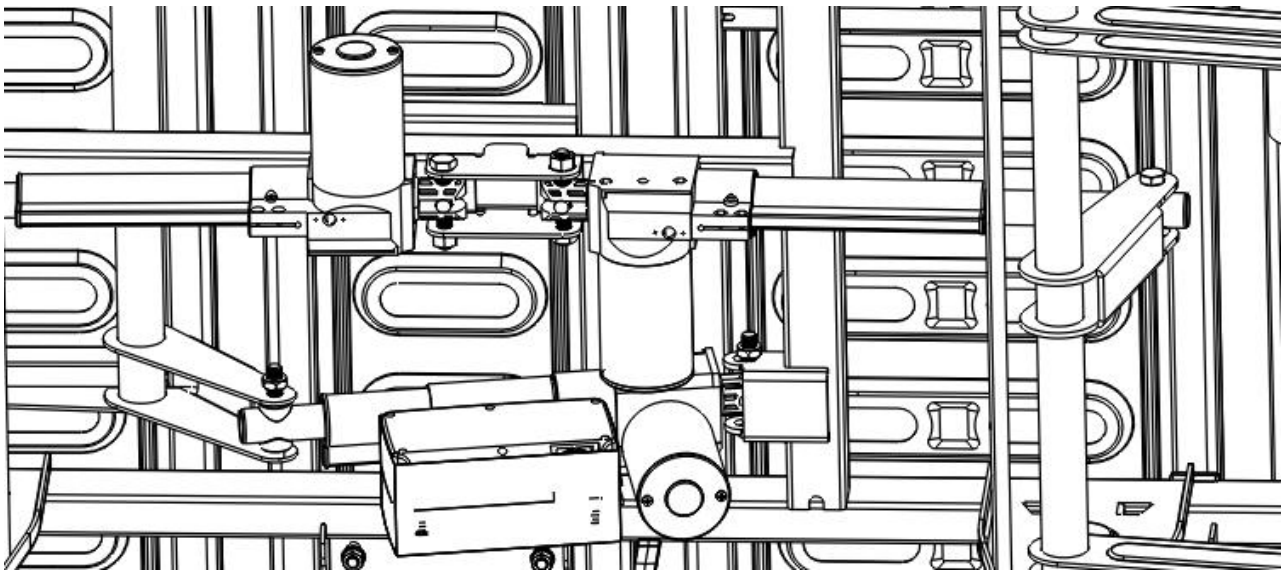
### 3.2 Guardrail disassembly and replacement

Use 2pcs 14-inch wrenches to loosen the nut, remove the nut, spring washer, flat washer, and bolt in turn, and then remove the guardrail. (The picture is for reference only, the actual product shall prevail)

### 3.3 Motor disassembly and replacement

Use 2pcs 17-inch Allen wrenches  to unscrew the motor, take out the bolt, and replace the motor. When the replacement is completed, install the bolts in the order shown in the figure, and use the wrench to tighten the knot.

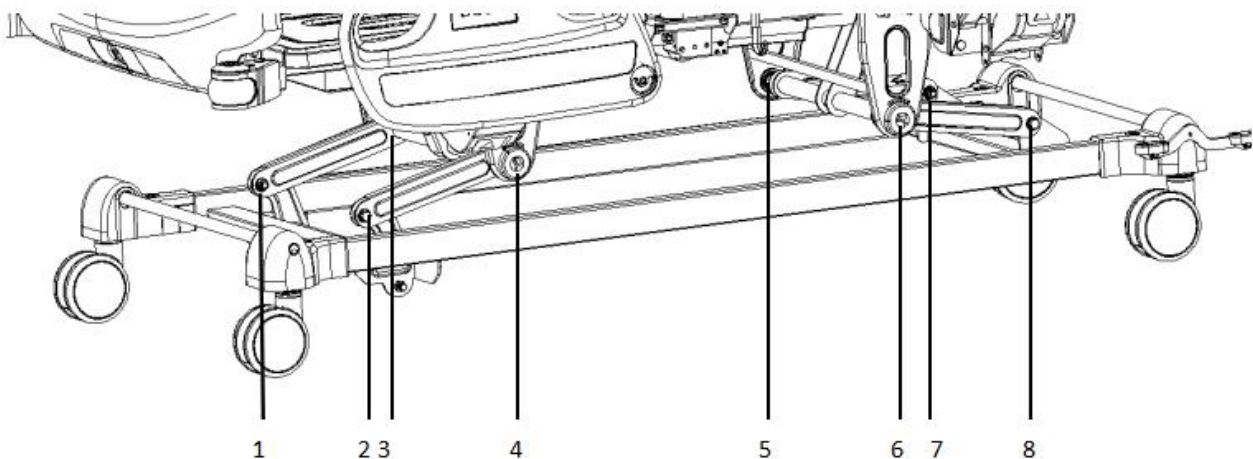
#### 4. Faults exclusion

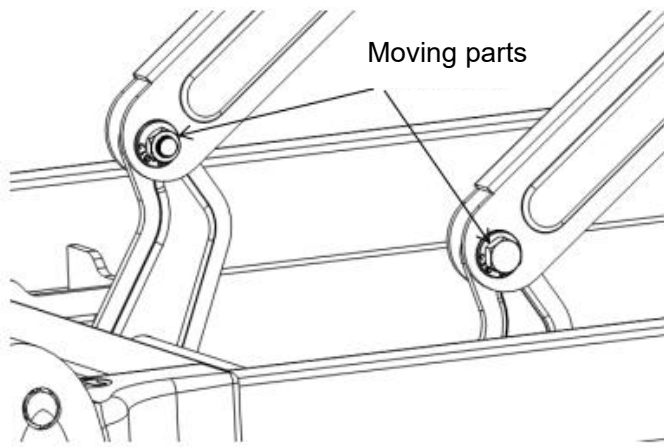
Common failures	Probable reasons	Exclusion methods
Unable to adjust the bed using the position buttons	Power button not pressed	Press the power button
	Drive motor power off	Check the power connection
	Drive motor failure	Replace the motor, refer to 3.3 Motor Disassembly and Replacement
	Plug not plugged in	Insert the power plug correctly
	Control element failure	Notify maintenance
Mattress platform height adjustment failure	Drive motor power off	Check the power connection
	Drive motor failure	Replace the motor, refer to 3.3 Motor Disassembly and Replacement
	Plug not plugged in	Insert the power plug correctly
	Power box failure	Notify maintenance

	Control element failure	Notify maintenance
Unable to adjust guardrail	There are foreign objects under the backboard or in the drive unit	Remove foreign objects
Casters cannot turn	The rotating shaft is stuck by strips of dirt	Clean the rotating mechanism
	Internal mechanism failure of casters	Replace casters, refer to 3.1 Casters Disassembly and Replacement
Unable to install headboard and footboard	Wrong installation method of headboard or footboard	Check the locking mechanism Place the headboard and footboard correctly, refer to 1.2 Head board & Foot board assembly method
	Mechanical failure	Notify maintenance

## 5. Bed-maintenance

1. In order to use the bed safely, safety inspections must be done regularly. Conduct a comprehensive inspection every six months, to ensure that there is no looseness in the connection parts and the function of up & down for the bed can be operated normally.
2. At the joints of the frequently moving parts of the bed, an appropriate amount of lubricating oil needs to be added during daily inspection; if serious damage is found, it should be replaced immediately to ensure safety, add lubricated parts as show. Add some lubrication oil at the moving parts as show in the figure.

3. Avoid hitting the bed and scratching the coating on the bed with sharp objects.
4. Please avoid inhaling moisture into the bed and mattress, which will cause rust, abnormal noise and bacteria breeding. If water adheres, wipe it off immediately.
5. When the bed frame ages and it reaches a certain servicing time, the metal part of the bed body and the plastic parts of the headboard or foot board, guardrail can be recycle.

## 6. Mattress-cleaning and maintenance

1. When using it, avoid scratching the fabric with sharp-angled tools or knives, cover it with sheets or cleaning pads, and clean it frequently to protect it from dryne
2. Clean the mattress regularly with a vacuum cleaner, Mattress cover can be washed in 100°C.
3. Expose the mattress often to keep the sleep fresh and comfortable, but be careful not to expose the mattress to too long.
4. If the fabric is accidentally stained, please refer to the corresponding recommended cleaning methods in the instructions; do not use corrosive chemicals to clean the fabric.
5. If accidentally the chewing gum or other jellies get on the fabric, do not wipe it forcibly. Use an ice pack to allow it harden and gently peel it off.
6. If the thread is found to be loose due to wear and tear, do not pull it off by hand, use scissors to cut it together.

\* If your order does not have a mattress, please refer to the contract configuration.

## 7. Caster-detection and maintenance

1. Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
2. Grease the casters or wheels on a regularly, and the frequency of addition should be determined according to the specific actual use conditions. In general, lubricating grease is added every six months.

3. After cleaning the caster equipment, pay attention to adding grease. The selection of lubricating grease should be based on the actual use environment, especially high and low temperature environment, and the corresponding lubricating grease should be selected.
4. It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
5. The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
6. Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
7. For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a hospital bed equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the bed also check whether the brake performance of each caster is good. If the function of the caster brakes fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
8. If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

## **8. Guardrail-maintenance**

1. Please check whether the side rails are installed correctly to prevent the lying patient from shifting and rolling down.
2. Please always check the screws of the total position of the guardrail and the wire of the guardrail to prevent loosening and falling off.
3. Please use the plastic parts of the guardrail switch correctly, and do not use brute force to prevent it from damaging the guardrail.

## **9. Package, transport, storage**

1. The packaging of electric bed is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the electric bed.
3. The electric bed should be stored:
  - a. Ambient temperature:  $+5^{\circ}\text{C} \sim +40^{\circ}\text{C}$ .
  - b. Relative humidity:  $\leq 85\%$ .

- c. Atmospheric pressure: 70 kPa ~ 106 kPa.
  - d. Power voltage: AC 100-220±22V;
  - e. Power frequency: 50±5 Hz
4. The equipment is with internal power supply, and the protection type is Type B.
- a. Working voltage: DC 24V.
  - b. Input power: 300VA.
  - c. Fuse model and rating: F3AL 250V, 5×20mm
  - d. Working schedule: intermittent loading and continuous operation
  - e. Ingress protection: IPX4
  - f. Non-AP /APG devices

Symbol Interpretation:

Note! Check the file come with the bed

IPX4 Ingress protection

Class II equipment

Protective grounding

## 10. After-sales service

1. Please keep the files that comes with the machine and invoices of this product properly and you need to present these materials when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. Do not disassemble the internal device of this product by yourself to avoid unnecessary damage. If you find that the use is affected due to quality problems, please contact our after-sales service department or our authorized dealer.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will repair the product for the user for free.

### Correct Disposal

This product is subject to the provision of european Directive 2012/19/EU. The symbol showing a wheellie bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.

JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

**Imported to AUS:** SIHAO PTY LTD.

1 ROKEVA STREET EASTWOOD NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd.

Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,  
Centurion House, London Road,  
Staines-upon-Thames, Surrey, TW18 4AX

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Mainzer Landstr.69,  
60329 Frankfurt am Main.

**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate  
[www.vevor.com/support](http://www.vevor.com/support)

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JIANGSU SAIKANG MEDICINISK UTRUSTNING CO LTD
**Importerad till AUS:** SIHAO PTY LTD.
**Importerad till USA:** Sanven Technology Ltd.
E-CrossStu GmbH