


DOG HIP BRACE

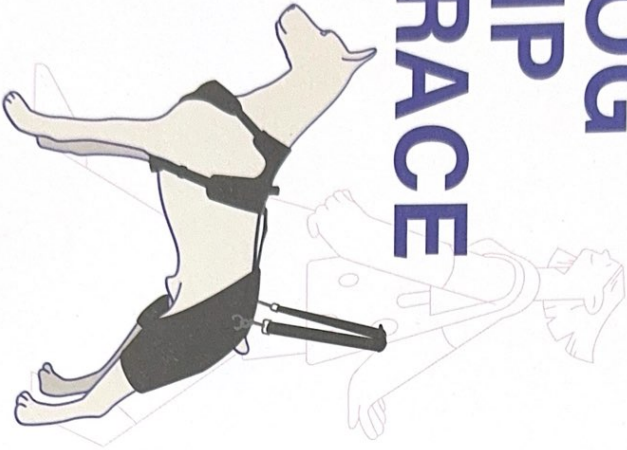

If you encounter any problems, you can contact LISPOO OFFICIAL. We will provide lifelong service.


• LETTER



Dear Friend

Thank you for choosing us. We all wish your beloved dogs are happy and get better sooner. We have also donated knee pads to over 1,000 dog shelters around the world. Thanks to our tireless efforts, these over 1,000 dogs have the ability to get healthier and live happier, healthier lives. To each and every pet owner, we offer our sincerest respect. Thank you for your endless love for your dogs! Please do not hesitate to contact us via account email. If you met any problems with size or special dog breeds, we will try our best to help you.

Best Regards

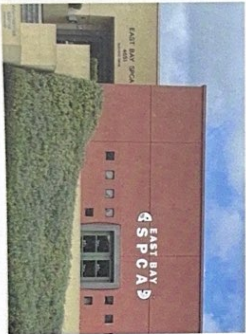

• AFTER SALES SERVICE



Dear Friend

Have you met any problems with the product? While if you do, please do not feel down, as we would love to help with any kinds of problems with it. All we could do is try our best to give you the most customer service and solve the problem with root. Do remember to contact us promptly, if you need us, we will be here within 24 hours. If happened with returning, please help us clean the dog hairs with accidentally stucked on the product.

Thank you so much.

• EXAMPLE DIAGRAM



Before adhesion

Cleaning with adhesive tape

After adhesion

HOW TO USE

1 Put the dog's head through the collar with the logo facing up.



2 Fix the left and right chest buckles and adjust to the appropriate length.



3 According to the shape of the dog's buttocks, adjust to the appropriate position.



4 Find the left and right leg labels, and fix the velcro to be attached.



5 Pass the rear velcro through the metal part of the front strap, and stick the velcro tightly.




6 Check the final wearing result.



SIZE CHART

Size	Girth of chest	Girth of neck	Girth of A	Length of B
M	24"-32"	20"-24.5"	7"-10.5"	12.5"-14"
L	25"-36"	21"-28"	8"-12"	14.5"-15.5"
XL	30"-41"	25"-35"	9"-14"	16"-18"
XXL	32"-42"	25.5"-36"	9.5"-16"	16.75"-19"

MEASURE FOR SIZING

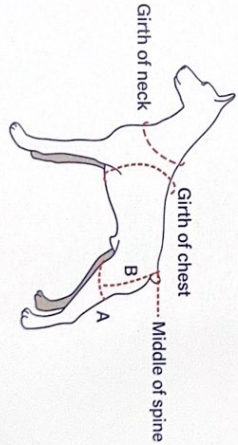

SYMPTOMS

It helps dog arthritis, mild to moderate hip dysplasia, decreased endurance during activity, hip pain, lower back pain or weakness, and back leg limping.

It stabilises and warms the hip articulation, reduces the pain and boosts the muscles activity avoiding atrophy and increasing the bond between the femur head and the acetabulum.

HOW TO MEASURE

