



XLIFE
SMART WIRELESS
EAR CLEANING OTOSCOPE

USER MANUAL


Step 1: Download the APP

iOS : Search " Xlife " in App Store
Android : Search " Xlife " in Google Play

Or scan the QR code to download the app




Step 2: Press and hold the power button
to turn on the ear cleaning rod

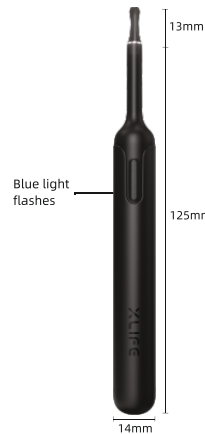

Step 3: Connect product WiFi

1 Connect Product Wi-Fi
"Xlife-XO-XXXXXX"
2 Back to Xlife app
Enter and start to use it

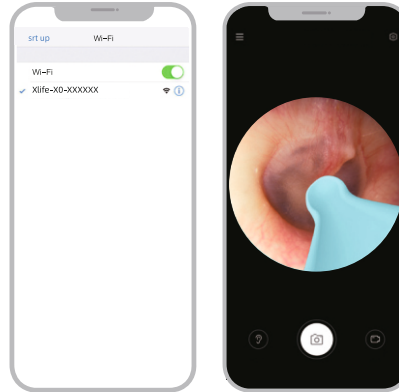

Accessory

1 Ear Otoscope x 1
2 Charging Cable x 1
3 User Manual x 1



Silicone Ear Spoon Cover x 8

Note: Do not use the charging equipment with output voltage exceeding 5V

Troubleshooting

Problem	Solution
No response after startup	Please use the incidental charging cable to charge. The charging time is 30mins
The app exits quickly	Delete the existing app and scan QR to reinstall it
Indicator light flashes	Please use the incidental charging cable to charge. The charging time is 30mins
Abnormal display in app	Interrupt the connection, restart the product to connect again
If you still have problems with the above procedures, please contact our Customer Service Representatives directly for further assistance	

Hazardous substance

Part name	Hazardous material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed Circuit Components	x	o	o	o	o	o
Case	o	o	o	o	o	o
Plastic rubber accessory	o	o	o	o	o	o

This form is formulated in accordance with SJ/T 11364
o : represent the hazardous material ratio is below GB/T26572-2011
x : represent the hazardous material ratio is over GB/T26572-2011

Product parameters

Product Name : XLife
Pixels : 3 Mega-pixels
Depth of field : 10-50mm
Lens diameter : 3.5mm
Image transmission rate : 20fps
Battery : 100mAh
Battery life : 30 minutes
Charging time : 30 minutes
Input current : 5V 0.5A
Gyroscope : 3-axis
Network standard : IEEE 802.11b/g
Working temperature : -10- 45 C
Rod weight : 11g
Working frequency : 2.4Ghz
Image sensor : CMOS

Attention

- When cleaning lens, please wipe it carefully with professional alcohol cotton swabs.
- Please observe your surroundings before use. Stay at least 2ft away from people, animals, or moving objects when using. Don't use in a moving vehicle or other unstable environments. Please don't move around when using.
- This product is unsuitable for children under 3 years old.
- This product is not suitable for children to use alone.
- Do not place in a humid environment for a long time or soak in liquids.
- There is a built-in rechargeable lithium-ion battery. Even if you do not use it, you should charge it once a month to avoid long-term battery damage.
- Please use a 5V charger to charge the device. Do not use this product while being charged.
- Avoid direct exposure to the sun. 9. Do not put heavy objects on the product.
- It is normal for the temperature to rise slightly when using (up to 95°F).