

Regulatory Information

The following additional information is provided in compliance with U.S. Food and Drug Administration (FDA) regulations:

WARNING: If you are younger than 18, do not use this.

You should go to a doctor, preferably an ear-nose-throat doctor (an ENT), because your condition needs specialized care. Over-the-counter hearing aids are only for users who are 18 or older.

This OTC hearing aid is for users who are 18 and older. People who are younger than 18 with hearing loss should see a doctor, preferably an ENT, because they may need medical testing and management. Hearing loss can affect speech and learning, so professional fitting and continuing care are also important.

Intended Use

Good Voice Nexus Hearing Aids are designed to enhance and deliver sound directly to the ear, addressing mild to moderate hearing loss in individuals aged 18 and above. No prior programming or hearing assessment is required. These aids are intended for use independently, without the need for hearing care professional's assistance.

WARNING: When to See a Doctor

If you have any of the problems listed below, please see a doctor, preferably an ear nose throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus, or fluid coming out of your ear in the past 6 months
- Your ear feels painful or uncomfortable
- You have a lot of ear wax, or you think something could be in your ear
- You get really dizzy or have a feeling of spinning or swaying (called vertigo)
- Your hearing changed suddenly in the past 6 months
- Your hearing changes: it gets worse then gets better again
- You have worse hearing in one ear
- You hear ringing or buzzing in only one ear

WARNING: This hearing aid should not cause pain when inserting it.

Remove this device from your ear if it causes pain or discomfort when you insert or place it. To try again, make sure to follow the instructions. If you feel pain or discomfort again, contact the manufacturer. If your pain or discomfort does not go away, contact your hearing healthcare professional. You can also report this to FDA as an adverse event according to the instructions that appear later.

Caution: This is not hearing protection.

You should remove this device if you experience overly loud sounds, whether short or long-lasting. If you're in a loud place, you should use the right kind of hearing protection instead of wearing this device. In general, if you would use ear plugs in a loud place, you should remove this device and use ear plugs.

Caution: The sound output should not be uncomfortable or painful.

You should turn down the volume or remove the device if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your device.

Caution: You might need medical help if a piece gets stuck in your ear.

If any part of your hearing aid, like the eartip (dome), gets stuck in your ear, and you can't easily remove it with your fingers, get medical help as soon as you can. You should not try to use tweezers or cotton swabs because they can push the part farther into your ear, injuring your eardrum or ear canal, possibly seriously.

Note: If you remain concerned, consult a professional

If you try this device and continue to struggle with or remain concerned about your hearing, you should consult with a hearing healthcare professional.

Note: What you might expect when you start using a hearing aid

A hearing aid can benefit many people with hearing loss. However, you should know it will not restore normal hearing, and you may still have some difficulty hearing over noise. Further, a hearing aid will not prevent or improve a medical condition that causes hearing loss.

People who start using hearing aids sometimes need a few weeks to get used to them. Similarly, many people find that training or counseling can help them get more out of their devices.

If you have hearing loss in both ears, you might get more out of using hearing aids in both, especially in situations that make you tired from listening—for example, noisy environments.

Note: Tell FDA about injuries, malfunctions, or other adverse events.

To report a problem involving your hearing aid, you should submit information to FDA as soon as possible after the problem. FDA calls them “adverse events,” and they might include: skin irritation in your ear, injury from the device (like cuts or scratches, or burns from an overheated battery), pieces of the device getting stuck in your ear, suddenly worsening hearing loss from using the device, etc.

Instructions for reporting are available at <https://www.fda.gov/Safety/MedWatch>, or call 1-800-FDA-1088.

You can also download a form to mail to FDA.

Possible Side Effects

You may experience minor itchiness, redness, or irritation as your ear adjusts to having an object in it. If these or any related symptoms occur, please see the back page for full contact details. Should you experience an allergic reaction, inflammation, ear discharge, excessive wax buildup, or any other unusual conditions, seek medical help from a physician.

Table of Contents:

Regulatory Information	1
Intended Use.....	1
Table of Contents:.....	5
Welcome.....	8
Important Guidelines for Optimal Performance.....	8
Operating Principle.....	10
Getting to Know Your Good Voice Nexus	11
Included Accessories	13
Care and Maintenance Guidelines.....	14
Instruction for Product Use.....	16
Getting the Right Fit with Earbuds and Ear tips.....	17
Device Setting	20
Choose Display style.....	20
Adjust the Screen Orientation.....	20
Set the date display format.....	21
Adjust screen brightness.....	22

Set the date and time	23
Microphone-input selection	24
Device Control.....	25
Adjust the Volume	25
Activate Noise Reduction Level	26
Power Off the Watch (Controller)	27
Bluetooth Devices Pairing Instructions.....	28
Audio Streaming Bluetooth Devices.....	29
Answering Phone Calls.....	30
Daily Use of the Hearing Aids.....	31
Charging Instructions	32
Charging Your Watch (Controller).....	32
Charging Your Bluetooth Earbuds	34
Troubleshooting Guide	36
Symbol Information.....	41
Wireless Technical Description	43
Technical Specifications.....	45
FCC Warning Statement	48



Manufacturer Information.....	49
Service and Warranty	49
General Warranty Information:	49
Warranty Period and Coverage:.....	50
Customer Responsibilities:	50
How to Claim Warranty Service:.....	51
Out-of-Warranty Cases:	51
Your Warranty Information	52
Contact Details.....	52



Welcome


Thank You for Choosing Good Voice Nexus Hearing Aid. We appreciate your purchase as a greater step toward hearing what MATTERS.

Please thoroughly review this manual before using your Good Voice Nexus Hearing Aid. It contains important information on product features, safety precautions, and regulatory guidelines. Similar to trying on any new devices, it may take some time to get used to the Good Voice Nexus Hearing Aid.

Important: Please retain this manual for future reference. If you have any questions, please see the back page for full contact details. Follow the safety information to avoid damage or injury.

Important Guidelines for Optimal Performance

- Ensure the earbuds fit securely and comfortably in your ears. Improper fitting may result in discomfort or reduced performance. Gradually increase the volume to a comfortable level to avoid discomfort.
- Regularly clean or replace the ear tips as instructed to maintain optimal functionality. Accumulation of earwax or residue may cause irritation or impair performance.
- To ensure the best performance of your Good Voice Nexus use only authentic Good Voice accessories purchased from authorized dealers.

- This hearing device is intended exclusively for individuals with hearing loss. It is not recommended for use by individuals with normal hearing.
- Remove the earbuds immediately if exposed to excessively loud sounds.
- “Enhanced Noise Reduction Mode”  reduces environmental sound awareness. Avoid using this mode when full awareness of the user's surroundings is necessary
- Consult your healthcare professional before using the device if you have an active implanted medical device (e.g. pacemaker).
- Avoid using the hearing devices or charger in areas with oxygen enrichment or a risk of explosion.
- Protect the product from exposure to electromagnetic fields, humidity, smoke, direct sunlight, fire, and extreme temperatures.
- This devices complies with the IP54 rating, offering limited protection against dust and dripping water. It is not fully waterproof, so avoid exposing it to water or immersing it in any liquid.

CAUTION: Please keep away from children and pets, because small parts may be inhaled or swallowed.

CAUTION: Please keep this device away from pets, pests, and children.

WARNING: The equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous: Not AP or APG Category

WARNING: No modification of this equipment is allowed.

IMPORTANT: Accessory equipment connected to the analog and digital interfaces must be in compliance with the respective nationally harmonized IEC standards (i.e. IEC 60950 for data processing equipment, IEC 60065 for video equipment, IEC 62368-1 for audio, video, information and communication technology, IEC 61010-1 for laboratory equipment, and IEC 60601-1 for medical equipment.) Furthermore, all configurations shall comply with the system standard IEC 60601-1. Everybody who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore, responsible that the system complies with the requirements of the system standard IEC 60601-1. The unit is for exclusive interconnection with IEC 60601-1 certified equipment in the patient environment and IEC 60XXX certified equipment outside of the patient environment. If in doubt, consult the technical services department or your local representative.

Operating Principle

Good Voice Nexus hearing aid amplifies sounds to help users with mild to moderate hearing loss hear more clearly.


The Watch (Controller) uses a built-in microphone to pick up sound, a digital signal processor (DSP) to analyze and enhance speech while reducing unwanted noise, and a speaker (receiver) to deliver the amplified sound into the ear by Bluetooth earbuds, allowing wireless audio streaming from compatible device

Getting to Know Your Good Voice Nexus

Watch (Controller)



Bluetooth Earbuds (Model: HT-002)

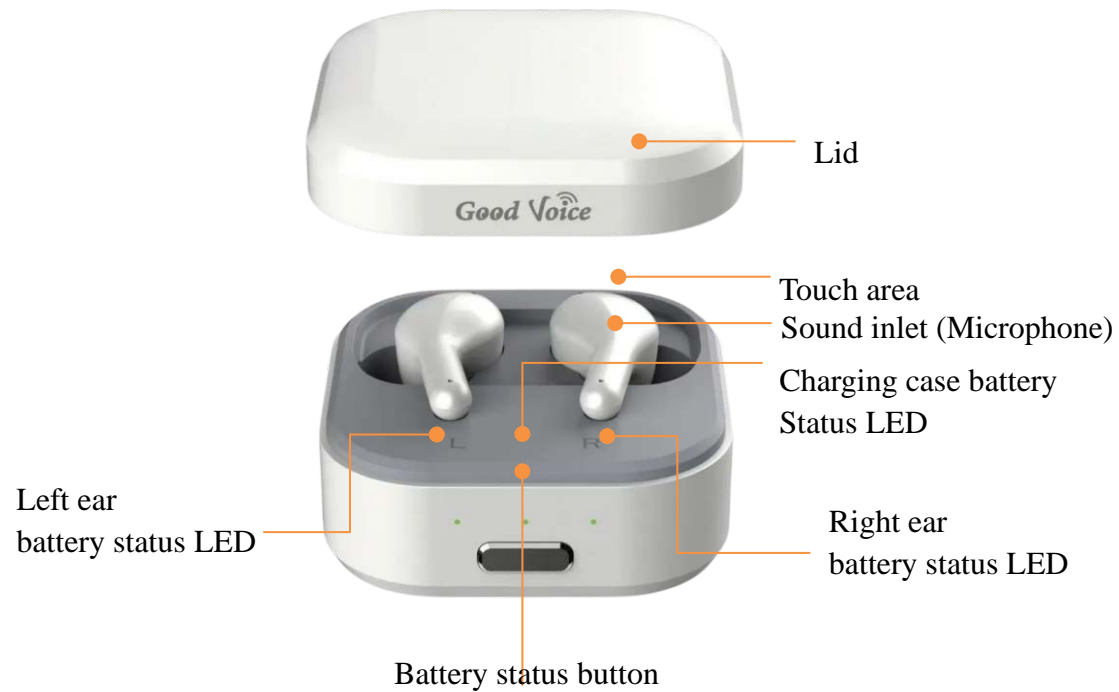

Note: The included Bluetooth earbuds are exclusively designed for pairing with the Good Voice Nexus Watch (Controller)

Included Accessories


Please ensure all the following items are included in the package. If any of the following accessories are missing, please see the back page for full contact details.



Watch (Controller)



Bluetooth Earbuds

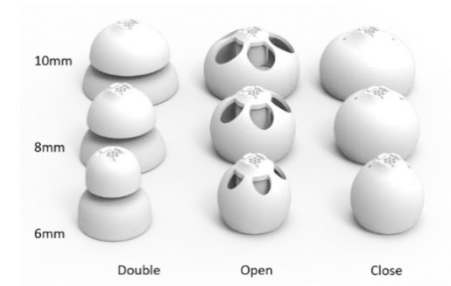


Replacement Ear Tips



Charging Cable
(USB-A to Dual Type-C)



Watch strap & strap holder

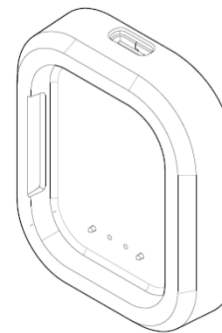

Watch charger dock

**OTC
HEARING AID**

**Good Voice
Nexus**
User manual

Model: GV-NXT

OKB



User Manual

Care and Maintenance Guidelines

To maximize the life of your rechargeable hearing aid batteries, do not store the devices for extended periods without recharging them. Recharge at least once every three months if it is not used.

- Avoid storing the hearing aids in extreme temperatures or harsh weather conditions to prevent battery deterioration.
- Do not expose the hearing aids or charging case to sand, dust, or other fine particulates, as this may cause permanent damage to the device.
- Store and charge your devices in a dry location. Prolonged exposure to moisture can harm the hearing aids and charging case.
- Avoid using a hairdryer or other heating devices to dry the hearing aids as excessive heat may cause damage.
- Clean the hearing aids as instructed to prevent earwax or residue buildup for maximum product performance.
- To prevent a voided warranty, do not attempt to repair, disassemble, or modify the product.
- To Clean the Watch (Controller), Earbuds and Charging case: Wipe with a cloth lightly dampened with water. Do not use alcohol for cleaning.
- It is recommended to clean the Watch (Controller), Earbuds and Charging case whenever visible dirt is observed to maintain hygiene and proper function.

Battery Safety Warning

- The in-built lithium battery can only be replaced by the manufacturer or an authorized service provider.
- Replacement of batteries with an incorrect type that can defeat a safeguard, especially with some lithium battery types.

- Disposal of batteries into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion.
- Leaving batteries in an extremely high-temperature environment can result in an explosion or the leakage of flammable liquid or gas.
- Batteries are subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas.

CAUTION: Risk of fire or explosion

- Do not replace the battery with an incorrect type.
- Dispose of used batteries according to the instructions.

Instruction for Product Use


1. Turn on the Controller

Press and hold the Power button for 2 seconds to turn on the Watch (Controller), you will see the “Good Voice” shows on the screen.

2. Open the lid of charging case.



3. Automatic Pairing

- The left and right earbuds will automatically attempt to connect to the Watch (Controller) once the lid is opened.
- If successful, you will hear the "connected" voice prompt, indicating that the earbuds and watch have been successfully paired. Then the connected icon  will appear at the top of the Controller (watch) screen.

4. Wearing the Earbuds

- Remove the earbuds from the charging case.
- Place the earbuds in your ears according to the left (L) and right (R) indicators for proper fit and function.

Getting the Right Fit with Earbuds and Ear tips


Wearing Your Earbud

1. Consider practicing in front of a mirror
 - Hold the earbuds stem with the ear tips facing inward.
 - The “L” (Left) or “R” (Right) symbol on the stem will be facing you.
2. Place the Earbuds into Your Ear
3. Rotate the Earphone Stem
 - Rotate about 45 degrees toward your face and gently press the earbuds further into your ear.



Finding the Suitable Ear Tips

- The package comes with three types of ear tips. Each type is available in three sizes (6 mm, 8 mm, and 10 mm) for replacement.

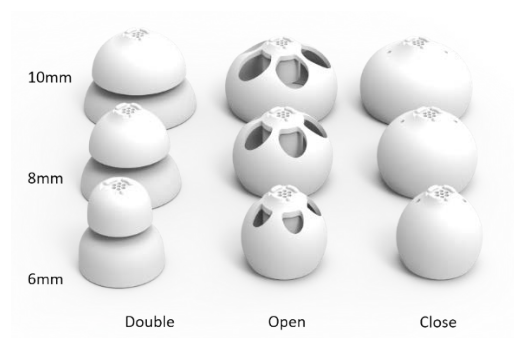

- Double-flange (Double): Provide two physical barriers for the most secure seal, helping prevent the tips from slipping out during movement. Suitable for users with more severe hearing loss or those who require higher volume.
- Open dome (Open): Feature large side vents to reduce the feeling of blockage, helping to avoid echo or “plugged” sensations when speaking. As they allow natural ambient sound to enter the ear canal, some interference may occur. Suitable for users with mild hearing loss.
- Closed dome (Closed): Have a full dome structure. Compared to open ear tips, they more effectively retain sound energy within the ear canal.
- The earbuds are pre-installed with 8 mm closed ear tips. If they do not fit comfortably, please try other sizes.
- Choose the ear tip type and size that provide the best fit and comfort to achieve optimal performance.

Removing Ear Tips

1. Hold the earbud stem securely in one hand.
2. Use the other hand to grasp the ear tip.
3. Gently pull the ear tip away from the earbud to detach.

Replacing Ear Tips


1. Align the chosen ear tip with the sound outlet of the earbud.
2. Press the ear tip firmly onto the sound outlet.
3. Ensure the ear tip is fully and securely mounted on the sound outlet before use.



When to Discard an Ear Tip

Dispose of the ear tips when:

- Ear tips appear noticeably dirty or stained.
- Signs of wear and tear, such as cracks or deformation, are visible.
- Ear tips no longer fit securely or comfortably.

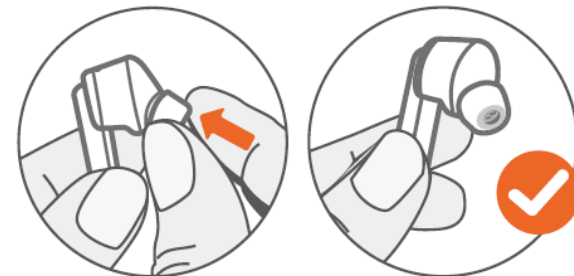

When to Reuse an Ear Tip

Ear tips can be reused if they remain intact and show minimal signs of wear or damage. To clean:

- Soak the ear tip in a mixture 1:100 ratio of soap to warm water
- Rinse thoroughly with clean water.
- Allow the ear tip to dry completely before reattaching it to the earbud.

Device Setting

Choose Display style




Swipe left choose the wallpaper display style.




Adjust the Screen Orientation




Since everyone wears their watch differently, the screen may sometimes appear upside down.

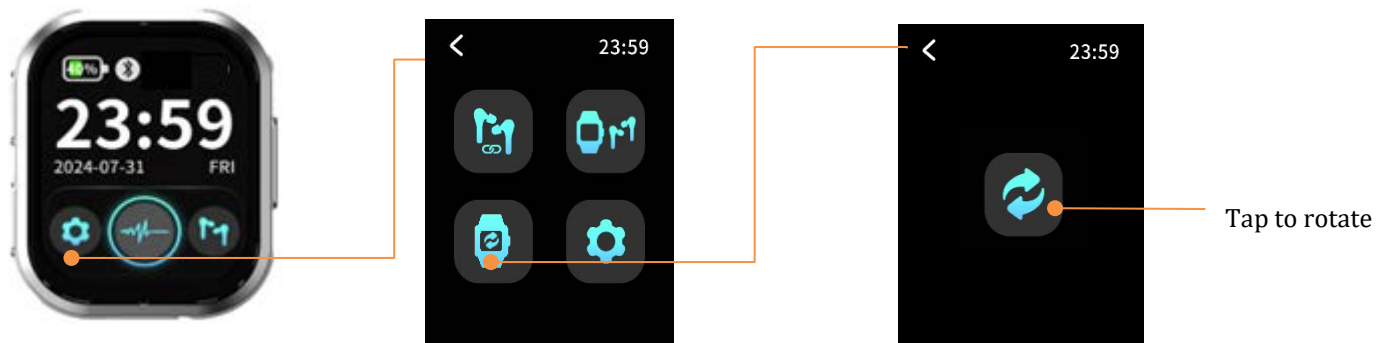


You can follow the steps below to adjust the screen orientation and ensure it displays correctly at all times.


1. Tap the Settings icon  to enter the menu.
2. Select the watch rotation icon , then tap  to set the screen to the correct orientation.






Set the date display format.


1. Tap the Settings icon  to enter the menu.
2. Select the Settings icon  again.
3. Tap the Globe icon  to Select date format:

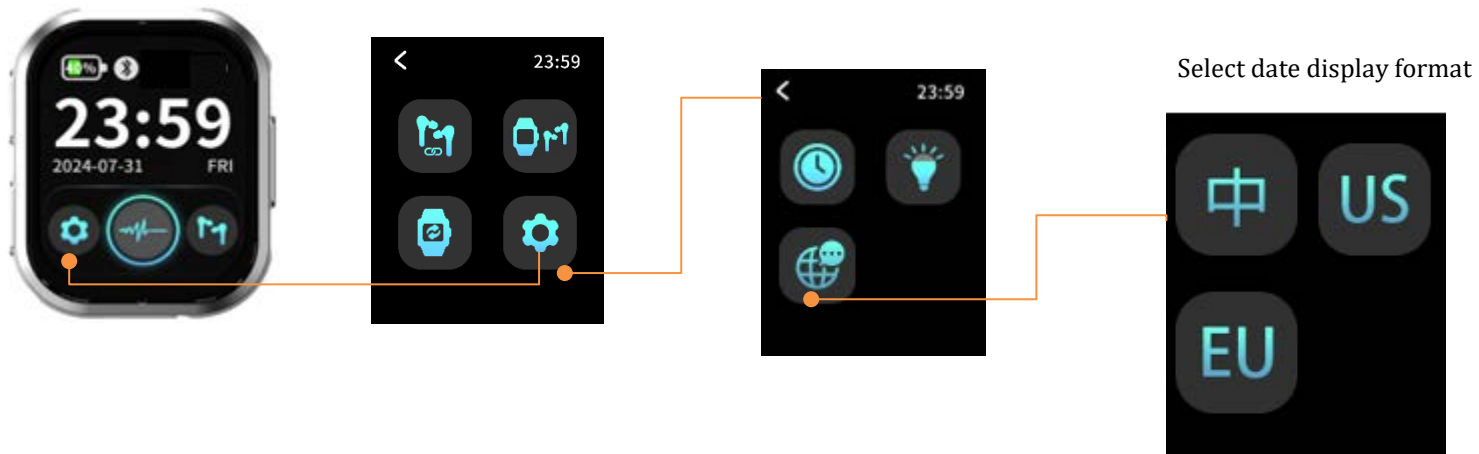
-  Standard date format (YY/MM/DD)
-  US date format (MM/DD/YY)
-  European date format (DD/MM/YY)




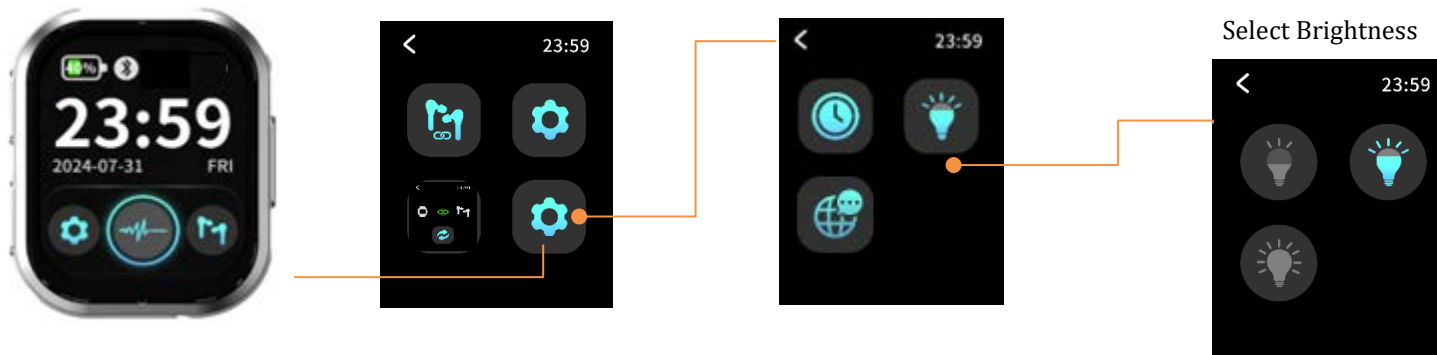
Adjust screen brightness

1. Tap the Settings icon  to enter the menu.
2. Select the Settings icon  again.
3. Tap the Brightness icon  to select preferred brightness






 The brightness is set to 50%


 The brightness is set to 75%

 The brightness is set to 100%



Set the date and time



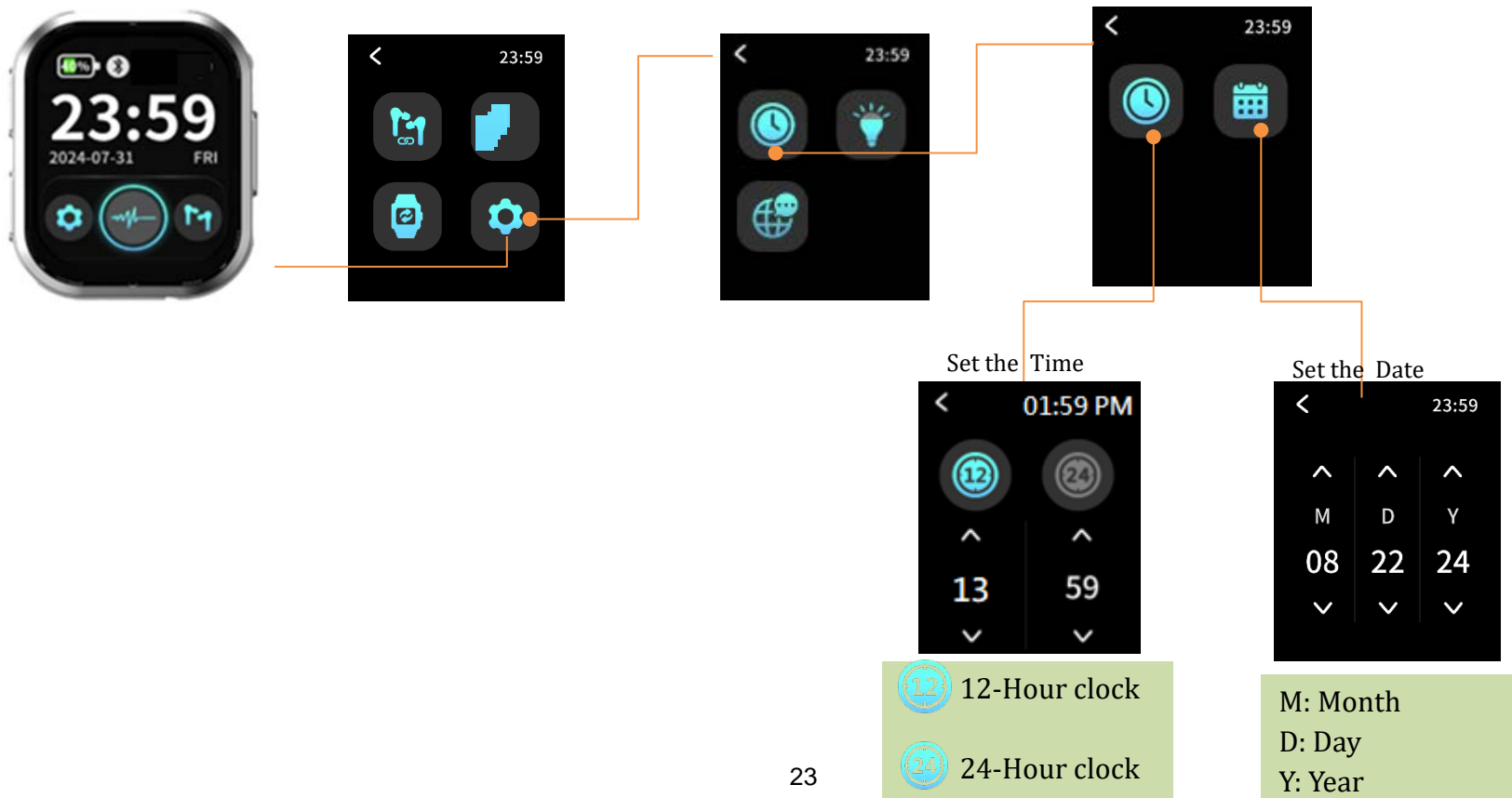

1. Tap the Settings icon  to enter the menu.
2. Select the Settings icon  again
3. Tap the clock icon 
4. Select the clock icon  again to set the time
5. Select the calendar icon  to set the date



Microphone-input selection

The Good Voice Nexus hearing aid uses the Watch (Controller) microphone as the default audio input. If the Watch (Controller) is out of range or the Bluetooth connection is lost, the system will seamlessly and automatically switch to the earbuds' microphones to ensure continuous sound pickup.

You have full flexibility to choose your preferred microphone source at any time. You can freely select either the Watch (Controller) microphone or the earbuds' microphones as the audio input, depending on your environment and preference. To manually select your preferred source, simply follow the steps below:

1. Tap the Settings icon  to enter the menu.
2. Select the Microphone switch icon .
3. Select  to use Watch's (controller) microphone as the default audio input; Select  to use earbuds' microphone as the default audio input




Note: For optimal hearing experience, we recommend using the Watch's (Controller) microphone to receive sound.

Device Control

Adjust the Volume


➤ By Touch control

4. Tap the earbuds icon  to enter for volume control page, where you can adjust the volume for the left and right ears independently.
5. Tap the "+" icon to increase the volume, up to a maximum of level 16.
6. Tap the "-" icon to decrease the volume, down to a minimum of level 1.



➤ By Physical button

Press the side volume button on the watch to adjust the volume of both the left and right ears up or down simultaneously.

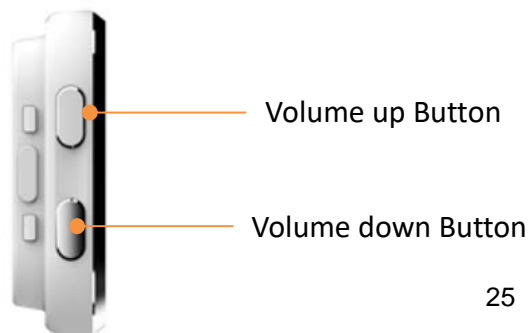

Activate Noise Reduction Level


To ensure an optimal hearing experience, you can freely adjust the noise reduction mode to suit your current surroundings. Whether in a quiet room or a noisy crowd, these settings allow you to maintain crystal-clear sound quality at all times.

1. Tap the Noise Reduction icon once to activate “Standard Noise Reduction Mode.”
2. Tap the icon again to switch to “Enhanced Noise Reduction Mode.”
3. Tap the icon once more to return to “Regular Mode.”



Icon	Meaning	Usage Scenario
	Regular mode	Moderately quiet environments, like home or office
	Standard Noise Reduction mode	Use in relatively noisy environments when conversations are easily disturbed by background noise
	Enhanced Noise Reduction mode	Use in very noisy environments, like restaurants, markets, or social gatherings. ⚠️ Enhanced Noise Reduction Mode reduces environmental sound awareness. Avoid using this mode when full awareness of the user's surroundings is necessary.

Power Off the Watch (Controller)

1. Press and hold the side power button on the watch for 3 seconds.
2. The power-off icon  will appear on the screen, press the icon to turn off the watch.



Press and hold the power button for 3 seconds.




Tap to power off



Watch is powered off.


Bluetooth Devices Pairing Instructions

For first-time use, please follow the steps below to pair your hearing aid with your mobile phone or other Bluetooth devices (such as a tablet). Once the setup is complete, the Bluetooth connection will automatically reconnect whenever both the hearing aid and the mobile device are turned on.

1. Turn on the Watch (Controller) and open the charging case lid of earbuds.
2. Once the Watch (Controller) and earbuds are successfully paired, the connection icon  will appear at the top of the screen.
3. Enable the Bluetooth function on your phone or tablet
4. Find “GV-NX1” in the list of available Bluetooth devices and tap to connect.

Watch (Controller) and earbuds are successfully paired



-  If “GV-NX1” does not appear in the Bluetooth device list on your mobile phone, please place the earbuds back into the charging case and press and hold the “battery status button” on the charging case for 5 seconds until the battery status LED flashes orange. Then release the button. Once the LED light turns off, repeat the pairing steps above.

Audio Streaming Bluetooth Devices

Once your Good Voice Nexus Hearing Aid has successfully paired with your mobile phone or Bluetooth device, any audio or video streaming from the phone or connected Bluetooth device will automatically switch the hearing aid to streaming mode.

When the audio or video playback is stopped, the hearing aid will automatically return to hearing mode within approximately **10** seconds.


Answering Phone Calls

Once your hearing aids are paired with your phone via Bluetooth, phone call audio will be automatically streamed to your hearing aids for maximum clarity.

Answering Calls via your Mobile Phone


Press  on the Watch (Controller) to pick up the phone, or press  to reject the call.

Incoming phone

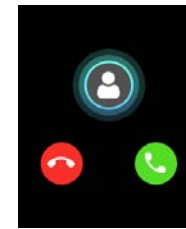

Answer the phone call via your Mobile Phone

1. Swipe right or Tap "Accept" to answer the call
2. Tap "audio"
3. Choose "GV-NX1" as the audio source.

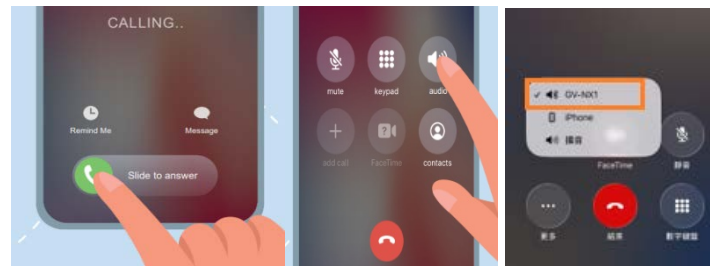

General Warnings:

- During Bluetooth streaming, ensure that the volume is adjusted directly on the phone or Bluetooth devices.
- IOS 6.0 or higher is required to use the watch (controller) with an iPhone, and Android 5.0 or later for Android phones. Please ensure your device meets the required operation system version before use.
- Bluetooth streaming will lead to higher power consumption on your Good Voice Nexus Hearing aids.

Daily Use of the Hearing Aids

Extension of Your Ear

The device is suitable for various environments, ensuring seamless communication with others and helping users to hear **WHAT MATTERS**. Consider the Watch (Controller) as an extension of your ears. For optimal audio reception, position the Watch (controller) close to the speaker or sound source in different situations.


In Meetings

For effective participation and easier follow-up on conversations, place the Watch (Controller) in the center of the conference room table or close to the speaker to capture discussions clear

When Watching TV

Position the Watch (Controller) near the TV to enhance audio clarity. If the Watch (Controller) cannot be placed close to the sound source, adjust the volume for each ear to achieve optimal audio performance.

Note: When watching TV, please do not enable “Enhanced Noise Reduction Mode” to ensure the best listening experience.

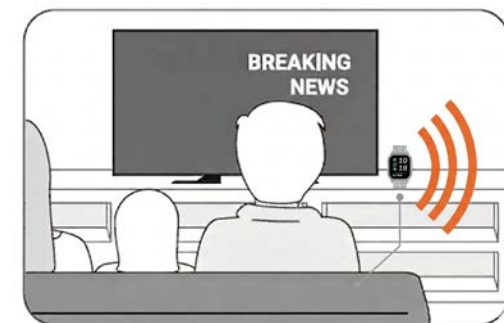


Charging Instructions

Charging Your Watch (Controller)

Battery brand/Model: Zhongshan Zhongwangde New Energy Technology Co., Ltd./702528

Ensure the charging contacts on the back of the Watch (Controller) are aligned with the pins on the charging cradle. Then, use the included USB cable to connect the charging port to a power source for charging.

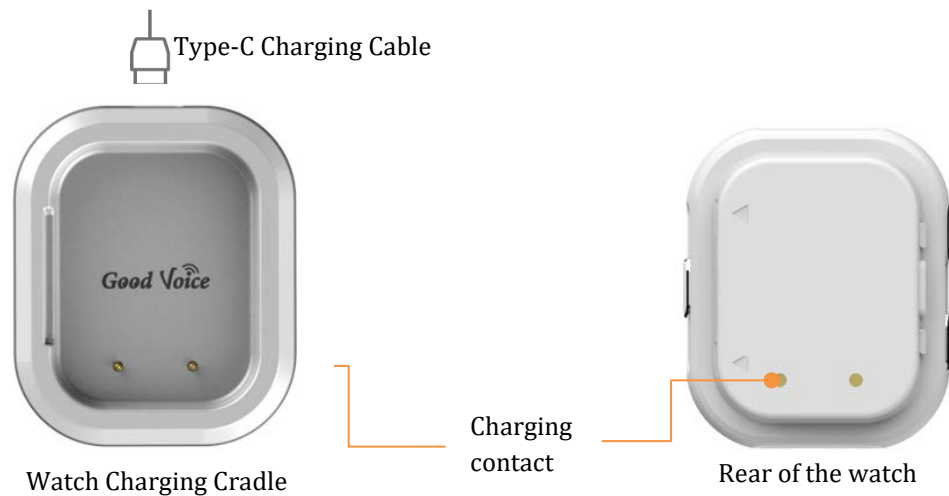

The Watch (Controller) is designed to provide a battery life of up to 28 hours on a full charge.

A fully depleted Watch (Controller) takes approximately 1.5 hours to recharge completely.


Watch (Controller) Battery Status Indicators

- Low Battery  :

The battery level of the Watch (controller) is displayed at the top-left corner of the screen. When it drops below 20%, the battery icon turns red, indicating a low battery that needs to be charged.

- Charging  :

The red charging icon appears in the center of the screen to indicate that the device is charging. When the battery level exceeds 20%, the charging icon turns light blue.

- Charging Completed  :

A progress bar moves along the outer edge of the circle to show the charging progress. When the charging is complete, the progress bar forms a full circle and the battery level shows 100%.


Battery Life

The Good Voice Nexus Watch's battery capacity gradually decreases with use. Lithium-ion batteries lose a small amount of capacity with each full charge cycle, which consists of a complete discharge and recharge. After 500 cycles, the battery retains 80% of its capacity. However, it can be recharged at any time without needing a full discharge, ensuring long-lasting performance.

Warning: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

Charging Your Bluetooth Earbuds

- Place the left earbud in the left cradle and the right earbud in the right cradle of the charging case.
- Ensure the charging contacts on the earbuds and the charging cradles are properly aligned.
- Once the earbuds are properly seated and make good contact with the charging pins, the indicator lights on the charging case will remain solid, confirming a secure connection. The leftmost light indicates the left earbud, and the rightmost light indicates the right earbud.
- Each light color reflects the battery level of that earbud. Once the lid is closed, the earbuds will turn off automatically while inside the charging case.

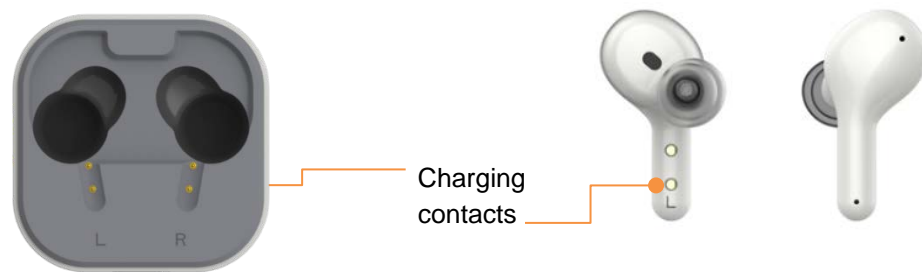

- It takes approximately 2 hours to fully charge a fully depleted charging case.
- Fully charged earbuds provide up to 8 hours of use. The on-the-go charging case offers up to three additional full charges (approximately 24 hours), ensuring all-day use.
- A 5-minute charge in the charging case provides approximately 35 minutes of use.

Charging case battery indicator

- Left LED: Indicates the battery level of the left earbud.
- Center LED: Indicates the battery level of the charging case.
- Right LED: Indicates the battery level of the right earbud.

LED Color Meaning

- Green light: Battery level is sufficient, above 80%.
- Yellow light: Battery level is approximately 30~80%.
- Amber light: Battery level is below 30%.




Note: You will hear a “Low battery please charge.” alert when the battery level falls below 20%




Notes:

- Always use the USB cable provided in the packaging.
- Do not use non-specified chargers or adapters, as this may damage the device.
- The shelf life of your Good Voice Nexus Watch (controller) and wireless earbuds (including the charging case) is 1 years. During this time, if stored properly under recommended conditions, the product is expected to remain functional and safe for use.
- You can expect reliable operation of Your Good Voice Nexus Watch (controller) and wireless earbuds (included charging case) for up to 1 years, beyond this decreased battery life may occur.

Troubleshooting Guide


If you encounter any issues with your device, please try the following steps:

No.	Description	Possible Cause	Solution
1	The Watch (Controller) and earbuds have never been paired.	Earbuds were replaced with a new set.	<p>Step 1:</p> <p>Place the earbuds back to the charging case and close the lid.</p> <p>If the hearing aids are paired with a mobile phone, please disconnect the Bluetooth pairing between the phone and the hearing aids first.</p> <p>Step 2:</p> <p>Turn on your Watch (controller).</p> <p>On the touchscreen, tap the <i>Setting</i> icon 「」, then tap the <i>Earbud Pairing</i> icon</p>

			<p>「」, and finally tap the <i>Pair</i> icon 「」 to start pairing.</p> <p>Step 3:</p> <p>Remove the earbuds from the charging case. You will hear a "Connected" prompt within 5 seconds, At the same time, the connection icon 「」 will appear in the top-right corner of the watch screen, indicating that the earbuds and the Watch(controller) have been successfully paired.</p>
--	--	--	--










2	The Watch (controller) and earbuds have been paired, but they cannot re-pair when both are powered on.	The Bluetooth connect issue.	Place the earbuds into the charging case and close the lid. After a few seconds, remove earbuds from the charging case; the earbuds and Watch (Controller) should automatically re-pair. If the connection is not restored, please repeat the first troubleshooting steps to manually pair the devices.
3	Loud feedback noise.	Earbuds are too close to the Watch (Controller).	Increase the distance between the earbuds and the Watch (Controller), especially when the volume is set high or when replacing the ear tips.









4	No sound coming from one or both earbuds	Audio signal transmission error between the Watch (Controller) and earbuds.	Place the earbuds back in the charging case and close the lid for a few seconds. Then, remove them from the case again.
		The earbuds were not properly seated in the ear, causing no sound.	Insert the earbuds into the ear canal, ensuring they fit snugly against the concha.
5	No sound in phone calls or audio streaming mode, even when your hearing aids is connected to the mobile phone or Bluetooth device	The Bluetooth device is muted.	During Bluetooth streaming, ensure that the volume is adjusted directly on the connected phone or Bluetooth device.
6	The watch (controller) screen is frozen, and tapping any button	The system crashed.	Step 1: When the Watch (Controller) is

	has no response.		<p>already on, press and hold the physical Power button on the right side for 10 seconds until the red power-off icon  disappears from the screen to reset the system.</p> <p>Step 2:</p> <p>After the screen goes completely dark, wait 5 seconds, then press the physical power button to turn on the Watch (Controller) again.</p>
--	------------------	--	---

If there is any other problem, please consult or return the device to your distributor. Do not try to repair a defective device by yourself.

Symbol Information

Symbol	Description
	Caution: Consult accompanying document.
	Consult instructions for use.
	Medical device
	CE marking indicates that this product complies with applicable European Union directives regarding health, safety, and environmental protection requirements.
	Please follow the national guidelines for electronic product disposal.
	Keep away from direct sunlight
	Protected from moisture
	Do not use it if packaging is damaged.
	Humidity limitation: Maintain within recommended levels.

	Temperature limits: Maintain within recommended levels.
	Manufacturer.
	Catalogue number.
	Serial number.
	Federal Communications Commission symbol
	This device complies with the technical standards of the Radio Law and the Telecommunications Business Law of Japan.
	Class II devices
	This product complies with the Japanese Electrical Appliance and Material Safety Law (DENAN) and bears the PSE Circle Mark, indicating it meets the required safety standards for electrical products in Japan
IP54	Protected against limited dust ingress. Dust may enter but will not interfere with the normal operation of the device. Protected against water splashes from any direction. Important: The device is not waterproof or completely dustproof. Do not submerge it in water or expose it to heavy moisture, such as strong rain or water jets.

Wireless Technical Description

Your Good Voice Nexus hearing aid system is classified as Class II Exempt per 21 CFR 874 and Class II Type BF per IEC 60601-1 Medical Electrical Equipment.

This hearing aid model has been tested to, and has passed, the following EMC, emissions, and immunity tests:

<p>Electromagnetic Emission</p> <p>Good Voice Nexus hearing aids are intended for use in the Electromagnetic environment specified below.</p> <p>The user should assure that your Good Voice Nexus is used in such an environment</p>		
Emission	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The Good Voice Nexus uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Good Voice Nexus is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage
Harmonic current emissions IEC 61000-3-2	Class A	

Voltage Fluctuations/flicker emissions IEC61000-3-2	Compliance	power supply network that supplies buildings used for domestic purposes.
--	------------	--

Electromagnetic Immunity

Good Voice hearing aids are intended for use in the Electromagnetic environment specified below.

The user should assure that your Good Voice Nexus is used in such an environment

Immunity Test	IEC60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 6100-1-2	±8 kV contact ±- 15 kV .	Compliant	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.



Technical Specifications

Audio Specs	Max Output: 116dB (OSPL90)		
	Max Output Gain: 39 dB (HFA-FOG)		
	Frequency Range: 200~8KHz		
	THD+N @ 500Hz	70dB	1.6%
		100 dB	<1%
	Equivalent Input Noise <32dBA		
	Latency : 5.6 ms		
Frequency response smoothness : 4.4 dB			
Compatibility	Bluetooth Version: BT 5.3		
	Bluetooth Codecs: LC3, SBC, AAC		
Input Rating	Watch (Controller): 5Vdc, 0.63A		

	Charging case: I/P 5Vdc, 0.6A; O/P:5Vdc, 0.32A
	Bluetooth Earbuds: 5Vdc, 0.095A
Battery Life	Watch (Controller): Approx. 28 hours
	Wireless Earbuds: 8 hours
	With Charging Case: Approx. 32 hours
Charging Time	Watch (Controller): Approx. 1.5 hours
	Wireless Earphones: Approx. 1 hours
	With Charging Case: Approx. 2 hours
Battery Capacity	Watch (Controller)r: 600 mAh / 3.7V
	Wireless Earbuds: 60mAh / 3.7V
	Charging Case: 600mAh / 3.7V
Charging Port	Type-C
Operating Conditions	Temperature: 32°F to 122°F (0°C to 50°C)
	Humidity: 5% to 85% RH (must consider all components: Watch (controller), and earbuds)

	Atmospheric pressure: 70~106KPa
Storage/Transportation Conditions	Temperature: -4°F to 158°F (-20°C to 70°C) Humidity: 5% to 85% RH Atmospheric pressure: 70~106KPa

FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Manufacturer Information

OK Biotech Co., Ltd.

Address: No. 91, Sec. 2, Gongdao 5th Rd., Hsinchu City 30070, Taiwan (ROC)

Website: www.okbiotech.com

Tel: +886-3-5160258

Made in Taiwan

Service and Warranty

General Warranty Information:

OKB warrants the product to be free from defects in workmanship and materials for the duration of the warranty period. If the product fails during normal and proper use within the warranty period, OKB may, at its discretion, repair or replace the defective parts or the product itself with new or reconditioned parts/products of equal or superior functionality.

The warranty applies only to products newly manufactured at the time of purchase and does not cover used, refurbished, or manufacturing seconds. Please retain your original purchase invoice for warranty claims.

Warranty Period and Coverage:

- 30-Day Money-Back Guarantee: Begins on the delivery date and expires on the 31st day.
- 1-Year Manufacturer's Warranty: Starts on the delivery date and lasts 365 days. For replacements, the warranty duration is calculated from the original purchase date.
- If proof of purchase is unavailable, the manufacturer's recorded date will be used as the warranty start date.

This warranty covers all OKB Hearing aids and accessories purchased directly from OKB and includes defects in materials or workmanship under normal use.

This warranty excludes damage or failure caused by:

- Improper installation, operation, cleaning, or maintenance
- Tampering, unauthorized repairs, or modifications
- Loss or theft
- Wear and tear, including changes in texture, color, or finish
- External factors such as accidents, natural disasters, or electrical faults
- Use outside recommended operational conditions as stated in the user manual

Customer Responsibilities:

- Read the user manual and follow instructions carefully.

- Avoid overcharging; unplug the product once fully charged.
- Keep the original packaging for transportation if servicing is required.

How to Claim Warranty Service:

1. Contact customer support in the back page with a description of the issue.
2. Provide proof of purchase.
3. Follow instructions for sending the product for evaluation.

Out-of-Warranty Cases:

If the warranty period has expired or exclusions apply, OKB will offer a service estimate. Repairs will proceed only after customer approval and payment.

International Warranty:

The warranty applies only in the country of purchase.

Limitation of Liability:

OKB is not responsible for incidental or consequential damages beyond the scope of this warranty.



Your Warranty Information

Thank You for Choosing OKB OTC Hearing Aid. This warranty is valid for a period of 12 months from the date of purchase. To ensure smooth service, please keep your warranty card in a safe place.

- Name:
- Address:
- Phone:
- E-mail:
- Production Lot Number:
- Date of Purchase:
- Purchased from (Dealer's Name)

Contact Details

Email: support@okbhearing.com

Phone: +1 626-225-0621

Business Hours: 9:00 AM – 6:00 PM (PST) | Monday - Friday

Business Address: 414C Cloverleaf Dr, Baldwin Park, CA 91706

GV-NX1_UG_EN_V2APR26