Enjoy!

Please visit the PioCreat website for more products operation and maintenance tutorials.


Note: The UI interface is only for reference, and shall be subject to the actual UI.

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Quick Start Guide 快速入门指南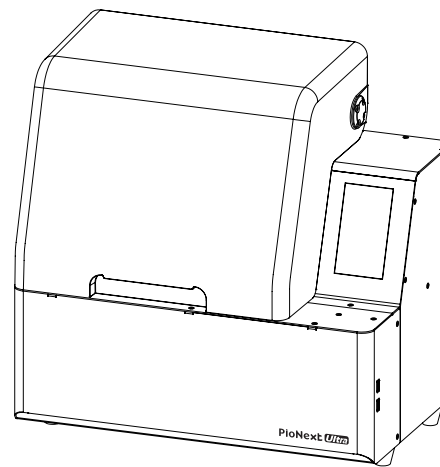


Usage scenarios

This product is designed for indoor use only


Please refer to this guide to complete the installation and printing of the printer.

*Safety Notice: Do not power on the printer before installation is complete.


Component Introduction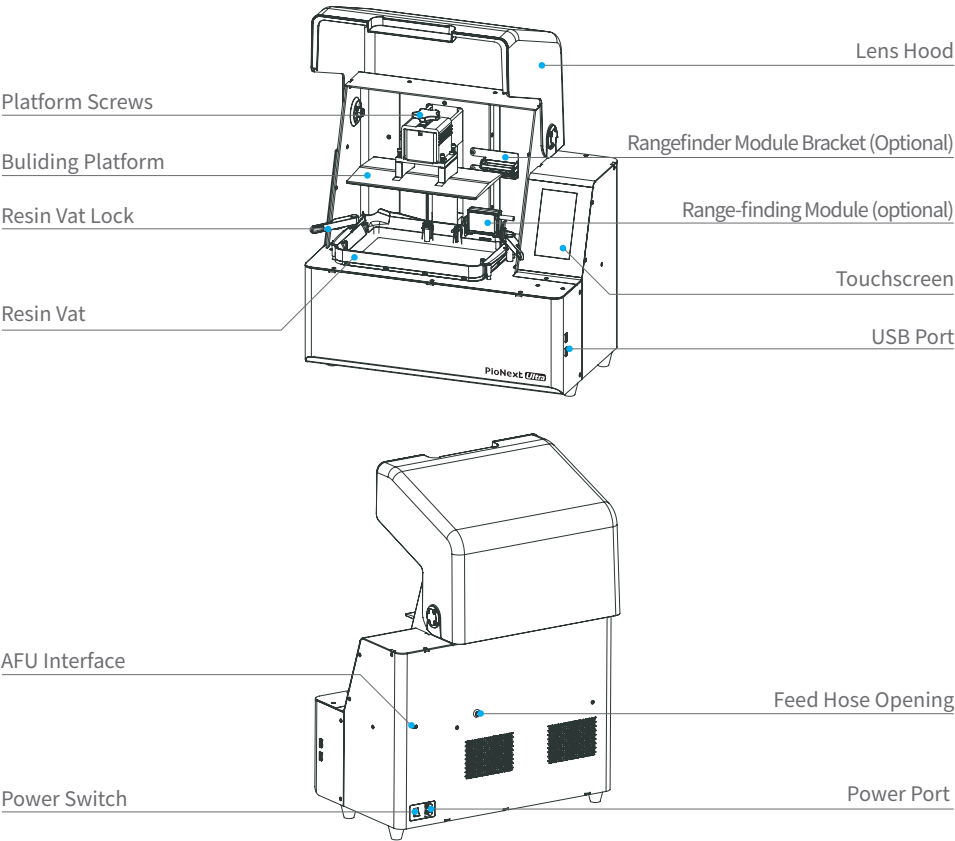


List of Accessories

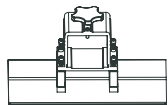Printing Platform
×1

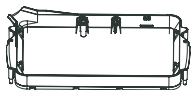Resin Vat
×1

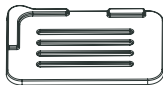
Resin Vat Dust Cover
×1

Power Cable
×1

U Disk
×1

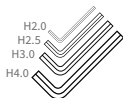
Hex Keys Set
×1


Disposable Latex Gloves
×1

Filters
×4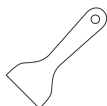

Plastic Blade
×1

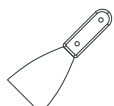Metal Blade
×1

Lubricant
×1

Paintbrush
×1

Calibration Card
×1

Qualified Certificate
×1


Quick Start
Guide ×1


After-Sales
Service Card ×1

Quick Start Operation Schematic


- 1 Unpack the accessories such as the dust cover of the material tank and the power cable, remove the protective foam from the top and the right side of the machine.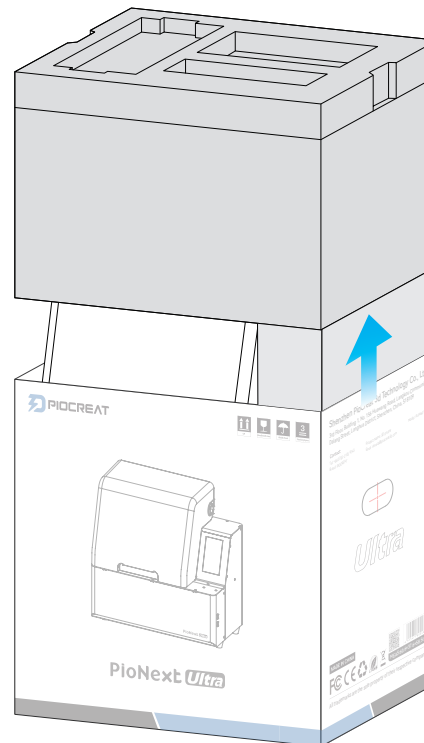


- 2 Remove the seamless tape between the hood and the base, open the hood, and take out all the foam inside the machine.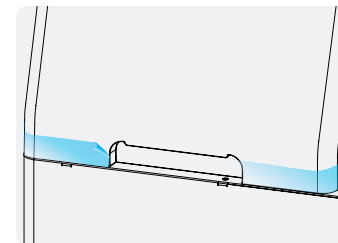

- 3 Once the power cord is connected, turn on the device switch to complete the startup operation.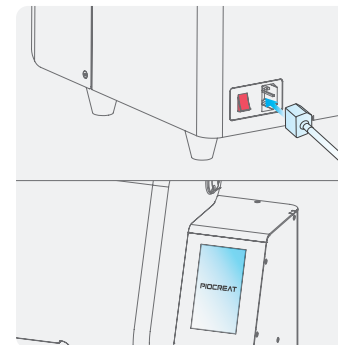


First Printing

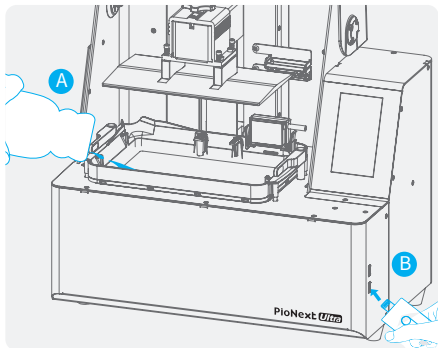A. Fill the resin vat.
B. Insert the included USB drive.

1


2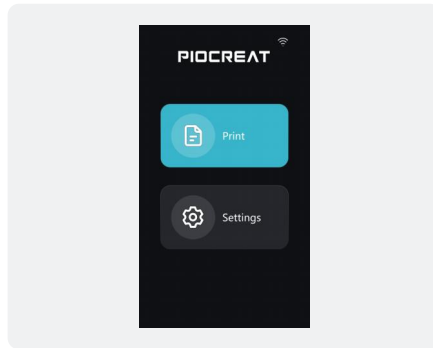


Tap "Print" on the home page.

3

4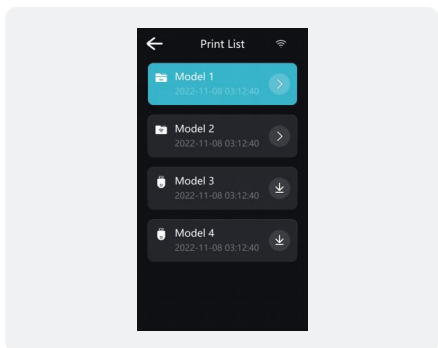


Select the pre-sliced file stored in the device.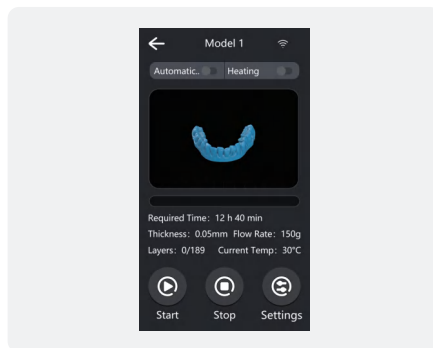

Click "Start" to start the first print.

4


Connect AFU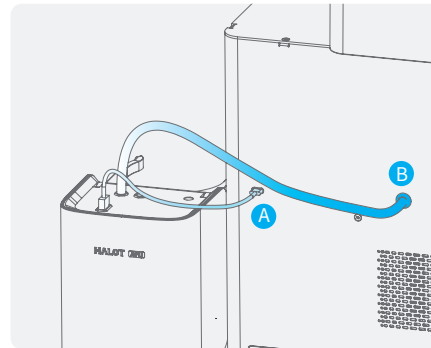

A. Plug one end of the AFU control cable into the I/O port.
B. Thread the feed hose through the AFU material pipe through-hole.

1


2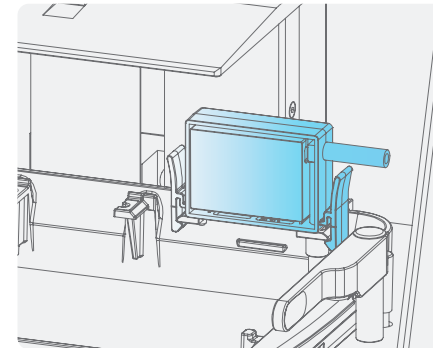


Remove the detection module and install it into the fixed position on the material tank.
Press down until the buckle makes a "click" sound.

3

4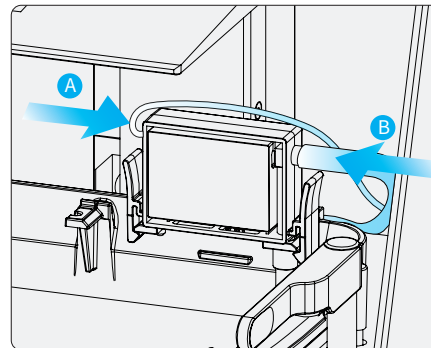


A- Insert the control cable vertically into the connection port of the detection module.
B- Insert the feed hose vertically into the feed port of the detection module.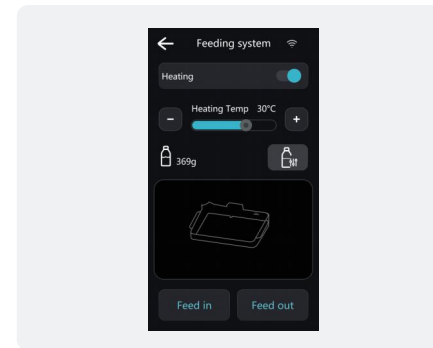

Click "Settings" → "Feeding System" on the main interface.
If you can successfully access the "Feeding System", it indicates that the AFU has been connected.

5

Connect Device to Network & Install Software

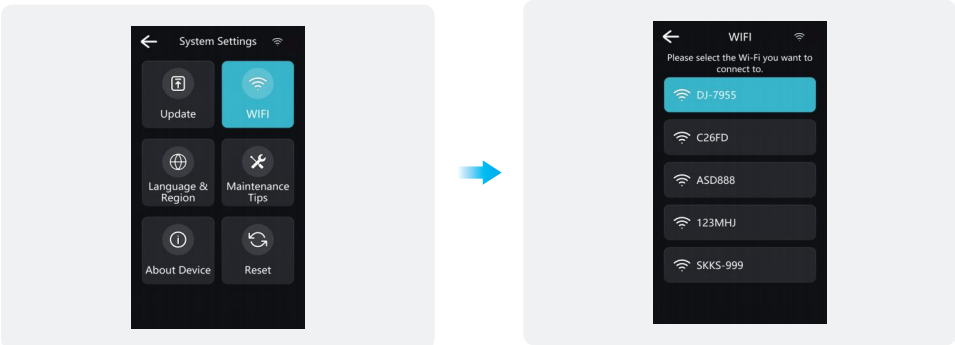In the main interface of the device, go to “Settings” → “System Settings” → “Wi-Fi”, select the Wi-Fi you want to connect to, and network the device.

Piocreat BOX

Download the installation package from www.piocreat3d.com
Read the installation package from a random USB flash drive.

Tips: This product supports CHITUBOX Dental. Contact customer service to receive a free 3-month CHITUBOX Dental slicing software license.

PioNext Ultra Technical Specifications

PioNext Ultra		
Body Parameters	Printing Size	211.68mm×118.37mm×100mm
	Machine Size	468mm×286mm×470mm
	Package Size	600mm×370mm×585mm
	N.W.	19.78kg
	G.W.	23.67kg
Printing Parameters	Printing Technology	LCD
	Light Source Wavelength	385nm、405nm
	Resolution	16K (15120*6230px)
	Pixel Size	14*19μm
	Layer Thickness	0.01-0.1mm
	Printing Speed	Max.60mm/h (0.1mm)
	Touch Screen	5-inch Capacitive Color Touch Screen
System Parameters	Slicing Software	Piocreat BOX
	Data Transmission	USB/WIFI
	Input Voltage	100-240V~, 50/60Hz
Supported Consumables	Power	660W
	Removable Die Model Resin / Ortho Model Resin / Ortho Model 2.0 Resin / Dental Casting Resin / Surgical Guide Resin / Surgical Guide Pro Resin / Temporary Restoration Resin / Denture Base Resin / Crown & Bridge Resin and other official authorized resin	

IC ID Requirement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference;
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

The device meets the exemption from the routine evaluation limits in section 6 of RSS 102 and compliance with RSS-102 RF exposure, users can obtain Canadian information on RF exposure and compliance.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter. This equipment should be installed and operated with a minimum distance of 20 centimeters between the radiator and your body.

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FCC Requirement

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference;
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20cm between the radiator & your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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