VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

Full Body Lift Sling

MODEL: 13223M, 13221M,
13223L, 13223XL, 13222M, 13222L,
13224XL, 30132, 13221L, 13221XL

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

Full Body Lift Sling

MODEL: 13223M, 13221M, 13223L, 13223XL, 13222M,
13222L, 13224XL, 30132, 13221L, 13221XL

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Full Body Lift Sling
Please read all of the instructions before using it. The information will help you achieve the best possible results.

WARNING:

Do not use your Vevor sling without first reading and understanding the operator's manual supplied with your Vevor sling. If you are unable to understand the warnings or instructions contact a healthcare professional before attempting to use your sling. Using your sling without properly understanding the instructions and the warnings can cause injury to the patient and/or caregiver, and cause damage to the sling.

 **WARNING:** If any part of the sling is damaged or missing, contact Vevor customer service immediately.

 **WARNING:** Carefully inspect the sling before each use and after each laundering for wear and damage to seams, fabric, straps, and strap loops. Torn, cut, frayed or broken slings can fail, resulting in serious personal injury to the user or caregiver. Use only slings that are in good

condition. Discard and destroy old, unusable slings.

 **WARNING:** DO NOT exceed the patient lift or the patient sling's maximum weight capacity.

 **WARNING:** Notice for California Customers—California Proposition 65
—This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

 **WARNING:** Vevor assumes no responsibility for any damage or injury caused by improper application or use of this product.

 **WARNING:** Never leave a patient unattended while in a Patient Sling

 **WARNING:** Never attempt to re-position a patient by pulling on the sling loops.

 **WARNING:** Check the sling for shipping damage before using. In case of damage, DO NOT use the sling. Contact Vevor Customer Service for further instructions.

General Guidelines for Using the Sling:

Use an appropriate patient sling that is recommended by the individual's

doctor,nurse,or medical assistant for the comfort and safety of the individual being lifted and their specific medical need.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion)between the patient and any patient sling as this may cause the patient to slide out of the sling during transferring.

The loops of the sling are color coded and can be used to place the patient in various positions.The colors make it easy to connect both sides of the sling equally.Make sure that there is sufficient head support when lifting a patient.

USING A ONE-PIECE STYLE FULL BODY PATIENT SLING:

Positioning the Sling:

Note:Please make sure the patients weight does not exceed the weight capacity for the patient sling or lift before you attempt to lift the patient.

NOTE:For this procedure,refer to FIGURE 1.1 and FIGURE 1.2

NOTE:Use the following method to easily move the patient and avoid strain to the caregiver.

1.Position the patient in the center of the bed and lying flat

on his/her back.

2. Fold the sling in half (length-wise) and place the sling beside the patient. Refer to Detail "A".

NOTE: The closed-end or commode opening of the sling with positioning handle should be facing the patient when folded.

NOTE: The top edge of the sling fabric should be slightly above the patient's head. The bottom edge of the sling fabric should then be a few inches above the back of the patient's knees.

3. With one assistant holding the patient, the second assistant pushes the folded sling under the patient without rolling him/her over. Refer to Detail "B".

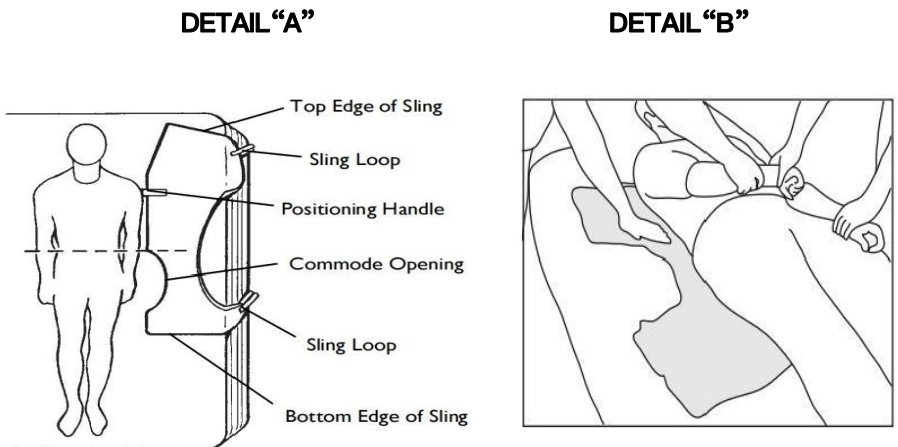
FIGURE 1.1 Positioning the Sling

4. With an assistant on each side of the bed and up against the mattress, the

assistant on the left-hand side of the bed will position his/her right-hand on the elevated knee and his/her left-hand under the patient's right shoulder, slowly push on the knee and assist with a slight lift of the shoulder. The patient will easily roll onto his/her side. Refer to Detail "A" below in **Figure 1.2**.

1. With the patient on their side, push the fabrics of the seat and back gently under them. Refer to Detail "B".

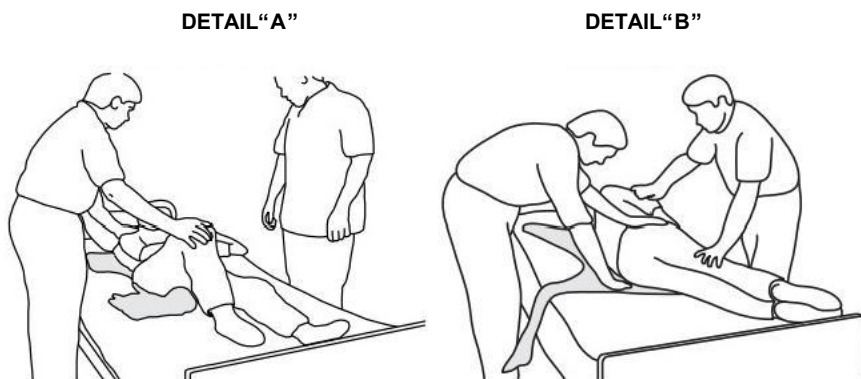
FIGURE 1.2 Rolling Patient Onto Their Side

NOTE: The patient's head should be positioned in the headrest just below the top edge for maximum comfort and the lower edge of the seat section positioned a few inches above the back of the patient's knees.

2. Roll the patient on to his/her back.

NOTE: Assistants will reverse roles.

3. After the patient has been positioned once again on his/her back, you now need to roll the patient to their right-side (facing the assistant on the left).

4. With an assistant on each side of the bed and up against the mattress, the assistant on the right-hand side of the bed will elevate the left knee and position his/her left-hand on the elevated knee and his/her right-hand under the patient's left shoulder, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side. Refer to Detail "A".

5. Pull the fabrics of the seat and back across the mattress until they are smooth.

6. Roll the patient onto their back and they should be approximately centered on the sling.

7. Position the lift for use.

Attaching the One-Piece Style Sling to the Patient Lift

NOTE: For this procedure, refer to FIGURE 1.3.

1. Place the straps of the sling over hooks of the swivel bar. Match the corresponding colors on each side of the sling for an even lift of the patient.

CAUTION:

When connecting sling to the patient lift, the shortest of the straps **MUST** be at the back of patient for support. Using long section will leave little or no support for patient's back. The loops of the sling are color coded and can be used to place patient in various positions. The colors make it easy to connect both sides of the sling equally. Make sure that there is sufficient head support when lifting a patient.

Note: Spreader bars have up to three hookup points per side. The middle hookup is **ONLY** used for slings that have three sets of straps per side or slings that use chains or adjustable straps.

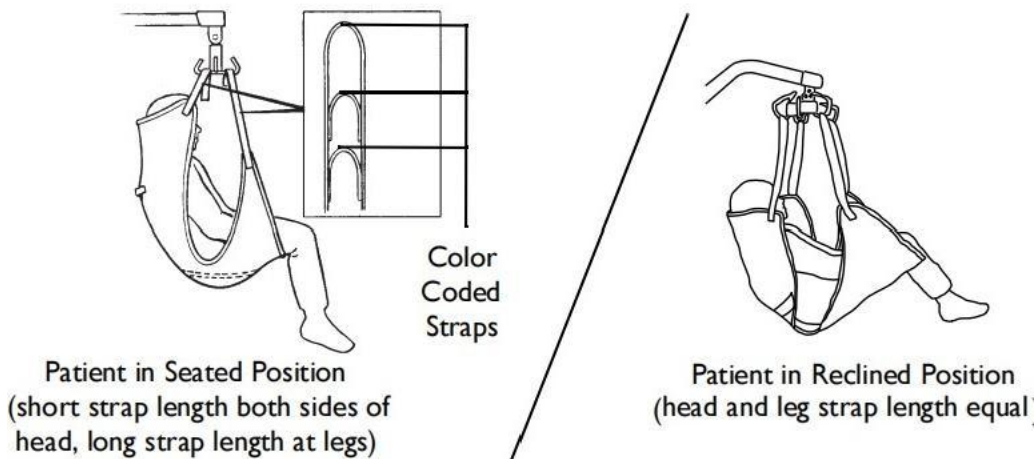
FIGURE 1.3 Attaching the One-Piece Style Sling to the Patient Lift

WASHING INSTRUCTIONS: Machine wash in warm or cold water. Maximum water temperature of 120°F(48°C). Air dry or tumble dry at cool or very low temperature. **DO NOT** use bleach or wash with other colors. Inspect after each wash. **DO NOT** dry clean.

WARNING: Inspect patient slings for wear prior to each use. If signs of tearing, fraying, or wear are found discard the sling immediately; worn out slings are not safe for use and may result in injury or death. **Do not alter slings in any way.**

LIMITED LIFETIME WARRANTY: Your Vevor sling is warranted to be free from defects in materials and workmanship for a period of one year from the date of purchase. This warranty is extended only to the original purchaser/user of our Vevor Slings. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Vevor's option. This warranty does not include any labor or shipping charges incurred in replacement or repair of any such product. Vevor's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement. Useful life of this product is 6 months from date of purchase under normal use, however, heavy use or excessive washing may reduce the useful life of the product. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear, or incorrect washing and drying at temperatures higher than

120°F(48°C),or failure to adhere to the instructions in this manual.Vevor shall not be liable for any consequential or incidental damages whatsoever.Some states do not allow exclusion or limitation of incidental or consequential damage,or limitation on how long an implied warranty lasts,so the above exclusions and limitations may not apply to you.This warranty shall be extended to comply with state or provincial laws and requirements.If you have any questions about your Vevor sling or warranty service,please contact customer service

Manufacturer: Dongguan Chuangguo Daily Products Co. Ltd

Address: No.169.Kangyi Road, Qingxi Town,Dongguan

City,Guangdong.CN 523657

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD

NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,

Centurion House,

London Road, Staines-upon-Thames, Surrey,

TW18 4AX

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Mainzer Landstr.69, 60329 Frankfurt am

Main.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Produttore: Dongguan Chuangguo Daily Products Co. Ltd
Importato in AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importato negli USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH