


kinetik
WELLBEING



LACTATION
MASSAGER

Instruction Manual

SKU WW-400

CONTENTS

Brand Story	3
Introduction	4
Getting to Know	5
Features	5
Box Includes	5
How to Guide	6
Care & Maintenance	8
Specification	9
Warnings	10
Support	11

BRAND STORY


At Kinetik Wellbeing we are driven by the belief that everyone deserves to take control of their health from the comfort of their own home. Designed with simplicity in mind, our clinically validated devices empower you to proactively monitor and manage your long-term health journey.

We created our Women's Wellness range to empower and support women through every stage of life, addressing everything from menstrual discomfort to menopausal symptoms.

INTRODUCTION

Introducing the Lactation Massager, the perfect, compact device empowering new mums during their breastfeeding journey.

With 10 soothing vibration modes and 3 gentle heat settings, you can customise the combination to ease clogged milk ducts, reduce breast pain, stimulate milk production, and improve latching.

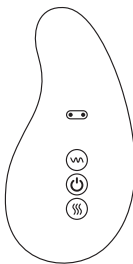


The massager's waterproof design makes it easy to clean and ideal for use in the shower, offering convenience and relief whenever you need it.

Designed for portability and crafted from soft silicone, the Lactation Massager is shaped to fit your breast perfectly, making breastfeeding

and pumping easier and more enjoyable for both you and your baby.

GETTING TO KNOW YOUR LACTATION MASSAGER

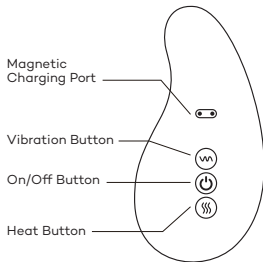

Features:

- Helps to stimulate and improve milk supply
- Helps to improve clogged ducts
- Breast pain relief
- Rechargeable & reusable
- 10 vibration modes
- 3 heat temperature modes

Box Includes:

- Lactation Massager
- USB Charging Cable
- Instruction Manual

HOW TO GUIDE

Switching the Lactation Massager On/Off

Press the On/Off switch and hold for 2 seconds, a red light will light up, indicating that the Lactation Massager is on. To turn it off press and hold the button for 2 seconds.



Using the Vibration Mode

After switching on the device, press and hold the vibration button for 2 seconds, a red light will light up, indicating that the vibration setting is on. Press the vibration button in order to switch between modes. To turn off the vibration mode, press and hold the vibration button.



Using the Heat Mode

After switching on the device, press and hold the heat button for 2 seconds, a red light will light up, indicating that the heat setting is on. It will take a few

minutes to heat up. Press the heat button in order to switch between modes. Press and hold to turn off the heat mode.

Unclogging milk ducts

Turn on the device using your preferred vibration and heat setting. Use the thinner end and apply light pressure to the clogged ducts.




Improving milk supply

Turn on the device using your preferred vibration and heat setting. Use the length of the Lactation Massager and rub breast towards the nipple.



Place the Lactation Massager in your bra before breastfeeding or pumping.

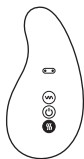

Charging

Place the magnetic charger on to the Lactation Massager port above the buttons.

Plug the USB side into an appropriate plug and power socket and switch it on.

CARE & MAINTENANCE

- Ensure you clean the Lactation Massager after every use.
- Wipe with a soft cloth.
- Avoid direct sunlight or excessive heat.

SPECIFICATION

Product Model : WW-400

Material : Liquid Silicone, BPA-free

Features : Warming & Vibration

Size : 4.85 x 2.26 x 1.11 inch

Waterproof : IPX7 Rated Design

Vibration : 10 Vibration Modes

Heating : 3 Heat Levels Adjustable
(45°C-50°C-55°C)

Charging : Rechargeable
USB Magnetic Charging

Working time: 60 minutes

Charging Voltage: 5V

Battery Voltage: 3.7V

Battery Capacity: 700mAh (Li-ion battery)



WARNING

- Please read the instruction manual carefully before use and keep it for future reference to ensure the Lactation Breast Massager is used safely.
- Charge for a full cycle before use. Only use the USB cable provided.
- Keep away from children.
- Check the device is not damaged before use, disposing of it if damaged.
- Do not use while sleeping.
- Do not use on sensitive skin or open wounds.
- Do not drop the device.
- The appliance should be used with caution by people with reduced physical or sensory capabilities where reduced heat perception could lead to the user unknowingly burning themselves.
- If anything goes wrong with your device, do not attempt to repair it yourself. Stop using it immediately and contact the seller you bought the device from.

SUPPORT

Please feel free to contact us at:



+44 (0) 1483 937 967



customer care@kinetikwellbeing.com



www.kinetikwellbeing.com



Kinetik Medical Devices Ltd
Unit 11, Perrywood Business Park
Honeycrook Lane
Salfords, Redhill
RH1 5JQ
United Kingdom



Share Info GmbH
Heerdter Lohweg 83,
40549 Düsseldorf