VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

SICK BED

MODEL:V8V8C-CPR

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us.

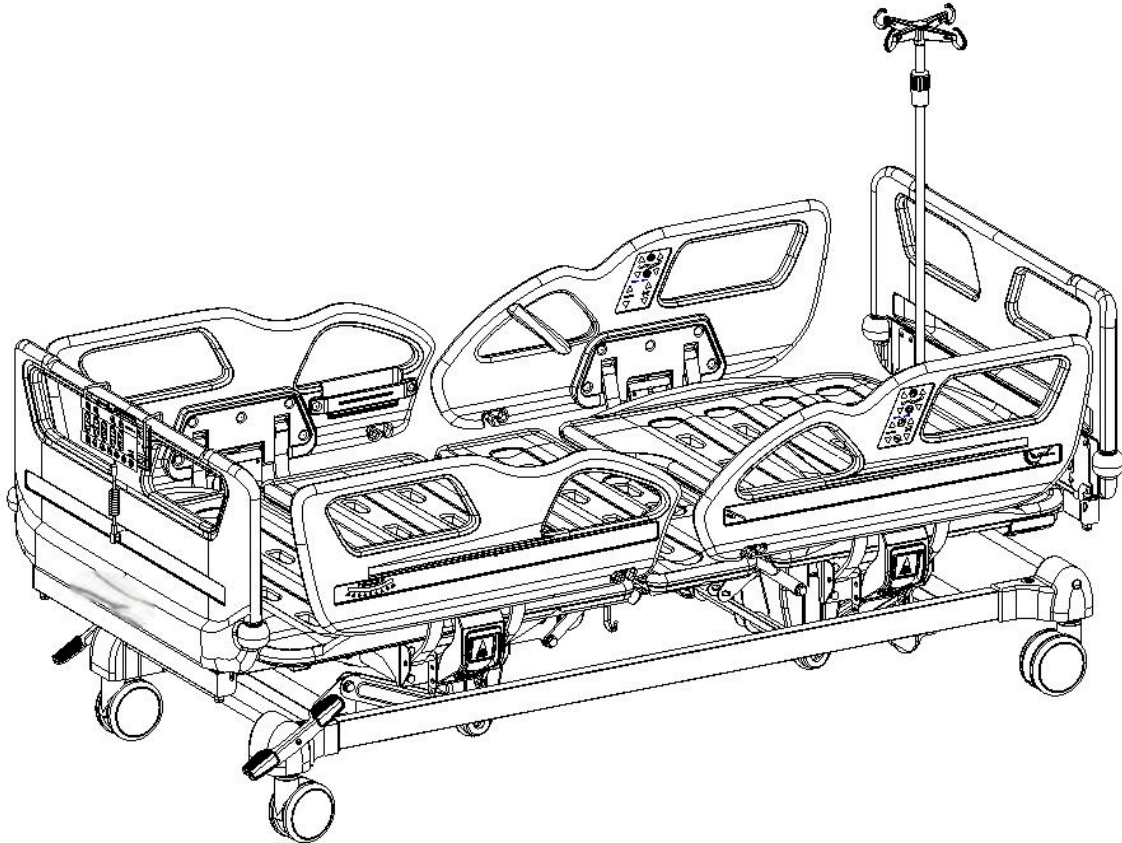
You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

SICK BED

MODEL: V8V8C-CPR

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate

www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Overview

This manual is applicable to V8v8c electric hospital bed, and includes installation method, usage and maintenance instructions, etc.

The graphics in this manual are for user reference only, please refer to the actual product during operation.

This manual includes product description, instructions, maintenance and some technical data and information.

If you have any questions, please contact our after-sale service or your distributor.

Anyone using this manual should read it carefully and strictly follow the instructions.

Except for some unexpected situations, we require users to be familiar with usage guides and matters needing attention.

1. Product installation guide

1.1 List of standard accessories

After opening the packing case, please check whether the following parts are complete:

Table a:


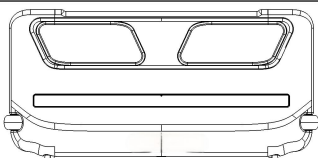
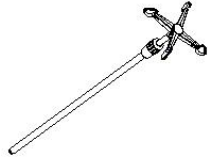
No	Name	Picture	Specification	Qty	Unit
1	Head foot board packing box		check table a.1 below	1	box
2	IV pole		Φ16-19/off-white	1	pc
3	Bed platform		2048*1047	1	pc
4	Screw pack	/	Check table a.2 below	1	bag

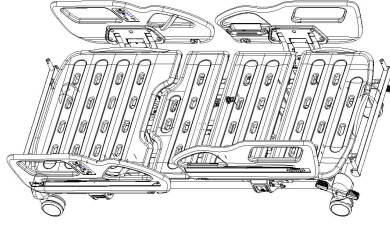Table a.1

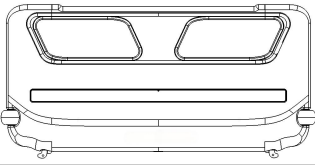
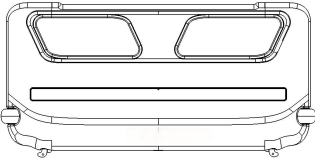
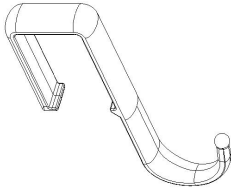
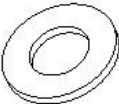
No	Name	Picture	Specification	Qty	Unit
1	Head board		990*460	1	pc
2	Foot board		990*460	1	pc



Table a.2



No	Name	Picture	Specification	Qty	Unit
1	30-50 drainage hook		30-50(plastic)	2	pc
2	Pole 154		φ21.7*154	2	pc
3	Pole 184		φ21.7*184	2	pc
4	Flat washer		φ12	4	pc
5	Jam nut		M12	4	pc

1.2 Head and foot board installation

(the picture does not exactly match the real object, for installation guide reference only)

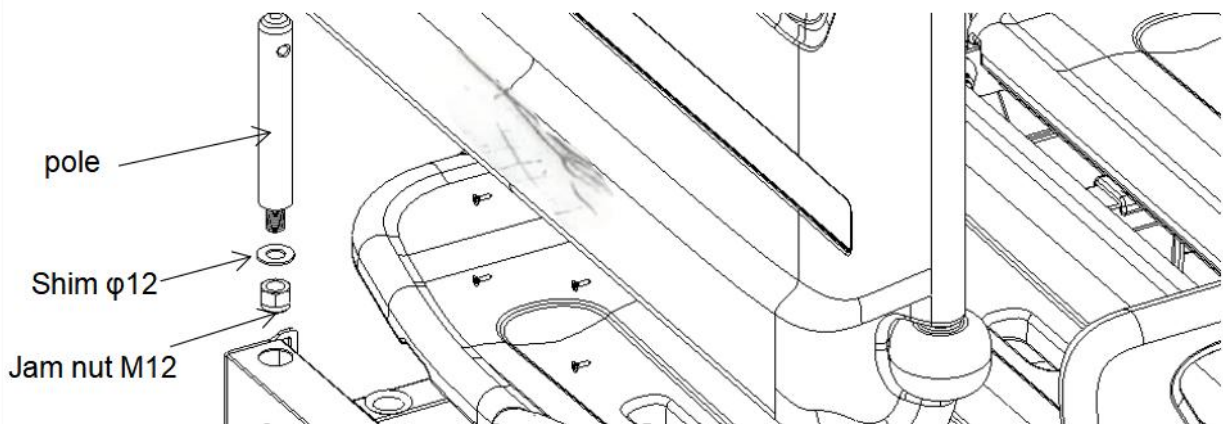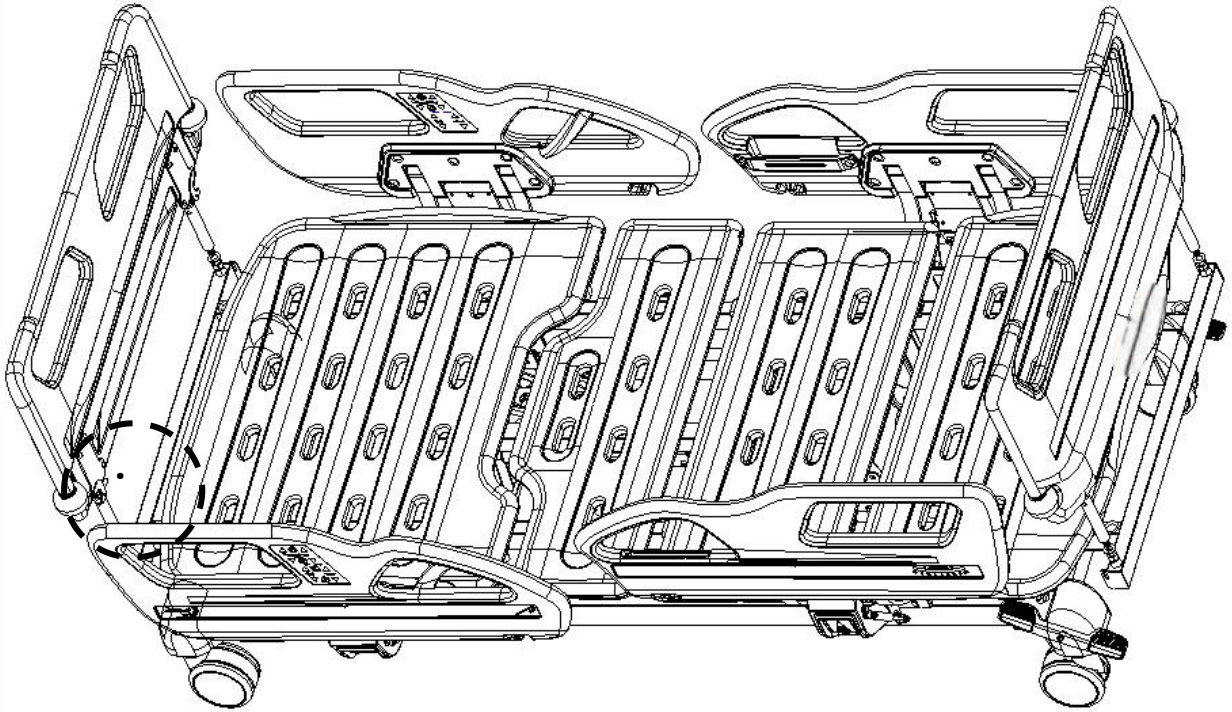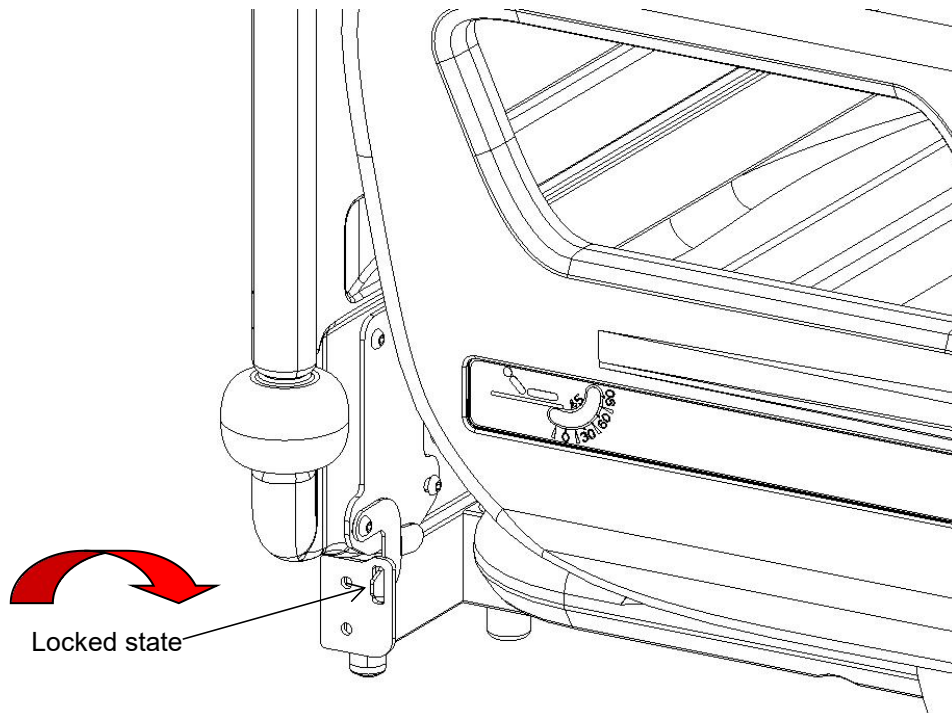
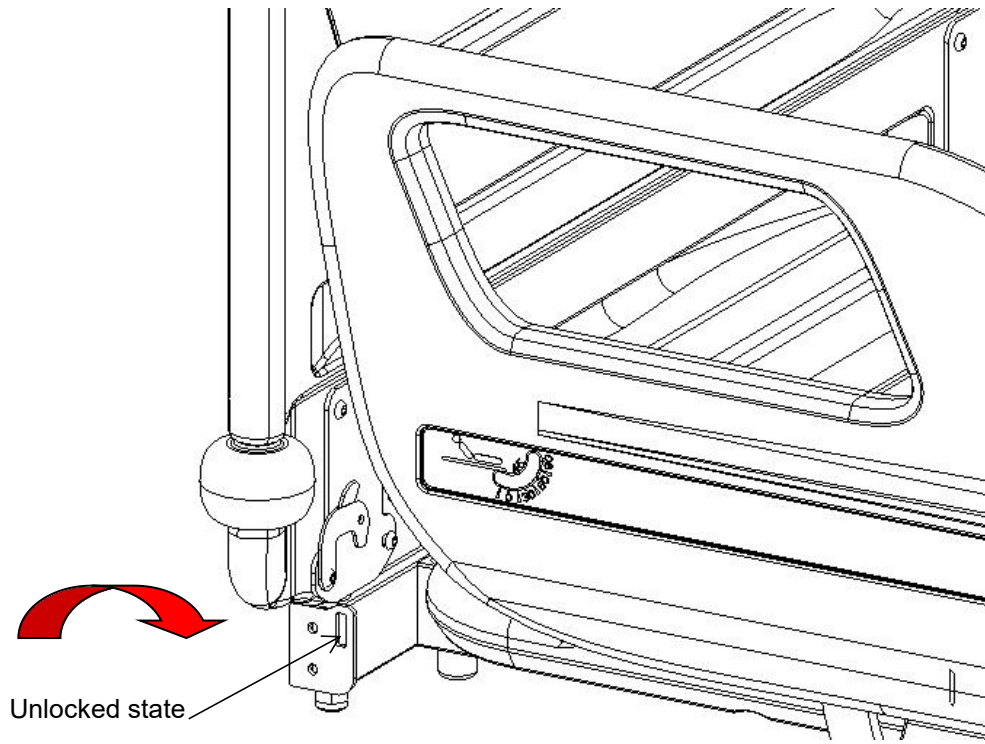
Head board poles: $\varnothing 21.7 \times 154$, foot board poles: $\varnothing 21.7 \times 184$, the installation method is as shown in the figure below:

1.3 Drainage hook installation

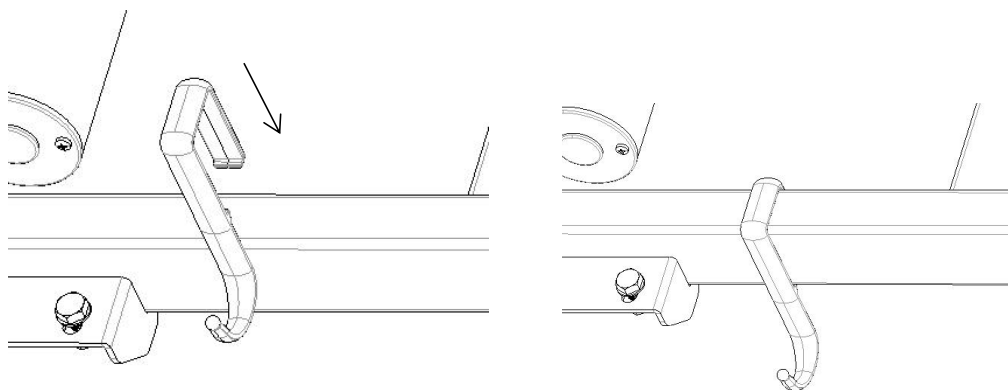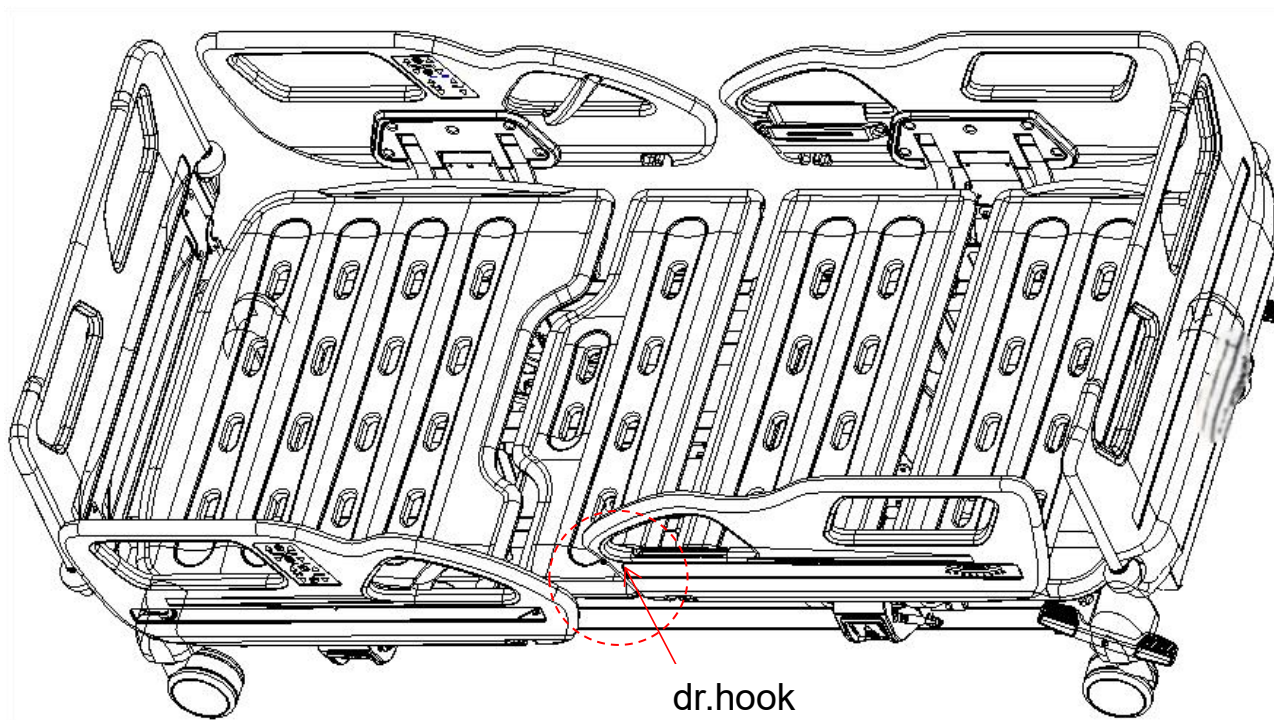Prepare drainage hook, turn the concave buckle of the drainage hook downwards, then buckle it down on the beam of the bed frame, see picture:

1.4 IV pole installation

Primary part installation: place the primary plastic hook part on top of the 16mm Ø stainless steel pole and press it down.

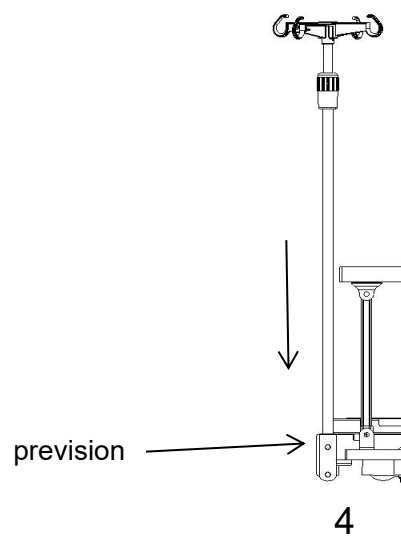
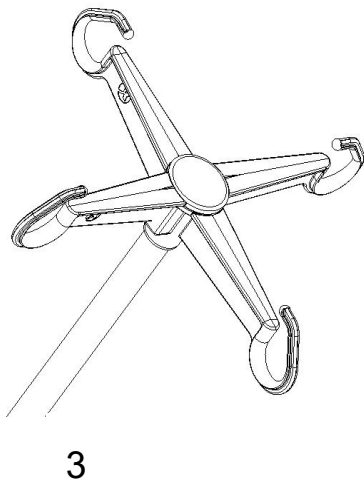
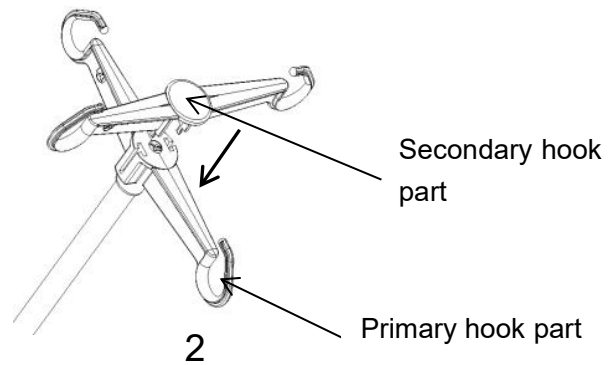
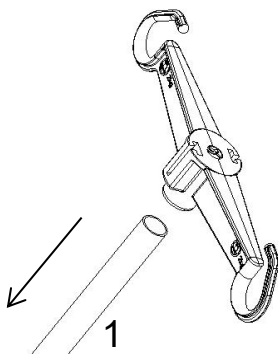
Secondary part installation: place the secondary plastic hook part on top of primary, align the secondary hook part's snap with the primary's socket and press it down, after you hear a "click" sound, check if the clip is tight, if so, the installation is completed, if not, continue to press down. Place IV pole vertically in one of the IV pole provisions.

IV pole prevision 1


IV pole prevision 3

IV pole prevision 2

IV pole prevision 4

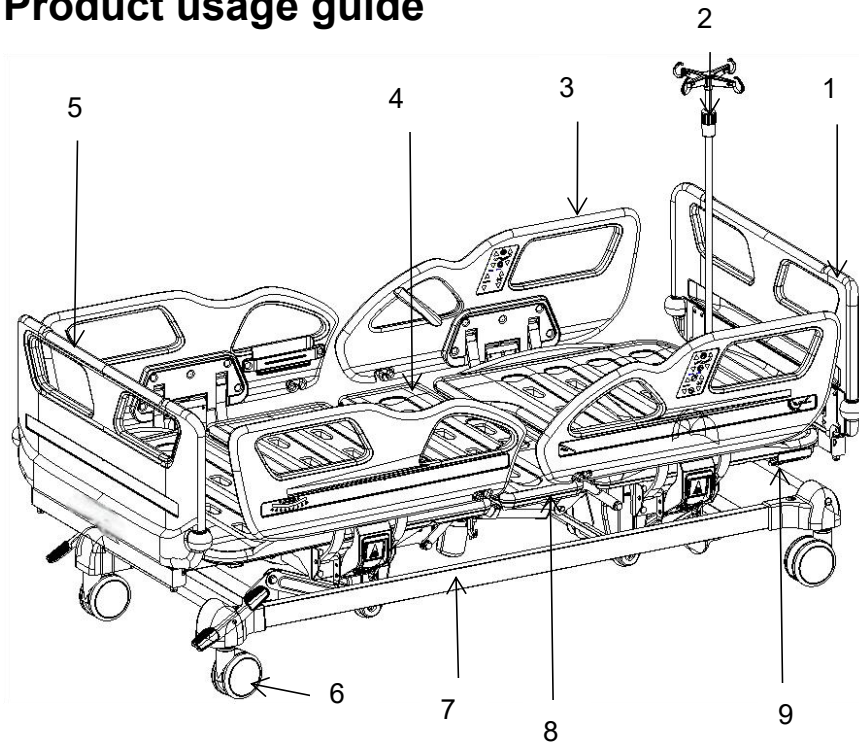2. Product usage guide

1- Head board 2- IV pole 3- Side rail 4 - Bed board

5- Foot board 6- Caster 7- Bed frame 8- Drainage hook 9- CPR


Technical parameters:

1	Bed platform dimension	length	mm	1925±10
		width		900±10
		height		455±10
2	Overall dimension	length		2085±10
		width		1025±10
		height		(455-745) ±10
3	Tilt range	back rest	/	0 ~ 75±10°
		knee rest		0 ~ 32±10°
		reverse trendelenburg		0 ~ 12±2°
		trendelenburg		0 ~ 12±2°
4	Weight	Safe working load	/	170


2.1 Casters usage

In this product the central locking system is used, which can lock all four wheels by controlling the brake pedal on one side. Press the green pedal down to unlock casters, press the red pedal down to lock casters.

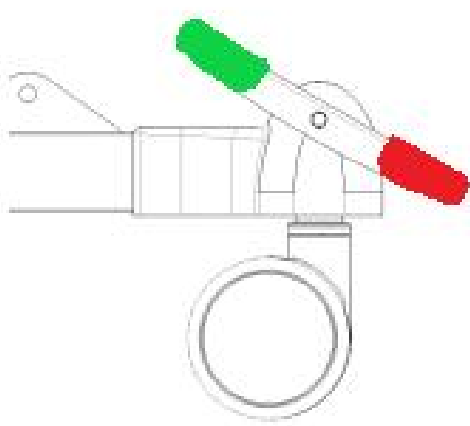The figures below show two states of the wheels:

Locked state

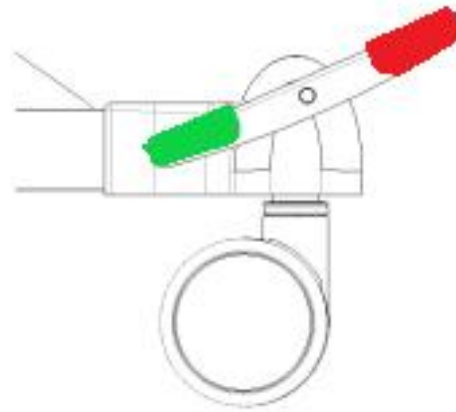
Unlocked state


2.2 Side rail usage

(The picture is for reference only, the actual product shall prevail.)

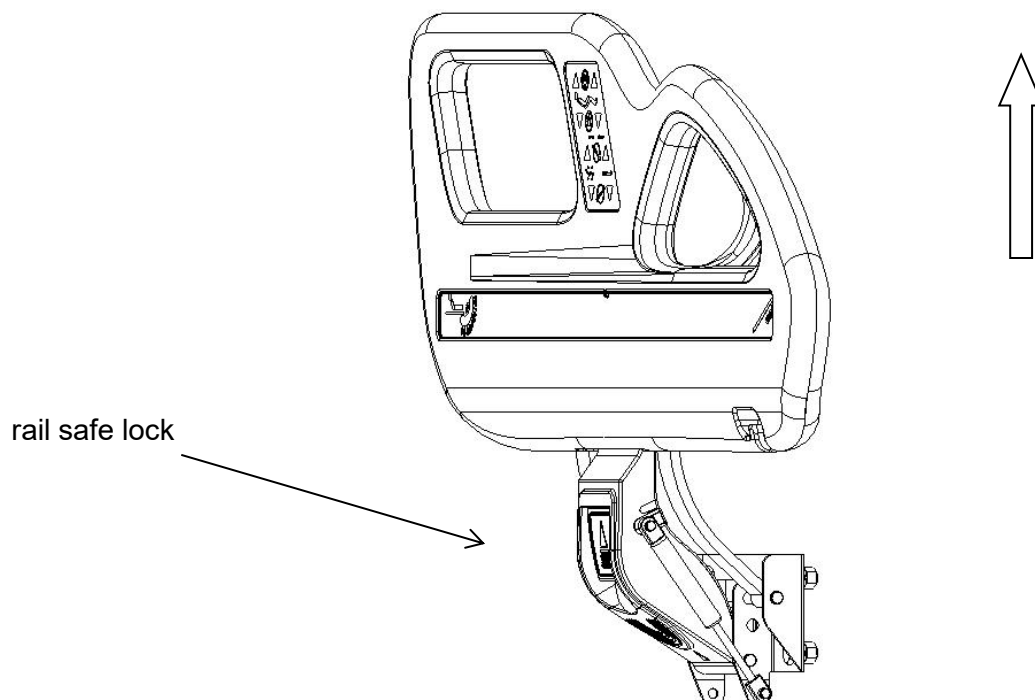To raise side rail: hold the rail and pull it up, when you hear “click” sound means the rail is locked, see the pic:


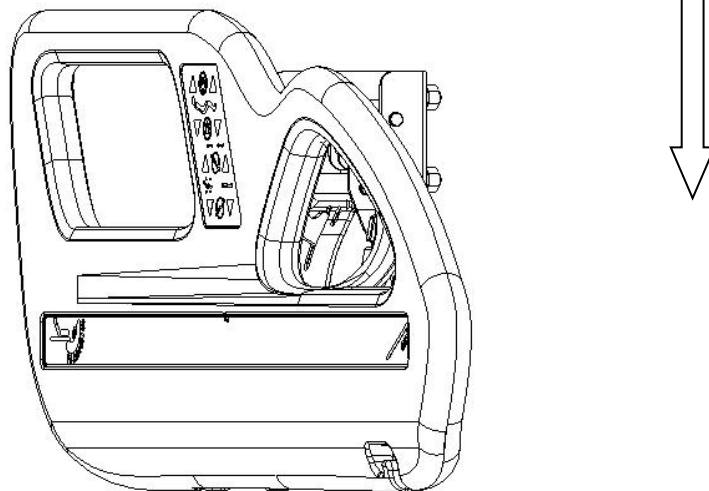
To lower side rail: unlock the rail, slowly lower it down, see the pic:

2.3 CPR usage

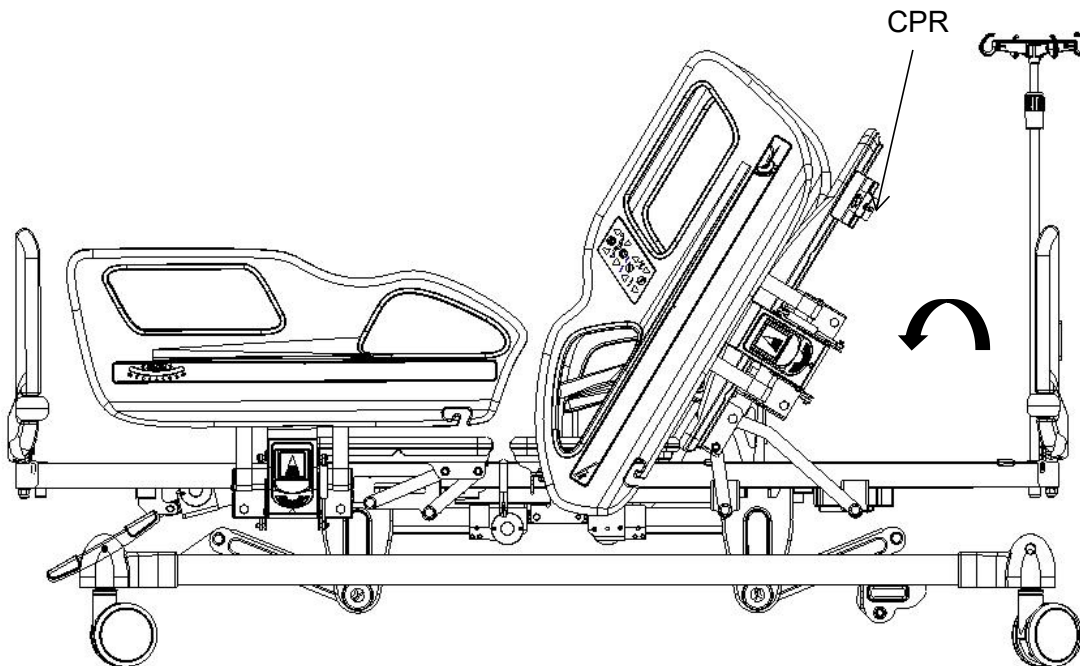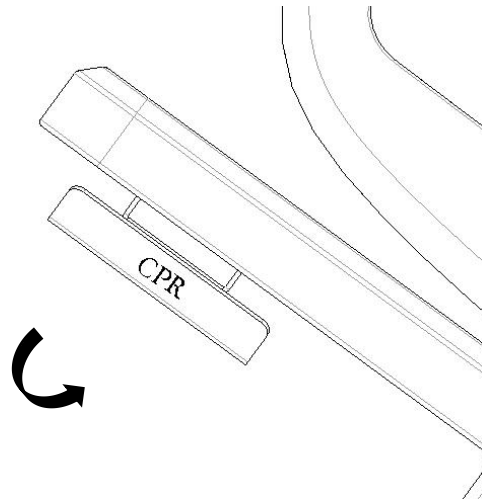The back rest of this bed can be instantly lowered during emergency treatment. Operation method: pull the red CPR crank upward to realize the rapid lowering of the back rest board.


Back rest instant release



2.4 IV pole usage

IV pole height adjustment: first unscrew the outer rotating sleeve, counterclockwise, insert the inner tube of the IV pole into the rotating sleeve, and then adjust the IV pole height as needed. After adjusting the required height, tighten the outer rotating sleeve, clockwise. Safe working load of each individual hook is 2kg, total safe working load is 8kg.

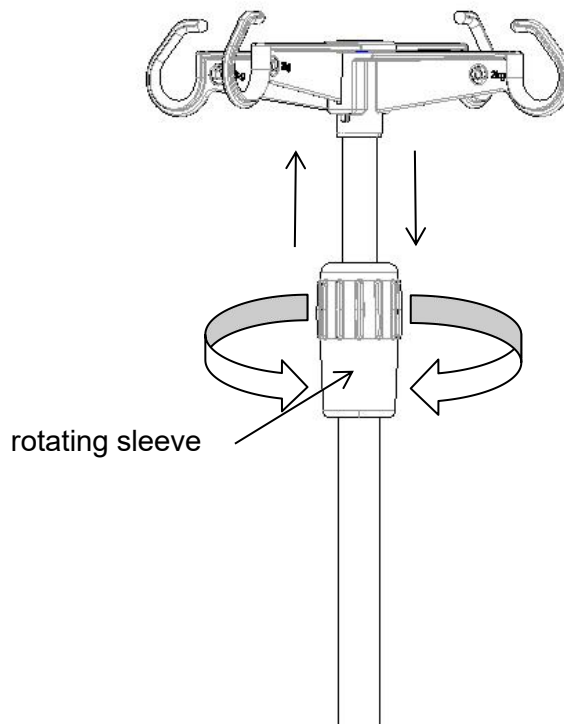
(height adjustment range: 890-1540mm, the picture is for reference only)

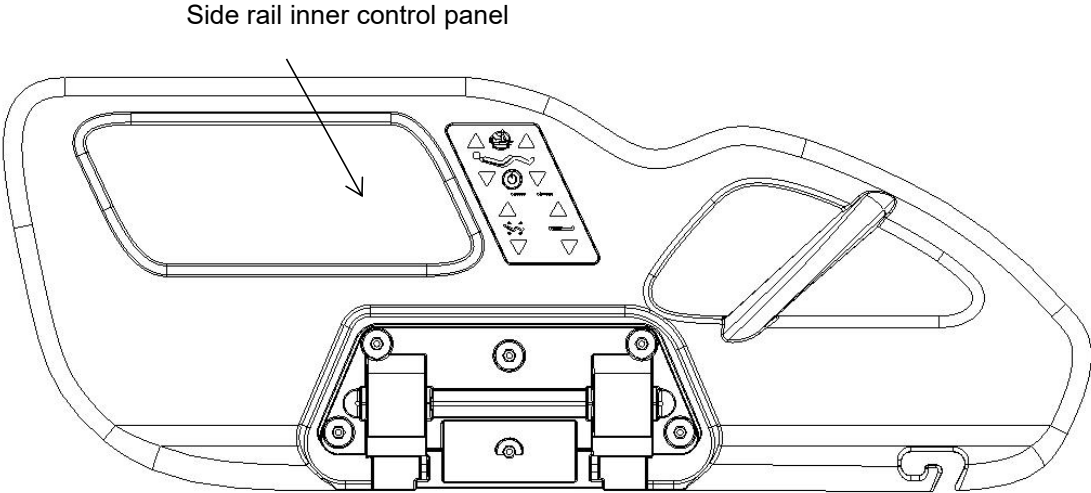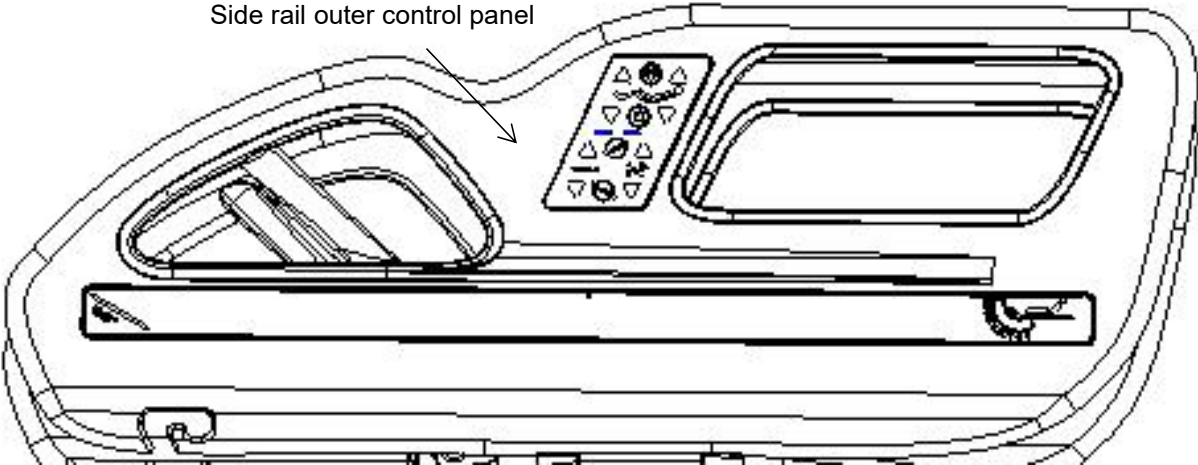2.5 Side rail control panel usage

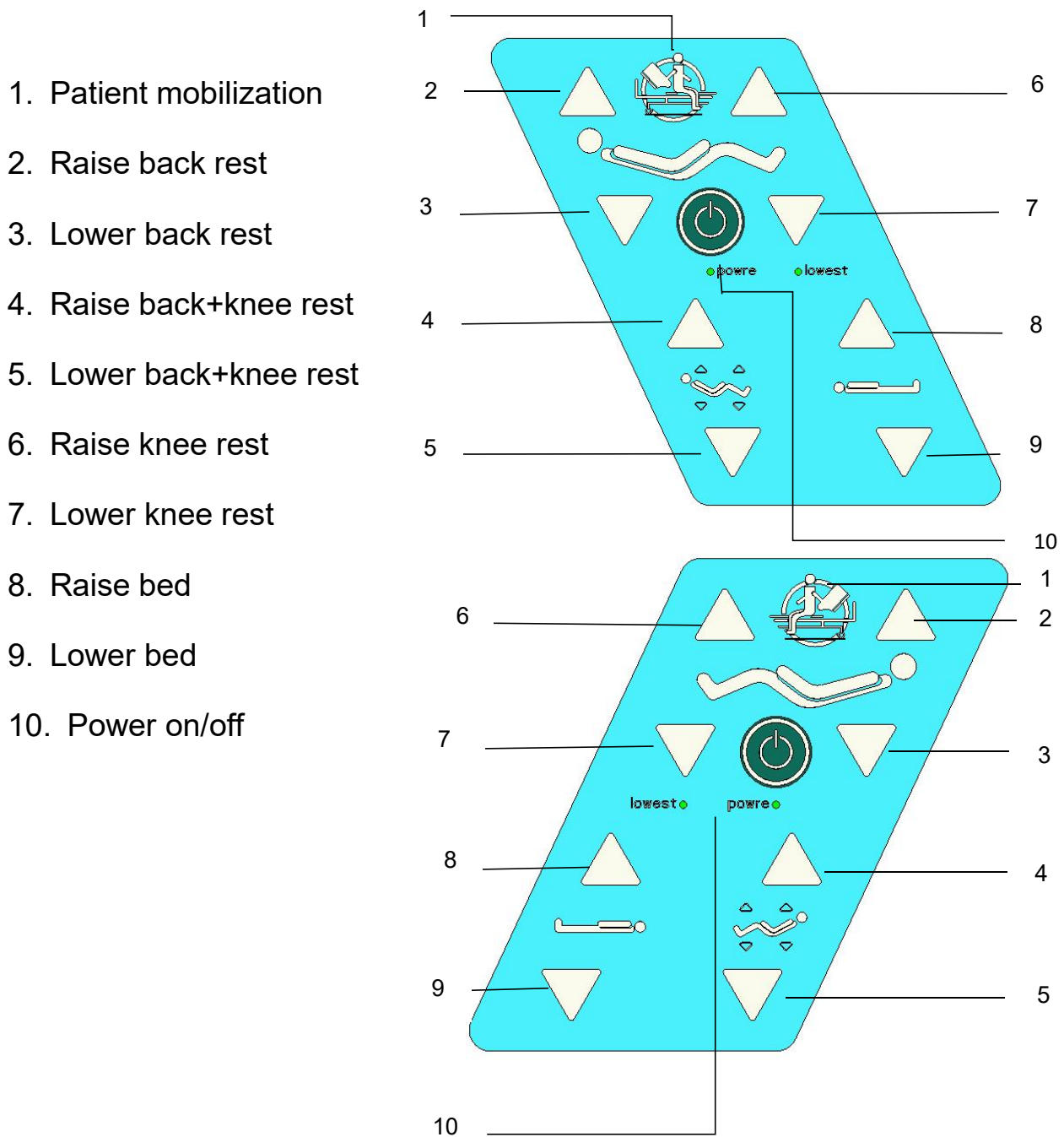2.5.1 Side rail inner control panel

2.5.2 Side rail outer control panel

1. CPR

2. Raise back rest

3. Lower back rest

4. Power on/off

5. Raise back+knee rest

6. Lower back+knee rest

7. Trendelenburg

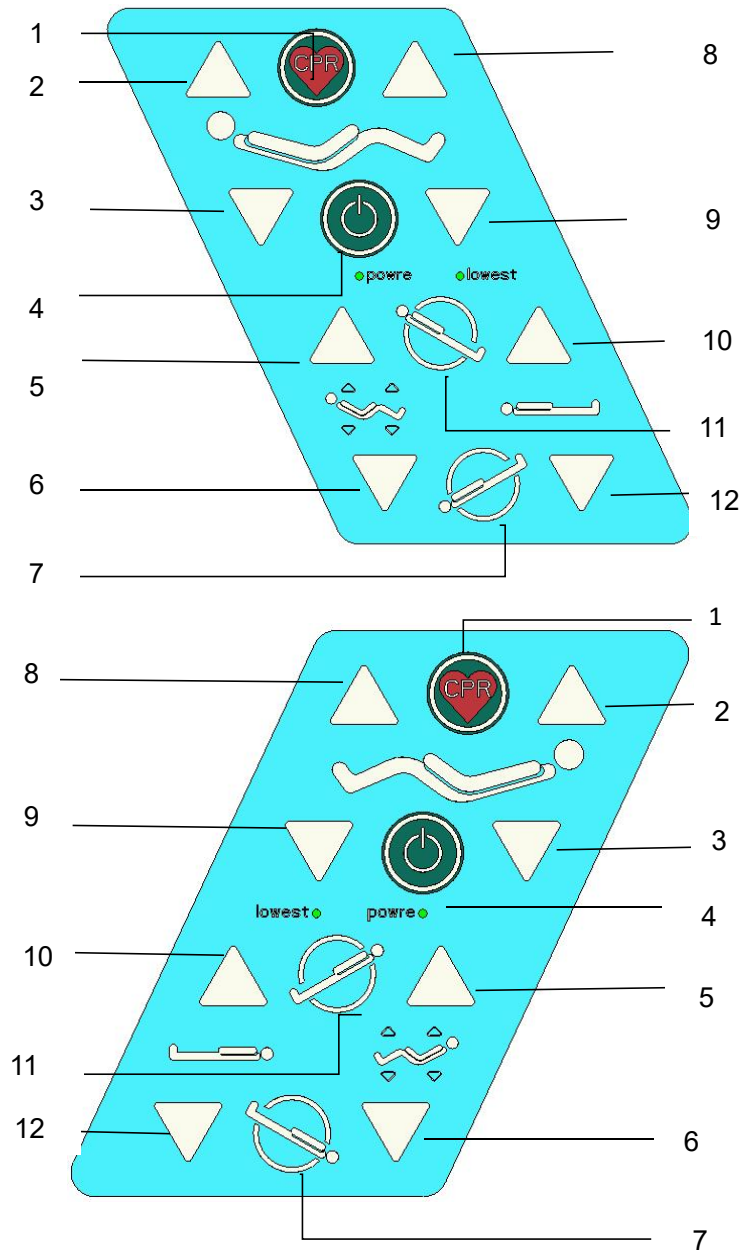
8. Raise knee rest

9. Lower knee rest


10. Raise bed

11. Reverse Trendelenburg

12. Lower bed

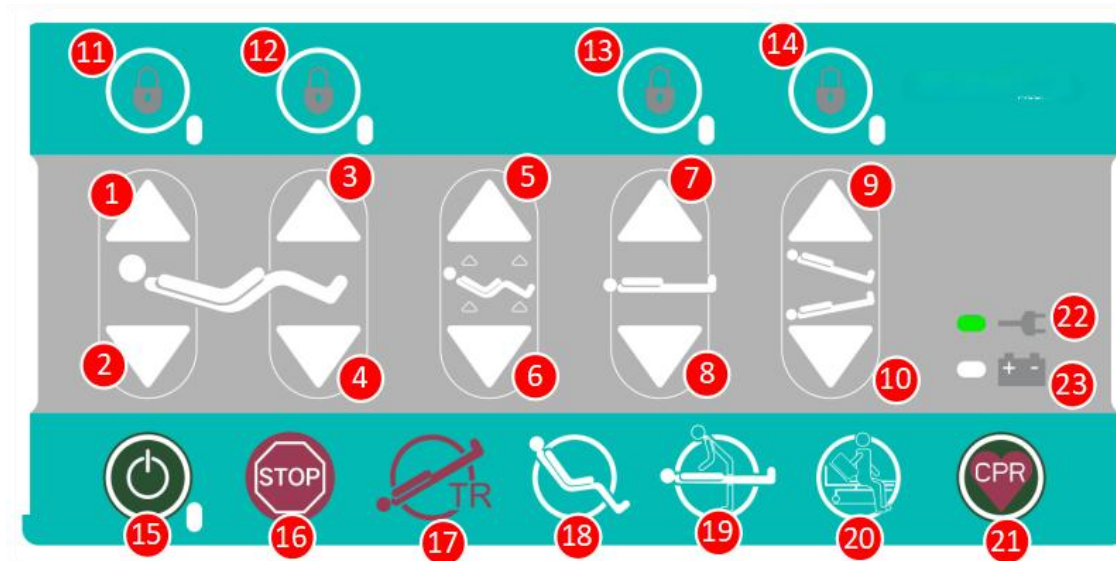
2.6 Nurse control panel usage

- | | |
|--------------------------|--------------------------|
| 1. Raise back rest | 14. Lock Trendelenburg |
| 2. Lower back rest | 15. Power on/off |
| 3. Raise knee rest | 16. Emergency stop |
| 4. Lower knee rest | 17. Trendelenburg |
| 5. Raise back+knee rest | 18. Cardiac position |
| 6. Lower back+knee rest | 19. Nursing position |
| 7. Raise bed | 20. Patient mobilization |
| 8. Lower bed | 21. CPR |
| 9. Reverse Trendelenburg | |
| 10. Trendelenburg | |
| 11. Lock back rest | |
| 12. Lock knee rest | |

2.7 Electronic control system composition

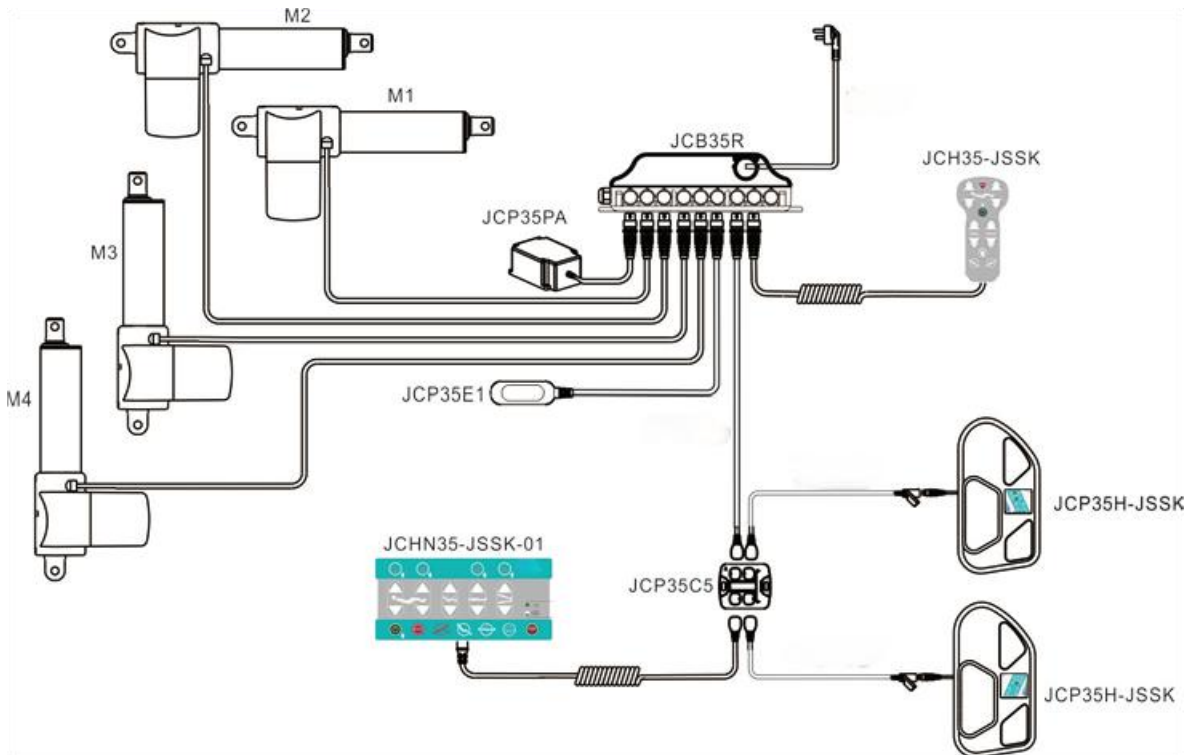2.7.1 Electronic control system diagram

Operator priority:

- (1) Nurse control panel > Hand controller > Side rail control panel;
- (2) One priority button can interrupt and terminated the same priority button operation (except hand controller), when one button is released and another one is still pressed, the operation of the last one is continued;
- (3) High-priority keys (except CPR, TR, and STOP) can interrupt and terminate low-priority keys. When one button is released and another one is still pressed, the operation of the last one is continued. When CPR, TR or STOP button is pressed during other button operation, CPR, TR or STOP operation is executed.

2.8.2 Electronic control parts list


Name	Qty
JC motor 200 vertical(manual CPR)	1
JC knee rest motor 100 horizontal	1
JC bed platform height motor 150 horizontal	2
JC 5-functions control panel	1
JC 3m power cord (environmental)	1

JC 5-functions nurse control panel	1
JC 5-functions hand controller	1
Emergency stop switch box	1
JC night light	1
Adapter box JCP35C5	1
JC battery box	1
JC Adapter cable	2
JC Adapter cable	1
V21 side rail 5-functions control panel, left	1
V21side rail 5-functions control panel, right	1

3. Spare parts replacement and maintenance

3.1 Caster disassembly and replacement

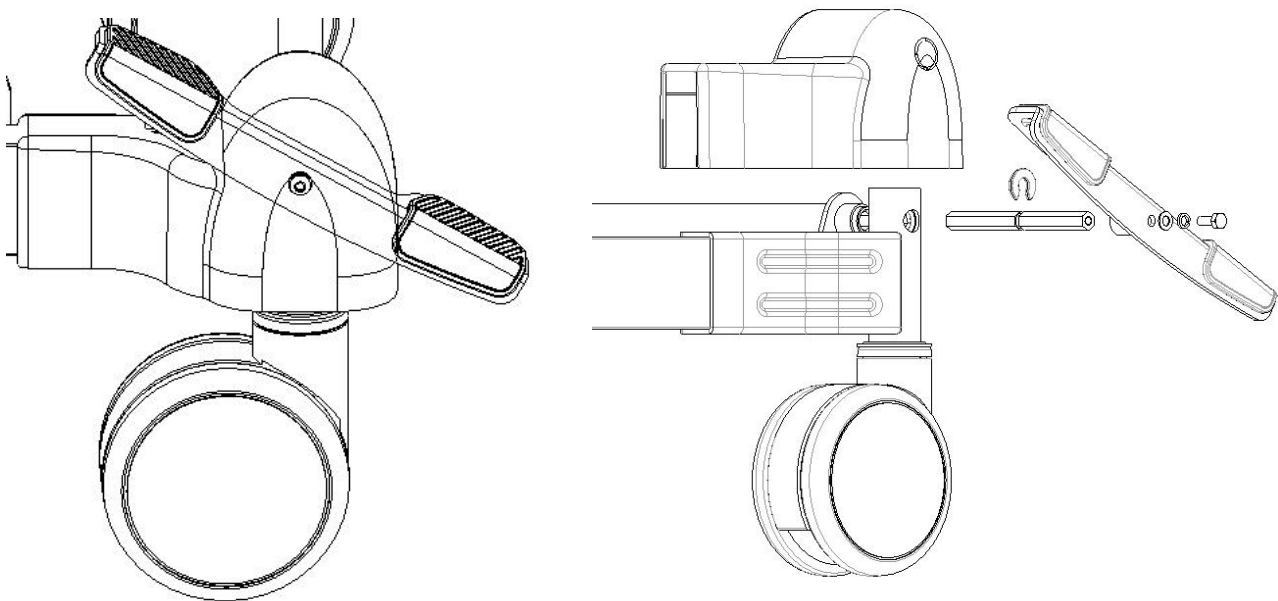
Use 10"inch wrench  unscrew the bolt, pull out the screw, flat spring washers, brake pedal, plastic retaining ring, hexagonal rod, etc, replace with spare caster, place the bolt back into the socket, tighten the nut.

3.2 Side rail gas spring disassembly and replacement


(the picture is for reference only)

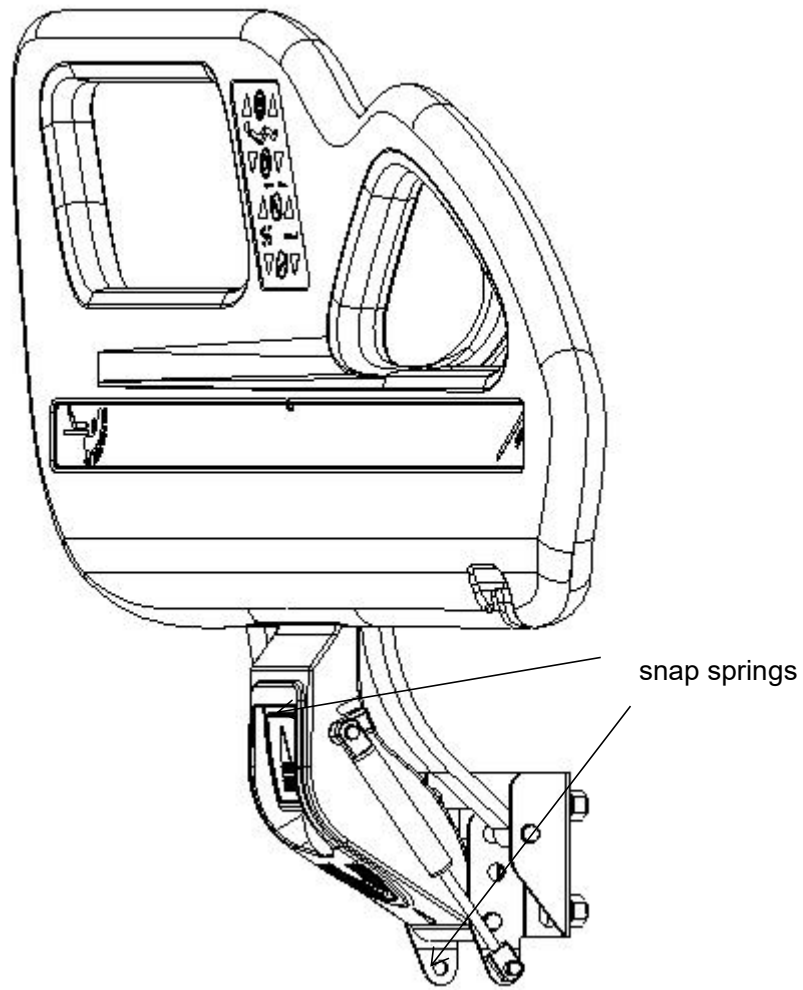Use external calipers to take out the head and tail pins' snap springs, remove the gas spring, replace it with the spare one, reinstall it, test the rail.



3.3 Side rail disassembly and replacement

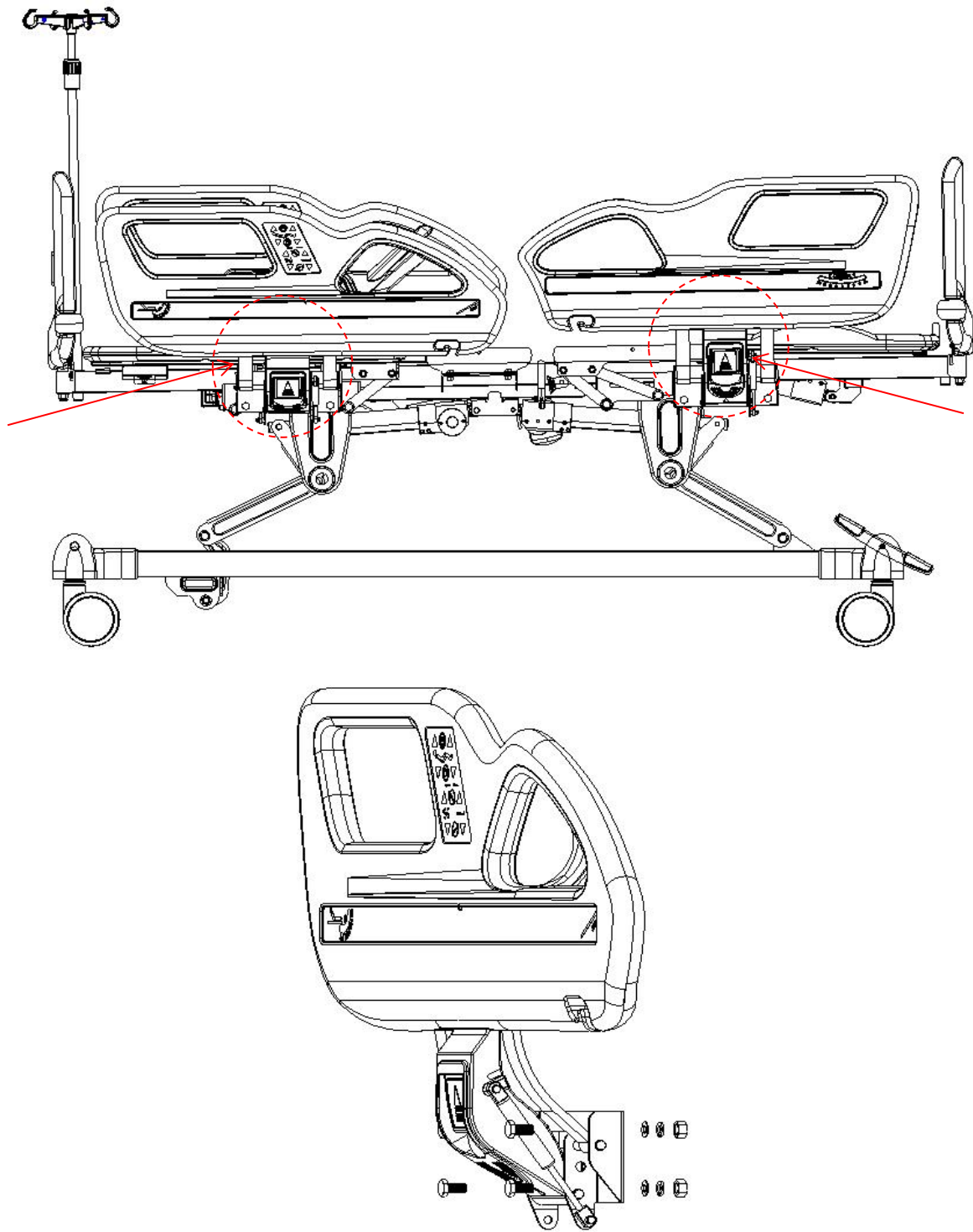First remove the gas spring, using two 14"inch wrenches  loosen the nut, remove the nut, remove the bearing, remove the rail. (The picture is for reference only, the actual product shall prevail)

3.4 Motor removal and replacement

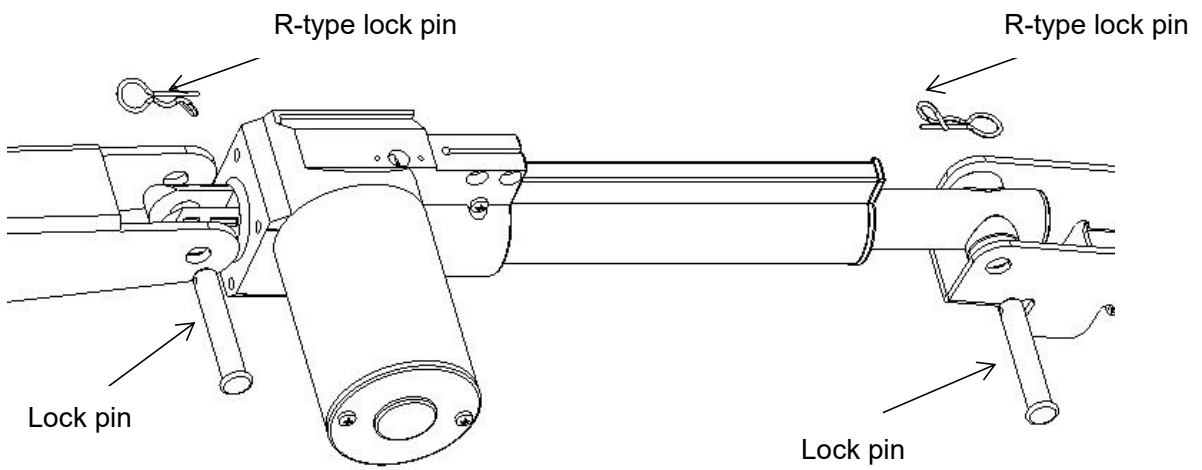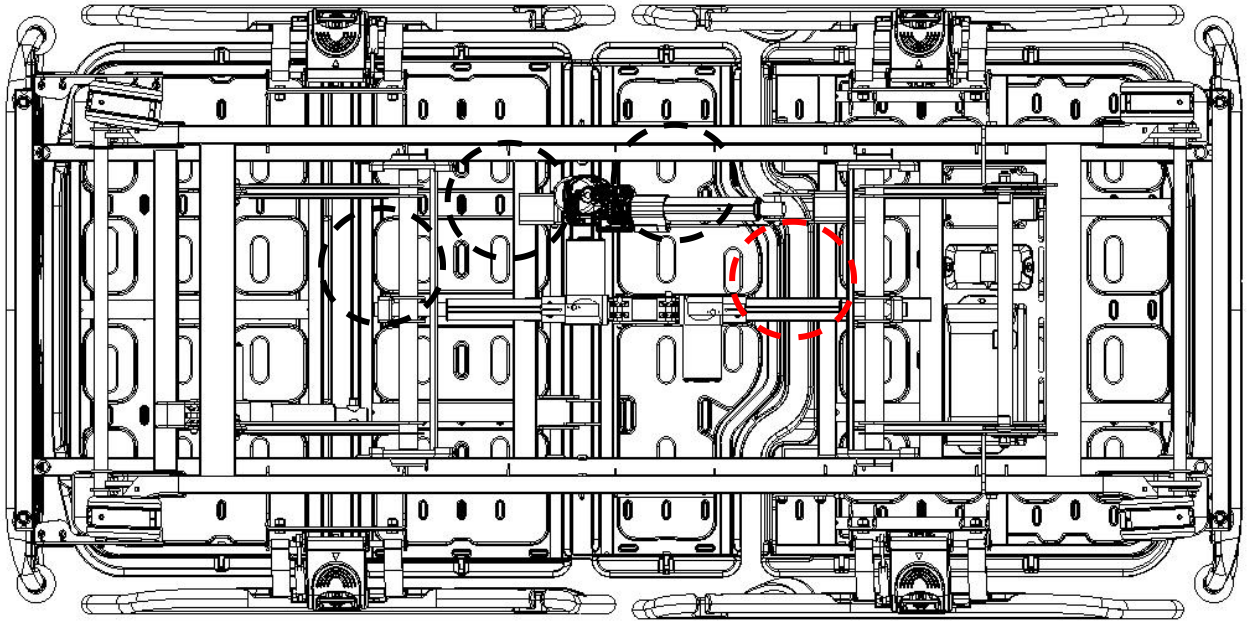After removing the R-type lock pins at both ends of the motor, pull out the lock pins. After motor replacement, install the bolts in the order shown below. (the pic. is for reference only)

4. Troubleshooting

Issue	Possible reason	Troubleshooting
Bed cannot be adjusted using positioning buttons	Did not power on	Press power on button
	Motor is powered off, Motor malfunction	Check power cords Replace motor, see 3.4
	Power plug isn't plugged correctly	Plug in power plug correctly
	Control system malfunction	Contact technical support
Bed height adjustment failure	Motor is powered off	Check power cords
	Motor malfunction	Replace motor, see 3.4
	Power plug isn't plugged correctly	Plug in power plug correctly
	Power box malfunction	Contact technical support
	Control system malfunction	Contact technical support
Back plate cannot be lowered from upright position	Foreign objects under the backplane or in the drive unit	Remove foreign objects
The guardrail cannot be adjusted	Dirty guardrail lock	Clean the locking mechanism
	Side rail safe lock malfunction	Replace side rail, see 3.3
Casters can't turn	The rotating shaft is stuck by strips of dirt	Clean the rotating mechanism
	Malfunction of the internal mechanism of the caster	Replace caster, see 3.1
Unable to install head and foot board	Wrong installation method	Check the locking mechanism, properly position the head and foot board, see 1.2
	Mechanical malfunction	Contact technical support

4.1 Push rod malfunction handling

Push rod malfunction handling:

(1) Causes of push rods malfunction:

1. push rod's abnormal Hall signal;
2. first time programming;
3. nurse desk, hand controller, guardrails control panel 6S system malfunction.

(2) Push rod malfunction prompting method: if the M1, M3, M4 malfunctions are detected, JCH35-JSSK Saikang custom hand controller blue indicator is flashing.

(3) Push rod malfunction troubleshooting:

1. By pressing corresponding buttons, raise the M1 push rod to the top position or lower it to the bottom position, so the push rod completes the "rebound", and the malfunction is fixed;
2. By pressing corresponding buttons, lower the M3, M4 push rods to the bottom position, make them complete "rebound", and fix the malfunction;
3. When the M3, M4 rods are completely malfunctioned, Reverse TR/TR buttons are not valid;
4. When any push rod is malfunctioned, patient mobilization, cardiac position, TR, emergency buttons are not valid.

4.2 Test method for unresponsive side rail control panel

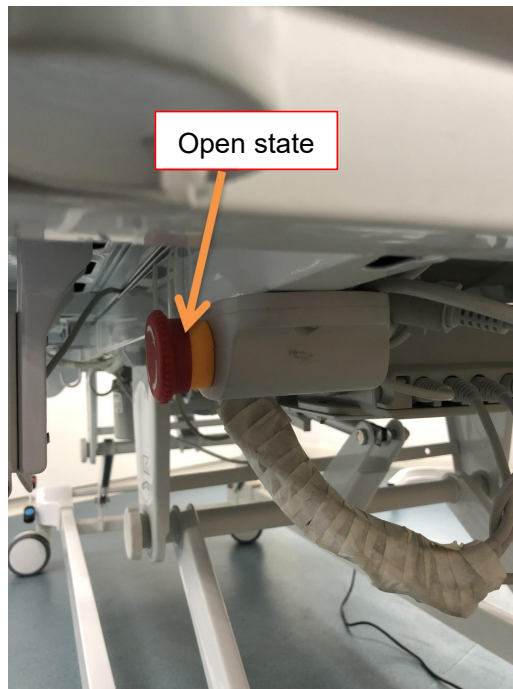
Step 1, check if the emergency stop switch is pressed

A. Check: if the emergency stop switch is pressed;

B. Check method: turn the emergency stop switch clockwise to open(it bounces out for a short distance), see pic 1-1;

C. Define a problem: if the emergency stop switch pops up, it means that it has been pressed; if the emergency stop switch doesn't pop up, it means that it is already in the open state, and there is no need to rotate it;



D. Conclusion: after the above three steps are finished, test the guardrail, if there is still no response, go back to step 2.


Pic. 1-1

Step 2, check whether the guardrail buckle is loose or the pin cores are deformed.

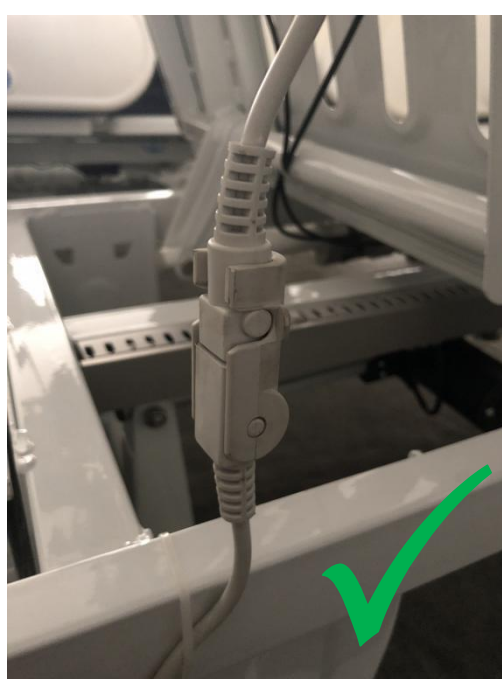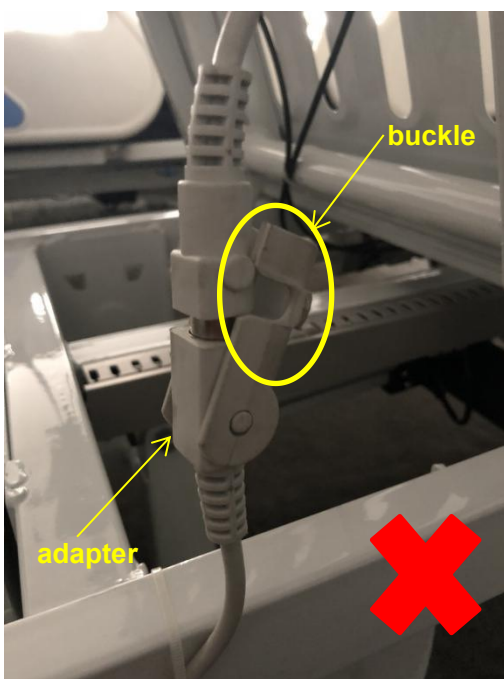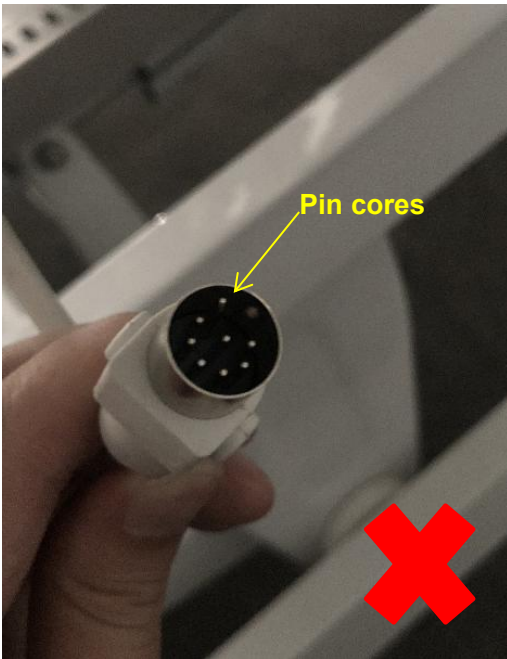- A. Check the content: check whether side rail buckle is loose, or pin cores are deformed;
- B. Check method: follow the connecting line under back board side rail, find the side rail adapter, see pic.1-2;
- C. Problem determination: the buckle is loose, as on pic.1-2 lock the buckle; pin cores are deformed, using a flat-blade screwdriver rectify pin cores, make sure it's connected tightly with the adapter;
- D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, move to the Step 3.


Pic. 1-2

Step 3, Check if the controller plug is loose

A. Check the content: check if the controller plug is loose;

B. Check method:

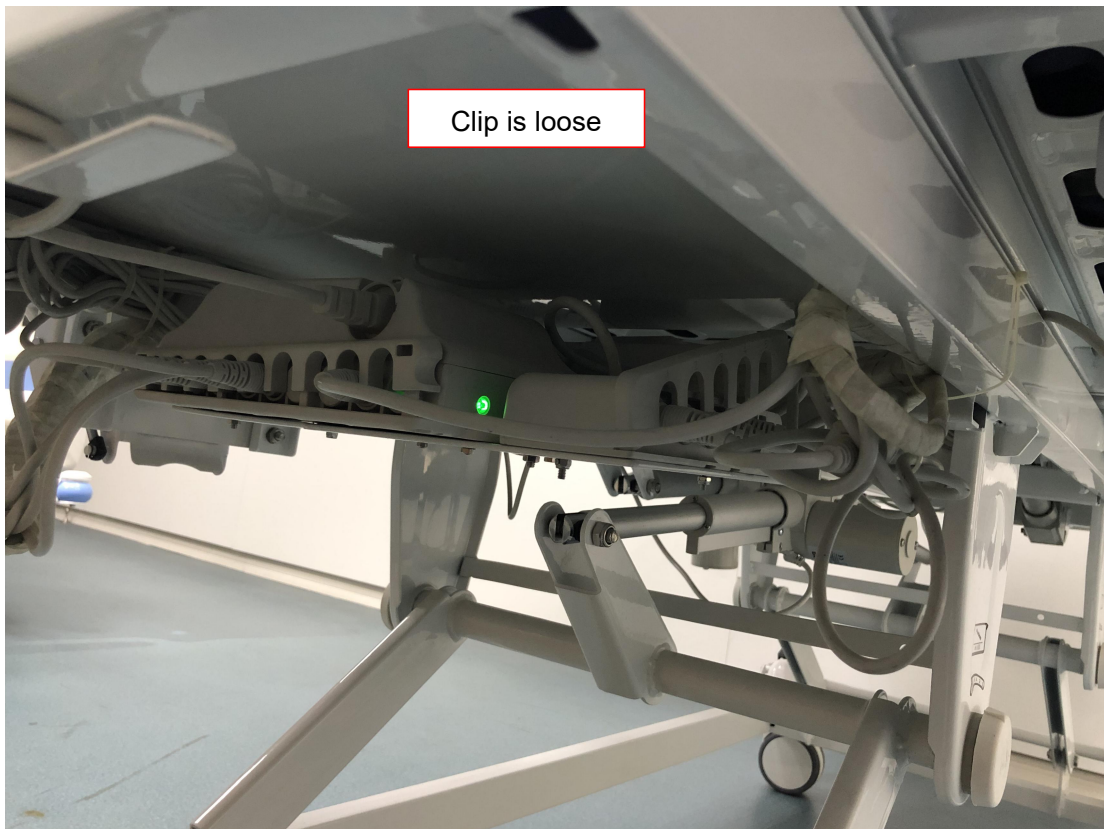- ① Check if the controller clip on the bed frame under the back plate is loose or dislodged, see pic.1-3;
- ② Check if the plug is loose;

Pic.1-3

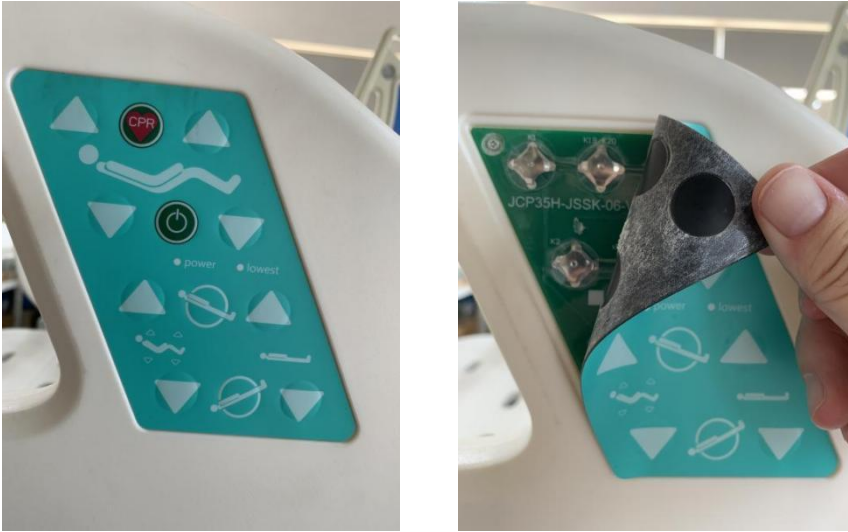
C. Problem determination: loose clips can easily lead to loose plugs, please fasten the clips after the plugs are firmly inserted;

D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, move to the Step 4.

Step 4, Check if the side rail control circuit board connector is loose


A. Check the content: side rail circuit board connector is loose;

B. Check method: peel off the cover sticker from the side rail control panel, using screwdriver unscrew 4 screws, see pic.1-4;

Pic. 1-4

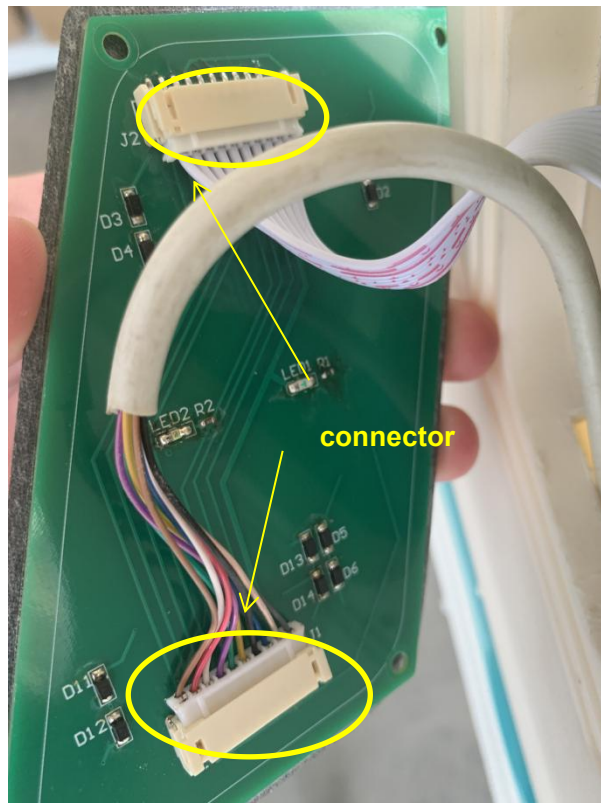C. Problem determination: if the circuit board connector is loose, plug it in tightly, see pic.1-5;

Pic. 1-5

D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, means the control panel is damaged, and should be replaced.

4.3 Test method for unresponsive foot board control panel

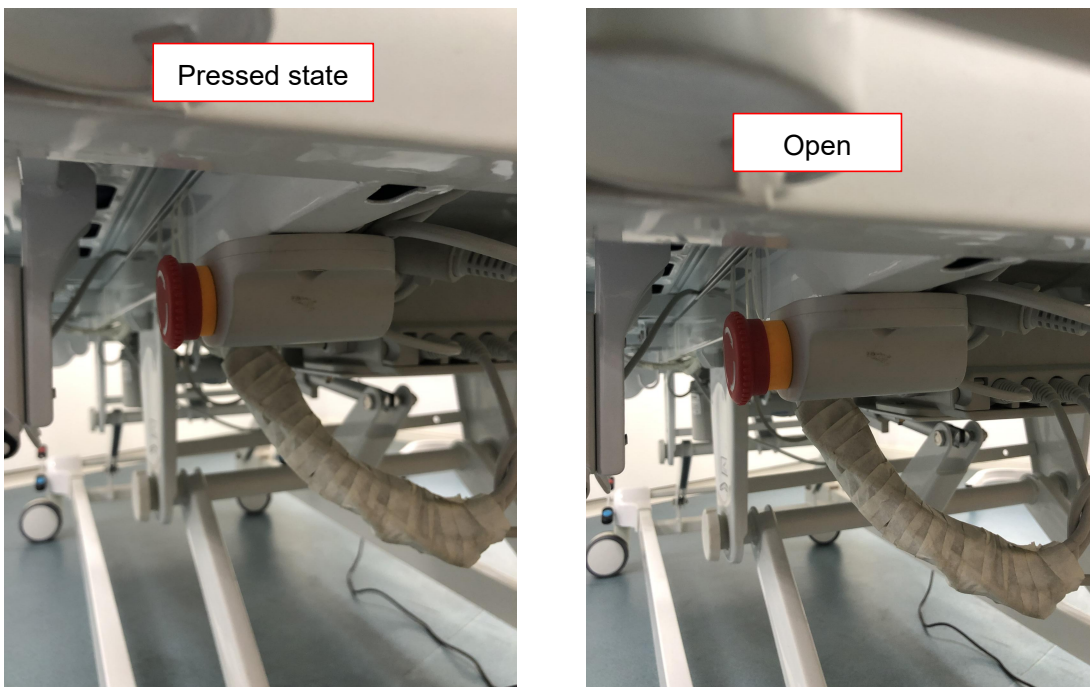
Step 1, check if the emergency stop switch is pressed

A. Check the content: if the emergency stop switch is pressed;

B. Check method: turn the emergency stop switch clockwise to open(it bounces out for a short distance), see pic.2-1;

C. Problem determination: if the emergency stop switch pops up, it means that it has been pressed; if the emergency stop switch doesn't pop up, it means that it is already in the open state, and there is no need to rotate it;

D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, move to the Step 2.

Pic. 2-1

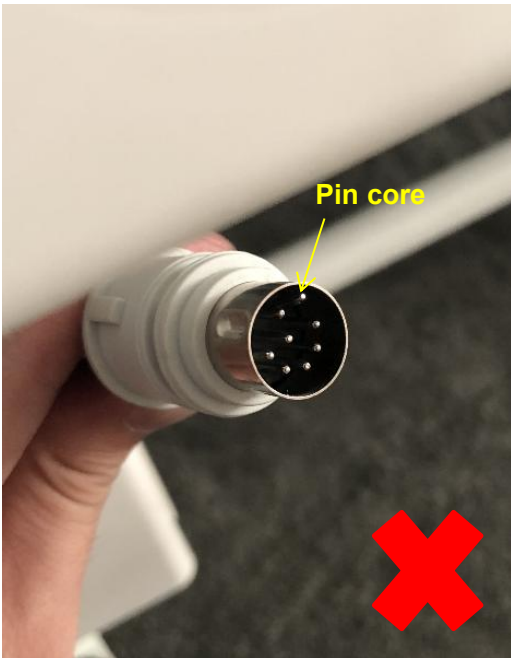
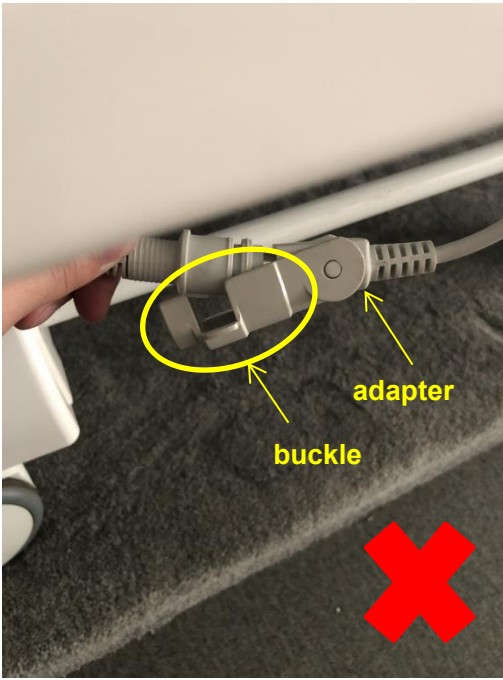
Step 2, check whether the guardrail buckle is loose or the pin cores are deformed

A. Check the content: foot board buckle is loose, pin cores are deformed;

B. Check method: follow the connecting line of the foot board, find the foot board adapter, see pic.2-2;

C. Problem determination: buckle is loose, as on pic.2-2 lock the buckle; pin cores are deformed, using a flat-blade screwdriver rectify pin cores, make sure it's connected tightly with the adapter;

D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, move to the Step 3.


Pic. 2-2

Step 3, Check if the controller plug is loose

A. Check the content: controller plug is loose;

B. Check method:

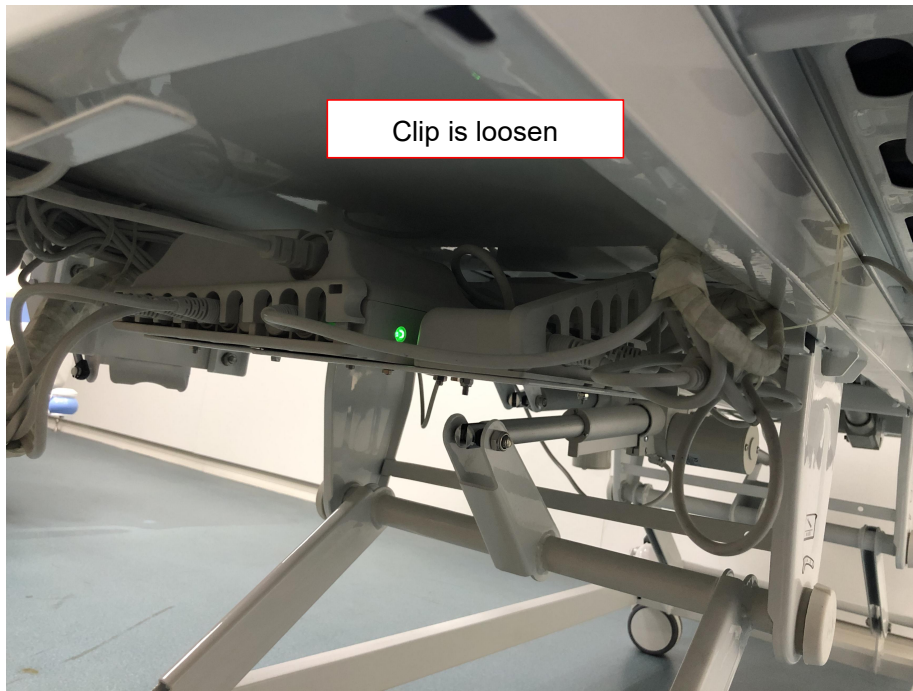- ① Check if the controller clip on the bed frame under the back plate is loose or dislodged, see pic.2-3;
- ② Check if the plug is loose;



Pic. 2-3

C. Problem determination: loose clips can easily lead to loose plugs, please fasten the clips after the plugs are firmly inserted;

D. Conclusion: after the above three steps, check the side rail control panel, if it still unresponsive, means the control panel is damaged, and should be replaced.

5. Bed maintenance

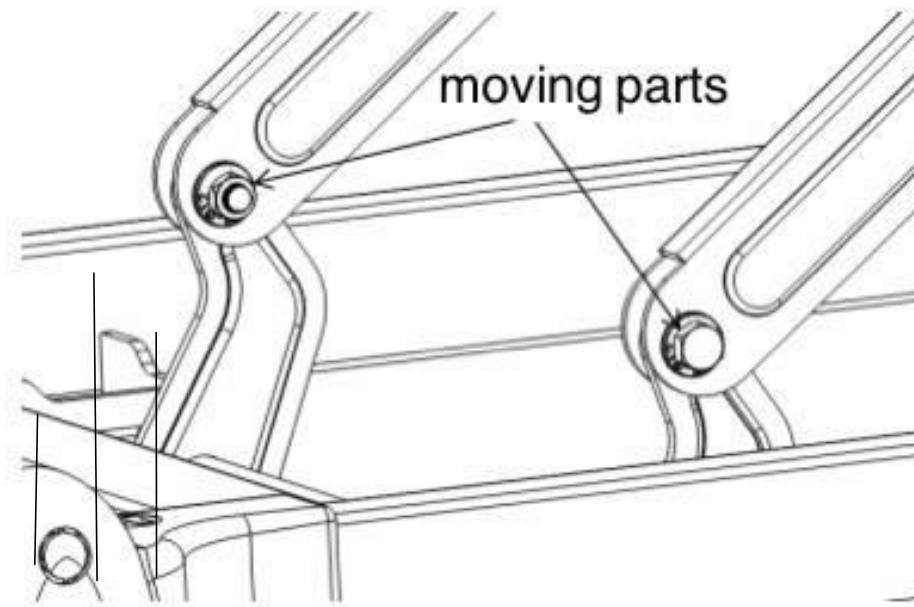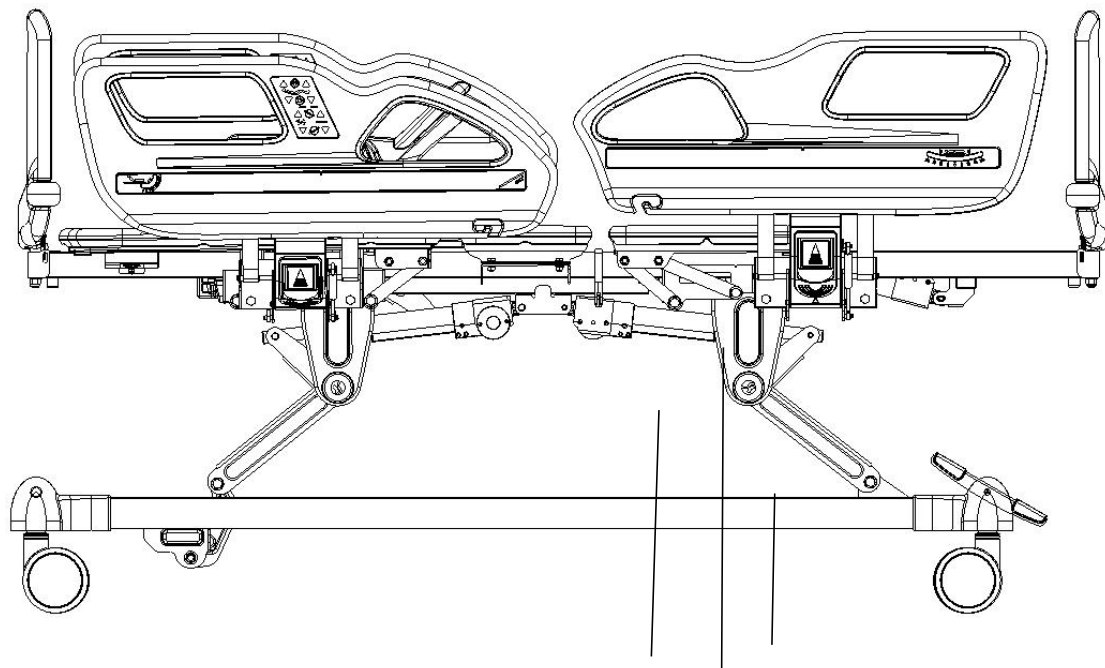1. In order to use the bed safely, it is necessary to conduct regular safety inspections on the bed, it's recommended to conduct a comprehensive inspection every six months to ensure the connection parts are not loose and the lifting function of the bed is operating normally.
2. The connection of the frequently moving parts of the bed needs to be lubricated with an appropriate amount of oil during daily inspection; is serious wear is found, it should be replaced immediately to ensure safety. The moving part in figure below:

3. Avoid hitting the bed and scratching the coating on the bed surface with sharp objects.
4. Please prevent the bed and mattress from absorbing water and moisture, which may cause rust, abnormal noises and bacterial growth. If there was water spilled, please wipe it off immediately.
5. When the bed ages and reaches its service life time, the metal part and the plastic parts can be recycled.

6. Bed clearing precautions

1. When cleaning, please wring the cloth soaked in neutral detergent diluted with water, wipe it dry, then wring the cloth soaked in clean water, wipe off the residual detergent ingredients, and finally wipe with a dry cloth.
2. Do not use volatile items (thinners, propellants, gasoline, etc) that may cause chemical reactions and damage the bed.

3. When cleaning with a disinfectant, be sure to use it after diluting it at the specified concentration. Depending on the composition of the disinfectant, it may corrode metal parts, resin parts, etc, causing discoloration, deformation and other undesirable consequences. Therefore, the recommended content of the disinfectant is as follows:

0.05~0.2% ammonium chloride

0.05~0.2% chlorinated phenyl

0.05% dichlorobenzene biguanide ethane

0.05~0.2% sodium hypochlorite

Do not use a smoking sterilizer or autoclave, and do not use methyl phenol to clean the head foot board, bed feet, etc, which may cause corrosion, discoloration, deterioration.

Note:

1. Unplug power cable before cleaning and maintenance
2. During the cleaning, DO NOT splash water to prevent short circuit and electric shock
3. When cleaning the bed or changing bedding, pay attention to the corners, edges and screw parts of the frame to prevent scratches.

7. Mattress maintenance

1. Avoid scratching the fabric with sharp-angled appliances or knives when using it, cover it with sheets or cleaning pads, and clean it frequently to keep it clean and dry.
2. Regularly clean the mattress cover with a vacuum cleaner, it can be washed directly with water or detergent.
3. Air the mattress often to keep your sleep fresh and comfortable, but be careful not to expose the mattress to the sun for too long.
4. If the fabric is accidentally stained, please refer to the corresponding recommended cleaning method in the instructions. Do not use corrosive chemicals to clean the fabric.
5. If you accidentally get chewing gum or other glue-like objects on the fabric, do not wipe it forcibly. You can use an ice pack to allow it to harden and then peel it off gently.
6. If the thread is found to be loose due to wear and tear, it cannot be broken by hand, and it should be cut short with scissors.

8. Casters inspection and maintenance

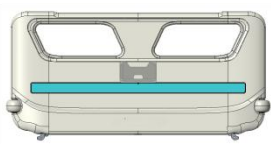
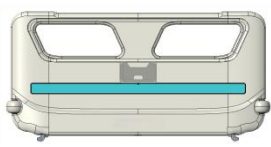
1. Check the casters regularly, avoid hitting heavy objects or overloading, which will cause damage to the casters.
2. The casters should be applied with lubricating grease regularly, the frequency of appliance should be determined according to the specific actual use situation. Under normal conditions, the grease should be applied every six months.
3. After cleaning the caster equipment, pay attention to adding lubricating grease. The selection of lubricating grease should be based on the actual use environment, special high low temperature environment, etc.

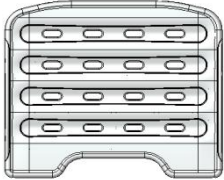
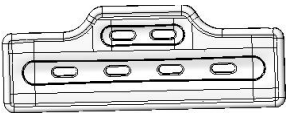
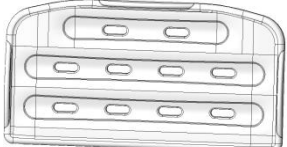
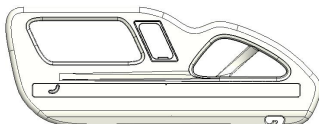
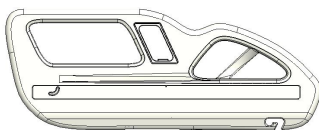
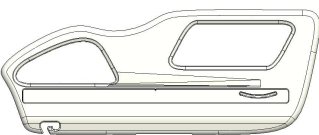
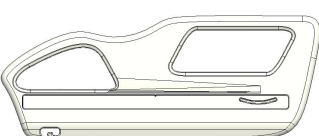
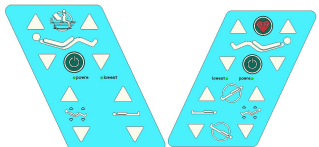
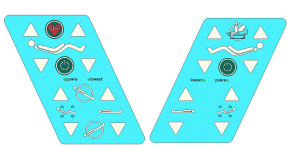
4. It's forbidden to push the bed with force when the casters are braked, which will cause damage to the braking system.
5. The wear of the tire tread of the caster can be detected by visual inspection. Some accumulations such as thread and spun yarn may wrap around the caster. Remove the bolts and nuts of the caster, clean up the debris, and check whether the bearing of the caster is damaged. If the parts are not damaged, the caster can be reassembled and used.
6. Decision whether to replace entire caster is according to the wear condition. After the caster is replaced or reinstalled, make sure that the axle bolt and nut are connected reliably, and use lock washers or lock nuts as much as possible.
7. For casters equipped with brakes, it is necessary to regularly check whether the brakes work normally, and check the brakes every day or before each use. For a hospital bed equipped with multiple brake casters, you can lock only one brake caster at a time and try to push the bed to check whether the braking performance of each caster is good. If the caster brake fails to function due to wear or damage to the caster, replace the caster immediately and retest the brake system.
8. If the mechanism of the brake system is damaged, and the brake needs to be repaired or replaced, please contact our after-sales or an authorized dealer. Every time the brakes are replaced, the brake performance of the casters needs to be retested.

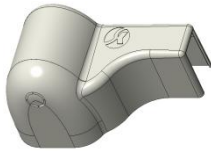
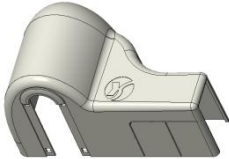
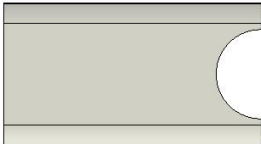
9. Side rails maintenance

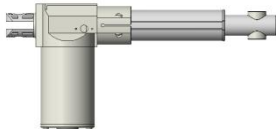
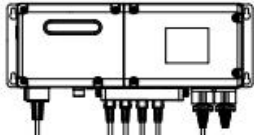
1. Please check whether the side rails are installed correctly to prevent the lying patient from shifting and rolling off.
2. Please always check the screws of the side rail and the side rail fittings, to prevent loosening and falling off.
3. Please use the red part of the side rail handle correctly, do not use brute force to pull hard, which will cause damage to the side rail.

10. List of wearing and spare parts

No	Name	Material code	Picture	Specification
1	V21 head board	GPSL101TS648		990*460
2	V21 foot board	GPSL101TS648		990*460

3	V21 bed board 1	GPSL101TS610		733*903/PP/white
4	V21 bed board 2	GPSL101TS611		333*903/PP/white
5	V21 bed board 3	GPSL101TS613		373*903/PP/white
6	V21 bed board 4	GPSL101TS614		473*903/PP/white
7	V21 back board guardrail,with control panel, left, blow molding	GPSL101TS604		924*350mm/PP/white
8	V21 back board guardrail,with control panel, right, blow molding	GPSL101TS605		924*350mm/PP/white
9	V21 foot board guardrail, left, blow molding	GPSL101TS608		924*350mm/PP/white
10	V21 foot board guardrail, right, blow molding	GPSL101TS609		924*350mm/PP/white
11	V21 guardrail 5-functions control panel, left	LGPDJ101TS820		Back rest, knee rest, back+knee rest, bed height, lock, CPR, patient mobilization
12	V21 guardrail 5-functions control panel, right	LGPDJ101TS819		Back rest, knee rest, back+knee rest, bed height, lock, CPR, patient mobilization
13	Nurse control panel	/		/

14	V21 back board guardrail sticker	LGPFU101TS416		780*35*0.4
15	V21 foot board guardrail sticker	LGPFU101TS417		750*35*0.4
16	Guardrail damper(white)	LGPQT101TS024		18-58-168-35N
17	30-50 drainage hook	BYKBGKM02806		30-50(plastic)
18	Stainless Steel IV pole with 4 plastic hooks	S0FJSY010003		Φ16-19/off-white
19	Foot cover (left)	BYKBGKM02402		white
20	Foot cover (right)	BYKBGKM02403		white
21	bed foot cover	BYKBGKM02404		white
22	Brake pedal (left) A	BYKBGKM02817		green
23	Brake pedal (left) B	BYKBGKM02818		red
24	Brake pedal (right) A	BYKBGKM02819		green
25	Brake pedal (right) B	BYKBGKM02820		red

26	5 inch central control double-sided wheel	BYDYGX105134		/
27	JC motor 200 vertical (manual CPR, Hall)	LGPDJ101TS821		JC35L10Y4424200/385VG DZK2102D0
28	JC knee rest motor 100 horizontal	LGPDJ101TS805		JC35D106324100/275HGD ZD0102D0
29	JC height motor 150 horizontal	LGPDJ101TS806		JC35D1Y6324150/325HGD ZD0102D0
30	JC 3m power cord	LGPDJ101TS694	/	GB 3 plug
31	JC 5-functions controller	LGPDJ101TS818		JCB35R-AM-4-3-G-R0088- 100-03-14
32	Emergency stop switch box	LGPDJ101TS590		JCP35Q-G-4D1-2D0
33	JC 5-functions nurse control panel	LGPDJ101TS721		/
34	JC night light	LGPDJ101TS714	/	JCP35E-LED-5D1-2D0
35	Adapter box JCP35C5	LGPDJ101TS768	/	JCP35C5-4-G
36	JC battery box	LGPDJ101TS713	/	JCP35PA-B-1-G-1.3AH-24- 6D1-2D0

37	JC Adapter cable	LGPDJ101TS722	/	1.5m off-white 10 core crystal head waterproof bend 9-pin waterproof motor socket plug 10-pin transfer cable
38	JC Adapter cable	LGPDJ101TS839	/	1.5m off-white 10 core crystal head waterproof bend 8 core yellow motor straight plug 6-pin patch cord

Note: Due to the renewal of product lineup, the parts of the original product may not be available. (The company ensures availability of the parts for sold products within 6 years after their production ends.)

11. Packaging, transportation and storage conditions

1. The packaging of the manual bed is carried out according to the contract or product standard.
2. During the transportation of the product, avoid rushing out, violent vibration, prevent direct sun and rain access.
3. Store conditions:
 - a) Ambient temperature: $-40^{\circ}\text{C} \sim 70^{\circ}\text{C}$
 - b) Relative humidity: $\leq 95\%$
 - c) Atmospheric pressure: $500\text{hPa} \sim 1060\text{hPa}$
 - d) Voltage: $100-220\pm 22\text{V}$
 - e) Power frequency: $50\pm 5\text{ Hz}$
4. Equipment safety classification is class I, with internal power supply, protection type B.
 - a) operating voltage: 24V
 - b) input power: 300VA
 - c) fuse protector, nominal value: F3AL 250V, 5×20mm
 - d) operating system: intermittent loading and continuous operation
 - e) liquid ingress protection: IPX4
 - f) non-AP/APG type equipment

12. After-sales service

1. Please preserve the attachment papers and invoice of this
2. product, you need to present those to enjoy after-sales
3. maintenance service.
4. If you have any question during use, please contact us any time,
5. so that our company can provide you with accurate and efficient

6. technical support and maintenance services in a timely manner.
7. Starting from the purchase date and during one year, if any malfunction or damage occurs when the product is installed and used correctly according to the instructions, you can enjoy free warranty and lifetime maintenance service presenting a “certificate” or invoice.
8. Do not disassemble the internal parts of this product by yourself to avoid unnecessary damage. If you find that the use is affected by quality issues, please contact the company’s after-sales service department or the authorized distributor.
9. Starting from the date of purchase within 1 year, if the product cannot work normally due to quality problems, the company will repair it for clients for free.
10. The warranty is guaranteed according to the contract requirements. Outside the warranty period, the company is responsible for maintenance, providing accessories, and implementing life-long service for clients.

Correct Disposal

This product is subject to the provision of European Directive 2012/19/EU. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.

Jiangsu Saikang Medical Equipment Co., Ltd.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate
www.vevor.com/support

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australien
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH