

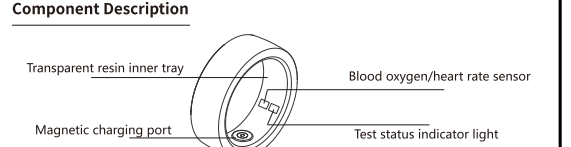


smart ring USER MANUAL

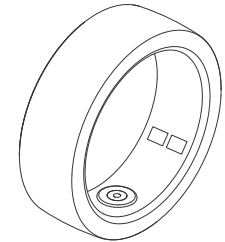

Power on

Align the charging port of the smart ring with the charging compartment (magnetic charging cable) and place the charging port. The indicator light flashes and the machine is turned on.

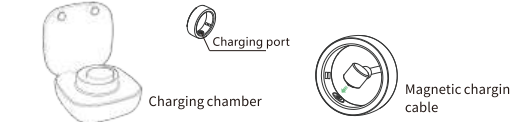

Connect APP

1. Search and download the SmartHealth app, Or scan the QR code on the right to select the appropriate app Version Download




2. Open the software and pair the rings according to the prompts




Lighting status

The red light flashes during charging, and the light turns off when fully charged
The green light flashes during heart rate testing, and the red light flashes during blood oxygen testing

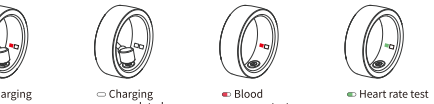

Wearing method

Wear the ring on your suitable finger and choose a comfortable size. It is recommended to wear it on the index finger, middle finger, or none. On the index, it can better monitor data.
Wearing the sensor directly under the finger for more accurate heart rate and blood oxygen measurement

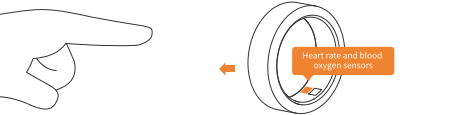

***Note:**
The measurement results of this product are for reference only and are not intended for any medical purposes. Please follow your guidance, doctor. Do not use the measurement results for self diagnosis and treatment.



FCC Requirement

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) this device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation. Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
– Reorient or relocate the receiving antenna.
– Increase the separation between the equipment and receiver.
– Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
– Consult the dealer or an experienced radio/TV technician for help.